Science.
Applied to Life.™

# Total versatility. Uncompromising performance.


3M™ Scotchbond™ Universal Adhesive




#### Simplifying the bonding step is no easy task.

That's why 3M developed 3M™ Scotchbond™ Universal Adhesive. It's a single-bottle solution that offers a simple one-step, one-coat, 35-second application—without compromising strength. Years of clinical evaluations have stacked the evidence. It can be used in all etching techniques, including total-etch, self-etch and selective-etch; in both direct and indirect applications; and on all dental surfaces, without any extra primer—taking versatility to a whole new level.


Use your preferred technique without compromising high, reliable and durable bonds.

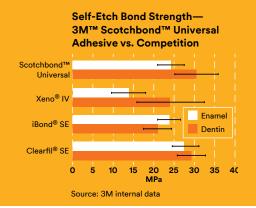

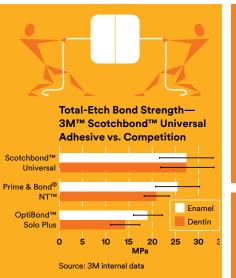

#### Bond to any surface.

Scotchbond Universal adhesive bonds to all surfaces, including enamel, dentin, glass ceramic, zirconia, noble and non-precious alloys and composites—without additional primer. In combination with 3M™ Scotchbond™ Universal DCA Dual Cure Activator, Scotchbond Universal adhesive is compatible with all resin cements, core build-up materials and even self-cure composites!

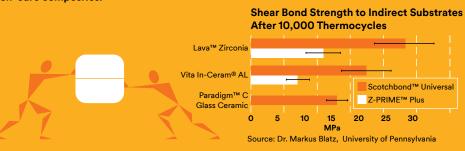

Quick 35-second application time



Virtually no post-operative sensitivity in total-etch or self-etch applications.

Total number of total-etch restorations:

3,467

Total number

of self-etch

restorations:

3,495

0.4%
Percent of total-etch restorations with

restorations with 3M™ Scotchbond™ Universal Adhesive having post-op sensitivity.

0.06%
Percent of

Percent of self-etch restorations with 3M™ Scotchbond™ Universal Adhesive having post-op sensitivity.

Source: 3M internal data

## 3M™ Scotchbond™ Universal Adhesive performs across a range of variables, giving you true versatility.

#### **Direct Placement Indications:**

- Bonding light cured composite or compomer for all classes of direct restorations
- Root surface desensitization
- Sealing of dentin prior to amalgam restorations
- Protective varnish for glass ionomer restorative materials
- Repair of composite or compomer restorations
- Bonding sealants

#### **Indirect Placement Indications:**

- Bonding veneers with 3M™ RelyX™ Veneer Cement
- Adhesive/primer for chemical adhesion when bonding zirconia, alumina, metal or glass ceramic restorations
- Intraoral repair of existing indirect restorations
- Bonding self- or dual-cure core build-up materials and resin cements (with 3M™ Scotchbond™ Universal DCA Dual Cure Activator)
- Sealing of dentin prior to temporization for indirect restoration placement

Refer to Instructions for Use for full list of indications.

- One-step, one-coat adhesive application
- Combined total-etch, self-etch and selective-etch adhesive
- Consistent bond strength to both moist and dry etched dentin
- Virtually no post-op sensitivity
- High bond strength to all indirect surfaces (zirconia, alumina, glass ceramics and metals) without a separate primer
- Excellent marginal integrity for highly esthetic restorations
- Self- and dual-cure compatible (with 3M<sup>™</sup> Scotchbond<sup>™</sup> Universal DCA Dual Cure Activator)
- No refrigeration needed; store at room temperature for two years
- Bonds indirect restorations with 3M™ RelyX™ Ultimate Adhesive Resin Cement— without the need to light cure the adhesive
- Flip-top vial and unit dose delivery



#### Ordering Information









Item #	Product Information
41254	3M™ Scotchbond™ Universal Adhesive Intro Kit—Vial Includes: 1 – 5ml adhesive vial; 50 disposable mixing wells; 50 disposable microbrushes; 1 – 3ml etchant syringe; 25 dispensing tips
41258	3M™ Scotchbond™ Universal Adhesive Refill Includes: 1 – 5ml adhesive vial
41255	3M™ Scotchbond™ Universal Adhesive Intro Kit—Unit Dose 40 Includes: 40 unit dose; 1 – 3ml etchant syringe
41256	3M™ Scotchbond™ Universal Adhesive Unit Dose 100 Includes: 100 unit dose; 1 – 3ml etchant syringe
41264	3M™ Scotchbond™ Universal DCA Dual Cure Activator Includes: 1 – 5ml vial
41263	3M™ Scotchbond™ Universal Etchant Etching Gel Includes: 2 – 3ml syringes; 50 dispensing tips

### www.3M.com/Scotchbond



3M Oral Care 2510 Conway Avenue St. Paul, MN 55144-1000 USA

Phone 1-800-634-2249 Web 3M.com/dental 3M Canada Post Office Box 5757 London, Ontario N6A 4T1 Canada

Phone 1-888-363-3685

3M, ESPE, Lava, Paradigm, RelyX and Scotchbond are trademarks of 3M or 3M Deutschland GmbH. Used under license in Canada. Please recycle. Printed in USA. © 3M 2016. All rights reserved. All other trademarks are not trademarks of 3M. 70-2013-0795-9