Scotchbond™ Universal

Scotchbond™ Universal DCA

- (en) Adhesive / Dual Cure Activator
- (de) Adhäsiv / Aktivator für Dualhärtung
- (fr) Adhésif / Activateur pour la prise duale
- (it) Adesivo / Attivatore per polimerizzazione duale
- (es) Adhesivo / Activador para polimerización dual
- (pt) Adesivo / Activador para dupla polimerização

Instructions for Use
Gebräuchsinformation
Mode d'emploi

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ENGLISH

Product Description
Scotchbond™ Universal is a single-component, light-curing adhesive which is available in L-Pop blisters for single dosing or in bottles for multiple doses.

Depending on the indication, this adhesive is used:

In a "self-curing" procedure to achieve the best possible treatment time and to minimize post-operative sensitivities.

- With selective enamel etching to maximize the adhesion to the tooth enamel and to minimize post-operative sensitivities.

- In a "total etching" procedure with a prior phosphoric acid etching step, e.g., if the etching cannot be restricted to the tooth enamel.

Scotchbond™ Universal activates the dual curing mechanism of Scotchbond Universal and is mixed with the adhesive whenever dual-cure or self-cure composite filling materials, cements, or core build-up materials are being used.

The addition of the activator to the adhesive is not required when using RelyX™ Ultimate Adhesive Resin Cement because the activator is already integrated into the cement.

► These Instructions for Use must be kept for reference for the duration of product use. Please see the pertinent information for use for details on all of the products mentioned below.

Indications

• All classes of fillings (according to Black) with light-curing composite or compomer filling materials

• Cementation of indirect restorations when combined with RelyX Ultimate Adhesive Resin Cement

• Cementation of veneers when combined with RelyX Veneer Cement

• Bonding of core build-ups made of light-curing composite or core build-up materials

• Bonding of dual-cure cements and core build-up materials and self-cure composites when combined with Scotchbond Universal DCA

• Repair of compomer or compomer fillings

• Intraoperative repair of composite restorations, porcelain fused to metal, and all-ceramic restorations without extra primer

• Root surface desensitization

• Sealing of cavities and preparation of tooth stumps prior to temporary cementation of indirect restorations

• Bonding of fissure sealants

• Protective varnish for glass ionomer fillings

Precautionary Measures

For Patients and Dental Personnel

• Scotchbond Universal: contact with eyes may cause severe eye damage.

Wear eye protection to prevent injury. In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.

For Patients

• Scotchbond Universal contains substances that may cause an allergic reaction by skin contact in certain individuals. Avoid use of this product in patients with known acrylic allergies.

• If prolonged contact with oral tissue occurs, flush with large amounts of water. If allergic reaction occurs, seek medical attention as needed, remove the product if necessary, and discontinue future use.

For Dental Personnel

• Scotchbond Universal contains substances that may cause an allergic response, minimize exposure to these materials. In particular, avoid exposure to the tooth structure.

• Use protective gloves and a no-touch technique is recommended.

Acrylates may penetrate common utility gloves. If Scotchbond Universal contacts the glove, remove and discard the gloves. Wash your hands immediately with soap and water or re-glove.

If allergic reaction occurs, seek medical attention as needed.

• Scotchbond Universal and Scotchbond Universal DCA are highly flammable. Avoid static electricity build-up. Keep away from heat sparks, open flame, pilot lights, and other ignition sources.

3M MSDS can be obtained from www.3m.com, or contact your local subsidiary.

Precautions during Processing

• Seal temporary restorations using a eugenol-free product (e.g., with RelyX™ Temp NE). The use of temporary cements containing eugenol can inhibit the polymerization process of Scotchbond Universal during the final cementing.

• For the best possible bond, do not treat the surface of Zirconia, aluminum oxide, or metal restorations with phosphoric acid.

Pulp Protection

To avoid pulpal irritation, cover areas in close proximity to the pulp by applying small amounts of hard-setting calcium hydroxide material. To avoid bite increases during the cementing of the later restoration, the pulp protection should be applied prior to taking an impression for the final restoration.

Tooth Preparation

Preparation of the Cavity/Tooth Stump

To avoid pulpal irritation, cover areas in close proximity to the pulp by applying small amounts of hard-setting calcium hydroxide material. To avoid bite increases during the cementing of the later restoration, the pulp protection should be applied prior to taking an impression for the final restoration.

Procedure of the Cavity/Tooth Stump

► Clean the tooth/stump area with water and lightly air dry.

► The stump or the cavity in only 2-3 bursts of water-free and oil-free air, or use cotton pellets to dry off. Do not overdry.

► The cavity should be just dry enough that the surface has a slightly glossy appearance. Overdrying can lead to post-operative sensitivity.

We advise against the use of substances such as desensitizers, disinfectants, astringents, dentin sealants, rinsing solutions containing EDTA, hydrogen peroxide, etc. (with the exception of chlorhexidine solutions) after the final cleaning. The residues of these substances can be detrimental to the bonding strength and setting reaction of the adhesive, especially of dual- and self-cure composites.

Unprepared Tooth Enamel

Clean the tooth enamel as usual (e.g., with polishing paste or powder blasting), and thoroughly rinse by spraying with water.

► Then etch and rinse, see section "Selective Enamel Etching".

Etching of the Tooth Structure (optional)

Clinically adequate adhesive bonds are achieved by the application of the adhesive. Even higher adhesive values can be achieved by selectively etching the enamel or using the "total etching procedure".

Selective Enamel Etching

Accidental etching of the dentin does not have any detrimental effect on the adhesion.

► Apply a commonly used phosphoric acid etching gel (about 35%), e.g., Scotchbond Universal Etilentac to the prepared and unprepared (if present) tooth and enamel and allow to react for 15 sec.

► Rinse thoroughly with water and dry with water-free and oil-free air or with cotton pellets; do not overdry.

Total Etching Procedure

► Apply a commonly used phosphoric acid etching gel (about 35%), e.g., Scotchbond Universal Etilentac to the prepared and unprepared (if present) tooth structure (enamel and dentin) and allow to react for 15 sec.

► Rinse thoroughly with water and dry with water-free and oil-free air or with cotton pellets; do not overdry.

Root Surface Desensitization

Clean the root surface as usual (e.g., with polishing paste or powder blasting).

► Clean thoroughly by spraying with water, and dry with water-free and oil-free air or with cotton pellets.

► Continue with the application of Scotchbond Universal Adhesive.

Pre-treatment of Fillings or Restorations During Repair

(most of composite, glass ceramic, Zirconia ceramic, aluminum oxide ceramic, metal (incl. amalgam))

► Clean the surface to be sealed with polishing paste or by powder blasting (e.g., using Clinpro Prophy Paste).

► Use the disposable applicator to apply Scotchbond Universal to the surface to be sealed and rub it in for 20 sec, applying pressure. Continue to rub in for a longer period of time when sealing larger surfaces. The stickiness of the adhesive will gradually decrease, so that the surface feels seidematt (smooth). To starker Trocken kann zu postoperativen Sensibilitäten führen.

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No do light-cure

► Carefully cement the veneer with RelyX™ Veneer Cement following the pertinent instructions for use. Remove any excess from the veneer edges and light-cure in accordance with manufacturer's instructions. We recommend light-curing gingival edges first, then the middle and the incisal edges. Avoid eye contact with the light source.

Procedure for light-cure sealing of fissures

In contrast to a composite filling, light-cure sealants can be bonded to untreated tooth enamel using Scotchbond Universal without prior etching with phosphoric acid.

► Clean the surface to be sealed with polishing paste or by powder blasting (e.g., using Clinpro Prophy Paste).

► Use the disposable applicator to apply Scotchbond Universal to the surface to be sealed and rub it in for 20 sec, applying pressure. Continue to rub in for a longer period of time when sealing larger surfaces. The stickiness of the adhesive will gradually decrease, so that the surface feels seidematt (smooth). To starker Trocken kann zu postoperativen Sensibilitäten führen.

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3M, ESPE, Clinpro and RelyX are trademarks of 3M or
© 2013, 3M. All rights reserved.
3M, Clinpro and RelyX are trademarks of 3M or 3M Deutschland GmbH. Used under license in Canada. © 2013, 3M. All rights reserved.