Rx ONLY

Description

Integra™ Miltex® Aspirating Syringes are used in conjunction with a needle and a 1.8cc pre-filled anesthetic cartridge (not supplied by Integra™ Miltex®). Aspirating syringes are available in Type CW (Cook-Waite) and Type A (Astra). The syringes are reusable and are provided clean and supplied non-sterile.

Indications

Integra™ Miltex® Aspirating Syringes are indicated to be used in conjunction with anesthetic needles and anesthetic cartridges for injection of anesthetic solutions in the oral cavity.

Contraindications

None known.

Warnings

- To avoid needle sticks, re-cap needle after injection has been given.
- To avoid leakage during an injection, ensure that needle is straight as it penetrates the cartridge diaphragm.

Precautions

- Syringes must be sterilized prior to use. See sterilization instructions.
- Hubs cannot be removed. Do not attempt to remove hub from syringe for sterilization or for any reason as damage to the syringe will occur.
- Do not mix metals when sterilizing syringes. Sterilizing syringes with other non chrome-plated brass instruments may cause damage to syringes.
- Do not use cold sterilants, as they will react to the chrome plating of the syringe.
- Keep needle capped while affixing or removing from syringe.

Instructions

Loading Instructions

1. Retract piston rod fully.
2. Insert 1.8cc cartridge, plunger end first, into syringe barrel.
3. Hold cartridge firmly in place. Push piston rod forward with a quick jab.
4. Affix needle to aspirating syringe.
5. Expel a few drops of solution to clear needle.
6. Aspirate before injecting solution after inserting needle into tissue.
7. After positioning the needle, pull back on thumb ring to aspirate.

Unloading Instructions

1. Remove needle from syringe.
2. Withdraw piston rod to clear cartridge for removal.
3. Tilt syringe to expel cartridge.

Cleaning Instructions

- For best and long service life, syringes should be processed according to Integra™ Miltex® Care and Cleaning guidelines.
- Remove all traces of blood, saliva, and anesthesia solution.
- Remove paraffin that has accumulated on the harpoon.

Sterilization Instructions

- Syringes are supplied non-sterile and must be sterilized prior to each use.
- Syringes should not be sterilized with other instruments to avoid syringe damage.
- Before each use, sterilize and inspect instrument. Sterilize syringes according to an approved National/International Standard. Integra™ Miltex® syringes have been tested with no adverse effects under the following sterilization parameters in a standard, gravity steam, sterilizer:
 - Temperature: 250°F (121°C)
 - Exposure Time: 30 minutes
 - Dry Time: 20 minutes
- Do not use cold sterilants, as they will react to the chrome plating of the syringe.

Symbols

Symbols may be used on some package labeling for easy identification.

Manufacturer

Authorized Representative
in the European Community

Catalog number

Lot number

Caution: see warnings or precautions

Product complies with requirements of directive
93/42/EEC for medical devices

Rx ONLY

Federal (USA) law restricts this device to sale by or
on the order of a physician or practitioner

Consult instructions for use and website address

Manufacturer
Integra York PA, Inc. • 589 Davies Drive, York, PA 17402 USA
866-854-8300 USA • 717-840-2763 outside USA • 717-840-9347 fax
integralife.com/integra-miltex

Integra LifeSciences Services (France) SAS Immeuble Sequoia 2
97 allée Alexandre Borodine
Parc Technologique de la Porte des Alpes
69800 Saint Priest-France
Telephone: 33 (0) 4 37 47 59 10
Fax: 33 (0) 4 37 47 59 29

Integra and the Integra logo are trademarks of, and Miltex is a registered trademark of, Integra LifeSciences Corporation or its subsidiaries.
©2011 Integra LifeSciences Corporation. All rights reserved.

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Integra York PA, Inc. • 589 Davies Drive, York, PA 17402 VS
©2011 Integra LifeSciences Corporation. Alle rechten voorbehouden.