3M ESPE

CoJet™ System

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3M, ESPE, CoJet, Elipar and Sinfony are trademarks of 3M Company or 3M Deutschland GmbH.
Used under license in Canada. © 2012, 3M. All rights reserved.

ENGLISH

Product Description

CoJet System is a set of products for intraoral repair of composite fillings, fixed restorations, and for pre-treatment of restorations prior to adhesive or conventional cementation. The system consists of the following components:

- CoJet™ Sand: blast-coating agent for silicization of metal, ceramic or composite restorations.
- ESPE™ Sil: silane solution for silanization of silicized metal, ceramic or composite surfaces.
- Visio Bond: bonding between ceramic or composite restorations and a composite filling material.
- Sinfony™ Opaque®: for covering of metal surfaces prior to building-up a composite.

☞ Instructions for Use should not be discarded for the duration of product use.

- (1) Adhesive repair of defects in ceramic or composite restorations without exposed metal
- (2) Adhesive repair of defects in ceramic or composite restorations with exposed metal
- (3) Pre-treatment of ceramic, composite or metal restorations prior to adhesive cementation
- (4) Mechanical increase of the retentive surface of metal restorations prior to conventional cementation

Precautionary Measures

When processing the individual products, please observe the section Precautionary Measures shown in the respective Instructions for Use.

3M MSDSs can be obtained from www.mmm.com or contact your local subsidiary.

Preparation

- In intraoral applications place a rubber dam.
- Use e.g. a matrix to cover dental areas that are not to be coated.
- Dental personnel and patient must wear suitable protective goggles; the dental personnel must also wear respiratory protection.
- Gently smooth the surface of ceramic or composite restorations to be repaired using rotary instruments to render the surface evenly accessible to blasting. Then clean with a water spray and dry with air.

Processing

Please use the products as required for the application at hand, and comply with the specified sequence of use. The details of processing of the individual products are shown after the following list:

- Adhesive repair of defects in ceramic or composite restorations without exposed metal:**
CoJet Sand – ESPE Sil – Visio Bond – standard composite filling material used in compliance with the recommendations of the manufacturer.
- Adhesive repair of defects in ceramic or composite restorations with exposed metal:**
CoJet Sand – ESPE Sil – Sinfony Opaque® – Visio Bond – standard composite filling material used in compliance with the recommendations of the manufacturer.
- Pre-treatment of ceramic, composite or metal restorations prior to adhesive cementation**
CoJet Sand – ESPE Sil – standard composite fixation cement used in compliance with the recommendations of the manufacturer.
- Mechanical increase of the retentive surface of metal restorations prior to conventional cementation**
CoJet Sand – conventional cement used in compliance with the recommendations of the manufacturer.

Processing of CoJet Sand:
For details on the processing of CoJet Sand, please refer to the material's Instructions for Use.

- Fill the CoJet Sand into the micro-blasting unit.
 - The blast-coating agent must be clean and dry!
 - Set the air pressure to 2–3 bar (30–45 psi). If the air pressure is less than 2 bar, the level of impact energy will be too low and no coating with silicates will occur. If the air pressure exceeds 3 bar, the workpiece will be coated with silicates, but there will also be unnecessary abrasion.
- Test the operation of the blasting unit on a work sample
- Blast from a distance of 2–10 mm and perpendicular to the surface of the work sample while providing for aspiration.
- Evenly coat the entire surface for 15 sec – include exposed metal surfaces in coating.
- Fifteen sec of coating are sufficient for surfaces of the size of a veneering unit, whereas smaller or larger surfaces should be treated for proportionally shorter or longer times, respectively.
- If the pressure is appropriate and the entire surface is coated, metal surfaces become evenly dark. Working with ceramic or composite parts, the success of coating cannot be ascertained by inspecting the color of the dental work. Thus, it is essential to strictly comply with the specified coating times!

- Remove any residual blast-coating agent with a stream of dry, oil-free air.
 - Avoid any contamination by fluids, e.g. saliva, to prevent detrimental effects on bonding.
- Processing of ESPE Sil:**
For details on the processing of ESPE Sil, please refer to the material's Instructions for Use.

The use of ESPE Sil is not required in CoJet Sand applications, which only aim at roughening the surface of metal or metal-ceramic restorations for subsequent conventional cementation.

- Directly after coating, dose ESPE Sil into a Dappen dish and use a clean brush to wet the entire surface with the solution.

- Allow the silane solution to dry for **30 sec in intraoral applications** or for **5 min if applied extraorally**.
- Avoid any contamination by fluids, e.g. saliva, to prevent detrimental effects on bonding.

Processing Sinfony Opaque:

- For details on the processing of Sinfony, please refer to the material's Instructions for Use.
- Dose the powder and liquid of the desired opaque® color in a Dappen dish at 1:1 ratio.
 - Do not use coated mixing pads, since any abraded material has a detrimental effect on the bonding to metal.
 - Do not stir the opaque® on a level surface to prevent the solvent from evaporating too rapidly, which would negatively affect the product properties.
- Use a plastic spatula and stir for 45 sec until a creamy consistency is attained.
- Then use a fine clean brush to apply a thin layer of opaque® exclusively to the metal surface.
 - The metal must be completely covered. For aesthetic reasons, do not cover ceramic or composite surfaces with opaque®.
 - The processing time is 5 min; a 0.1 mm layer of opaque® forms a continuous coat.
- Subsequently cure with a halogen light unit, e.g. Elipar™ for 10 sec.

Processing of Visio Bond:

- For details on the processing of Visio Bond, please refer to the material's Instructions for Use.
- Dose Visio Bond onto a mixing pad.
- Use a disposable brush to apply a thin layer of the material to the silanized surface.
 - Any exposed metal coated with opaque® during the repair process does not need to be wetted with Visio Bond, but it is not detrimental to wet the opaque®.
- In pairs, light-cure Visio Bond for 20 sec for each veneer surface.

Processing of Filling Material/Fixation Cement

- In repairs and cementations, apply the desired composite or cement, respectively, complying with the corresponding Instructions for Use.

Storage and Shelf-life

Store the product at 15–25°C/59–77°F.

Do not use after the expiration date.

Customer Information

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

Warranty

3M Deutschland GmbH warrants this product will be free from defects in material and manufacture. 3M Deutschland GmbH MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M Deutschland GmbH's sole obligation shall be repair or replacement of the 3M Deutschland GmbH product.

Limitation of Liability

Except where prohibited by law, 3M Deutschland GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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