Brownie Greenie Supergreenie

03:2018-02
300030455

EN

Brownie, Greenie, Supergreenie silicone polishers are for polishing dental restorations. Brownies are for prepolishing, Greenies are for polishing and Supergreenies (with yellow banded shank) are for superpolishing.

PRECAUTIONS

- Never exceed the maximum permissible rotation speeds described in Table A.
- Use FG type with lower speed airmotor handpiece.
- High speed air turbine units can break cups or points.
- Read all instructions on the back side carefully before use.

STERILIZATION

- Perform autoclave under the conditions described in Table B.

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

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(CAUTIONS)

- Never exceed the maximum permissible rotation speeds. ●Use FG type with lower speed airtight handpiece. High speed air turbine units can break cups or points. ●Mount an unmounted wheel onto the mandrel securely. ●Insert the shank completely into the chuck of the handpiece. ●Do not use damaged or defective instruments i.e. corrosion, surface flaw, deformation, dirt and exposure of shank due to repeated use. ●Perform test run to check safety and smooth operation without run-out prior to use. ●Avoid any use at acute angles or with excessive pressure. ●Always use an unmounted wheel in clockwise motion. ●Use local dust extractor and protective equipment while using. ●Amalgam should be polished under running water to avoid heat generation. Thoroughly remove amalgam dust. ●Avoid exposure to moisture, corrosive chemical and its vapor. Keep away from external pressure (physical load) and avoid contamination. ●This product is intended for use by dental professionals only. ●Do not use this product for any purposes other than the intended use.

(STERILIZATION)

- Perform autoclave for 3 min at 134°C (273°F) or 30 min at 121°C (250°F) for sterilization. ●When using intraorally, sterilize before each use. ●For reuse after using intraorally, remove any contaminants immediately with cleaning liquid or disinfectant, then sterilize. ●Do not immerse into chlorine disinfectant for a prolonged time to avoid corrosion. ●Do not perform dry heat sterilization. ●Follow the instructions for use of cleaning liquid, disinfectant and sterilizer, respectively.

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