ENGLISH
FLUORIDE RELEASING FLOWABLE RESTORATIVE MATERIAL

PRECAUTIONS FOR DENTAL PERSONNEL AND PATIENTS

- Do not use this product for any purposes other than specifically outlined in the Indications. This product is intended for use by dental professionals only.
- If any inflammation or other allergic reactions occur on either patient or operator, immediately discontinue use and seek medical advice.
- Do not use this material on patients or operators with known allergies to methacrylate monomer.
- Avoid contact with soft tissue, skin or eyes. In case of accidental contact, immediately rinse with copious amounts of water and seek medical advice.
- Use medical (dental) gloves and protective glasses to avoid sensitization to this material. In case of accidental contact, immediately blot with an alcohol moistened cotton ball, and rinse with plenty of water. In case of contact with eyes, immediately rinse with copious amounts of water and seek medical advice. Medical (dental) gloves protect against direct contact with this material; however, some monomers penetrate very quickly to reach skin through the gloves. Therefore, should this material contact even the gloves, immediately remove them and wash hands with copious amounts of water.

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- After cleaning and removing stains from the teeth, take shade while the tooth is still moist.
- Prepare the cavity in conventional method and isolate the prepared tooth with a rubber dam. If pulp exposure occurred or if the cavity is deep, cap the pulp with calcium hydroxide material. (Do not use Eugenol lining material.)
- Apply bonding agent following the manufacturer's Instructions for Use. Either FL-BOND II or BeautiBond are recommended.
- Pull off the shield cap from the syringe and attach a needle tip by turning it clockwise until it is securely attached. Apply necessary amount of BEAUTIFIL Flow Plus directly into the cavity. BEAUTIFIL Flow Plus may also be extruded onto a dispensing pad to be applied into cavity with a suitable instrument. When the cavity is deep, incremental filling and light-curing not exceeding 2mm is recommended. Syringe should be stored according to 3 in [Note on Use] after use.
- Light-cure each layer for 20 seconds with a halogen light unit or for 10 seconds with LED light-curing unit. When the cavity surface is large, light-cure in segments.
- Remove excess material with suitable finishers such as "Dura White Stones", "T&F Hybrid points" or "Super-Snap strips" for interproximal removal and after adjusting occlusion, use polishers such as "CompoMaster", "Super-Snap" or "OneGloss" for final polishing.

Repair of restorations and prostheses:

- Prepare according to the kind of material (resin, porcelain, metal or tooth). Follow the manufacturer's Instructions for Use for application and curing procedures of etching, priming and bonding. Either FL-BOND II, BeautiBond, M.L. Primer or Porcelain Primer are recommended.
- Follow the steps 4-6 in [Direct restorations for anterior and posterior teeth]) as described above.

NOTE ON USE

- Wear protective glasses when light-curing.
- Needle tip is for single use. It can be used only on one patient. Discard after each use.
- After use, remove the needle tip from the syringe by turning it counter-clockwise and discard. Attach a new needle tip and insert the needle into the small hole on the head of the shield cap.
- Carefully handle the shield caps since they are fragile.
- Keep out of direct sunlight and store at room temperature (1-30 °C / 34-86 °F).
- The paste dispensed onto the paper pad should be shielded from ambient light by light-shielding cover.
- Only dental professionals must be in charge of use and storage of this product.
- Use this product within the expiration date.

COMPOSITION	
Base-resin:	Bis-GMA/TEGDMA resin
Filler:	Multi-functional glass filler and S-PRG filler based on fluorobororaluminosilicate glass
Filler loading:	67.3wt%, 47.0vol% (F00) and 66.8wt%, 46.3vol% (F03)

SHADES	
F00	A1, A2, A3, A3.5, A4, B1, B2, C2, D2 / SA0.5, SA0.50, A10, A20, Inc, BW
F03	A1, A2, A3, A3.5, A4, B1, B2, C2, D2 / SA0.5, A20, A30, Inc, BW, MI (milky), CV (cervical)

STORAGE

Store in a dark room at temperature between 1-30 °C / 34-86 °F. Avoid high temperatures and high humidity. Keep out of direct sunlight and away from any source of ignition.

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

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