OneGloss® PS

English

Finishing and Polishing

The OneGloss® PS polisher was developed for finishing and polishing of composite restorations, enamel surfaces and removal of excess cements. To complete the finishing and polishing steps, alter the contact pressure of the OneGloss® PS polisher.

Indications

Finishing and polishing of composite restorations.
Removal of excess resin or cements on tooth surfaces without marring the enamel.
To remove cement and to polish enamel after orthodontic bracket removal and de-banding.
Polishing enamel after tooth scaling.
To remove stains from tooth surface.

Directions for Use

• Polishing composite

1. Apply heavy pressure for finishing.
For gross reduction, the use of Shofu's T&F Hybrid points or Super-Snap coarse (black disks) is recommended.
2. Apply light pressure for polishing.

• Removal of excess resin and cements

1. Remove excess cements and polish tooth surfaces using moderate to a light touch.

• Polishing enamel and stain removal

1. Use a light touch to polish enamel and remove stains.

NOTE: Use intermittent water spray when applying heavy pressure or polishing enamel. OneGloss® PS is a completely disposable product and should be discarded after single patient use to avoid weakening or breakage of the plastic shank due to multiple uses. The plastic shank may slightly discolor over time while in storage. This will not effect the quality or performance of the OneGloss® PS.

OneGloss® PS: Details and Order Information

Packaging: 50 pcs./box

Product Name	Recommended regions to polish	PN
1 OneGloss® PS Assorted Cup & IC: 15 pcs. Midi: 20 pcs.		0175
2 OneGloss® PS Cup	cusp, incisal, occlusal	0176
3 OneGloss® PS Midi	occlusal, labial, lingual, buccal	0177
4 OneGloss® PS IC (Inverted Cone)	labial, lingual, buccal, interproximal	0178

OneGloss® PS

Recommended r. p. m.	Maximum r. p. m.	Recommended pressure
3,000 – 5,000	5,000	0.3 N – for polishing 1.0 N – for finishing

Caution

- Operate each polisher up to maximum permissible speed only.
- Insert the shank completely into the chuck of the handpiece.
- Rotate each polisher each time prior to use to check smooth run and safety.
- Operate at recommended pressure only.
- Do not use at an acute angle.
- Wear eye protecting glasses when using.
- Use these products for indicated purpose only.
- Discard OneGloss® PS after single use.

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Manufactured by
SHOFU INC. 11 Kamikamatsu-cho, Fukuro, 154-8544 Tokyo, Japan
SHOFU INC. Singapore Branch 10 Science Park Road #02-12, The Alpha, Science Park II, Singapore 117664
SHOFU DENTAL CORPORATION 1225 Stone Drive, San Marcos, CA 92076-4093 USA
SHOFU DENTAL PRODUCTS LTD. Duke's Factory, Chiddingfold, Tonbridge, Kent, TN11 8 JLUK
SHOFU DENTAL SUPPLIES (Shanghai) CO. LTD. No. 645 Jiyue Road, Sheshan Industry Park, Songjiang 201802, Shanghai, China
SHOFU DENTAL GmbH Am Brühl 17, 40878 Ratingen, Germany
www.shofu.de