


BEAUTIFIL II

Continually Fluoride Releasing
Aesthetic Dental Restorative Material

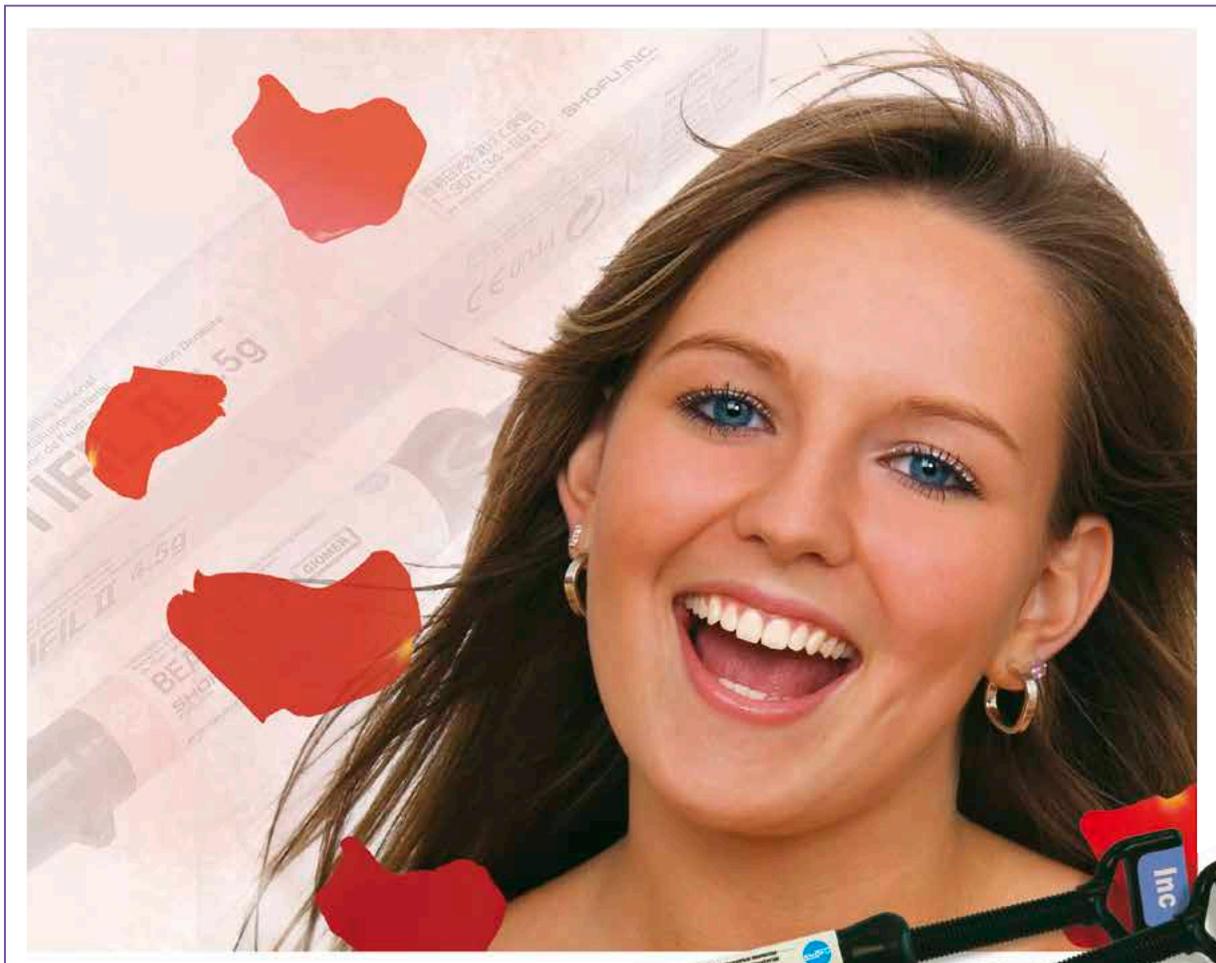


PRG
Technology
Giomer



Research is our best product



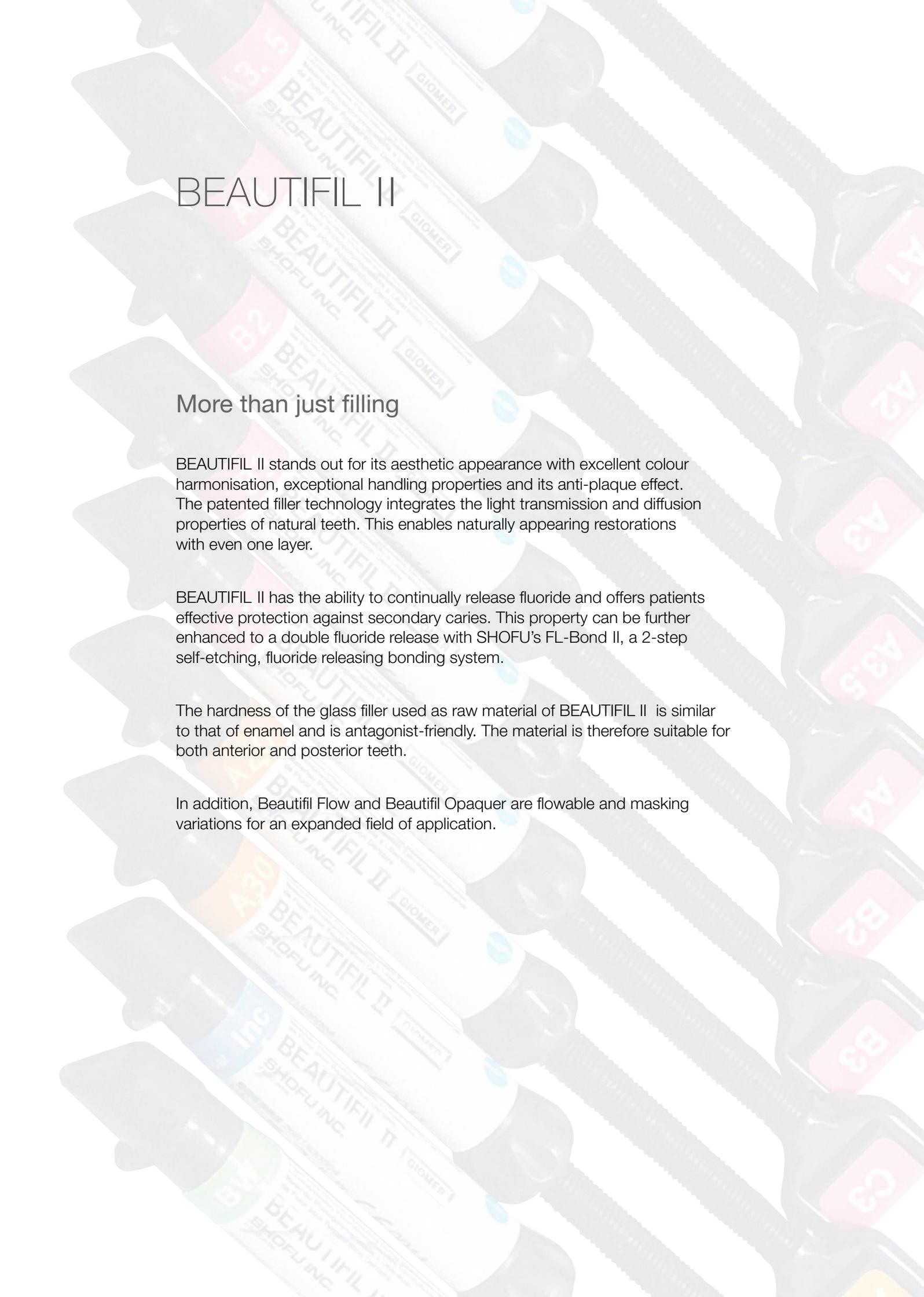

BEAUTIFIL II

More than just filling

BEAUTIFIL II stands out for its aesthetic appearance with excellent colour harmonisation, exceptional handling properties and its anti-plaque effect. The patented filler technology integrates the light transmission and diffusion properties of natural teeth. This enables naturally appearing restorations with even one layer.

BEAUTIFIL II has the ability to continually release fluoride and offers patients effective protection against secondary caries. This property can be further enhanced to a double fluoride release with SHOFU's FL-Bond II, a 2-step self-etching, fluoride releasing bonding system.


The hardness of the glass filler used as raw material of BEAUTIFIL II is similar to that of enamel and is antagonist-friendly. The material is therefore suitable for both anterior and posterior teeth.

In addition, Beautifil Flow and Beautifil Opaquer are flowable and masking variations for an expanded field of application.

Giomer

As a Giomer, BEAUTIFIL II is based on the PRG (Pre-Reacted Glass-ionomer) technology. The inorganic filler consists of pre-reacted micro-glass particles. By integrating the reaction into the manufacturing process, a stable glass-ionomer phase is formed on the surface of the glass particles. Any further reaction in the oral environment (water absorption) is excluded. The S-PRG fillers (Surface Pre-Reacted Glass-ionomer) give the composite specific features, such as the recharge and release of fluoride or the anti-plaque effect, without impairing its physical parameters.

Production process

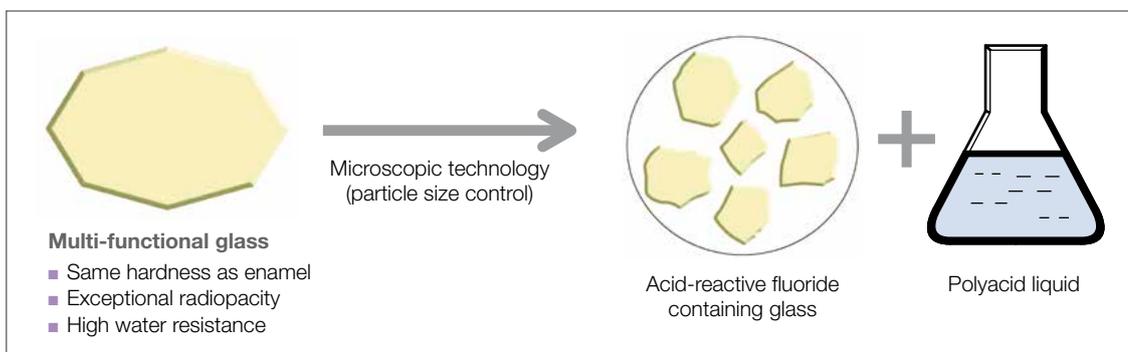


S-PRG technology

SPECIAL FEATURES

- Fluoride release and recharge
- Aesthetics close to natural teeth
- Anti-plaque effect
- High level of radiopacity
- Long term clinical stability
- Resistance to wear of posterior teeth

Reaction on the surface

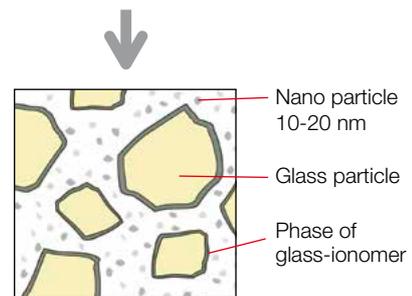

S-PRG Filler

S-PRG filler

+

Resin

PRG
Technology
Giomer

BEAUTIFIL II

Release and recharge of fluoride


The S-PRG fillers contained in BEAUTIFIL II have the ability to release and recharge with fluoride depending on the fluoride concentration in the mouth. The strength and stability of the composite remain unaffected.

The release of fluoride takes place exactly where the effect can best develop. This means that the adjacent contact areas of the natural hard tooth structure are effectively protected from caries due to permanent local remineralisation.

BEAUTIFIL II recharges with fluoride ions via external sources such as fluoride rinses, professional treatment or toothpastes.

This property and effect was clinically proven in an 8-year study.¹

The use of the coordinated bonding agent FL-Bond II enhances the release of fluoride and improves the protection against caries.

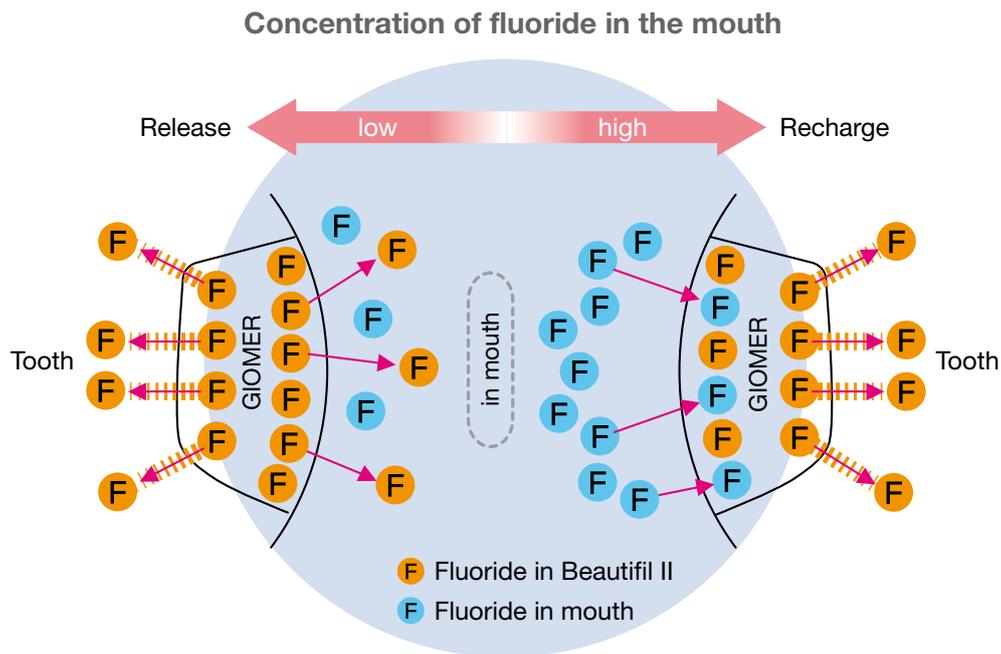


¹ Gordan V.V., Mondragon E., Watson R.E., Garvan C., Mjör I. A.: A clinical evaluation of a self-etching primer and a giomer restorative material. Result at eight years. J Am Dent Assoc 138: 621-627 (2007) (Available on request)


Characteristic features

Conducts light naturally

BEAUTIFIL II can recreate the effects of light transmission and diffusion of natural teeth due to its unique filler technology. Thanks to the perfect adaption you will achieve invisible restorations with the optical features of natural teeth.

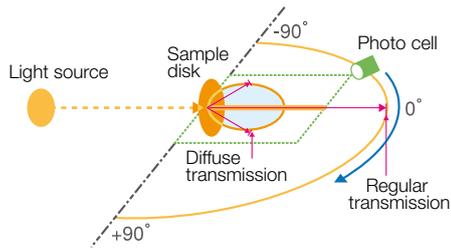

Two Dimensional Goniophotometry

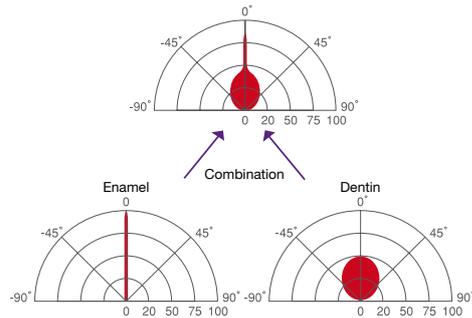

Colour stability

The shade of BEAUTIFIL II negligibly differs during modelling as well as before and after curing.

This ensures the right colour match even while layering.



Core before curing and outer ring after curing

Recommended use of colours

BEAUTIFIL II comes in 12 different colours.

The universal colours – A1, A2, A3, A3.5, A4, B1, B2, B3, C2 and C3 – have been specially designed for a rational layering technique.

In addition, the opacious dentin colours – A20 and A30 – are particularly suitable for multi-colour layering.

The enamel shade – Inc – offers highest translucency for aesthetically demanding anterior tooth restorations.

For the treatment of bleached teeth, use the Bleaching White shade – BW.


The indicators of the shade guides are made of genuine BEAUTIFIL II composite. They simulate the thickness of natural teeth to simplify the determination of the shade.




Anti-plaque effect

By the interaction between the components of saliva and the elements from S-PRG fillers in BEAUTIFIL II, a material film is formed on the area restored with BEAUTIFIL II. The material film that is continuously regenerated repeatedly minimizes the plaque adhesion and thus the colonization of bacteria.

Even if this film is removed, for example by teeth cleaning, subsequent protective layers are produced by the saliva.

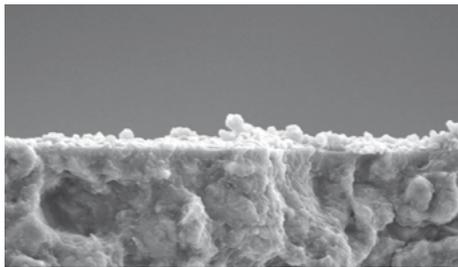

BEAUTIFIL II (with S-PRG filler): sporadic plaque formation

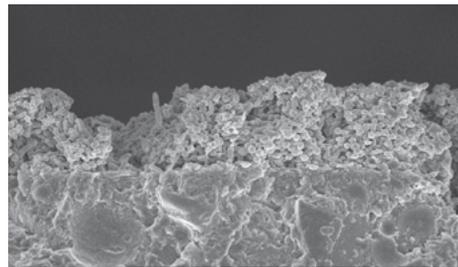

Conventional composite resin (without S-PRG filler): extensive plaque formation

Image provided by Prof. Koji Yamamoto,
ASAHI University


Optimum polishability

Due to the average particle size of $0.8\ \mu\text{m}$ and the integration of nanofillers (10-20 nm), BEAUTIFIL II is quick and easy to polish. The homogenous surface supports the anti-plaque effect.




Fluorescence

The fluorescence of BEAUTIFIL II is similar to that of natural teeth. The impact of visual influences such as UV light is minimized.

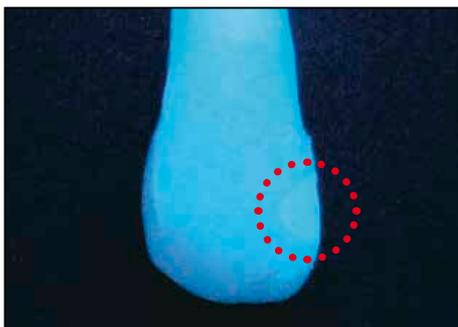

Extraordinary radiopacity

The radiopacity is 1.7 times higher than enamel and 3 times higher than dentin and permits reliable visual X-ray diagnostics.

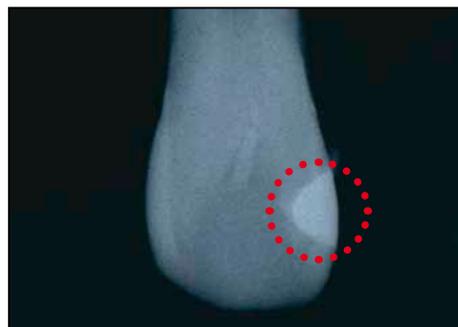

Outstanding handling properties

BEAUTIFIL II is easy to form and can easily be attached to the prepared hard tooth structure. The viscosity of the material, which is only slightly dependent on temperature, ensures stable modelling properties without sticking to the instruments.

The “one touch cap” of the syringe has been enhanced for easy dispensing and accurate application.



Reliable strength and durability

The homogenous microstructure of BEAUTIFIL II, with its high filler loading, ensures excellent physical properties with long-term stability and a high degree of flexural strength. These outstanding material properties have been clinically proven and documented in an 8-year study.¹

Physical properties

Compressive strength	Flexural strength	Vickers Hardness	Wear resistance	Filler loading	Average filler size	Radiopacity	Fluoride release and recharge
330 MPa	130 MPa	62 HV	0.52 weight %	83.3 weight %	0.8 µm	3.4 Al:mm	yes

(In-house data: SHOFU Inc., R&D Dept.)



¹ Gordon V.V., Mondragon E., Watson R.E., Garvan C., Mjör I. A.: A clinical evaluation of a self-etching primer and a glomer restorative material. Result at eight years. J Am Dent Assoc 138: 621-627 (2007) (Available on request)

Confidence starts here

Before restoration



After restoration



Images kindly provided by Dr. Sushil Koirala

Indications

Direct restorations that demand highest aesthetics and biocompatibility such as:

- Restorations of class I to V cavities
- Restoration of cervical erosion and root caries
- Repair of fractured incisal edges
- Veneers and posts
- Direct cosmetic repairs



System components

Beautiful Flow - Beautiful Opaquer - FL-Bond II

The flowable versions **Beautiful Flow** increase the range of indications. The moderately viscous F02 and the smooth-flowing F10 are coordinated to match the colours of BEAUTIFIL II precisely.

Our **Beautiful Opaquer** is available in two colours for masking discolourations, even with very thin film layers.

Beautiful Flow		BEAUTIFIL II
F02	F10	
A3	A3	A3




Shade coordination of BEAUTIFIL II and Beautiful Flow



The self-etching, two-step bonding system **FL-Bond II** also contains fluoride releasing S-PRG fillers and, in combination with BEAUTIFIL II, enhances the release of fluoride. FL-Bond II Primer is free of HEMA and acetone. FL-Bond II Bonding Agent is radiopaque and, thanks to its flowable consistency, easy to apply.




Low viscosity
(other bonding agent)

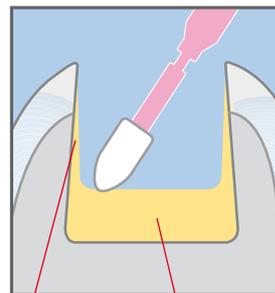

Extremely thin bonding layer
Collection of liquid at cavity floor

Moderate viscosity
(FL-Bond II)

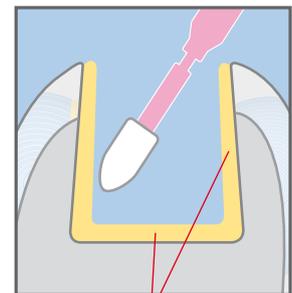

Equal bonding layer thickness

Order information



BEAUTIFIL II - 6 Color Set

- 6 BEAUTIFIL II syringes (A2, A3, A3.5, B2, A3O, Inc), 4.5 g each
- 1 Dura-White Stone (FL2)
- 1 Super-Snap Single Kit
- 1 OneGloss Trial Set
- Paper pad

PN 1775

PN 1776 **Tips** (6x 20 Tips at 0.25 g each)

BEAUTIFIL II - Cosmetic Kit

- 3 BEAUTIFIL II syringes (A2, A3, Inc), 4.5 g each
- 1 syringe Beautifil Flow F 02 (A2), 2 g
- 1 syringe Beautifil Opaquer (LO), 2 g
- 1 Dura-White Stone (FL2)
- 1 Super-Snap Single Kit
- 1 OneGloss Trial Set
- 2x 5 pcs syringe tips
- Paper pad

PN 1777



BEAUTIFIL II

Syringes 4.5 g each / **Tips** 20 pcs, 0.25 g each

Shade	PN	
	Syringes	Tips
A1	1401	1752
A2	1402	1753
A3	1403	1754
A3.5	1404	1755
A4	1405	1756
B1	1408	1759
B2	1409	1760
B3	1410	
C2	1412	1763
C3	1413	
Inc (Incisal)	1415	1766
BW (Bleaching White)	1416	1767
A2O (opacious A2)	1419	1770
A3O (opacious A3)	1420	1771



OneGloss Set

One step finishing and polishing system of silicone polishers for composite restorations

PN 0180



SHOFU INC. 11 Kamitakamatsu-cho, Fukuine, Higashiyama-ku, Kyoto 605-0983, Japan (www.shofu.co.jp)
SHOFU DENTAL ASIA-PACIFIC PTE. LTD. 10 Science Park Road, #03-12 The Alpha, Science Park II, Singapore 117684 (www.shofu.com.sg)
SHOFU DENTAL TRADING (SHANGHAI) CO., LTD. No. 645 Jiye Road, Sheshan Industrial Park, Songjiang, Shanghai 201602, China (www.shofu.com.cn)
SHOFU DENTAL CORPORATION 1225 Stone Drive, San Marcos, CA 92078-4059, USA (www.shofu.com)
SHOFU UK Riverside House, River Lawn Road, Tonbridge, Kent, TN9 1EP, UK (www.shofu.co.uk)
SHOFU DENTAL GmbH  Am Brüll 17, 40878 Ratingen, Germany (www.shofu.de)
SHOFU is a registered trademark of SHOFU INC. All other trademarks and registered trademarks are the property of their respective holders. SHOFU INC. reserves the right to change specifications without notice.