**BEAUTIFIL II**

DEUTSCH

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ENGLISH

FLUORIDE RELEASING DENTAL RESTORATIVE MATERIAL

**BEAUTIFIL II** offers greatly improved handling properties and superb polishability. Other properties like fluoride release and recharge, wear resistance, outstanding radio-opacity, brilliant aesthetics because of controlled light diffusion, remain characteristics of **BEAUTIFIL**.

**PRECAUTIONS FOR DENTAL PERSONNEL AND PATIENTS**

- Do not use this product for any purpose other than specifically outlined in the Indications. This product is intended for use by dental professionals only.
- If any inflammation or other allergic reactions occur on either patient or operator, immediately discontinue use and seek medical advice.
- Do not use this material on patients or operators with known allergies to methacrylate monomer.
- Avoid contact with soft tissue, skin or eyes. In case of accidental contact, immediately rinse with copious amounts of water and seek medical advice.
- Use medical (dental) gloves and protective glasses to avoid sensitization to this material. In case of accidental contact, immediately blot with an alcohol moistened cotton ball, and rinse with plenty of water. In case of contact with eyes, immediately rinse with copious amounts of water and seek medical advice. Medical (dental) gloves protect against direct contact with this material; however, some monomers penetrate very quickly to reach skin through the gloves. Therefore, should this material contact even the gloves, immediately remove them and wash hands with copious amounts of water.

**INDICATIONS**

Restoration of all anterior and posterior cavity classes I through V, wedge shaped defects and root caries.  
Direct laminate veneer, Core build-up, Repair of restorations.

**DIRECTIONS FOR USE**

- After cleaning tooth surface, take shade while the tooth is still moist, make cavity preparation, isolate prepared tooth with a rubber dam and protect the dentin close to the pulp with calcium hydroxide.
- Apply bonding agent according to the manufacturer's instructions.
- Syringe : Dispense the necessary amount of material from the syringe. After dispensing, immediately close the syringe cap then fill the cavity. Tips : Insert the selected shade tip into the Shofu TIPS dispensing gun on an other appropriate gun. Dispense slowly with consistent pressure.
- If cavities exceeding 2.0mm depth, incremental placement no more than 2.0mm is recommended.
- Light cure for 20 sec (halogen lamp) or 10 sec (high power type LED light). If the surface of the cavity is large, draw the light curing tip with a fanning action several times across the restoration.
- Remove excess material with "Dura White Stones" or "T&F Hybrid Points". Final polishing is obtained by "CompoMaster".

**NOTE IN USE**

- Use this product within the expiration date.
- Keep away from direct sunlight and store at room temperature (1-30°C/ 34-86°F).
- If a cavity is deep, protect it with a pulp capping agent. (Do not use eugenol containing liner.)
- Wear protective glasses when light curing.
- Do not use this product in any other purpose except the purpose mentioned in the application.
- This product is intended for dental operators use only.

**COMPOSITION**

Filler: Multi-Functional glass filler and S-PRG filler based on fluoroboroaluminosilicate glass  
Filler loading: 83.3wt% (68.6 vol%)  
Base resin: Bis-GMA/TEGDMA resin

**SHADES**

A1, A2, A3, A3.5, A4, B1, B2, B3, C2, C3, Inc (Incisal), BW (Bleaching White), A20, A30

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