



February,
2013

IV sets and accessories catalog

featuring:

MaxPlus® clear needleless connector

SmartSite® needle-free valve

Alaris® products





CareFusion

Key terms

0.2 micron filter

Removes particulate matter larger than 0.2 microns and virtually all fungi and bacteria. Often used as final filter for air elimination in the NICU. Recommended for use with non-lipid TPN infusions. Required for use with Taxol infusions.

Notes:

- CareFusion filters should not be inverted when priming.
- CareFusion 0.2 micron filters contain Supor (Polyethersulfone) membranes which have low protein and drug binding characteristics.

1.2 micron filter

Removes particulate matter larger than 1.2 microns. Used to filter lipids and/or 3 in 1 solutions; Amiodarone; Mannitol; Remicade requires either 1.2 or 0.2 micron filter.

Notes:

- CareFusion filters should not be inverted when priming.
- CareFusion 1.2 micron filters contain Supor (Polyethersulfone) membranes which have low protein and drug binding characteristics.

15 micron filter

Filters particulates >15 micron from solution (i.e., albumin).

10-drop

Spike is designed so that 10 drops equal approximately 1 mL. Used often by Anesthesiologists in the OR.

15-drop

Spike is designed so that 15 drops equal approximately 1 mL.

20-drop

Spike is designed so that 20 drops equal approximately 1 mL.

60-drop

Spike is designed so that 60 drops equal approximately 1 mL. Used often for Pediatric and Neonatal applications.

Check valve

A device which allows flow in only one direction. Commonly used for secondary piggyback drug administration or to protect certain elements of the set from high pressures seen when a syringe injects into the set. Reduces risk of contaminants flowing towards solution bag.

DEHP (Diethyl hexyl Phthalate)

A chemical commonly used in PVC tubing; used to yield clarity and soft pliable tubing.

Hat valve

Also known as duck bill valve, helps to reduce the chance of wet out of the air vent in the drip chamber when used with bottles.



Tubing does not contain DEHP in the fluid path.

Non-DEHP/DEHP-free

Tubing/set does not contain DEHP.

Non-PVC

Tubing either manufactured completely without polyvinyl chloride or which has an inert lining that helps prevent exposure of infused fluids to polyvinyl chloride.

Latex free

Contains no latex. Latex is known to cause hypersensitivity reactions in significant percentages of patients and health care workers.

Low sorbing

PVC free tubing in the fluid path designed to minimize absorption of drugs (for example nitroglycerin) into the matrix of tubing materials. Also prevents leaching (has inert internal surface).

Microbore

Tubing with smaller internal diameter ≤ 0.04 " to minimize the contained volume. Often used in Pediatric and Neonatal areas. Promotes more efficient drug delivery at low flow rates. Some companies will interchange with smallbore or minibore; there is no standardized internal diameter.

Smallbore

Tubing with smaller internal diameter > 0.04 " and < 0.1 " to minimize the contained volume. Some companies use interchangeably with microbore.

Screen blood filter

Special filter designed for blood and blood products to remove debris such as clots.

Micro aggregate filter

Filters are typically in the 20 to 80 micron range for specialty applications.

Note: CareFusion current product offering does not include these filter types.

Key terms

Non-vented	Characteristic of the spike that enters the solution container. Designed for collapsible plastic containers as no air is required to enter the container in order for fluid to flow.*
Vented	Characteristic of the spike that enters the solution container. Designed for containers that do not collapse when emptying (glass or semi-rigid plastic). Vent allows air to enter the container in order to allow fluid to flow.*

**Note: All sets with drip chambers are vented/non-vented unless otherwise noted.*

CareFusion has made every effort to assure the accuracy of the information in this Catalog. Information on products contained within this catalog may change and these changes may alter the information contained in this publication. Any corrections and changes CareFusion is made aware of after the publication date of this Catalog issuance will be completed in subsequent issues.

Contents

Section	Page
01.00 Needleless connectors and needle-free valves	1
02.00 Access devices and accessories	
02.01 Blood collection devices	2
02.02 Accessories	3
03.00 Chemo-Safety System	
03.01 Texium® closed male luer and SmartSite® access devices and accessories	4
03.02 Vial and bag access devices	8
03.03 Secondary administration sets with SmartSite bag access port and Texium closed male luer	10
03.04 Alaris® Pump module administration sets with SmartSite bag access port and Texium closed male luer	11
03.05 Alaris SE pump administration sets with SmartSite bag access port	14
04.00 Specialty extension sets	
04.01 Filters	
04.01.01 MaxPlus® and MaxGuard® connector sets	15
04.01.02 SmartSite valve sets	20
04.01.03 VersaSafe™ split septum sets	22
04.01.04 No port sets	23
04.02 Stopcocks and manifolds	
04.02.01 Stand alone	26
04.02.02 MaxPlus needleless connector and y-site sets	28
04.02.03 SmartSite valve sets	30
04.02.04 Injection site sets	32
04.02.05 No port sets	33
04.03 T-connectors	
04.03.01 MaxPlus needleless connector sets	36
04.03.02 SmartSite valve sets	40
04.03.03 VersaSafe split septum sets	42
04.03.04 Other access sets	43
04.04 Specialty tubing	
04.04.01 Filters and Flow Regulators	44
04.04.02 No port sets	45
04.05 Labeled for use with low pressure power injectors	
04.05.01 MaxPlus and MaxGuard needleless connector sets	46
04.05.02 SmartSite valve sets	55
04.05.03 No port sets	56

Contents

Section		Page
05.00	Extension sets	
05.01	Standard tubing	
05.01.01	MaxPlus and MaxGuard needleless connector sets	57
05.01.02	SmartSite valve sets	60
05.01.03	VersaSafe split septum sets	66
05.01.04	No port sets	67
05.02	Kink resistant tubing	
05.02.01	MaxPlus and MaxGuard needleless connector sets	70
05.03	Minibore/Smallbore tubing	
05.03.01	MaxPlus and MaxGuard needleless connector sets	74
05.03.02	SmartSite valve sets	85
05.03.03	VersaSafe split septum sets	89
05.03.04	No port sets	90
05.04	Microbore tubing	
05.04.01	MaxPlus and MaxGuard needleless connector sets	95
05.04.02	No port sets	97

Contents

Section	Page
06.00 Alaris Pump module	
06.01 Stopcocks and manifolds	
06.01.01 SmartSite valve sets	99
06.01.02 VersaSafe split septum sets	104
06.02 Burettes	
06.02.01 SmartSite valve sets	105
06.02.02 No port sets	110
06.02.03 VersaSafe split septum sets	111
06.03 Low sorbing tubing	
06.03.01 SmartSite valve sets	113
06.03.02 No port sets	118
06.03.03 VersaSafe split septum sets	120
06.04 Filters	
06.04.01 Blood filters	
06.04.01.01 SmartSite valve sets	121
06.04.01.02 No port sets	122
06.04.01.03 VersaSafe split septum sets	123
06.04.02 0.2 micron filters	
06.04.02.01 SmartSite valve sets	124
06.04.02.02 No port sets	129
06.04.02.03 VersaSafe split septum sets	130
06.04.03 1.2 micron filters	
06.04.03.01 SmartSite valve sets	131
06.04.03.02 No port sets	132
06.04.03.03 VersaSafe split septum sets	133
06.04.04 15 micron filters	
06.04.04.01 SmartSite valve sets	134
06.04.04.02 No port sets	135
06.04.04.03 VersaSafe split septum sets	136
06.05 Special applications	
06.05.01 Yellow striped tubing sets	137
06.05.02 Amber tubing sets	138
06.05.03 SmartSite bag access port sets	139
06.05.04 Ball valve sets	141
06.05.05 Syringe adapter sets	144
06.06 Primary administration sets	
06.06.01 SmartSite valve sets	145
06.06.02 No port sets	152
06.06.03 VersaSafe split septum sets	153

Contents

Section	Page
07.00 Alaris Syringe module sets	
07.01 With pressure sensing disc	156
07.02 Without pressure sensing disc	159
08.00 Alaris PCA module sets	160
09.00 Alaris SE pump	
09.01 Burettes	
09.01.01 SmartSite valve sets	163
09.01.02 VersaSafe split septum sets	165
09.02 Low sorbing tubing	
09.02.01 SmartSite valve sets	166
09.02.02 No port sets	167
09.03 Filters	
09.03.01 Blood filters	
09.03.01.01 SmartSite valve sets	168
09.03.01.02 No port sets	169
09.03.01.03 VersaSafe split septum sets	170
09.03.02 0.2 micron filters	
09.03.02.01 SmartSite valve sets	171
09.03.02.02 No port sets	172
09.03.02.03 VersaSafe split septum sets	173
09.03.03 1.2 micron filters	
09.03.03.01 SmartSite valve sets	174
09.03.03.02 VersaSafe split septum sets	175
09.03.04 15 micron filters	
09.03.04.01 No port sets	176
09.04 Special applications	
09.04.01 Yellow striped tubing sets	177
09.04.02 Amber tubing sets	178
09.04.03 SmartSite bag access port sets	179
09.04.04 Syringe adapter sets	180
09.05 Primary administration sets	
09.05.01 SmartSite valve sets	181
09.05.02 No port sets	185
09.05.03 VersaSafe split septum sets	186

Contents

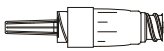
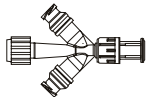
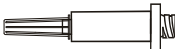
Section			Page
10.00	MedSystem III® infusion pump		
10.01	Burettes		
10.01.01	SmartSite valve sets		189
10.01.02	Standard injection port sets		190
10.02	Low sorbing tubing		
10.02.01	SmartSite valve sets		191
10.02.02	No port sets		192
10.03	Filters		
10.03.01	Blood filters		
10.03.01.01	SmartSite valve sets		193
10.03.01.02	Standard injection port sets		194
10.03.02	0.2 micron filters		
10.03.02.01	SmartSite valve sets		195
10.03.02.02	Standard injection port sets		196
10.04	Primary administration sets		
10.04.01	SmartSite valve sets		197
10.04.02	No port sets		198
10.04.03	Standard injection port sets		200
11.00	Secondary administration sets		
11.01	Filters		
11.01.01	Blood filters no port sets		201
11.01.02	15 micron filters no port sets		202
11.02	Special applications		
11.02.01	SmartSite bag access port sets		203
11.02.02	Bag access transfer sets		204
11.02.03	Syringe adapter sets		205
11.03	Secondary administration sets		206

Contents

Section	Page
12.00 Gravity sets	
12.01 Stopcocks and manifolds	
12.01.01 MaxPlus needleless connectors and y-site sets	209
12.01.02 SmartSite valve sets	211
12.01.03 VersaSafe split septum sets	216
12.02 Burettes	
12.02.01 MaxPlus needleless connector and y-site sets	217
12.02.02 SmartSite valve sets	219
12.02.03 VersaSafe split septum sets	220
12.03 Low sorbing tubing SmartSite valve sets	224
12.04 Filters	
12.04.01 Blood filters	
12.04.01.01 SmartSite valve sets	222
12.04.01.02 No port sets	223
12.04.02 0.2 micron filters	
12.04.02.01 SmartSite valve sets	224
12.04.02.02 MaxPlus needleless connector and y-site sets	225
12.04.02.03 No port sets	226
12.04.03 1.2 micron filters	227
12.04.04 15 micron filters	228
12.05 Special applications	
12.05.01 Ball valve sets	229
12.05.02 Flow regulator sets	231
12.06 Primary administration sets	
12.06.01 MaxPlus needleless connector and y-site sets	234
12.06.02 SmartSite valve sets	237
12.06.03 No port sets	242
12.06.04 VersaSafe split septum sets	243

01.00 Needleless connectors and needle-free valves

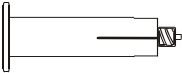
Needleless connectors and needle-free valves eliminate the need for needles, caps, blunt cannulas and add-on components. All devices are latex and DEHP free.

Model #	Drops	Description	Case quantity
MP1000-C		 MaxPlus clear needleless connector Approximate priming volume: 0.28 mL Labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second.	 100
MP1000A		 MaxGuard antimicrobial needleless connector Approximate priming volume: 0.28 mL Labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second.	 100
MP1000		 MaxPlus needleless connector Approximate priming volume: 0.28 mL Labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second.	 100
2000E		 SmartSite needle-free valve Approximate priming volume: 0.11 mL Labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second.	 100
20558E		 SmartSite tri-port connector Approximate priming volume: 0.46 mL	 50
MX1002		 Standard needleless connector	 100

02.00 Access devices and accessories

02.01 Blood collection devices

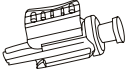
Access devices and accessories provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All devices are latex and DEHP free. Blood collection devices assist healthcare facilities comply with OSHA directives.

Model #	Drops	Description	Case quantity	
MBC6000				200
		Blood collection assembly, male luer lock		
MBC6008				200
		Blood transfer assembly, female luer lock		

02.00 Access devices and accessories

02.02 Accessories

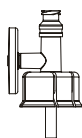
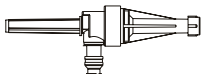
Access devices and accessories provide flexibility and additional infusion capabilities to gravity and infusion instrument sets.
All access devices on this page are latex and DEHP free.

Model #	Drops	Description	Case quantity	
9391-0200		VersaSafe™ split septum plastic cannula,-11 gauge		200
70200		Female cap		600
70804		Female/male cap		600
MPC-1A		Breather cap for male luer slips, locks, spin locks		100
MY0240		Pre slit injection port with male luer lock connector		100

03.00 Chemo-Safety System

03.01 Texium closed male luer and SmartSite access devices and accessories

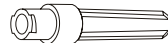
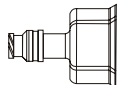
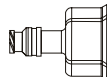
The Texium closed male luer partners with SmartSite needle-free valve to deliver an easy-to-use closed system. The SmartSite IV sets and accessories eliminate the need for needles, caps, blunt cannulas and add-on components. Access devices and accessories provide flexibility and additional infusion capabilities to gravity and infusion instruments sets. All access devices are latex and DEHP free.

Model #	Drops	Description	Case quantity
10012240-0500		 Texium female cap	 100
10012241-0500		 Texium closed male luer with female cap Approximate priming volume: 0.12 mL	 100
MV0400		 Chemo-Safety Universal Vented Vial Access Device, 0.2 micron hydrophobic air-venting filter, with SmartSite needle-free valve, compatible with 13 mm, 20 mm and 28 mm vial closures. Approximate flow rate > 3000 mL per hour, residual fluid < 1.2 mL Approximate priming volume: < 0.15 mL	 400
10012392-0500		 SmartSite 20mm vented vial access device, 0.2 micron hydrophobic air-venting filter Approximate priming volume: 0.14 mL	 80
10013365		 SmartSite add-on bag access device, spike adapter, with 1 needle-free valve bag access port Approximate length: 3.8" 9.6 cm. approx. Approximate priming volume: 0.61 mL	 100

03.00 Chemo-Safety System

03.01 Texium closed male luer and SmartSite access devices and accessories

The Texium closed male luer partners with the SmartSite needle-free valve to deliver an easy-to-use closed system. SmartSite IV sets and accessories eliminate the need for needles, caps, blunt cannulas and add-on components. Access devices and accessories provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All access devices are latex and DEHP free.

Model #	Drops	Description	Case quantity
19MMTP			 100
	Vial dispensing pin with vent		
	Approximate length:	2" 5 cm. approx.	
	Approximate priming volume:	0.22 mL	
2000E			 100
	SmartSite® needle-free valve		
	Approximate priming volume:	0.11 mL	
	Labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second		
2201			 100
	Single-dose dispensing pin		
	Approximate priming volume:	0.3 mL	
2202E			 100
	SmartSite needle-free access pin		
	Approximate priming volume:	0.07 mL	
2203E			 100
	SmartSite 13mm vial access device		
	Approximate priming volume:	0.07 mL	
2205E			 100
	SmartSite 20mm vial access device		
	Approximate priming volume:	0.09 mL	

03.00 Chemo-Safety System

03.01 Texium closed male luer and SmartSite access devices and accessories

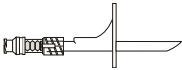
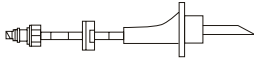
The Texium closed male luer partners with SmartSite needle free valve to deliver an easy-to-use closed system. The SmartSite IV sets and accessories eliminate the need for needles, caps, blunt cannulas and add-on components. Access devices and accessories provide flexibility and additional infusion capabilities to gravity and infusion instruments sets. All access devices are latex and DEHP free.

Model #	Drops	Description	Case quantity	
MY8003				100
		Needle-Free Syringe with bonded Texium closed male luer Approximate priming volume: 3 mL		
MY8005				100
		Needle-Free Syringe with bonded Texium closed male luer Approximate priming volume: 5 mL		
MY8010				100
		Needle-Free Syringe with bonded Texium closed male luer Approximate priming volume: 10 mL		
MY8020				100
		Needle-Free Syringe with bonded Texium closed male luer Approximate priming volume: 20 mL		
MY8060				50
		Needle-Free Syringe with bonded Texium closed male luer Approximate priming volume: 60 mL		

03.00 Chemo-Safety System

03.01 Texium closed male luer and SmartSite access devices and accessories

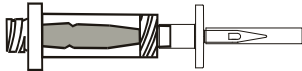
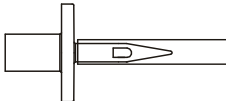
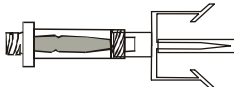
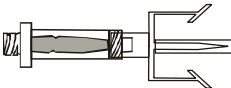
The Texium closed male luer partners with the SmartSite needle-free valve to deliver an easy-to-use closed system. SmartSite IV sets and accessories eliminate the need for needles, caps, blunt cannulas and add-on components. Access devices and accessories provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All access devices are latex and DEHP free.

Model #	Drops	Description	Case quantity
2300E-0500		 SmartSite bag spike Approximate priming volume: 0.51 mL	 100
2309E		 SmartSite bag spike with check valve Approximate priming volume: 0.93 mL	 100
MM4000		 Manifold with three 1-way valves Approximate length: 6" 15 cm. approx. Approximate priming volume: 1.5 mL	 50
70200		 Female cap	 600
70804		 Female/male cap	 600

03.00 Chemo-Safety System

03.02 Vial and bag access devices

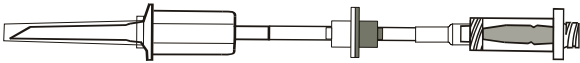
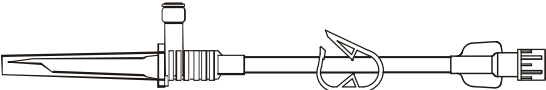
Vial and bag access devices eliminate the need for needles, caps, blunt cannulas and add-on components. Access devices and accessories provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All access devices are latex and DEHP free.

Model #	Drops	Description	Case quantity	
MV0500				50
		Multi-dose vial access spike, standard clear needleless connector		
MV0510				100
		Single dose vial access spike, standard clear needleless connector		
MV0600				100
		14 mm multi-dose vial access adapter, standard clear needleless connector		
30-day lead time				
MV0700				100
		20 mm multi-dose vial access adapter, standard clear needleless connector		

03.00 Chemo-Safety System

03.02 Vial and bag access devices

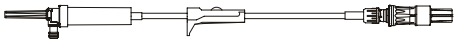
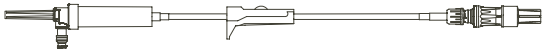
Vial and bag access devices eliminate the need for needles, caps, blunt cannulas and add-on components. Access devices and accessories provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All access devices are latex and DEHP free.

Model #	Drops	Description	Case quantity	
MY3001		 Bag access spike with cap, back check valve, standard clear needleless connector		100
MY3003		 Bag access spike with cap, standard clear needleless connector		100
MY3004		 Bag access spike with cap, pinch clamp		50
30-day lead time		Approximate set length: 7" 18 cm. approx.		

03.00 Chemo-Safety System

03.03 Secondary administration sets with SmartSite bag access port and Texium closed male luer

The Texium closed male luer partners with the SmartSite needle-free valve to deliver an easy-to-use closed system. The SmartSite IV sets and accessories eliminate the need for needles, caps, blunt cannulas and add-on components. Access devices and accessories provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All access devices are latex free.

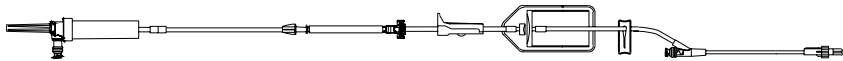
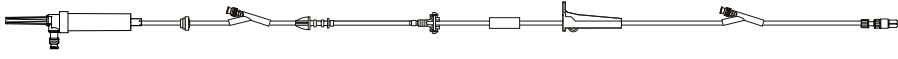
Model #	Drops	Description	Case quantity
10013364	20	 Non-vented secondary set, needle-free valve bag access port, fixed male luer lock and hanger Approximate set length: 32" 81 cm. approx. Approximate priming volume: 12 mL	 100
10013364T	20	 Non-vented secondary set, needle-free valve bag access port, with bonded Texium closed male luer and hanger Approximate set length: 32" 81 cm. approx. Approximate priming volume: 12 mL	100
10014881	20	 Low sorbing non-vented secondary set, needle-free valve bag access port, fixed male luer lock and hanger Approximate set length: 31" 79 cm. approx. Approximate priming volume: 12 mL	 100
10014881T	20	 Low sorbing non-vented secondary set, needle-free valve bag access port, with bonded Texium closed male luer and hanger Approximate set length: 31" 79 cm. approx. Approximate priming volume: 12 mL	 100
11610341	20	 Non-vented secondary set, needle-free valve bag access port, spin male luer lock and hanger Approximate set length: 31" 79 cm. approx. Approximate priming volume: 12 mL	 100

30-day lead time

03.00 Chemo-Safety System

03.04 Alaris Pump module administration sets with SmartSite bag access port and Texium closed male luer

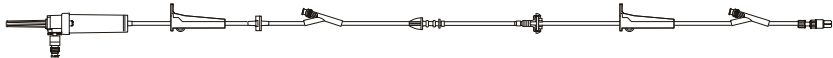
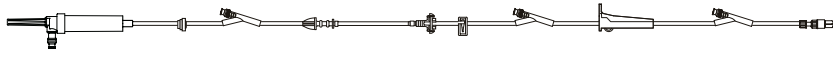
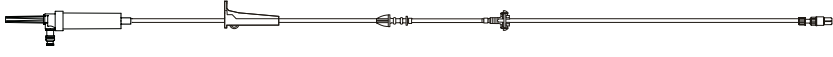
The Texium closed male luer partners with the SmartSite needle-free valve to deliver an easy-to-use closed system. The SmartSite IV sets and accessories eliminate the need for needles, caps, blunt cannulas and add-on components. Access devices and accessories provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All access devices are latex and DEHP free.

Model #	Drops	Description	Case quantity
2465-0007	20	 Low sorbing infusion set, non-vented, needle-free bag access port, 20 drop, 0.2 micron (low protein binding) filter, 1 needle-free valve 6" from 2-piece male luer lock Approximate set length: 116" 293 cm. approx. Approximate priming volume: 26 mL	 20
2466-0007	20	 Low sorbing infusion set, non-vented, needle-free bag access port, spin male luer lock Approximate set length: 86" 219 cm. approx. Approximate priming volume: 20 mL	 20
10013361	20	 Infusion set, non-vented, needle-free valve bag access port, check valve, 2 needle-free valves 78" and 6" from 2-piece male luer lock Approximate set length: 117" 297 cm. approx. Approximate priming volume: 25 mL	 20
10013854 30-day lead time	20	 Low sorbing infusion set, non-vented, 15 micron filter, needle-free valve bag access port, luer lock connector. Approximate set length: 113" 287 cm. approx. Approximate priming volume: 20 mL	 20

03.00 Chemo-Safety System

03.04 Alaris Pump module administration sets with SmartSite bag access port and Texium closed male luer

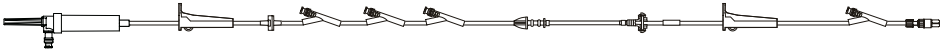
The Texium closed male luer partners with the SmartSite needle-free valve to deliver an easy-to-use closed system. The SmartSite IV sets and accessories eliminate the need for needles, caps, blunt cannulas and add-on components. Access devices and accessories provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All access devices are latex and DEHP free.

Model #	Drops	Description	Case quantity
10014855	20	 Low sorbing infusion set, non-vented, needle-free valve bag access port, check valve, 2 needle-free valves 95" and 6" from 2-piece male luer lock Approximate set length: 116" 295 cm. approx. Approximate priming volume: 24 mL	 20
10015861	20	 Infusion set, non-vented, needle-free valve bag access port, low sorbing tubing, check valve, 3 needle-free valves 95", 75" and 6" from 2-piece male luer lock Approximate set length: 128" 325 cm. approx. Approximate priming volume: 26 mL	 20
10015862	20	 Low sorbing infusion set, non-vented, needle-free valve bag access port, 2-piece male luer lock Approximate set length: 44" 112 cm. approx. Approximate priming volume: 14 mL	 20

03.00 Chemo-Safety System

03.04 Alaris Pump module administration sets with SmartSite bag access port and Texium closed male luer

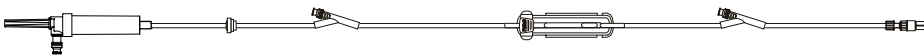
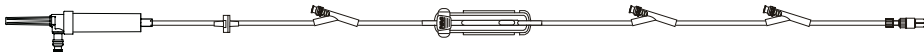
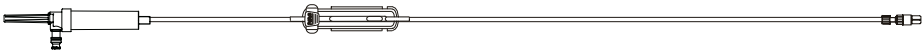
The Texium closed male luer partners with the SmartSite needle-free valve to deliver an easy-to-use closed system. The SmartSite IV sets and accessories eliminate the need for needles, caps, blunt cannulas and add-on components. Access devices and accessories provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All access devices are latex and DEHP free.

Model #	Drops	Description	Case quantity
10061661	20	 Low sorbing infusion set, non-vented, needle-free valve bag access port, check valve, 4 needle-free valves 94", 92", 90" and 6" from 2-piece male luer lock Approximate set length: 121" 307 cm. approx. Approximate priming volume: 25 mL	 20
10321213	20	 Infusion set, non-vented, needle-free valve bag access port, fixed male luer lock Approximate set length: 106" 269 cm. approx. Approximate priming volume: 23 mL	 20

03.00 Chemo-Safety System

03.05 Alaris SE pump administration sets with SmartSite bag access port and Texium closed male luer

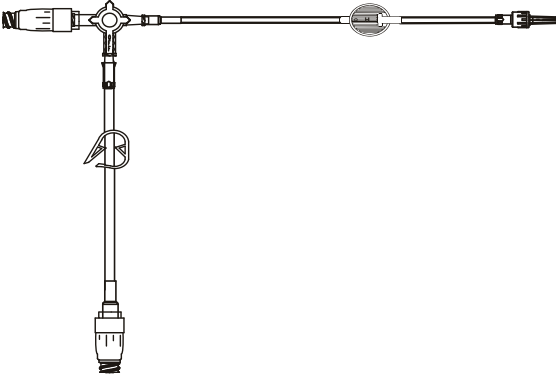
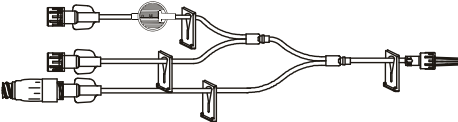
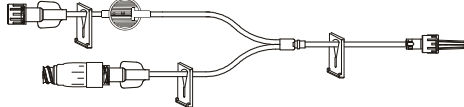
The Texium closed male luer partners with the SmartSite needle-free valve to deliver an easy-to-use closed system. The SmartSite IV sets and accessories eliminate the need for needles, caps, blunt cannulas and add-on components. Access devices and accessories provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All access devices are latex free.

Model #	Drops	Description	Case quantity
10013363	20	 Infusion set, non-vented, needle-free valve bag access port, smallbore tubing segment, check valve, 2 needle-free valves 90" and 9" from 2-piece male luer lock Approximate set length: 100" 254 cm. approx. Approximate priming volume: 22 mL	50
10015863	20	 Infusion set, low sorbing tubing segment, non-vented, needle-free valve bag access port, check valve, 3 needle-free valves 83", 37" and 9" from 2-piece male luer lock Approximate set length: 101" 257 cm. approx. Approximate priming volume: 22 mL	50
10026645	20	 Infusion set, low sorbing tubing segment, non-vented, needle-free valve bag access port, 2-piece male luer lock Approximate set length: 88" 224 cm. approx. Approximate priming volume: 20 mL	20
10784614	20	 Infusion set, low sorbing tubing segment, non-vented, needle-free valve bag access port, spin male luer lock Approximate set length: 88" 224 cm. approx. Approximate priming volume: 20 mL	20

04.00 Specialty extension sets

04.01.01 Filters | MaxPlus and MaxGuard connector sets

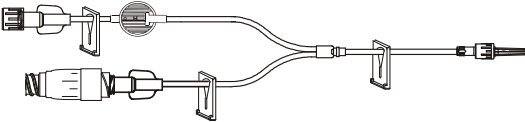
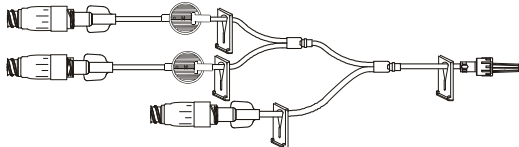
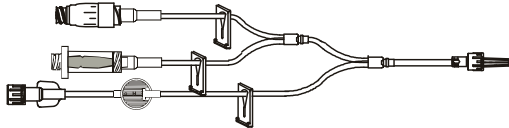
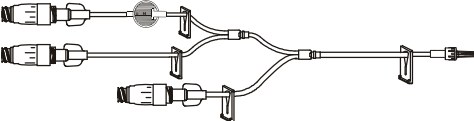
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves, VersaSafe split septum ports, filters, stopcocks, manifolds and/or T-connectors.

Model #	Drops	Description	Case quantity
			
MP4001-C		Extension set, 2 MaxPlus clear needleless connectors, 3-way stopcock, 0.2 µ filter Approximate set length: 77.75" 197 cm. approx. Approximate priming volume: 5 mL	 50
			
MP9008		Minibore tri-fuse extension set, removable MaxPlus needleless connector, 0.2 micron filter, 4 slide clamps, male spin lock Approximate set length: 9" 23 cm. approx. Approximate priming volume: 1.28 mL	 50
			
MP9009		Minibore bi-fuse extension set, removable MaxPlus needleless connector, 0.2 micron filter, 2 slide clamps, male spin lock Approximate set length: 8" 20 cm. approx. Approximate priming volume: 2.3 mL	 50
30-day lead time			

04.00 Specialty extension sets

04.01.01 Filters | MaxPlus and MaxGuard connector sets

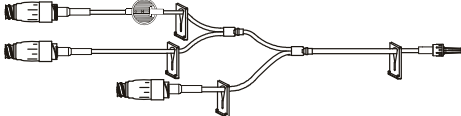
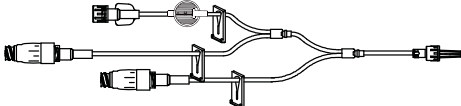
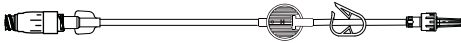
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves, VersaSafe split septum ports, filters, stopcocks, manifolds and/or T-connectors.

Model #	Drops	Description	Case quantity
MP9009-C 30-day lead time		 Bi-fuse minibore extension set, removable MaxPlus clear needleless connector, 0.2 micron filter, male spin lock Approximate set length: 8" 20 cm. approx. Approximate priming volume: 2.3 mL	 50
MP9204 30-day lead time		 Minibore tri-fuse extension set, 3 removable MaxPlus needleless connectors, two 0.2 micron filters, 4 slide clamps, male spin lock Approximate set length: 7" 18 cm. approx. Approximate priming volume: 0.98 mL	 50
MP9209 30-day lead time		 Minibore tri-fuse extension set, 1 MaxPlus needleless connector, 1 standard needleless connector, 0.2 micron filter, 3 slide clamps, male spin lock Approximate set length: 36" 91 cm. approx. Approximate priming volume: 3.2 mL	 50
MP9212-C 30-day lead time		 Minibore tri-fuse extension set, 3 MaxPlus removable clear needleless connectors, 0.2 micron filter, 4 slide clamps, male luer lock Approximate set length: 9" 23 cm. approx. Approximate priming volume: 1.9 mL	 50

04.00 Specialty extension sets

04.01.01 Filters | MaxPlus and MaxGuard connector sets

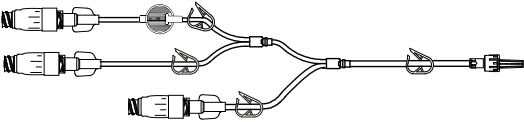
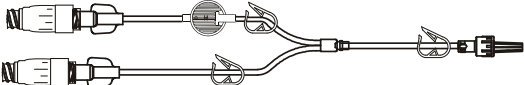
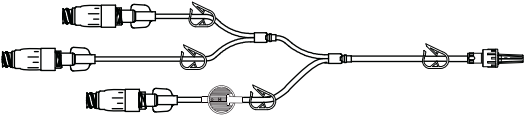
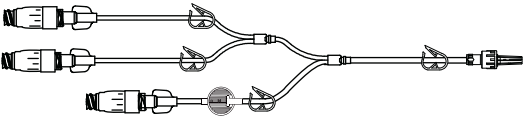
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves, VersaSafe split septum ports, filters, stopcocks, manifolds and/or T-connectors.

Model #	Drops	Description	Case quantity	
MP9234-C 30-day lead time				50
		Minibore extension set, clear needleless connector, 0.2 micron filter, male spin lock		
		Approximate set length: 60" 152 cm. approx. Approximate priming volume: 2.4 mL		
MP9254-C				50
		Minibore tri-fuse extension set, 3 clear needleless connectors, 0.2 micron filter, 4 slide clamps, male luer lock		
		Approximate set length: 9" 23 cm. approx. Approximate priming volume: 2.0 mL		
MP9264-C 30-day lead time				50
		Minibore tri-fuse extension set, 2 MaxPlus clear needleless connectors, 0.2 micron filter, 3 non-removable slide clamps, male spin lock		
		Approximate set length: 60" 152 cm approx. Approximate priming volume: 3.0 mL		
MP9269-C Contact CareFusion for lead time				50
		Microbore extension set, removable MaxPlus clear needleless connector, pinch clamp, 0.2 micron filter, male spin lock		
		Approximate set length: 60" 152 cm approx. Approximate priming volume: 1.0 mL		

04.00 Specialty extension sets

04.01.01 Filters | MaxPlus and MaxGuard connector sets

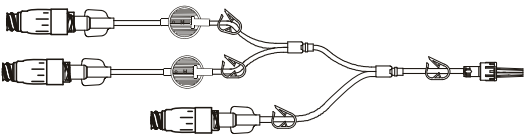
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves, VersaSafe split septum ports, filters, stopcocks, manifolds and/or T-connectors.

Model #	Drops	Description	Case quantity	
MP9270-C				50
		Microbore tri-fuse extension set, 3 removable MaxPlus clear needleless connectors, 0.2 micron filter, 4 pinch clamps, 2 white, 1 blue, 1 red, male spin lock		
		Approximate set length: 10" 25 cm approx. Approximate priming volume: 0.6 mL		
MP9271-C				50
		Microbore bi-fuse extension set, 2 removable MaxPlus clear needleless connectors, 0.2 micron filter, 3 pinch clamps, male spin lock		
		Approximate set length: 8.5" 22 cm approx. Approximate priming volume: 1.1 mL		
MP9272-C				50
		Microbore tri-fuse extension set, 3 removable MaxPlus clear needleless connectors, 0.2 micron filter, 4 pinch clamps, male spin lock		
		Approximate set length: 60" 152 cm approx. Approximate priming volume: 1.7 mL		
MP9273-C				50
		Microbore tri-fuse extension set, 3 removable MaxPlus clear needleless connectors, 0.2 micron filter, 4 pinch clamps, male spin lock		
		Approximate set length: 60" 152 cm approx. Approximate priming volume: 2.0 mL		

04.00 Specialty extension sets

04.01.01 Filters | MaxPlus and MaxGuard connector sets

Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves, VersaSafe split septum ports, filters, stopcocks, manifolds and/or T-connectors.

Model #	Drops	Description	Case quantity	
MP9274-C Contact CareFusion for lead time				50
		Microbore tri-fuse extension set, 3 removable MaxPlus clear needleless connectors, two 0.2 micron filters, 4 pinch clamps, 2 white, 1 blue, 1 red, male spin lock		
		Approximate set length: 8.5" 22 cm approx.		
		Approximate priming volume: 1.6 mL		
MX9166				50
		Extension set, 0.2 micron filter, needleless y-site, pinch clamp, male spin lock		
		Approximate set length: 16" 40.6 cm. approx. Approximate priming volume: 5.8 mL		
MX9171				50
		Extension set, 1.2 micron filter, needleless y-site, pinch clamp, male spin lock		
		Approximate set length: 16" 41 cm. approx. Approximate priming volume: 5.8 mL		

04.00 Specialty extension sets

04.01.02 Filters | SmartSite valve sets

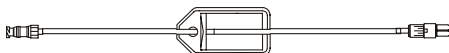
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves, VersaSafe split septum ports, filters, stopcocks, manifolds and/or T-connectors.

Model #	Drops	Description	Case quantity
20027E		Extension set, 0.2 micron (low-protein binding) filter, 1 needle-free valve 6" from spin male luer lock	 100
		Approximate set length: 18" 46 cm. approx.	
		Approximate priming volume: 5 mL	
20028E		Extension set, 0.2 micron (low-protein binding) filter, 1 needle-free valve 7" from spin male luer lock	100
		Approximate set length: 18" 46 cm. approx.	
		Approximate priming volume: 5 mL	
20029E		Smallbore extension set, 0.2 micron (low-protein binding) filter, 1 needle-free valve 1.5" from spin male luer lock	 100
		Approximate set length: 11" 28 cm. approx.	
		Approximate priming volume: 0.68 mL	
20127E		Extension set, 1.2 micron (low-protein binding) filter, 1 needle-free valve 6" from spin male luer lock	 100
		Approximate set length: 18" 46 cm. approx.	
		Approximate priming volume: 5 mL	
20128E		Extension set, 1.2 micron (low-protein binding) filter, 1 needle-free valve 7" from spin male luer lock	100
		Approximate set length: 18" 46 cm. approx.	
		Approximate priming volume: 5 mL	

04.00 Specialty extension sets

04.01.02 Filters | SmartSite valve sets

Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves, VersaSafe split septum ports, filters, stopcocks, manifolds and/or T-connectors.

Model #	Drops	Description	Case quantity
20350E		 Low sorbing extension set, 0.2 micron (low-protein binding) filter, 1 needle-free valve 6" from spin male luer lock Approximate set length: 19" 48 cm. approx. Approximate priming volume: 5 mL	 100
21028E		 Extension set, 0.2 micron (low-protein binding) filter, 1 needle-free valve 7" from fixed male luer lock Approximate set length: 18" 46 cm. approx. Approximate priming volume: 5 mL	100
10013902		 Low sorbing extension set, 0.2 micron (low-protein binding) filter, 1 needle-free valve, 2-piece male luer lock Approximate set length: 16" 41 cm. approx. Approximate priming volume: 5 mL	 100

04.00 Specialty extension sets

04.01.03 Filters | VersaSafe split septum sets

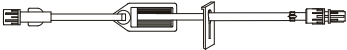
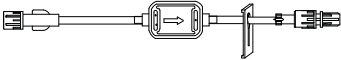
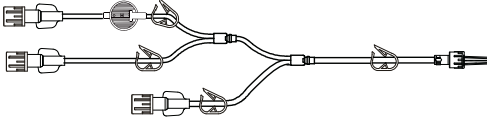
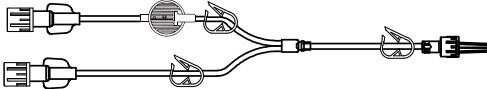
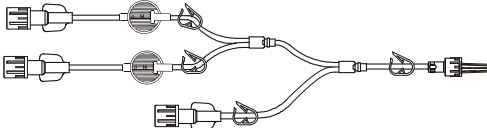
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves, VersaSafe split septum ports, filters, stopcocks, manifolds and/or T-connectors.

Model #	Drops	Description	Case quantity
10796815 30-day lead time		 Smallbore extension set, 0.2 micron (low-protein binding) filter, 1 split septum port 3" from spin male luer lock Approximate set length: 11" 28 cm. approx. Approximate priming volume: 1 mL	 100
10807851		 Extension set, 0.2 micron (low-protein binding) filter, 1 split septum port 6" from spin male luer lock Approximate set length: 18" 46 cm. approx. Approximate priming volume: 5 mL	 100
10807853		 Extension set, 1.2 micron (low-protein binding) filter, 1 split septum port 6" from spin male luer lock Approximate set length: 18" 46 cm. approx. Approximate priming volume: 6 mL	 100
11275183		 Low sorbing extension set, 0.2 micron (low-protein binding) filter, 1 split septum port 6" from 2-piece male luer lock Approximate set length: 20" 51 cm. approx. Approximate priming volume: 6 mL	 20

04.00 Specialty extension sets

04.01.04 Filters | No port sets

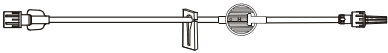
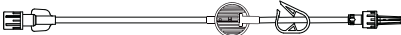
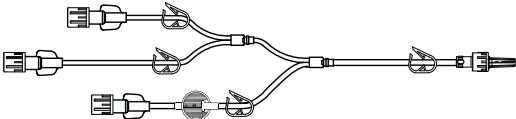
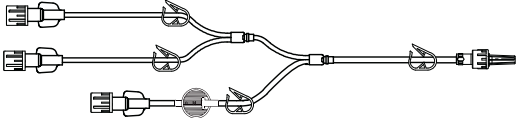
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves, VersaSafe split septum ports, filters, stopcocks, manifolds and/or T-connectors.

Model #	Drops	Description	Case quantity
ME1225 30-day lead time		 Extension set, 1.2 micron filter, slide clamp, male spin lock Approximate set length: 15" 38 cm. approx. Approximate priming volume: 4.8 mL	 50
ME1250 30-day lead time		 Extension set, 0.2 micron filter, non-removable slide clamp, male spin lock Approximate set length: 6" 15 cm approx. Approximate priming volume: 3.2 mL	 50
ME1254		 Microbore tri-fuse extension set, 0.2 micron filter, 4 pinch clamps, 2 white, 1 blue, 1 red, male spin lock Approximate set length: 10" 25 cm approx. Approximate priming volume: 0.6 mL	 50
ME1255 30-day lead time		 Microbore bi-fuse extension set, 0.2 micron filter, 3 white pinch clamps, male spin lock Approximate set length: 8.5" 22 cm approx. Approximate priming volume: 0.5 mL	 50
ME1256 30-day lead time		 Microbore tri-fuse extension set, two 0.2 micron filters, 4 pinch clamps, 2 white, 1 blue and 1 red, male spin lock Approximate set length: 8.5" 22 cm approx. Approximate priming volume: 0.8 mL	 50

04.00 Specialty extension sets

04.01.04 Filters | No port sets

Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves, VersaSafe split septum ports, filters, stopcocks, manifolds and/or T-connectors.

Model #	Drops	Description	Case quantity
ME2064 30-day lead time		 Microbore extension set, 0.2 micron filter, removable slide clamp, male spin lock Approximate set length: 60" 152 cm. approx. Approximate priming volume: 0.6 mL	 50
ME2069 30-day lead time		 Microbore extension set, 0.2 micron filter, white pinch clamp, male spin lock Approximate set length: 60" 152 cm approx. Approximate priming volume: 0.6 mL	 50
ME2070		 Microbore tri-fuse extension set, 0.2 micron filter, 4 white pinch clamps, male spin lock Approximate set length: 60" 152 cm approx. Approximate priming volume: 0.8 mL	 50
ME2071 30-day lead time		 Microbore tri-fuse extension set, 0.2 micron filter, 4 white pinch clamps, male spin lock Approximate set length: 60" 152 cm approx. Approximate priming volume: 1.1 mL	 50

04.00 Specialty extension sets

04.01.04 Filters | No port sets

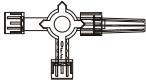
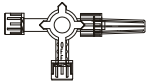
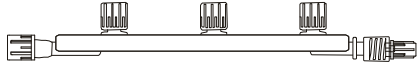
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves, VersaSafe split septum ports, filters, stopcocks, manifolds and/or T-connectors.

Model #	Drops	Description	Case quantity
20129E		 Smallbore extension set, 1.2 micron (low-protein binding) filter, fixed male luer lock Approximate set length: 7.5" 19 cm. approx. Approximate priming volume: 0.52 mL	 100
10010570		 Microbore extension set, 0.2 micron (low-protein binding) filter, fixed male luer lock Approximate set length: 60" 152 cm. approx. Approximate priming volume: 0.75 mL	 100
10011865		 Smallbore extension set, 0.2 micron (low-protein binding) filter, spin male luer lock Approximate set length: 7.5" 19 cm. approx. Approximate priming volume: 0.54 mL	 100
10012866		 Extension set, 1.2 micron (low-protein binding) filter, spin male luer lock Approximate set length: 18" 45 cm. approx. Approximate priming volume: 5 mL	 100

04.00 Specialty extension sets

04.02.01 Stopcocks and manifolds | Stand alone

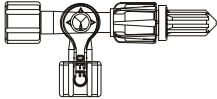
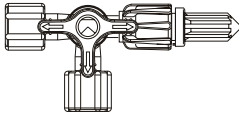
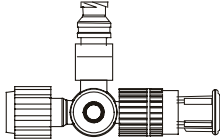
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves, VersaSafe split septum ports, filters, stopcocks, manifolds and/or T-connectors.

Model #	Drops	Description	Case quantity	
M4018		4-way stopcock, spin lock ("OFF" Indicator)		100
M4020		3-way stopcock, spin lock ("OFF" Indicator)		100
MM4000		3-port manifold, male spin lock, 3 one-way valves		50

04.00 Specialty extension sets

04.02.01 Stopcocks and manifolds | Stand alone

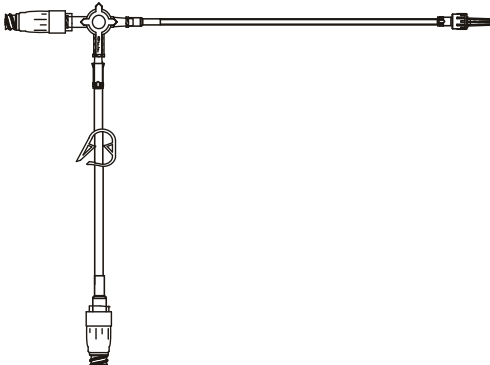
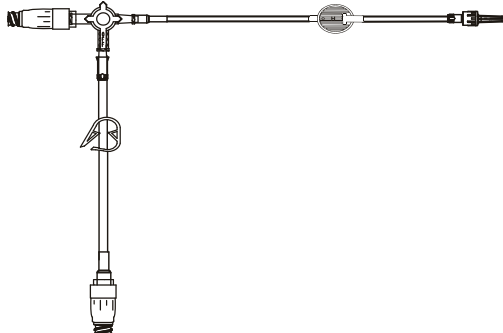
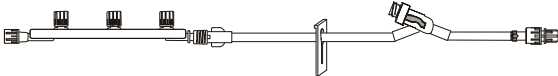
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves, VersaSafe split septum ports, filters, stopcocks, manifolds and/or T-connectors.

Model #	Drops	Description	Case quantity	
10012463				600
		3-way stopcock. Lipid resistant. ("OFF" Indicator)		
		Approximate priming volume: 0.2 mL		
10012496				600
		4-way stopcock. Lipid resistant		
		Approximate priming volume: 0.3 mL		
10237425				100
		SmartSite 4-way stopcock. Lipid resistant		
		Approximate priming volume: 0.34 mL		

04.00 Specialty extension sets

04.02.02 Stopcocks and manifolds | MaxPlus needleless connector and y-site sets

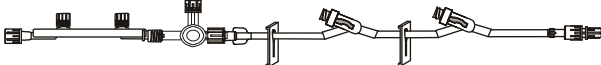
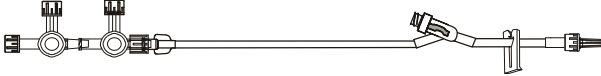
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves, VersaSafe split septum ports, filters, stopcocks, manifolds and/or T-connectors.

Model #	Drops	Description	Case quantity
MP4000-C			 50
		Extension set, 3-way stopcock ("OFF" Indicator), 2 MaxPlus clear needleless connectors, pinch clamp, male luer slip	
		Approximate set length: 77.5" 197 cm. approx.	
		Approximate priming volume: 4.9 mL	
MP4001-C			 50
		Extension set, 2 MaxPlus clear needleless connectors, 3-way stopcock ("OFF" Indicator), 0.2 µ filter	
		Approximate set length: 77.75" 197 cm. approx.	
		Approximate priming volume: 5 mL	
MM4300-1			 50
30-day lead time		Extension set, removable 3-port manifold, back check valve, needleless y-site, male spin lock	
		Approximate set length: 37" 94 cm. approx. Approximate priming volume: 6 mL	

04.00 Specialty extension sets

04.02.02 Stopcocks and manifolds | MaxPlus needleless connector and y-site sets

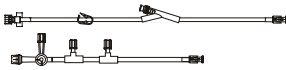
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves, VersaSafe split septum ports, filters, stopcocks, manifolds and/or T-connectors.

Model #	Drops	Description	Case quantity
MX4414 30-day lead time		 Extension set, 2 needleless y-sites, 4-way stopcock ("OFF" Indicator), removable 2-port manifold, 2 slide clamps, male spin lock Approximate set length: 37" 94 cm. approx. Approximate priming volume: 8 mL	 50
MX4439		 Extension set, 2 needleless y-sites, 4-way stopcock ("OFF" Indicator), slide clamp, male luer lock Approximate set length: 40" 102 cm. approx. Approximate priming volume: 7 mL	 50
MX4440 30-day lead time		 Extension set, needleless y-site, 3-way stopcock ("OFF" Indicator), slide clamp, male luer lock Approximate set length: 36" 91 cm. approx.	 50
MX4450		 Extension set, two 4-way stopcocks ("OFF" Indicator), needleless y-site, removable slide clamp, male luer lock Approximate set length: 64" 163 cm. approx. Approximate priming volume: 6.5 mL	 50

04.00 Specialty extension sets

04.02.03 Stopcocks and manifolds | SmartSite valve sets

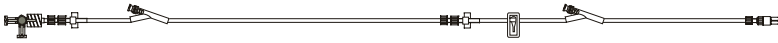
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves, VersaSafe split septum ports, filters, stopcocks, manifolds and/or T-connectors.

Model #	Drops	Description	Case quantity
20511E			 50
		Extension set, 1 needle-free valve, 2-piece male luer lock Extension set with two 1-way valves, one 4-way stopcock, fixed male luer lock	
		Approximate set length: 45" 114 cm. approx.	
		Approximate priming volume: 7 mL	
20519E			50
		Extension set, manifold with three 1-way valves, 1 needle-free valve 6" from 2-piece male luer lock	
		Approximate set length: 42" 107 cm. approx.	
		Approximate priming volume: 6 mL	
25202E			50
		Extension set, 4-way stopcock ("OFF" Indicator), 1 needle-free valve 6" from 2-piece male luer lock	
		Approximate set length: 37" 94 cm. approx.	
		Approximate priming volume: 5 mL	
25207E			 50
		Extension set, 2-ganged 4-way stopcock, 1 needle-free valve 6" from spin male luer lock	
		Approximate set length: 34" 86 cm. approx.	
		Approximate priming volume: 5 mL	
25208E			 50
		Extension set, 3-ganged 4-way stopcock, 1 needle-free valve 6" from spin male luer lock	
		Approximate set length: 34" 86 cm. approx.	
		Approximate priming volume: 5 mL	

04.00 Specialty extension sets

04.02.03 Stopcocks and manifolds | SmartSite valve sets

Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves, VersaSafe split septum ports, filters, stopcocks, manifolds and/or T-connectors.

Model #	Drops	Description	Case quantity
10016072	30-day lead time		50
		Extension set, 3-way stopcock, 1 needle-free valve	
		4" from spin male luer lock	
		Approximate set length: 39" 99 cm. approx. Approximate priming volume: 6 mL	
10108964			50
		Extension set, 4-way stopcock ("OFF" Indicator), 2 needle-free valves	
		28" and 4" from 2-piece male luer lock	
		Approximate set length: 38" 97 cm. approx. Approximate priming volume: 5 mL	

04.00 Specialty extension sets

04.02.04 Stopcocks and manifolds | Injection site sets

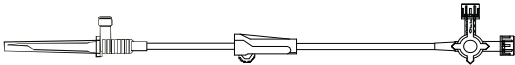
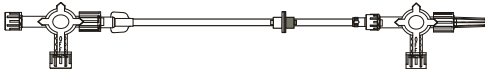
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves, VersaSafe split septum ports, filters, stopcocks, manifolds and/or T-connectors.

Model #	Drops	Description	Case quantity
MM4024		 Extension set, removable 3-port manifold, slide clamp, injection site, male spin lock Approximate set length: 40" 102 cm. approx.	 50

04.00 Specialty extension sets

04.02.05 Stopcocks and manifolds | No port sets

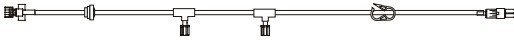
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves, VersaSafe split septum ports, filters, stopcocks, manifolds and/or T-connectors.

Model #	Drops	Description	Case quantity
M4025 30-day lead time		 Secondary transfer set, 4-way stopcock ("OFF" Indicator) roller clamp Approximate set length: 12" 30.4 cm. approx.	 50
M4026 30-day lead time		 Extension set, 3-way stopcock ("OFF" Indicator), male spin lock Approximate set length: 20" 50.8 cm. approx. Approximate priming volume: 3.2 mL	 50
M4058		 Extension set, 4-way stopcock ("OFF" Indicator), male spin lock Approximate set length: 36" 91.44 cm. approx.	 50
M4060 30-day lead time		 Extension set, 2 removable 4-way stopcocks ("OFF" Indicator), male spin lock Approximate set length: 35" 89 cm. approx. Approximate priming volume: 5.5 mL	 50
MM4021		 Extension set, 3-port manifold, male spin lock Approximate set length: 44" 112 cm. approx. Approximate priming volume: 7 mL	 50

04.00 Specialty extension sets

04.02.05 Stopcocks and manifolds | No port sets

Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves, VersaSafe split septum ports, filters, stopcocks, manifolds and/or T-connectors.

Model #	Drops	Description	Case quantity
25203		 Extension set, 3-way stopcock, spin male luer lock Approximate set length: 20" 51 cm. approx. Approximate priming volume: 2.9 mL	 72
25205	30-day lead time	 Extension set, 3-way stopcock, spin male luer lock Approximate set length: 32" 81 cm. approx. Approximate priming volume: 4 mL	 72
25206		 Extension set, 4-way stopcock, spin male luer lock Approximate set length: 32" 81 cm. approx. Approximate priming volume: 5 mL	 72
10012562	30-day lead time	 Extension set, two 1-way valves, 2-piece male luer lock Approximate set length: 39" 99 cm. approx. Approximate priming volume: 5 mL	 100
10013203		 Extension set, check valve, two 1-way valves, 2-piece male luer lock Approximate set length: 39" 99 cm. approx. Approximate priming volume: 6 mL	 100

04.00 Specialty extension sets

04.02.05 Stopcocks and manifolds | No port sets

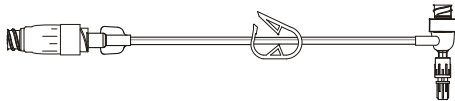
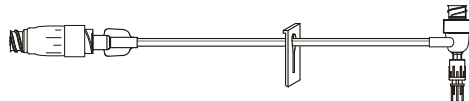
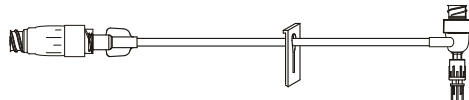
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves, VersaSafe split septum ports, filters, stopcocks, manifolds and/or T-connectors.

Model #	Drops	Description	Case quantity
10014472 30-day lead time		 Extension set, two 1-way valves, 4-way stopcock, spin male luer lock Approximate set length: 46" 117 cm. approx. Approximate priming volume: 8 mL	 100
10015487 30-day lead time		 Extension set, 3-way stopcock, spin male luer lock Approximate set length: 16" 41 cm. approx. Approximate priming volume: 2 mL	 72

04.00 Specialty extension sets

04.03.01 T-connectors | MaxPlus needleless connector sets

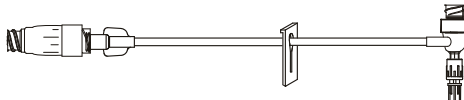
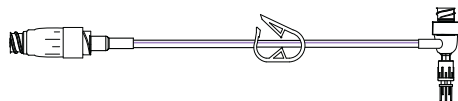
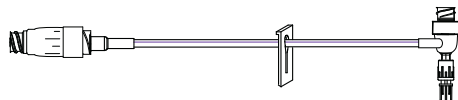
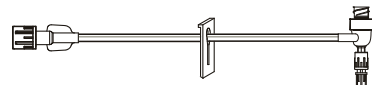
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves, VersaSafe split septum ports, filters, stopcocks, manifolds and/or T-connectors.

Model #	Drops	Description	Case quantity
MPXT1001-C			 50
		Pressure rated T-connector extension set, removable MaxPlus clear needleless connector, pinch clamp, male spin lock	
		Approximate set length: 6" 15 cm. approx.	
		Approximate priming volume: 1.3 mL	
		Labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second.	
MPXT1003-C			 50
		Minibore pressure rated T-connector extension set, removable MaxPlus clear needleless connector, slide clamp, male spin lock	
		Approximate set length: 6.5" 15 cm. approx.	
		Approximate priming volume: 1.1 mL	
		Labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second.	
MPXT1004-C			 50
		Minibore T-connector extension set, removable MaxPlus clear needleless connector, slide clamp, male luer slip	
		Approximate set length: 6" 15 cm. approx.	
		Approximate priming volume: 1.1 mL	
MPXT1005-C			 50
		Minibore T-connector extension set, removable MaxPlus clear needleless connector, slide clamp, male spin lock	
		Approximate set length: 6" 15 cm. approx.	
		Approximate priming volume: 1.1 mL	

04.00 Specialty extension sets

04.03.01 T-connectors | MaxPlus needleless connector sets

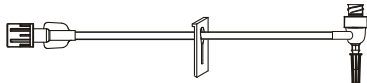
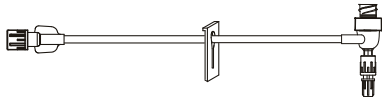
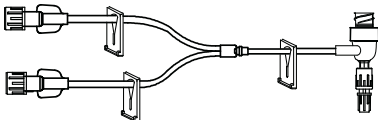
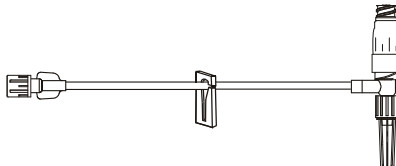
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves, VersaSafe split septum ports, filters, stopcocks, manifolds and/or T-connectors.

Model #	Drops	Description	Case quantity
MPXT1006			 50
		Minibore pressure rated T-connector extension set, removable needleless connector, slide clamp, male luer lock	
		Approximate set length: 6" 15 cm. approx.	
		Approximate priming volume: 1.1 mL	
		Labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second.	
MPXT5301-C			 50
		Pressure rated T-connector extension set, MaxPlus clear needleless connector, pinch clamp, male spin lock	
		Approximate set length: 7.5" 19 cm. approx.	
		Approximate priming volume: 1.3 mL	
		Labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second.	
MPXT5302-C			 50
		Minibore pressure rated T-connector extension set, with MaxPlus clear needleless connector, non-removable slide clamp, male spin lock	
		Approximate set length: 7.5" 19 cm. approx.	
		Approximate priming volume: 1.1 mL	
		Labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second.	
MXT1003			 50
		Minibore pressure rated T-connector extension set, slide clamp, male spin lock	
		Approximate set length: 6" 15 cm. approx.	
		Approximate priming volume: 0.8 mL	
		Labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second.	

04.00 Specialty extension sets

04.03.01 T-connectors | MaxPlus needleless connector sets

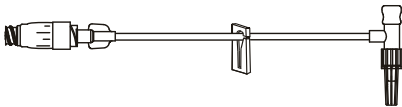
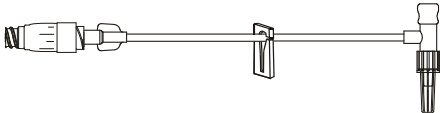
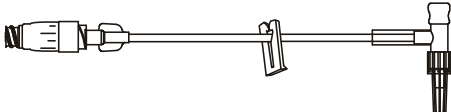
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves, VersaSafe split septum ports, filters, stopcocks, manifolds and/or T-connectors.

Model #	Drops	Description	Case quantity
MXT1004			 50
	Minibore T-connector extension set, slide clamp, male luer slip		
	Approximate set length:	6" 15 cm. approx.	
	Approximate priming volume:	0.8 mL	
MXT1005			 50
	Minibore pressure rated T-connector extension set, slide clamp, male luer lock		
	Approximate set length:	6" 15 cm. approx.	
	Approximate priming volume:	0.8 mL	
Labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second.			
MXT1006			 50
	Minibore bi-fuse t-connector extension set, 3 slide clamps, injection site, male luer lock		
	Approximate set length:	5.5" 14 cm. approx.	
	Approximate priming volume:	0.53 mL	
MP9208-C			 50
	Minibore extension set, clear needleless T-connector, removable slide clamp, male luer slip		
	Approximate set length:	6" 15.2 cm. approx.	
	Approximate priming volume:	1.28 mL	

04.00 Specialty extension sets

04.03.01 T-connectors | MaxPlus needleless connector sets

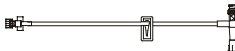
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves, VersaSafe split septum ports, filters, stopcocks, manifolds and/or T-connectors.

Model #	Drops	Description	Case quantity	
MP9216-C 30-day lead time				50
		Minibore t-connector extension set, removable clear needleless connector, removable slide clamp, injection site, male spin lock		
		Approximate set length: 7" 18 cm. approx.		
		Approximate priming volume: 0.8 mL		
MP9221-C				50
		Minibore T-connector extension set, removable MaxPlus clear needleless connector, removable slide clamp, injection site, male spin lock		
		Approximate set length: 5" 13 cm. approx.		
		Approximate priming volume: 0.3 mL		
MP9247-C				50
		T-connector extension set, removable MaxPlus clear needleless connector, removable slide clamp, injection site, male spin lock		
		Approximate set length: 6" 15 cm. approx.		
		Approximate priming volume: 1.3 mL		

04.00 Specialty extension sets

04.03.02 T-connectors | SmartSite valve sets

Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves, VersaSafe split septum ports, filters, stopcocks, manifolds and/or T-connectors.

Model #	Drops	Description	Case quantity
20041E			 100
	Smallbore extension set, 1 needle-free valve T-connector, spin male luer lock hub		
	Approximate set length:	6" 15 cm. approx.	
	Approximate priming volume:	0.23 mL	
Labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second.			
20043E			 100
	Smallbore extension set, 1 needle-free valve hi-flow T-connector, spin male luer lock hub		
	Approximate set length:	6" 15 cm. approx.	
	Approximate priming volume:	0.42 mL	
20051E			 100
	Smallbore extension set, 1 needle-free valve T-connector, luer slip adapter		
	Approximate set length:	6" 15 cm. approx.	
	Approximate priming volume:	0.23 mL	
20053E			 100
	Smallbore extension set, 1 needle-free valve hi-flow T-connector, luer slip adapter		
	Approximate set length:	5.5" 14 cm. approx.	
	Approximate priming volume:	0.4 mL	
10012476 30-day lead time			 100
	Smallbore extension set, 1 detachable needle free-valve, hi-flow T-connector needle free-valve, luer slip adapter		
	Approximate set length:	6" 15 cm. approx.	
	Approximate priming volume:	0.41 mL	

04.00 Specialty extension sets

04.03.02 T-connectors | SmartSite valve sets

Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves, VersaSafe split septum ports, filters, stopcocks, manifolds and/or T-connectors.

Model #	Drops	Description	Case quantity	
10013784 30-day lead time		Smallbore extension set, 1 needle free-valve 19" from T-connector needle-free valve, spin male luer lock hub		100
		Approximate set length: 23" 58 cm. approx.		
		Approximate priming volume: 0.75 mL		
C25013E		Smallbore extension set, 1 detachable needle-free valve, hi-flow T-connector needle-free valve, spin male luer lock hub		100
		Approximate set length: 7" 18 cm. approx.		
		Approximate priming volume: 0.48 mL		

04.00 Specialty extension sets

04.03.03 T-connectors | VersaSafe split septum sets

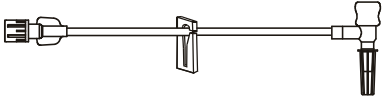
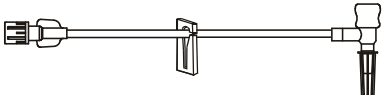
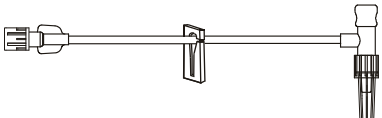
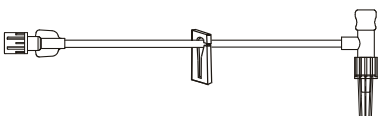
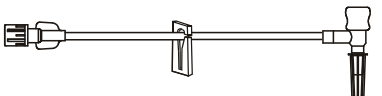
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus needleless connectors, SmartSite needle-free valves, VersaSafe split septum ports, filters, stopcocks, manifolds and/or T-connectors.

Model #	Drops	Description	Case quantity
10014813			 100
	Smallbore extension set, split septum injection port T-connector, luer slip adapter		
	Approximate set length:	6" 15 cm. approx.	
	Approximate priming volume:	0.3 mL	

04.00 Specialty extension sets

04.03.04 T-connectors | Other access sets

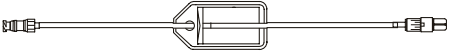
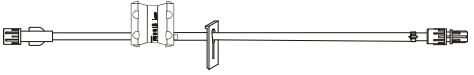
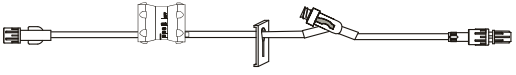
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves, VersaSafe split septum ports, filters, stopcocks, manifolds and/or T-connectors.

Model #	Drops	Description	Case quantity
ME1046		 Minibore T-connector extension set, injection site, male luer slip Approximate set length: 4" 10.2 cm. approx. Approximate priming volume: 0.33 mL	 50
ME1047 30-day lead time		 Minibore extension set, T-connector, injection site, male luer slip Approximate set length: 6" 15.2 cm. approx. Approximate priming volume: 0.4 mL	 50
ME1214		 Minibore T-connector extension set, removable slide clamp, injection site, male luer lock Approximate set length: 6" 15 cm. approx. Approximate priming volume: 0.3 mL	 50
ME1232		 Minibore T-connector extension set, removable slide clamp, injection site, male spin lock Approximate set length: 5" 13 cm. approx. Approximate priming volume: 0.3 mL	 50
ME1233		 Minibore T-connector extension set, removable slide clamp, injection site, male luer slip Approximate set length: 5" 13 cm. approx. Approximate priming volume: 0.3 mL	 50

04.00 Specialty extension sets

04.04.01 Specialty tubing | Filters and Flow Regulators

Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves, VersaSafe split septum ports, filters, stopcocks, manifolds and/or T-connectors.

Model #	Drops	Description	Case quantity
20350E			 100
		Low sorbing extension set, 0.2 micron (low-protein binding) filter, 1 needle-free valve 6" from spin male luer lock	
		Approximate set length: 19" 48 cm. approx.	
		Approximate priming volume: 5 mL	
MFS141			 100
		Extension set, flow regulator, 1 needleless y-site 7" from spin male lock	
		Approximate set length: 14" 36 cm. approx.	
		Approximate priming volume: 2.6 mL	
10013902			 100
		Low sorbing extension set, 0.2 micron (low-protein binding) filter, 1 needle free-valve, 2-piece male luer lock	
		Approximate set length: 16" 41 cm. approx.	
		Approximate priming volume: 5 mL	
MFS104			 50
		Extension set, flow regulator, non-removable slide clamp, male spin lock	
		Approximate set length: 20" 50.8 cm. approx.	
MFS106			 50
		Extension set, flow regulator, non-removable slide clamp, needleless y-site, male spin lock	
		Approximate set length: 23" 58.4 cm. approx.	
MFS107			 50
		Extension set, flow regulator, non-removable slide clamp, injection site, male spin lock	
		Approximate set length: 23" 58.42 cm. approx.	

04.00 Specialty extension sets

04.04.02 Specialty tubing | No port sets

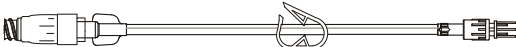
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves, VersaSafe split septum ports, filters, stopcocks, manifolds and/or T-connectors.

Model #	Drops	Description	Case quantity
30893		 Microbore administration set with yellow identification stripe, anti-siphon valve, fixed male luer lock Approximate set length: 112" 284 cm. approx. Approximate priming volume: 2.7 mL	 50
10241342		 Microbore extension set, amber tubing, fixed male luer lock Approximate set length: 62" 156 cm. approx. Approximate priming volume: 0.56 mL	 100
C20014		 Low sorbing extension set, 2-piece male luer lock Approximate set length: 30" 76 cm. approx. Approximate priming volume: 4 mL	 72

04.00 Specialty extension sets

04.05.01 Labeled for use with low pressure power injectors | MaxPlus and MaxGuard needleless connector sets

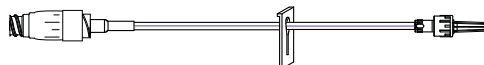
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves, VersaSafe split septum ports, filters, stopcocks, manifolds and/or T-connectors.

Model #	Drops	Description	Case quantity
MP1000-C			 100
		MaxPlus clear needleless connector	
		Approximate priming volume: 0.28 mL	
		Labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second.	
MP5301			 50
		Pressure rated extension set, MaxPlus needleless connector, pinch clamp, male spin lock	
		Approximate set length: 7.5" 19 cm. approx. Approximate priming volume: 0.8 mL	
		Labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second.	
MP5301-A			 50
		Pressure rated extension set, MaxGuard antimicrobial needleless connector, pinch clamp, male spin lock	
		Approximate set length: 8.5" 22 cm. approx. Approximate priming volume: 0.8 mL	
		Labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second.	
MP5301-C			 50
		Pressure rated extension set, MaxPlus clear needleless connector, pinch clamp, male spin lock	
		Approximate set length: 8.5" 22 cm. approx. Approximate priming volume: 0.8 mL	
		Labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second.	

04.00 Specialty extension sets

04.05.01 Labeled for use with low pressure power injectors | MaxPlus and MaxGuard needleless connector sets

Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves, VersaSafe split septum ports, filters, stopcocks, manifolds and/or T-connectors.

Model #	Drops	Description	Case quantity
MP5302			 50
		Minibore pressure rated extension set with MaxPlus needleless connector, non-removable slide clamp, male spin lock	
		Approximate set length: 6.5" 16 cm. approx.	
		Approximate priming volume: 0.59 mL	
		Labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second.	
MP5302-A			 50
		Minibore pressure rated extension set, MaxGuard antimicrobial needleless connector, non-removable slide clamp, male spin lock	
		Approximate set length: 8.5" 22 cm. approx.	
		Approximate priming volume: 0.6 mL	
		Labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second.	
MP5302-C			 50
		Minibore pressure rated extension set, MaxPlus clear needleless connector, non-removable slide clamp, male spin lock	
		Approximate set length: 8.5" 22 cm. approx.	
		Approximate priming volume: 0.59 mL	
		Labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second.	
MP5303	30-day lead time		 50
		Pressure rated extension set, removable MaxPlus needleless connector, non-removable pinch clamp, male spin lock	
		Approximate set length: 7" 18 cm. approx.	
		Approximate priming volume: 0.7 mL	
		Labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second.	

04.00 Specialty extension sets

04.05.01 Labeled for use with low pressure power injectors | MaxPlus and MaxGuard needleless connector sets

Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves, VersaSafe split septum ports, filters, stopcocks, manifolds and/or T-connectors.

Model #	Drops	Description	Case quantity
MP5303-A			 50
	Pressure rated extension set, removable MaxGuard antimicrobial needleless connector, pinch clamp, male spin lock		
	Approximate set length:	7" 18 cm. approx.	
	Approximate priming volume:	0.7 mL	
	Labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second.		
MP5303-C			 50
	Pressure rated extension set, removable MaxPlus clear needleless connector, pinch clamp, male spin lock		
	Approximate set length:	7" 18 cm. approx.	
	Approximate priming volume:	0.7 mL	
	Labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second.		
MP5304-C			 50
	Minibore pressure rated extension set with MaxPlus clear needleless connector, non-removable slide clamp, male spin lock		
	Approximate set length:	7" 18 cm. approx.	
	Approximate priming volume:	0.58 mL	
	Labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second.		
MP5305-C			 50
	Pressure rated extension set, removable MaxPlus clear needleless connector, male spin lock		
	Approximate set length:	7" 18 cm. approx.	
	Approximate priming volume:	0.7 mL	
	Labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second.		

04.00 Specialty extension sets

04.05.01 Labeled for use with low pressure power injectors | MaxPlus and MaxGuard needleless connector sets

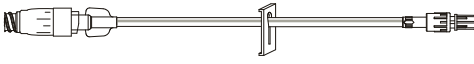
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves, VersaSafe split septum ports, filters, stopcocks, manifolds and/or T-connectors.

Model #	Drops	Description	Case quantity
MP5310-A 30-day lead time		 Minibore pressure rated extension set, MaxGuard antimicrobial needleless connector, male spin lock Approximate set length: 8.5" 22 cm. approx. Approximate priming volume: 0.59 mL Labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second.	 50
MP5310-C		 Minibore pressure rated extension set, MaxPlus clear needleless connector, male spin lock Approximate set length: 8.5" 22 cm. approx. Approximate priming volume: 0.59 mL Labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second.	 50
MP5311-C 30-day lead time		 Minibore pressure rated extension set, removable MaxPlus clear needleless connector, male spin lock Approximate set length: 8.5" 22 cm. approx. Approximate priming volume: 0.5 mL Labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second.	 50
MP5312-A		 Pressure rated extension set, MaxGuard antimicrobial needleless connector, male spin lock Approximate set length: 8.5" 22 cm. approx. Approximate priming volume: 0.8 mL Labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second.	 50

04.00 Specialty extension sets

04.05.01 Labeled for use with low pressure power injectors | MaxPlus and MaxGuard needleless connector sets

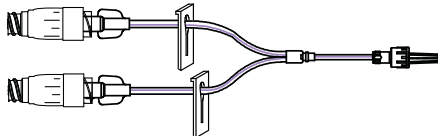
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves, VersaSafe split septum ports, filters, stopcocks, manifolds and/or T-connectors.

Model #	Drops	Description	Case quantity
MP5312-C			 50
		Pressure rated extension set, clear MaxPlus needleless connector, male spin lock	
		Approximate set length: 8.5" 22 cm. approx.	
		Approximate priming volume: 0.8 mL	
		Labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second.	
MP5313-A			 50
		Minibore pressure rated extension set, removable MaxGuard antimicrobial needleless connector, non-removable slide clamp, male spin lock	
		Approximate set length: 8.5" 22 cm. approx.	
		Approximate priming volume: 0.59 mL	
		Labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second.	
MP5313-C			 50
		Minibore pressure rated extension set, removable MaxPlus clear needleless connector, non-removable slide clamp, male spin lock	
		Approximate set length: 8.5" 22 cm. approx.	
		Approximate priming volume: 0.59 mL	
		Labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second.	
MP5314-C			 50
		Pressure rated extension set, removable MaxPlus clear needleless connector, non-removable slide clamp, male spin lock	
		Approximate set length: 7" 18 cm. approx.	
		Approximate priming volume: 0.7 mL	
		Labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second.	

04.00 Specialty extension sets

04.05.01 Labeled for use with low pressure power injectors | MaxPlus and MaxGuard needleless connector sets

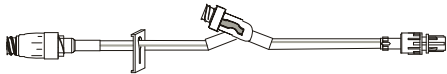
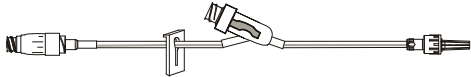
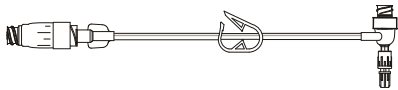
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves, VersaSafe split septum ports, filters, stopcocks, manifolds and/or T-connectors.

Model #	Drops	Description	Case quantity
MP5315-C			 50
		Bi-fuse pressure rated extension set, 2 removable MaxPlus clear needleless connectors, 2 non-removable slide clamps, male spin lock	
		Approximate set length: 8.375" 21 cm. approx.	
		Approximate priming volume: 1.1 mL	
		Labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second.	
MPX5300-C			 50
		Pressure rated extension set, MaxPlus clear needleless connector, pinch clamp, needleless y-site, male spin lock	
		Approximate set length: 9" 23 cm. approx.	
		Approximate priming volume: 1.2 mL	
		Labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second.	
MPX5301-C			 50
		Minibore pressure rated extension set, removable MaxPlus clear needleless connector, non-removable slide clamp, needleless y-site, male spin lock	
		Approximate set length: 7.5" 19 cm. approx.	
		Approximate priming volume: 1 mL	
		Labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second.	
MPX5302-C			 50
		Pressure rated extension set, removable MaxPlus needleless connector, pinch clamp, needleless y-site, male spin lock	
		Approximate set length: 9" 23 cm. approx.	
		Approximate priming volume: 1.2 mL	
		Labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second.	

04.00 Specialty extension sets

04.05.01 Labeled for use with low pressure power injectors | MaxPlus and MaxGuard needleless connector sets

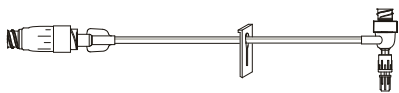
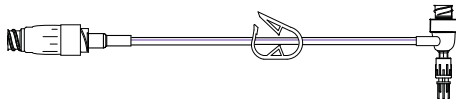
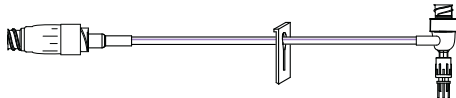
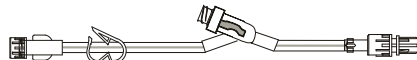
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves, VersaSafe split septum ports, filters, stopcocks, manifolds and/or T-connectors.

Model #	Drops	Description	Case quantity
MPX5303-C 30-day lead time		 Pressure rated extension set, y-site, MaxPlus clear needleless connector, needleless y-site, male spin lock Approximate set length: 9" 23 cm. approx. Approximate priming volume: 1.2 mL Labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second.	 50
MPX5304-C		 Pressure rated extension set, MaxcPlus clear needleless connector, non-removable slide clamp, needleless y-site, male spin lock Approximate set length: 11" 28 cm. approx. Approximate priming volume: 1.29 mL Labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second.	 50
MPX5305-C		 Minibore pressure rated extension set, MaxPlus clear needleless connector, non-removable slide clamp, needleless y-site, male spin lock Approximate set length: 7.5" 19 cm. approx. Approximate priming volume: 1 mL Labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second.	 50
MPXT1001-C		 Pressure rated T-connector extension set, removable MaxPlus clear needleless connector, pinch clamp, male spin lock Approximate set length: 6" 15 cm. approx. Approximate priming volume: 1.3 mL Labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second.	 50

04.00 Specialty extension sets

04.05.01 Labeled for use with low pressure power injectors | MaxPlus and MaxGuard needleless connector sets

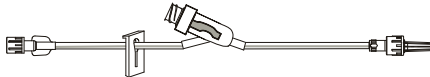
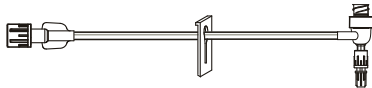
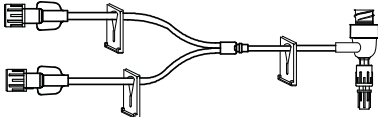
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves, VersaSafe split septum ports, filters, stopcocks, manifolds and/or T-connectors.

Model #	Drops	Description	Case quantity
MPXT1003-C			 50
		Minibore pressure rated T-connector extension set, removable MaxPlus clear needleless connector, non-removable slide clamp, male spin lock	
		Approximate set length: 6.5" 15 cm. approx.	
		Approximate priming volume: 1.1 mL	
Labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second.			
MPXT5301-C			 50
		Pressure rated T-connector extension set, MaxPlus clear needleless connector, pinch clamp, male spin lock	
		Approximate set length: 7.5" 19 cm. approx.	
		Approximate priming volume: 1.3 mL	
Labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second.			
MPXT5302-C			 50
		Minibore pressure rated T-connector extension set, with MaxPlus clear needleless connector, non-removable slide clamp, male spin lock	
		Approximate set length: 7.5" 19 cm. approx.	
		Approximate priming volume: 1.1 mL	
Labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second.			
MX5301			 50
		Pressure rated extension set, needleless y-site, pinch clamp, male spin lock	
		Approximate set length: 8" 20 cm. approx.	
		Approximate priming volume: 0.8 mL	
Labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second.			

04.00 Specialty extension sets

04.05.01 Labeled for use with low pressure power injectors | MaxPlus and MaxGuard needleless connector sets

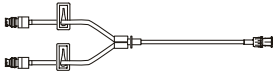
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves, VersaSafe split septum ports, filters, stopcocks, manifolds and/or T-connectors.

Model #	Drops	Description	Case quantity
MX5302			 50
	Minibore pressure rated extension set, needleless y-site, non-removable slide clamp, male spin lock		
	Approximate set length:	6" 15 cm. approx.	
	Approximate priming volume:	0.7 mL	
	Labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second.		
MXT1003			 50
	Minibore pressure rated T-connector extension set, non-removable slide clamp, male spin lock		
	Approximate set length:	6" 15 cm. approx.	
	Approximate priming volume:	0.8 mL	
	Labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second.		
MXT1006			 50
	Minibore pressure rated bi-fuse T-connector extension set, 3 non-removable slide clamps, male luer lock		
	Approximate set length:	5.5" 14 cm. approx.	
	Approximate priming volume:	0.53 mL	
	Labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second.		

04.00 Specialty extension sets

04.05.02 Labeled for use with low pressure power injectors | SmartSite valve sets

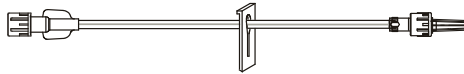
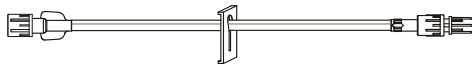
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves, VersaSafe split septum ports, filters, stopcocks, manifolds and/or T-connectors.

Model #	Drops	Description	Case quantity
2000E		 SmartSite needle-free valve Approximate priming volume: 0.11 mL Labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second.	 100
20019E		 Smallbore Y-connector extension set, 2 needle-free valves, spin male luer lock Approximate set length: 9" 23 cm. approx. Approximate priming volume: 0.33 mL Labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second.	 100
20039E		 Smallbore extension set, 1 needle-free valve, spin male luer lock Approximate set length: 6" 15 cm. approx. Approximate priming volume: 0.17 mL Labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second.	 100
20041E		 Smallbore extension set, 1 needle-free valve T-connector, spin male luer lock hub Approximate set length: 6" 15 cm. approx. Approximate priming volume: 0.23 mL Labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second.	 100
10011253		 Smallbore extension set, 1 detachable needle-free valve, spin male luer lock Approximate set length: 7.5" 19 cm. approx. Approximate priming volume: 0.29 mL Labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second.	 100

04.00 Specialty extension sets

04.05.03 Labeled for use with low pressure power injectors | No port sets

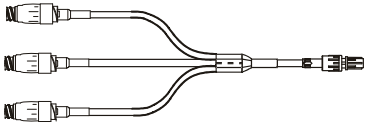
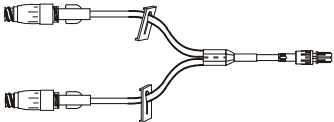
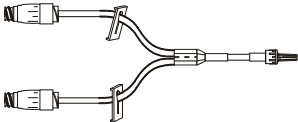
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves, VersaSafe split septum ports, filters, stopcocks, manifolds and/or T-connectors.

Model #	Drops	Description	Case quantity
ME5301			 50
	Pressure rated extension set, pinch clamp male spin lock		
	Approximate set length:	7.5" 19 cm. approx.	
	Approximate priming volume:	0.4 mL	
ME5302			 50
	Minibore pressure rated extension set, non-removable slide clamp, male spin lock		
	Approximate set length:	6.5" 16 cm. approx.	
	Approximate priming volume:	0.2 mL	
ME5303			 50
	Pressure rated extension set, non-removable slide clamp, male spin lock		
	Approximate set length:	7.5" 19 cm. approx.	
	Approximate priming volume:	0.4 mL	
ME5304			 50
	Pressure rated extension set, male spin lock		
	Approximate set length:	7.5" 19 cm. approx.	
	Approximate priming volume:	0.4 mL	

05.00 Extension sets

05.01.01 Standard tubing | MaxPlus and MaxGuard needleless connector sets

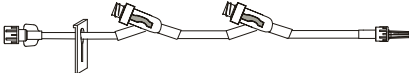
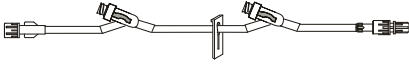
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves and VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
MP9202 30-day lead time		 Extension set, MaxPlus needleless connector, male luer lock Approximate set length: 7" 18 cm. approx.. Approximate priming volume: 1.3 mL	 50
MP9202-C 30-day lead time		 Extension set, MaxPlus clear needleless connector, male luer lock Approximate set length: 7" 18 cm. approx. Approximate priming volume: 1.3 mL	 50
MP9230 30-day lead time		 Tri-fuse extension set, 3 MaxPlus needleless connectors, male spin lock Approximate set length: 7" 18 cm. approx. Approximate priming volume: 2.4 mL	50
MP9238-C		 Bi-fuse extension set, 2 removable MaxPlus clear connectors, 2 non-removable slide clamps, male spin lock Approximate set length: 8.5" 21 cm. approx. Approximate priming volume: 2.3 mL	50
MP9246-C		 Bi-fuse extension set, 2 MaxPlus clear needleless connectors, 2 non-removable slide clamps, male luer lock Approximate set length: 8" 20 cm. approx. Approximate priming volume: 2.2 mL	50

05.00 Extension sets

05.01.01 Standard tubing | MaxPlus and MaxGuard needleless connector sets

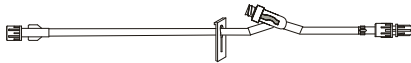
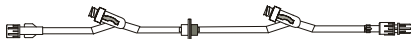
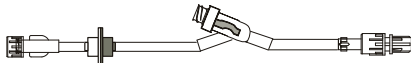
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves and VersaSafe split septum ports.

Model #	Drops	Description	Case quantity	
MX9037 30-day lead time				50
		Extension set, 2 needleless y-sites, non-removable slide clamp, male luer lock		
		Approximate set length: 8.5" 22 cm. approx. Approximate priming volume: 2 mL		
MX9059				50
		Extension set, 2 needleless y-sites, non-removable slide clamp, male spin lock		
		Approximate set length: 8.5" 22 cm. approx. Approximate priming volume: 2 mL		
MX9076 30-day lead time				50
		Extension set, 3 needleless y-sites, 2 pinch clamps, male spin lock		
		Approximate set length: 19" 48 cm. approx. Approximate priming volume: 3.8 mL		
MX9172				50
		Extension set, 2 needleless y-sites, 2 pinch clamps, male spin lock		
		Approximate set length: 40" 102 cm. approx. Approximate priming volume: 7.5 mL		

05.00 Extension sets

05.01.01 Standard tubing | MaxPlus and MaxGuard needleless connector sets

Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves and VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
MX9175			 50
		Extension set, needleless y-site, non-removable slide clamp, male spin lock	
	Approximate set length:	32" 81 cm. approx.	
	Approximate priming volume:	3.5 mL	
MX9178			 50
		Extension set, 2 needleless y-sites, back check valve, male spin lock	
	Approximate set length:	50" 127 cm. approx.	
	Approximate priming volume:	8.4 mL	
MX9182			 50
		Extension set, needleless y-site, check valve, male spin lock	
	Approximate set length:	8" 20 cm. approx.	
	Approximate priming volume:	1.4 mL	

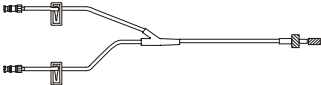
30-day lead time

30-day lead time

05.00 Extension sets

05.01.02 Standard tubing | SmartSite valve sets

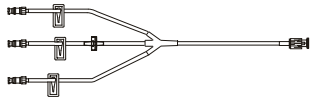
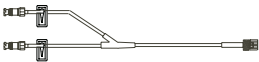
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves and VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
20057E			100
		Extension set, 1 needle-free valve, luer slip adapter	
		Approximate set length: 8.5" 22 cm. approx.	
		Approximate priming volume: 1 mL	
20059E			100
		Extension set, 1 needle-free valve, spin male luer lock	
		Approximate set length: 8.5" 22 cm. approx.	
		Approximate priming volume: 0.93 mL	
20060E	30-day lead time		 100
		Extension set, 1 needle-free valve, spin male luer lock	
		Approximate set length: 5" 13 cm. approx.	
		Approximate priming volume: 0.63 mL	
20061E	30-day lead time		 100
		Extension set, Y-connector, 2 needle-free valves, spin male luer lock	
		Approximate set length: 6.5" 17 cm. approx.	
		Approximate priming volume: 1.5 mL	

05.00 Extension sets

05.01.02 Standard tubing | SmartSite valve sets

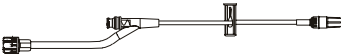
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves and VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
20062E		Tri-port extension set, smallbore tubing segments, 3 needle-free valves, check valve, spin male luer lock	 100
		Approximate set length: 7.5" 19 cm. approx.	
		Approximate priming volume: 1.5 mL	
20159E		Y-connector extension set, 2 needle-free valves, spin male luer lock	 100
		Approximate set length: 10" 25 cm. approx.	
		Approximate priming volume: 1.9 mL	
21059E		Extension set, 1 needle-free valve, fixed male luer lock	100
		Approximate set length: 8" 20 cm. approx.	
		Approximate priming volume: 0.87 mL	
22059E		Extension set, 1 detachable needle-free valve, spin male luer lock	100
		Approximate set length: 9" 23 cm. approx.	
		Approximate priming volume: 1 mL	
30202E		Extension set, 1 needle-free valve 10" from spin male luer lock	100
		Approximate set length: 21" 53 cm. approx.	
		Approximate priming volume: 2.6 mL	

05.00 Extension sets

05.01.02 Standard tubing | SmartSite valve sets

Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves and VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
30204E			100
	Extension set, 1 needle-free valve 6" from spin male luer lock		
	Approximate set length:	8" 20 cm. approx.	
	Approximate priming volume:	1 mL	
30205E			100
	Extension set, 1 needle-free valve 4" from spin male luer lock		
	Approximate set length:	8" 21 cm. approx.	
	Approximate priming volume:	0.93 mL	
30212E			 100
	Extension set, smallbore tubing segment, 1 needle-free valve 9" from spin male luer lock		
	Approximate set length:	21" 53 cm. approx.	
	Approximate priming volume:	2.2 mL	
30262E			100
	Extension set, 2 needle-free valves 32" and 8" from spin male luer lock		
	Approximate set length:	41" 104 cm. approx.	
	Approximate priming volume:	5 mL	
30263E			100
	Extension set, 2 needle-free valves 32" and 8" from 2-piece male luer lock		
	Approximate set length:	41" 104 cm. approx.	
	Approximate priming volume:	5 mL	
30842E			 100
	Extension set, smallbore tubing segment, check valve, 1 needle-free valve, fixed male luer lock		
	Approximate set length:	12" 30 cm. approx.	
	Approximate priming volume:	1.3 mL	

05.00 Extension sets

05.01.02 Standard tubing | SmartSite valve sets

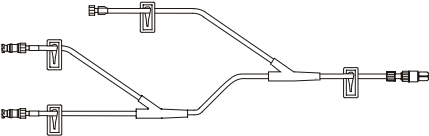
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves and VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
31202E			100
		Extension set, 1 needle-free valve 10" from fixed male luer lock	
		Approximate set length: 21" 53 cm. approx.	
		Approximate priming volume: 2.6 mL	
31262E			100
		Extension set, 2 needle-free valves 32" and 8" from fixed male luer lock	
		Approximate set length: 41" 104 cm. approx.	
		Approximate priming volume: 5 mL	
37262E			 100
		Extension set, 2 needle-free valves 32" and 7" from fixed male luer lock	
		Approximate set length: 41" 104 cm. approx.	
		Approximate priming volume: 6 mL	
10010912			 100
		Extension set, 1 needle-free valve, spin male luer lock	
		Approximate set length: 8.5" 22 cm. approx.	
		Approximate priming volume: 1.2 mL	

05.00 Extension sets

05.01.02 Standard tubing | SmartSite valve sets

Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves and VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
10010971 30-day lead time		 Extension set, detachable tri-port connector, 2 needle-free valves 32" and 7" from spin male luer. Approximate set length: 43" 109 cm. approx. Approximate priming volume: 6 mL	100
10010977 30-day lead time		 Extension set, 1 needle-free valve 10" from 2-piece fixed male luer lock Approximate set length: 21" 53 cm. approx. Approximate priming volume: 2.5 mL	100
10012143		 Y-connector extension set, 1 detachable needle free-valve, female luer port, spin male luer lock Approximate set length: 11" 28 cm. approx. Approximate priming volume: 1.7 mL	100
10014692 30-day lead time		 Tri-port extension set, 2 needle-free valves, 1 female luer lock, 2-piece fixed male luer lock Approximate set length: 10" 25 cm. approx. Approximate priming volume: 2.1 mL	 100
11590099 30-day lead time		 Extension set, 1 needle-free valve 4" from spin male luer lock Approximate set length: 10" 25 cm. approx. Approximate priming volume: 1.4 mL	 100

05.00 Extension sets

05.01.02 Standard tubing | SmartSite valve sets

Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves and VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
C25004E			100
		Extension set, 1 needle-free valve 10" from spin male luer lock	
		Approximate set length: 182" 462 cm. approx.	
		Approximate priming volume: 24 mL	
C25015E			100
		Extension set, 1 detachable needle-free valve, luer slip adapter	
		Approximate set length: 7.5" 19 cm. approx.	
		Approximate priming volume: 0.77 mL	

05.00 Extension sets

05.01.03 Standard tubing | VersaSafe split septum sets

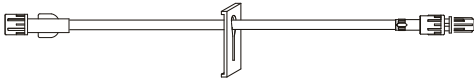
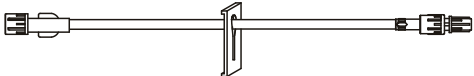
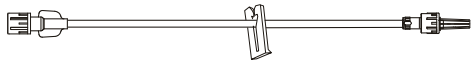
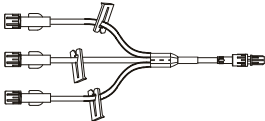
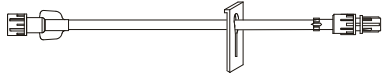
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves and VersaSafe split septum ports.

Model #	Drops	Description	Case quantity	
10796814				100
	Extension set, 2 split septum ports 23" and 7" from spin male luer lock			
	Approximate set length:	41" 103 cm. approx.		
	Approximate priming volume:	6 mL		
10807854				100
	Extension set, 1 split septum port 4" from spin male luer lock			
	Approximate set length:	9" 23 cm. approx.		
	Approximate priming volume:	1.4 mL		

05.00 Extension sets

05.01.04 Standard tubing | No port sets

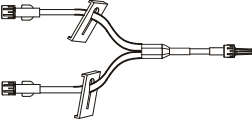
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves and VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
ME1055			 50
		Extension set, slide clamp, male spin lock	
		Approximate set length: 30" 76 cm. approx. Approximate priming volume: 4.9 mL	
ME1069			 50
		Extension set, slide clamp, male spin lock	
		Approximate set length: 20" 51 cm. approx. Approximate priming volume: 2.4 mL	
ME1223			 50
		Extension set, removable slide clamp, male spin lock	
		Approximate set length: 7" 18 cm. approx. Approximate priming volume: 1.0 mL	
ME1224	30-day lead time		50
		Tri-fuse extension set, 3 removable slide clamps, male spin lock	
		Approximate set length: 6" 15.2 cm. approx. Approximate priming volume: 2 mL	
ME1229	30-day lead time		 50
		Extension set, non-removable slide clamp, male spin lock	
		Approximate set length: 5" 12.7 cm. approx. Approximate priming volume: 0.7 mL	
ME1230	30-day lead time		 50
		Extension set, pinch clamp, 2 male luer locks	
		Approximate set length: 40" 102 cm. approx.	

05.00 Extension sets

05.01.04 Standard tubing | No port sets

Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves and VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
ME1242 30-day lead time		 Extension set, roller clamp, 2 male luers, spin locks Approximate set length: 40"102 cm. approx. Approximate priming volume: 6 mL	 50
ME1246 30-day lead time		 Extension set, male spin lock Approximate set length: 3"8 cm. approx. Approximate priming volume: 0.6 mL	 50
ME1247		 Bi-fuse extension set, 2 non-removable slide clamps, male luer lock Approximate set length: 7"18 cm. approx. Approximate priming volume: 1.6 mL	 50
ME1248		 Extension set, pinch clamp, male spin lock Approximate set length: 5"13 cm. approx. Approximate priming volume: 1 mL	 50
ME2068		 Extension set, male luer lock Approximate set length: 60"152 cm. approx. Approximate priming volume: 9.4 mL	 50
10012213		 Extension set, pinch clamp, 2-piece male luer lock Approximate set length: 60"152 cm. approx. Approximate priming volume: 8 mL	100

05.00 Extension sets

05.01.04 Standard tubing | No port sets

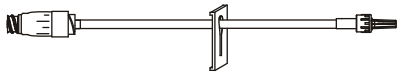
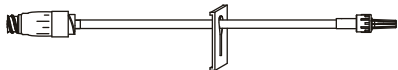
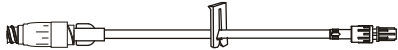
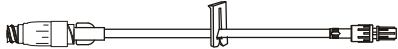
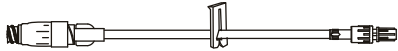
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves and VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
10015485 30-day lead time		 Extension set, pinch clamp, 2-piece male luer lock Approximate set length: 8.5" 20 cm. approx. Approximate priming volume: 1 mL	 100
		 Extension set, non-removable slide clamp, 2-piece male luer lock Approximate set length: 8.5" 22 cm. approx. Approximate priming volume: 0.91 mL	72
		 Extension set, 2-piece male luer lock Approximate set length: 20" 51 cm. approx. Approximate priming volume: 2.8 mL	 100
C20014		 Low sorbing extension set, 2-piece male luer lock Approximate set length: 30" 76 cm. approx. Approximate priming volume: 4 mL	 72
C20022		 Extension set, pinch clamp, 2-piece male luer lock Approximate set length: 20" 51 cm. approx. Approximate priming volume: 2.8 mL	 72
C20025		 Extension set, pinch clamp, 2-piece male luer lock Approximate set length: 10" 25 cm. approx. Approximate priming volume: 1.2 mL	72

05.00 Extension sets

05.02.01 Kink resistant tubing | MaxPlus and MaxGuard needleless connector sets

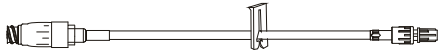
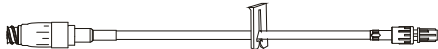
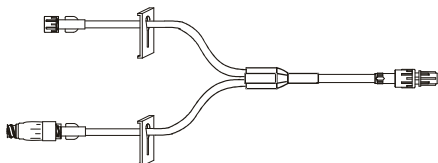
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves and VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
MP2005 30-day lead time			 50
		Kink resistant extension set, MaxPlus needleless connector, non-removable slide clamp, male luer lock Approximate set length: 7" 18 cm. approx. Approximate priming volume: 1 mL	
MP2005-C			 50
		Kink resistant extension set, MaxPlus clear needleless connector, non-removable slide clamp, male luer lock Approximate set length: 7" 18 cm. approx. Approximate priming volume: 1 mL	
MP9002			 50
		Kink resistant extension set, MaxPlus needleless connector, removable slide clamp, male spin lock Approximate set length: 7" 18 cm. approx. Approximate priming volume: 0.9 mL	
MP9002-A			 50
		Kink resistant extension set, removable MaxGuard antimicrobial needleless connector, removable slide clamp, male spin clamp Approximate set length: 7" 18 cm. approx. Approximate priming volume: 1 mL	
MP9002-C			 50
		Kink resistant extension set, MaxPlus clear needleless connector, removable slide clamp, male spin lock Approximate set length: 7" 18 cm. approx. Approximate priming volume: 0.9 mL	

05.00 Extension sets

05.02.01 Kink resistant tubing | MaxPlus and MaxGuard needleless connector sets

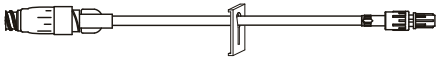
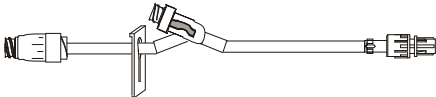
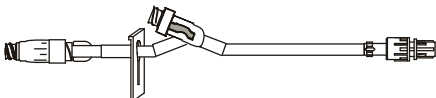
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves and VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
MP9081-A			 50
		Kink resistant extension set, MaxGuard antimicrobial needleless connector, removable slide clamp, male spin lock	
		Approximate set length: 8" 20 cm. approx. Approximate priming volume: 1.2 mL	
MP9081-C			 50
		Kink resistant extension set, MaxPlus clear needleless connector, removable slide clamp, male spin lock	
		Approximate set length: 8" 20 cm. approx. Approximate priming volume: 1.2 mL	
MP9081-SF			 50
		Kink resistant extension set, MaxPlus needleless connector, removable slide clamp, male spin lock	
		Approximate set length: 8" 20 cm. approx. Approximate priming volume: 1.2 mL	
MP9213-C 30-day lead time			 50
		Kink resistant extension set, MaxPlus clear needleless connector, pinch clamp, male spin lock	
		Approximate set length: 7" 18 cm. approx. Approximate priming volume: 1 mL	
MP9256-C 30-day lead time			 50
		Kink resistant bi-fuse extension set, removable MaxPlus clear needleless connector, removable slide clamp, male spin lock	
		Approximate set length: 7" 18 cm. approx. Approximate priming volume: 1.4 mL	

05.00 Extension sets

05.02.01 Kink resistant tubing | MaxPlus and MaxGuard needleless connector sets

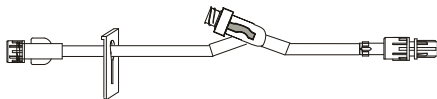
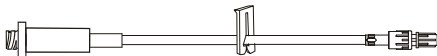
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves and VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
MP9259-C			 50
		Kink resistant extension set, removable MaxPlus clear needleless connector, non-removable slide clamp, male spin lock	
		Approximate set length: 7" 18 cm. approx.	
		Approximate priming volume: 0.9 mL	
MPX9100			 50
		Kink resistant extension set, MaxPlus needleless connector, non-removable slide clamp, needleless y-site, male spin lock	
		Approximate set length: 8" 20.3 cm. approx.	
		Approximate priming volume: 1.1 mL	
MPX9102-C			 50
		Kink resistant extension set, MaxPlus clear needleless connector, needleless y-site, male spin lock	
		Approximate set length: 9.5" 24 cm. approx.	
		Approximate priming volume: 1.6 mL	
MPX9102-SF-C 30-day lead time			 50
		Kink resistant extension set, MaxPlus clear needleless connector, needleless y-site, male spin lock	
		Approximate set length: 9.5" 24 cm. approx.	
		Approximate priming volume: 1.6 mL	
MPX9103			 50
		Kink resistant extension set, removable MaxPlus needleless connector, needleless y-site, non-removable slide clamp, male spin lock	
		Approximate set length: 8" 20.3 cm. approx.	
		Approximate priming volume: 1.1 mL	

05.00 Extension sets

05.02.01 Kink resistant tubing | MaxPlus and MaxGuard needleless connector sets

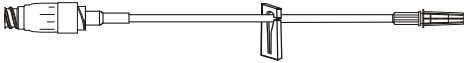
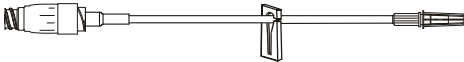
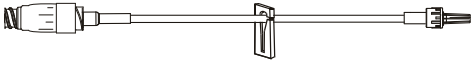
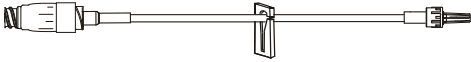
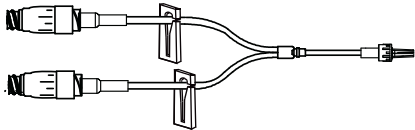
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves and VersaSafe split septum ports.

Model #	Drops	Description	Case quantity	
MX9060				50
	Kink resistant extension set, needleless y-site, non-removable slide clamp, male spin lock			
	Approximate set length:	8"20.3 cm. approx.		
	Approximate priming volume:	1.2 mL		
MX9164				50
	Kink resistant extension set, standard needleless connector, removable slide clamp, male spin lock			
	Approximate set length:	7"18 cm. approx.		
	Approximate priming volume:	1 mL		

05.00 Extension sets

05.03.01 Minibore/Smallbore tubing | MaxPlus and MaxGuard needleless connector sets

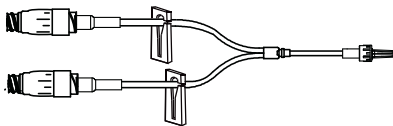
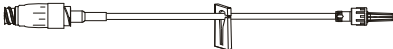
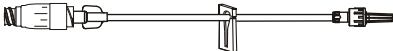
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves and VersaSafe split septum ports.

Model #	Drops	Description	Case quantity	
MP2000				50
		Minibore extension set, MaxPlus needleless connector, removable slide clamp, male luer slip		
		Approximate set length: 7" 17.8 cm. approx. Approximate priming volume: 0.5 mL		
MP2000-C				50
		Minibore extension set, MaxPlus clear needleless connector, removable slide clamp, male luer slip		
		Approximate set length: 7" 18 cm. approx. Approximate priming volume: 0.59 mL		
MP2002				50
		Minibore extension set, MaxPlus needleless connector, removable slide clamp, male luer lock		
		Approximate set length: 7" 18 cm. approx. Approximate priming volume: 0.51 mL		
MP2002-C				50
		Minibore extension set, MaxPlus clear needleless connector, removable slide clamp, male luer lock		
		Approximate set length: 7" 18 cm. approx. Approximate priming volume: 0.51 mL		
MP2202	30-day lead time			50
		Minibore bi-fuse extension set, 2 MaxPlus needleless connectors, 2 removable slide clamps, male luer lock		
		Approximate set length: 5" 13 cm. approx. Approximate priming volume: 1.2 mL		

05.00 Extension sets

05.03.01 Minibore/Smallbore tubing | MaxPlus and MaxGuard needleless connector sets

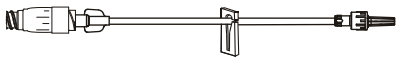
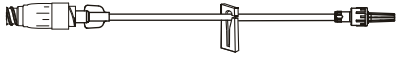
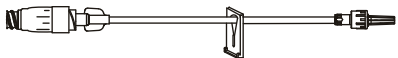
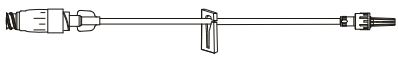
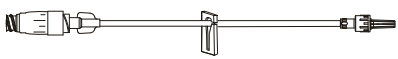
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves and VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
MP2202-C			 50
		Minibore bi-fuse extension set, 2 MaxPlus clear needleless connectors, 2 removable slide clamps, male luer lock	
		Approximate set length: 5" 13 cm. approx.	
		Approximate priming volume: 1.2 mL	
MP9001			 50
		Minibore extension set, removable MaxPlus needleless connector, removable slide clamp, male spin lock	
		Approximate set length: 7" 18 cm. approx.	
		Approximate priming volume: 0.51 mL	
MP9001-A			 50
		Minibore extension set, MaxGuard antimicrobial needleless connector, removable slide clamp, male spin lock	
		Approximate set length: 7" 18 cm. approx.	
		Approximate priming volume: 0.5 mL	
MP9001-C			 50
		Minibore extension set, MaxPlus clear needleless connector, removable slide clamp, male spin lock	
		Approximate set length: 7" 18 cm. approx.	
		Approximate priming volume: 0.51 mL	
MP9004			 50
		Minibore extension set, removable MaxPlus needleless connector, removable slide clamp, male spin lock	
		Approximate set length: 7" 18 cm. approx.	
		Approximate priming volume: 0.51 mL	

05.00 Extension sets

05.03.01 Minibore/Smallbore tubing | MaxPlus and MaxGuard needleless connector sets

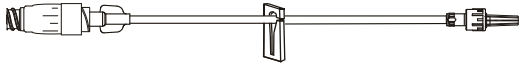
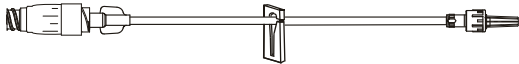
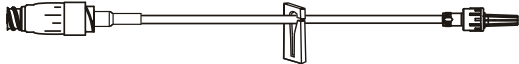
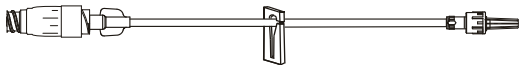
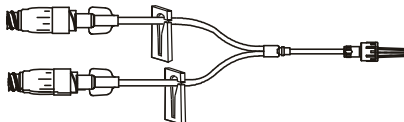
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves and VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
MP9004-A			 50
		Minibore extension set, removable MaxGuard antimicrobial needleless connector, removable slide clamp, male spin lock	
		Approximate set length: 7" 18 cm. approx. Approximate priming volume: 0.5 mL	
MP9004-C			 50
		Minibore extension set, removable MaxPlus clear needleless connector, removable slide clamp, male spin lock	
		Approximate set length: 7" 18 cm. approx. Approximate priming volume: 0.51 mL	
MP9004-NRC			 50
		Minibore extension set, removable MaxPlus needleless connector, non-removable slide clamp, male spin lock	
		Approximate set length: 7" 18 cm. approx. Approximate priming volume: 0.51 mL	
MP9014			 50
		Minibore extension set, removable MaxPlus needleless connector, removable slide clamp, male spin lock	
		Approximate set length: 14" 36 cm. approx. Approximate priming volume: 0.75 mL	
MP9014-A 30-day lead time			 50
		Minibore extension set, removable MaxGuard antimicrobial needleless connector, removable slide clamp, male spin lock	
		Approximate set length: 14" 36 cm. approx. Approximate priming volume: 0.7 mL	

05.00 Extension sets

05.03.01 Minibore/Smallbore tubing | MaxPlus and MaxGuard needleless connector sets

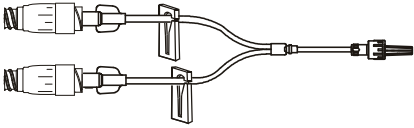
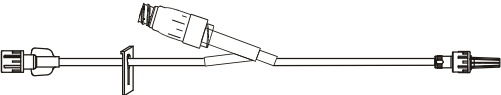
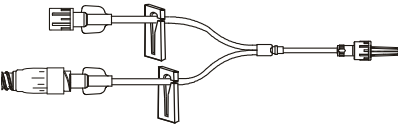
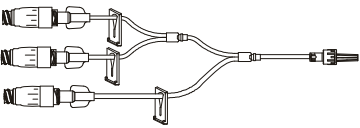
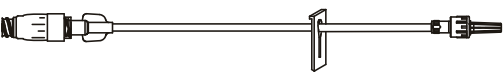
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves and VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
MP9014-C			 50
	Minibore extension set, removable MaxPlus clear needleless connector, removable slide clamp, male spin lock		
	Approximate set length:	14" 36 cm. approx.	
	Approximate priming volume:	0.75 mL	
MP9017			 50
	Minibore extension set, removable MaxPlus needleless connector, removable slide clamp, male spin lock		
	Approximate set length:	12" 30 cm. approx.	
	Approximate priming volume:	0.75 mL	
MP9017-B			 50
	Minibore extension set, MaxPlus needleless connector, removable slide clamp, male spin lock		
	Approximate set length:	12" 30 cm. approx.	
	Approximate priming volume:	0.75 mL	
MP9017-C	30-day lead time		 50
	Minibore extension set, removable MaxPlus clear needleless connector, removable slide clamp, male spin lock		
	Approximate set length:	12" 30 cm. approx.	
	Approximate priming volume:	0.7 mL	
MP9027			 50
	Minibore bi-fuse extension set, 2 removable MaxPlus needleless connectors, 2 removable slide clamps, male spin lock		
	Approximate set length:	5" 13 cm. approx.	
	Approximate priming volume:	0.8 mL	

05.00 Extension sets

05.03.01 Minibore/Smallbore tubing | MaxPlus and MaxGuard needleless connector sets

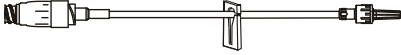
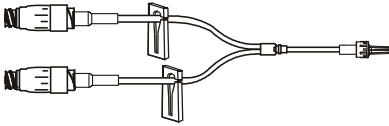
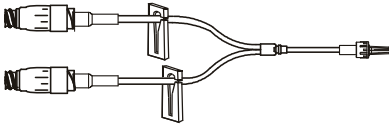
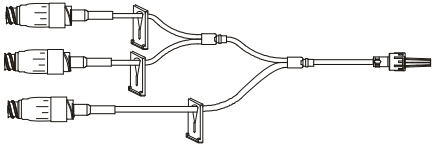
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves and VersaSafe split septum ports.

Model #	Drops	Description	Case quantity	
MP9027-C				50
		Minibore bi-fuse extension set, 2 removable MaxPlus clear needleless connectors, 2 removable slide clamps, male spin lock		
		Approximate set length: 5" 13 cm. approx.		
		Approximate priming volume: 0.8 mL		
MP9101				50
		Minibore extension set, MaxPlus needleless y-site, non-removable slide clamp, male spin lock		
		Approximate set length: 6" 15 cm. approx.		
		Approximate priming volume: 0.3 mL		
MP9218-C				50
		Minibore bi-fuse extension set, removable MaxPlus clear needleless connector, 2 removable slide clamps, male spin lock		
		Approximate set length: 7" 18 cm. approx.		
		Approximate priming volume: 0.7 mL		
MP9220-C				50
		Minibore tri-fuse extension set, 3 removable MaxPlus clear connectors, 3 removable slide clamps, male spin lock		
		Approximate set length: 7" 18 cm. approx.		
		Approximate priming volume: 1.5 mL		
MP9224	30-day lead time			50
		Minibore extension set, MaxPlus needleless connector, non-removable slide clamp, male spin lock		
		Approximate set length: 6" 15 cm. approx.		
		Approximate priming volume: 0.6 mL		

05.00 Extension sets

05.03.01 Minibore/Smallbore tubing | MaxPlus and MaxGuard needleless connector sets

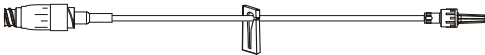
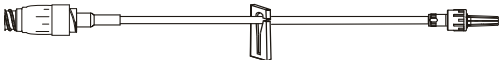
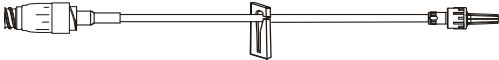
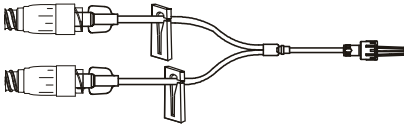
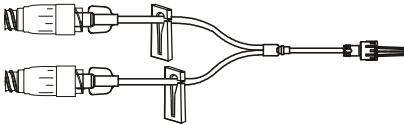
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves and VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
MP9229 30-day lead time		 Minibore extension set, MaxPlus needleless connector, removable slide clamp, male spin lock Approximate set length: 14" 36 cm. approx. Approximate priming volume: 0.8 mL	 50
MP9232-A		 Minibore bi-fuse extension set, 2 MaxGuard antimicrobial needleless connectors, 2 removable slide clamps, male luer lock Approximate set length: 6" 15 cm. approx. Approximate priming volume: 1.2 mL	 50
MP9232-C		 Minibore bi-fuse extension set, 2 MaxPlus clear needleless connectors, 2 removable slide clamps, male luer lock Approximate set length: 6" 15 cm. approx. Approximate priming volume: 1.2 mL	 50
MP9233-C		 Minibore tri-fuse extension set, 3 MaxPlus clear needleless connectors, 3 slide clamps, male spin lock Approximate set length: 7" 18 cm. approx. Approximate priming volume: 1.5 mL	 50

05.00 Extension sets

05.03.01 Minibore/Smallbore tubing | MaxPlus and MaxGuard needleless connector sets

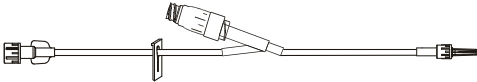
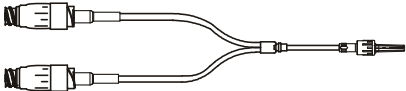
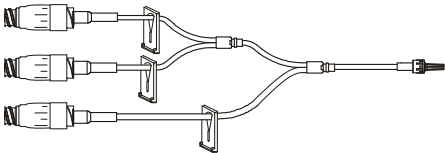
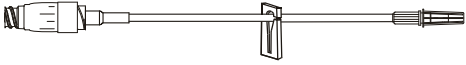
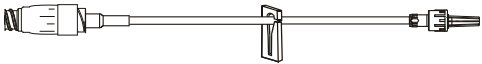
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves and VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
MP9235-C			 50
	Minibore extension set, MaxPlus clear needleless connector, removable slide clamp, male spin lock		
	Approximate set length:	60" 152 cm. approx.	
MP9236-A			 50
	Minibore extension set, MaxGuard antimicrobial needleless connector, removable slide clamp, male spin lock		
	Approximate set length:	14" 36 cm. approx.	
MP9236-C			 50
	Minibore extension set, MaxPlus clear needleless connector, removable slide clamp, male spin lock		
	Approximate set length:	14" 36 cm. approx.	
MP9240-C			 50
	Minibore bi-fuse extension set, 2 removable MaxPlus clear needleless connectors, removable slide clamp, male spin lock		
	Approximate set length:	8" 20 cm. approx.	
MP9250-C			 50
	Minibore bi-fuse extension set, 2 removable MaxPlus clear needleless connectors, 2 removable slide clamps, male spin lock		
	Approximate set length:	7" 18 cm. approx.	
Approximate priming volume:		1.5 mL	

05.00 Extension sets

05.03.01 Minibore/Smallbore tubing | MaxPlus and MaxGuard needleless connector sets

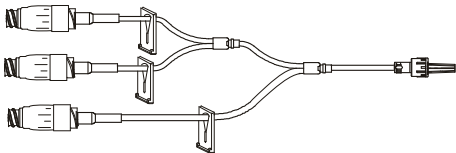
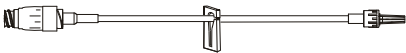
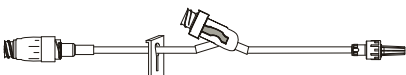
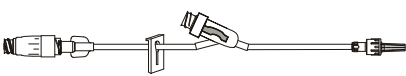
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves and VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
MP9251-C			 50
		Minibore extension set, MaxPlus clear needleless connector y-site, non-removable slide clamp, male luer lock	
		Approximate set length: 5.5" 14 cm. approx. Approximate priming volume: 0.3 mL	
MP9252-C	30-day lead time		 50
		Minibore bi-fuse extension set, 2 MaxPlus clear needleless connectors, male spin lock	
		Approximate set length: 6" 15 cm. approx. Approximate priming volume: 1.2 mL	
MP9253-C	30-day lead time		 50
		Minibore tri-fuse extension set, 3 MaxPlus clear needleless connectors, 3 non-removable slide clamps, male luer lock	
		Approximate set length: 7" 18 cm. approx.	
MP9255-C	30-day lead time		 50
		Minibore extension set, MaxPlus clear needleless connector, removable slide clamp, male luer slip	
		Approximate set length: 5" 13 cm. approx. Approximate priming volume: 0.6 mL	
MP9257-C			 50
		Minibore extension set, MaxPlus clear needleless connector, removable slide clamp, male spin lock	
		Approximate set length: 64" 163 cm. approx. Approximate priming volume: 2.2 mL	

05.00 Extension sets

05.03.01 Minibore/Smallbore tubing | MaxPlus and MaxGuard needleless connector sets

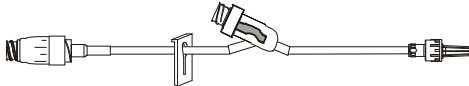
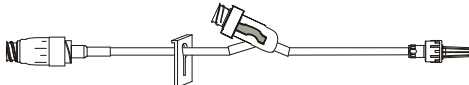
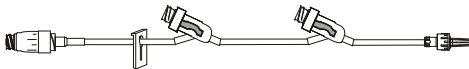
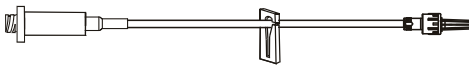
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves and VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
MP9260-A			 50
		Minibore tri-fuse extension set, 3 MaxGuard antimicrobial needleless connectors, 3 non-removable slide clamps, male spin lock	
		Approximate set length: 6" 15 cm. approx.	
		Approximate priming volume: 1.5 mL	
MP9262-C			 50
		Minibore extension set, MaxPlus clear needleless connector, removable slide clamp, male luer lock	
		Approximate set length: 14" 36 cm. approx.	
		Approximate priming volume: 0.75 mL	
MPX9101			 50
		Minibore extension set, MaxPlus needleless connector, needleless y-site, non-removable slide clamp, male spin lock	
		Approximate set length: 7" 18 cm. approx.	
		Approximate priming volume: 0.9 mL	
MPX9101-C			 50
		Minibore extension set, MaxPlus clear needleless connector, non-removable slide clamp, needleless y-site, male spin lock	
		Approximate set length: 7" 18 cm. approx.	
		Approximate priming volume: 1 mL	
MPX9104 30-day lead time			 50
		Minibore extension set, removable MaxPlus needleless connector, non-removable slide clamp, needleless y-site, male spin lock	
		Approximate set length: 7.5" 19 cm. approx.	
		Approximate priming volume: 1 mL	

05.00 Extension sets

05.03.01 Minibore/Smallbore tubing | MaxPlus and MaxGuard needleless connector sets

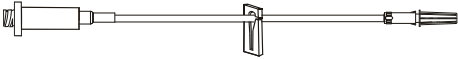
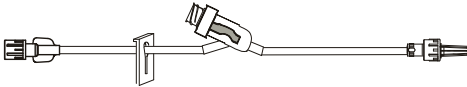
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves and VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
MPX9105 30-day lead time		 Minibore extension set, MaxPlus needleless connector, non-removable slide clamp, needleless y-site, male spin lock Approximate set length: 6.125" 16 cm. approx. Approximate priming volume: 1 mL	 50
MPX9105-C 30-day lead time		 Minibore extension set, MaxPlus clear needleless connector, non-removable slide clamp, needleless y-site, male spin lock Approximate set length: 6.125" 16 cm. approx. Approximate priming volume: 1 mL	 50
MPX9107-C 30-day lead time		 Minibore extension set, MaxPlus clear needleless connector, 2 needleless y-sites, non-removable slide clamp, male spin lock Approximate set length: 9.5" 24 cm. approx. Approximate priming volume: 1.5 mL	 50
MX9165		 Minibore extension set, standard needleless connector, removable slide clamp, male luer slip Approximate set length: 7" 18 cm. approx. Approximate priming volume: 0.51 mL	 50

05.00 Extension sets

05.03.01 Minibore/Smallbore tubing | MaxPlus and MaxGuard needleless connector sets

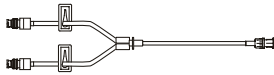
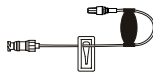
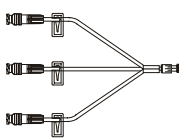
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves and VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
MX9167 30-day lead time		 Minibore extension set, standard needleless connector, removable slide clamp, male luer slip Approximate set length: 7" 18 cm. approx.	 50
MX9181 30-day lead time		 Minibore extension set, needleless y-site, slide clamp, male spin lock Approximate set length: 9" 23 cm. approx. Approximate priming volume: 0.73 mL	 50

05.00 Extension sets

05.03.02 Minibore/Smallbore tubing | SmartSite valve sets

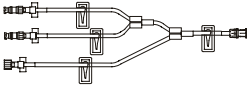
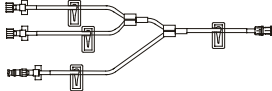
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves and VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
20019E			 100
	Minibore/Smallbore Y-connector extension set, 2 needle-free valves, spin male luer lock		
	Approximate set length:	9" 23 cm. approx.	
	Approximate priming volume:	0.33 mL	
Labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second.			
20021E			 100
	Minibore/Smallbore extension set, J-loop, 1 needle-free valve, spin male luer lock		
	Approximate set length:	6.5" 17 cm. approx.	
	Approximate priming volume:	0.19 mL	
20035E			 100
	Minibore/Smallbore extension set, 1 needle-free valve, spin male luer lock		
	Approximate set length:	6" 15 cm. approx.	
	Approximate priming volume:	0.35 mL	
20037E			 100
	Minibore/Smallbore extension set, 1 needle-free valve, luer slip adapter.		
	Approximate set length:	6.5" 17 cm. approx.	
	Approximate priming volume:	0.22 mL	
20038E			 100
	Minibore/Smallbore tri-port extension set, 3 needle-free valves, fixed male luer lock		
	Approximate set length:	4.5" 11 cm. approx.	
	Approximate priming volume:	0.37 mL	
20039E			 100
	Minibore/Smallbore extension set, 1 needle-free valve, spin male luer lock		
	Approximate set length:	6" 15 cm. approx.	
	Approximate priming volume:	0.17 mL	
Labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second.			

05.00 Extension sets

05.03.02 Minibore/Smallbore tubing | SmartSite valve sets

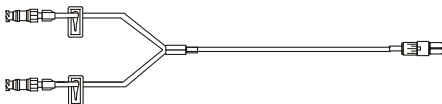
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves and VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
20049E			 100
		Minibore/Smallbore extension set, 1 needle-free valve, spin male luer lock	
		Approximate set length: 13" 32 cm. approx.	
		Approximate priming volume: 0.34 mL	
20337E			 100
		Minibore/Smallbore tri-port extension set, 2 needle-free valves, 1 female luer lock port, 2-piece fixed male luer lock	
		Approximate set length: 7.5" 19 cm. approx.	
		Approximate priming volume: 0.49 mL	
20338E			 100
		Minibore/Smallbore tri-port extension set, 1 detachable needle-free valve, 2 female luer lock ports, spin male luer lock	
		Approximate set length: 8" 20 cm. approx.	
		Approximate priming volume: 0.67 mL	
30203E			 100
		Minibore/Smallbore extension set, 1 needle-free valve 4" from fixed male luer lock	
		Approximate set length: 9" 24 cm. approx.	
		Approximate priming volume: 0.67 mL	
10010511			 100
		Minibore/Smallbore extension set, 1 needle-free valve, fixed male luer lock	
		Approximate set length: 6" 15 cm. approx.	
		Approximate priming volume: 0.18 mL	

05.00 Extension sets

05.03.02 Minibore/Smallbore tubing | SmartSite valve sets

Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves and VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
10010983			 100
		Minibore/Smallbore extension set, 1 detachable needle-free valve, fixed male luer lock	
		Approximate set length: 8.5" 22 cm. approx.	
		Approximate priming volume: 0.33 mL	
10011253			 100
		Minibore/Smallbore extension set, 1 detachable needle-free valve, spin male luer lock	
		Approximate set length: 7.5" 19 cm. approx.	
		Approximate priming volume: 0.29 mL	
Labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second			
10014766	30-day lead time		 100
		Minibore/Smallbore extension set, 1 needle-free valve, 2-piece male luer lock	
		Approximate set length: 6.5" 17 cm. approx.	
		Approximate priming volume: 0.22 mL	
10015488			 100
		Minibore/Smallbore Y-connector extension set, 2 detachable needle-free valves, spin male luer lock	
		Approximate set length: 8" 20 cm. approx.	
		Approximate priming volume: 0.54 mL	
10406890	30-day lead time		 100
		Minibore/Smallbore extension set, smallbore tubing segment, 1 needle-free valve 2" from spin male luer lock	
		Approximate set length: 111" 282 cm. approx.	
		Approximate priming volume: 6 mL	

05.00 Extension sets

05.03.02 Minibore/Smallbore tubing | SmartSite valve sets

Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves and VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
C25002E 30-day lead time		 Minibore/Smallbore extension set, 4 needle-free valves 227", 219", 15" and 6" from fixed male luer lock Approximate set length: 236" 599 cm. approx. Approximate priming volume: 13 mL	 50
C25005E		 Minibore/Smallbore extension set, 2 needle-free valves 136" and 5" from 2-piece male luer lock Approximate set length: 144" 366 cm. approx. Approximate priming volume: 8 mL	 100

05.00 Extension sets

05.03.03 Minibore/Smallbore tubing | VersaSafe split septum sets

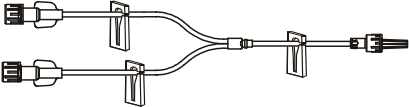
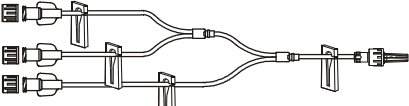
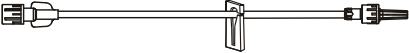
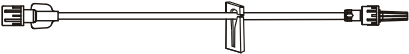
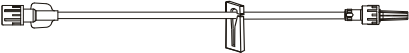
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves and VersaSafe split septum ports.

Model #	Drops	Description		Case quantity
10807855		 Minibore/Smallbore extension set, 1 split septum port 5" from spin male luer lock Approximate set length: 11" 28 cm. approx. Approximate priming volume: 0.6 mL		100
10807859 30-day lead time		 Minibore/Smallbore extension set, 1 split septum port 10" from spin male luer lock Approximate set length: 22" 56 cm. approx. Approximate priming volume: 0.9 mL		100
11202651		 Minibore/Smallbore extension set, 2 split septum ports 137" and 6" from 2-piece male luer lock Approximate set length: 146" 371 cm. approx. Approximate priming volume: 9 mL		100
11202652		 Minibore/Smallbore extension set, 1 split septum port 9" from fixed male luer lock Approximate set length: 33" 84 cm. approx. Approximate priming volume: 2 mL		72

05.00 Extension sets

05.03.04 Minibore/Smallbore tubing | No port sets

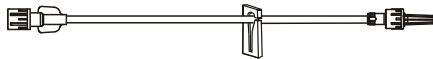
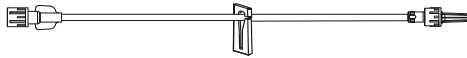
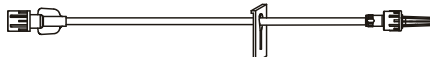
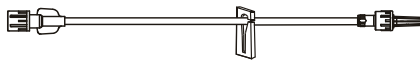
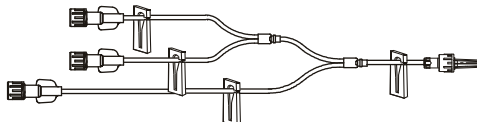
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves and VersaSafe split septum ports.

Model #	Drops	Description	Case quantity	
ME1001 30-day lead time				50
		Minibore bi-fuse extension set, 3 removable slide clamps, male spin lock		
		Approximate set length: 6" 15 cm. approx. Approximate priming volume: 0.6 mL		
ME1021				50
		Minibore tri-fuse extension set, 4 removable slide clamps, male spin lock		
		Approximate set length: 6" 15 cm. approx. Approximate priming volume: 0.5 mL		
ME1028 30-day lead time				50
		Minibore extension set, removable slide clamp, male spin lock		
		Approximate set length: 4" 10 cm. approx. Approximate priming volume: 0.27 mL		
ME1032				50
		Minibore extension set, removable slide clamp, male spin lock		
		Approximate set length: 6" 15 cm. approx. Approximate priming volume: 0.32 mL		
ME1048				50
		Minibore extension set, removable slide clamp, male spin lock		
		Approximate set length: 12" 30.5 cm. approx. Approximate priming volume: 0.4 mL		

05.00 Extension sets

05.03.04 Minibore/Smallbore tubing | No port sets

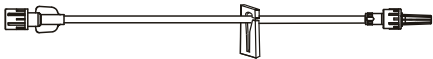
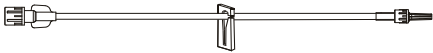
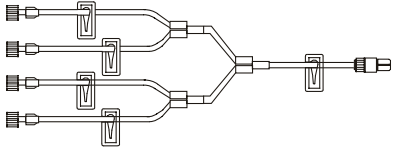
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves and VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
ME1050			 50
		Minibore extension set, removable slide clamp, male spin lock	
		Approximate set length: 30" 76 cm. approx.	
		Approximate priming volume: 1 mL	
ME1110			 50
		Minibore extension set, removable slide clamp, male spin lock	
		Approximate set length: 36" 91 cm. approx.	
		Approximate priming volume: 1.3 mL	
ME1227			 50
		Minibore extension set, non-removable slide clamp, male spin lock	
		Approximate set length: 6" 15 cm. approx.	
		Approximate priming volume: 0.32 mL	
ME1234			 300
		Minibore extension set, removable slide clamp, male spin lock	
		Approximate set length: 6" 15 cm. approx.	
		Approximate priming volume: 0.32 mL	
ME1251			 50
		Minibore extension set, pinch clamp, male luer slip	
		Approximate set length: 7" 18 cm. approx.	
		Approximate priming volume: 0.3 mL	
ME1253			 50
		Minibore tri-fuse extension set, 4 removable slide clamps, male spin lock	
		Approximate set length: 12.25" 31 cm. approx.	
		Approximate priming volume: 0.8 mL	

05.00 Extension sets

05.03.04 Minibore/Smallbore tubing | No port sets

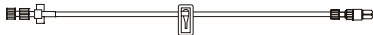
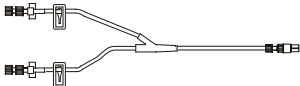
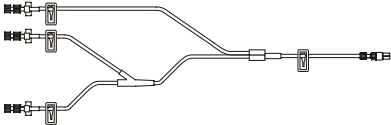
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves and VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
ME2010		 Minibore extension set, removable slide clamp, male spin lock Approximate set length: 60" 152.4 cm. approx. Approximate priming volume: 1.8 mL	 50
ME2017		 Ultra minibore extension set, pinch clamp, male luer lock Approximate set length: 60" 152.4 cm. approx. Approximate priming volume: 1 mL	 50
ME2050		 Minibore extension set, removable slide clamp, male luer lock Approximate set length: 60" 152 cm. approx. Approximate priming volume: 1.6 mL	 50
10010525		 Minibore/Smallbore extension set, anti-siphon valve, spin male luer lock Approximate set length: 31" 78 cm. approx. Approximate priming volume: 1.2 mL	 100
10011881		 Minibore/Smallbore extension set, luer slip adapter Approximate set length: 40" 102 cm. approx. Approximate priming volume: 1 mL	 100
10015817		 Minibore/Smallbore quadfuse extension set, 4 female luer locks, spin male luer lock Approximate set length: 7.5" 19 cm. approx. Approximate priming volume: 0.91 mL	 100

05.00 Extension sets

05.03.04 Minibore/Smallbore tubing | No port sets

Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves and VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
10031776 30-day lead time		 Minibore/Smallbore extension set, fixed male luer lock Approximate set length: 234" 594 cm. approx. Approximate priming volume: 13 mL	 50
10373772		 Minibore/Smallbore extension set, spin male luer lock Approximate set length: 6" 15 cm. approx. Approximate priming volume: 0.24 mL	 100
10794983		 Minibore/Smallbore extension set, 2 female luer locks, spin male luer lock Approximate set length: 7" 18 cm. approx. Approximate priming volume: 0.37 mL	 100
10839681		 Minibore/Smallbore extension set, 3 female luer locks, spin male luer lock Approximate set length: 7.5" 19 cm. approx. Approximate priming volume: 0.53 mL	 100
10896354 30-day lead time		 Minibore/Smallbore extension set, spin male luer lock Approximate set length: 110" 279 cm. approx. Approximate priming volume: 6 mL	 100

05.00 Extension sets

05.03.04 Minibore/Smallbore tubing | No port sets

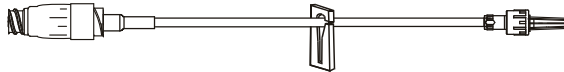
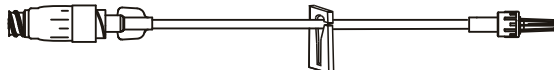
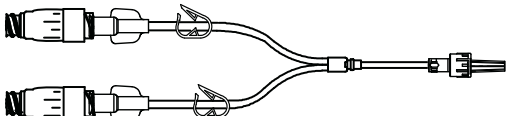
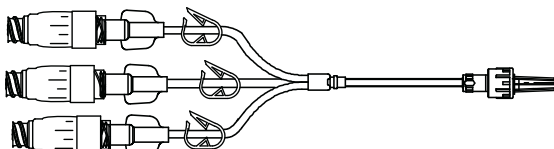
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves and VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
11088483		 Minibore/Smallbore extension set, fixed male luer lock Approximate set length: 62" 158 cm. approx. Approximate priming volume: 1.5 mL	 100
C20016		 Minibore/Smallbore extension set, fixed male luer lock Approximate set length: 32" 81 cm. approx. Approximate priming volume: 1.9 mL	 72
C25012		 Minibore/Smallbore extension set, detachable anti-siphon valve, fixed male luer lock Approximate set length: 5.5" 14 cm. approx. Approximate priming volume: 0.51 mL	 20
C30006		 Minibore/Smallbore extension set, spin male luer lock Approximate set length: 62" 157 cm. approx. Approximate priming volume: 1.5 mL	 100

05.00 Extension sets

05.04.01 Microbore tubing | MaxPlus and MaxGuard needleless connector sets

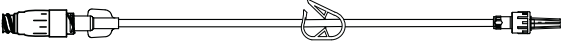
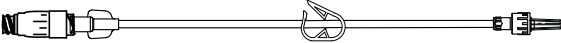
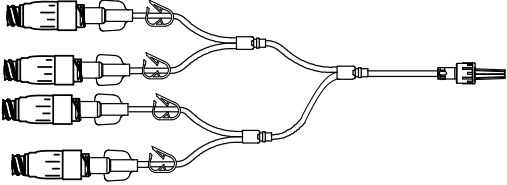
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves and VersaSafe split septum ports.

Model #	Drops	Description	Case quantity	
MP9249-C				50
	Microbore extension set, MaxPlus clear needleless connector, removable slide clamp, male spin lock			
	Approximate set length:	62" 157 cm. approx.		
	Approximate priming volume:	0.61 mL		
MP9258-C				50
	Microbore extension set, removable MaxPlus clear needleless connector, removable slide clamp, male luer lock			
	Approximate set length:	6" 15 cm. approx.		
	Approximate priming volume:	0.45 mL		
MP9265-C				50
	Microbore bi-fuse extension set, 2 removable MaxPlus clear needleless connectors, 2 pinch clamps, male spin lock			
	Approximate set length:	6" 15 cm approx.		
	Approximate priming volume:	0.8 mL		
Contact CareFusion for lead time				
MP9266-C				50
	Microbore tri-fuse extension set, 3 removable MaxPlus clear needleless connectors, 3 pinch clamps, male spin lock			
	Approximate set length:	5" 13 cm approx.		
	Approximate priming volume:	1.1 mL		
Contact CareFusion for lead time				

05.00 Extension sets

05.04.01 Microbore tubing | MaxPlus and MaxGuard needleless connector sets

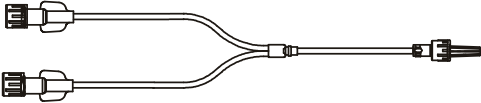
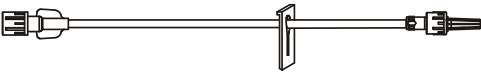
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves and VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
MP9267-C Contact CareFusion for lead time		 Microbore extension set, removable MaxPlus clear needleless connector, pinch clamp, male spin lock Approximate set length: 60" 152 cm approx. Approximate priming volume: 0.7 mL	 50
MP9268-C Contact CareFusion for lead time		 Microbore extension set, removable MaxPlus clear needleless connector, pinch clamp, male spin lock Approximate set length: 8" 20 cm approx. Approximate priming volume: 0.4 mL	 50
MP9275-C Contact CareFusion for lead time		 Microbore quad-fuse extension set, 4 removable MaxPlus clear needleless connectors, 4 pinch clamps, male spin lock Approximate set length: 6" 15 cm approx. Approximate priming volume: 1.6 mL	 50

05.00 Extension sets

05.04.02 Microbore tubing | No port sets

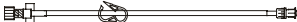
Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves and VersaSafe split septum ports.

Model #	Drops	Description	Case quantity	
ME1222 30-day lead time				50
		Microbore bi-fuse extension set, male spin lock		
		Approximate set length: 4" 10 cm. approx. Approximate priming volume: 0.2 mL		
ME2020				50
		Microbore extension set, non-removable slide clamp, male spin lock		
		Approximate set length: 60" 152 cm. approx. Approximate priming volume: 0.3 mL		
ME2059				50
		Microbore extension set, male spin lock		
		Approximate set length: 72" 183 cm. approx. Approximate priming volume: 0.4 mL		
20022				100
		Microbore extension set, spin male luer.		
		Approximate set length: 62" 158 cm. approx. Approximate priming volume: 0.57 mL		
30822				50
		Microbore extension set, fixed male luer lock		
		Approximate set length: 62" 157 cm. approx. Approximate priming volume: 0.58 mL		

05.00 Extension sets

05.04.02 Microbore tubing | No port sets

Extension sets provide flexibility and additional infusion capabilities to gravity and infusion instrument sets. All sets are latex free. Configurations are available with integrated MaxPlus and MaxGuard needleless connectors, SmartSite needle-free valves and VersaSafe split septum ports.

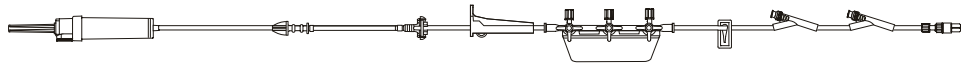
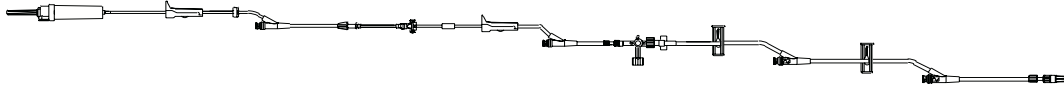
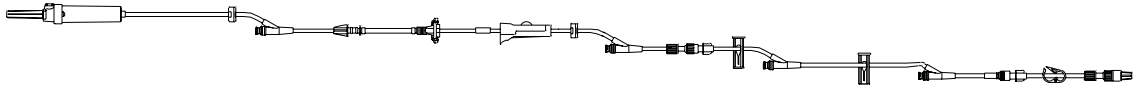
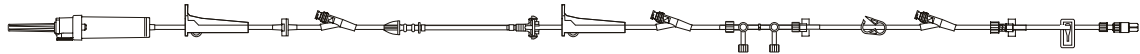
Model #	Drops	Description	Case quantity
30862-0500			50
		Microbore extension set, anti-siphon valve, fixed male luer lock	
		Approximate set length: 62" 157 cm. approx. Approximate priming volume: 0.67 mL	
30910			 100
		Microbore extension set, fixed male luer lock	
		Approximate set length: 38" 97 cm. approx. Approximate priming volume: 0.37 mL	
30914			 100
		Microbore extension set, fixed male luer lock	
		Approximate set length: 62" 156 cm. approx. Approximate priming volume: 0.57 mL	
C20032 30-day lead time			 100
		Microbore extension set, fixed male luer lock	
		Approximate set length: 7.5" 19 cm. approx. Approximate priming volume: 0.23 mL	
C25006			 100
		Microbore extension set, fixed male luer lock	
		Approximate set length: 84" 213 cm. approx. Approximate priming volume: 0.75 mL	

06.00 Alaris Pump module

06.01.01 Stopcocks and manifolds | SmartSite valve sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports.



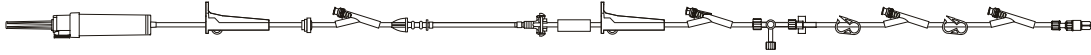
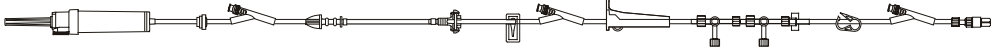
Model #	Drops	Description	Case quantity
2406-0500	20	 Infusion set, smallbore tubing segment, 3-port manifold, 2 needle-free valves 54" and 7" from 2-piece male luer lock Approximate set length: 112" 284 cm. approx. Approximate priming volume: 18 mL	20
2408-0500	10	 Infusion set, check valve, 4-way stopcock ("OFF" Indicator), 4 needle-free valves 105", 37", 24" and 6" from 2-piece male luer lock Approximate set length: 124" 315 cm. approx. Approximate priming volume: 25 mL	10
2428-0007	20	 Infusion set, 2 check valves, 4 needle-free valves 98", 50", 41" and 10" from 2-piece male luer lock Approximate set length: 127" 322 cm. approx. Approximate priming volume: 27 mL	 20
2450-0500	10	 Infusion set, check valve, 2-ganged 3-way stopcock, 3 needle-free valves 108", 45" and 12" from 2-piece male luer lock Approximate set length: 129" 327 cm. approx. Approximate priming volume: 27 mL	 10

06.00 Alaris Pump module

06.01.01 Stopcocks and manifolds | SmartSite valve sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports.



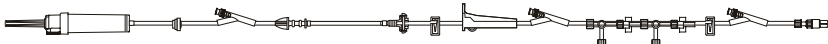
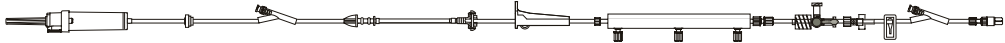
Model #	Drops	Description	Case quantity
10012182	10	 Infusion set, check valve, 4-way stopcock ("OFF" Indicator), 4 needle-free valves 124", 46", 32" and 7" from 2-piece male luer lock Approximate set length: 160" 406 cm. approx. Approximate priming volume: 30 mL	10
10013034	20	 Infusion set, check valve, 2-ganged 4-way stopcock, 3 needle-free valves 91", 73" and 8" from 2-piece male luer lock Approximate set length: 128" 325 cm. approx. Approximate priming volume: 27 mL	20
10015312	20	 Infusion set, check valve, 2-ganged 3-way stopcock, 3 needle-free valves 99", 79" and 6" from 2-piece male luer lock Approximate set length: 134" 340 cm. approx. Approximate priming volume: 28 mL	10
10015649	20	 Infusion set, check valve, 4-way stopcock ("OFF" Indicator), 3 needle-free valves 112", 40" and 5" from 2-piece fixed male luer lock Approximate set length: 129" 328 cm. approx. Approximate priming volume: 27 mL	20

06.00 Alaris Pump module

06.01.01 Stopcocks and manifolds | SmartSite valve sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports.



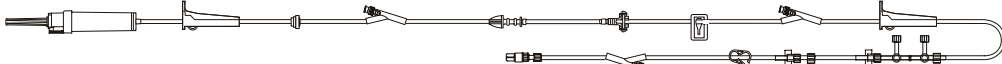
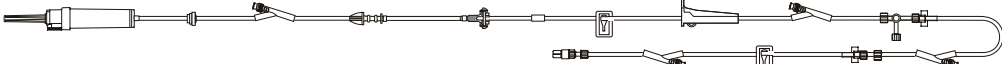
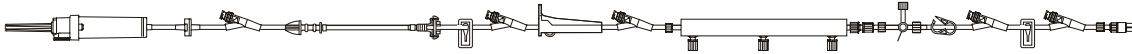
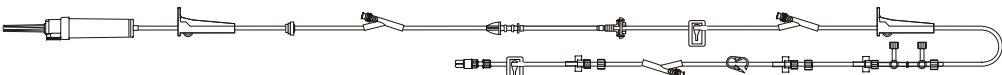
Model #	Drops	Description	Case quantity
10015896	10	 Infusion set, check valve, 2 needle-free valves 73" and 6" from 2-piece male luer lock Extension set, 2-ganged 4-way stopcock, 1 needle-free valve 6" from 2-piece male luer lock Approximate set length: 125" 318 cm. approx. Approximate priming volume: 25 mL	 10
10108961	20	 Infusion set, check valve, 4-way stopcock ("OFF" Indicator), 4 needle-free valves 98", 38", 27" and 4" from 2-piece male luer lock Approximate set length: 126" 320 cm. approx. Approximate priming volume: 26 mL	20
10142568	20	 Infusion set, check valve, 4-ganged 4-way stopcock, 4 needle-free valves 107", 49", 32" and 7" from spin male luer lock Approximate set length: 135" 343 cm. approx. Approximate priming volume: 28 mL	20
10382963	20	 Infusion set, check valve, manifold with three 1-way valves, 3-way stopcock, 2 needle-free valves 116" and 6" from 2-piece male luer lock Approximate set length: 139" 353 cm. approx. Approximate priming volume: 29 mL	20

06.00 Alaris Pump module

06.01.01 Stopcocks and manifolds | SmartSite valve sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports.



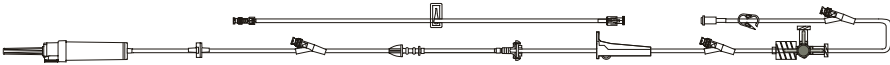
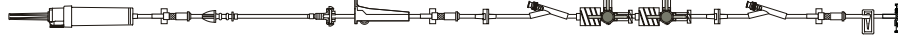
Model #	Drops	Description	Case quantity
10813621	20	 Infusion set, check valve, 3-ganged 4-way stopcock, 4 needle-free valves 68", 28", 12" and 6" from 2-piece male luer lock Approximate set length: 159" 404 cm. approx. Approximate priming volume: 30 mL DEHP	20
10934345	20	 Infusion set, check valve, 2-ganged 3-way stopcock, 3 needle-free valves 109", 90" and 8" from 2-piece male luer lock Approximate set length: 140" 357 cm. approx. Approximate priming volume: 28 mL	10
11059729	20	 Infusion set, check valve, 4-way stopcock ("OFF" Indicator), 4 needle-free valves 115", 47", 32" and 6" from 2-piece male luer lock Approximate set length: 130" 330 cm. approx. Approximate priming volume: 27 mL	20
11419365	20	 Infusion set, check valve, manifold with three 1-way valves, 3-way stopcock, 5 needle-free valves 120", 101", 47", 25" and 6" from 2-piece male luer lock Approximate set length: 159" 404 cm. approx. Approximate priming volume: 32 mL DEHP	20
11582773	20	 Infusion set, check valve, 2-ganged 3-way stopcock, 3 needle-free valves 87", 68" and 15" from 2-piece male luer lock Approximate set length: 148" 376 cm. approx. Approximate priming volume: 30 mL	10

06.00 Alaris Pump module

06.01.01 Stopcocks and manifolds | SmartSite valve sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports.



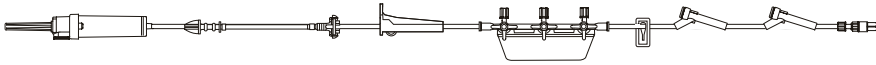
Model #	Drops	Description	Case quantity
C24104E	20	 Infusion set, check valve, 4-way stopcock ("OFF" Indicator), 3 needle-free valves 121", 42" and 6" from 2-piece male luer lock Smallbore extension set, 1 needle-free valve, fixed male luer lock Approximate set length: 161" 409 cm. approx. Approximate priming volume: 31 mL	10
C24106E	20	 Infusion set, 2 check valves, 2-ganged 4-way stopcock ("OFF" Indicator), 2 needle-free valves 26" and 2" from 2-piece male luer lock hub. Smallbore extension set, needle-free valve T-connector, spin male luer lock hub Approximate set length: 96" 244 cm. approx. Approximate priming volume: 21 mL	10
C24107E	60	 Infusion set, 2 check valves, 2-ganged 4-way stopcock ("OFF" Indicator), 2 needle-free valves 26" and 2" from 2-piece male luer lock hub. Smallbore extension set, needle-free valve T-connector, spin male luer lock hub Approximate set length: 96" 244 cm. approx. Approximate priming volume: 22 mL	10

06.00 Alaris Pump module

06.01.02 Stopcocks and manifolds | VersaSafe split septum sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports.



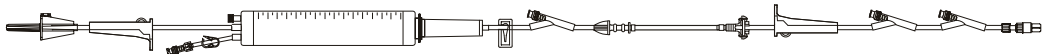
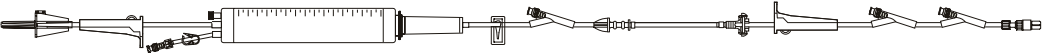
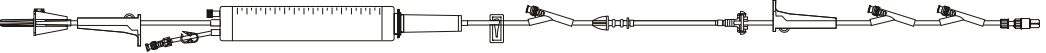
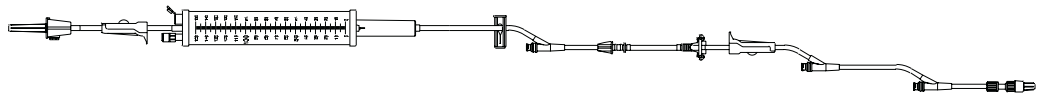
Model #	Drops	Description	Case quantity
2106-0500	20	 Infusion set, 3-port manifold, smallbore tubing segment, 2 split septum ports 54" and 7" from 2-piece male luer lock Approximate set length: 112" 286 cm. approx. Approximate priming volume: 19 mL	20
11147992	20	 Infusion set, smallbore tubing segment, manifold with three 1-way valves, 2 split septum ports 54" and 7" from 2-piece male luer lock Approximate set length: 113" 287 cm. approx. Approximate priming volume: 20 mL	20

06.00 Alaris Pump module

06.02.01 Burettes | SmartSite valve sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports.



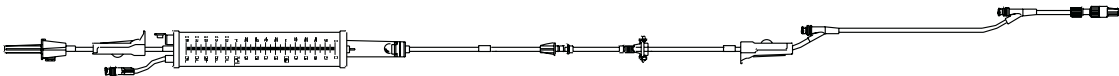
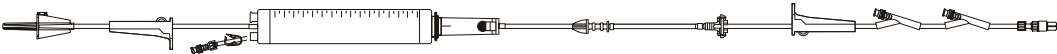
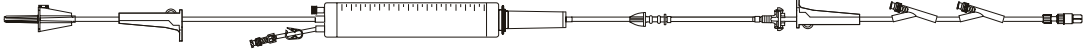
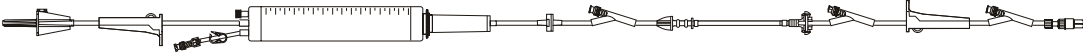
Model #	Drops	Description	Case quantity
2440-0600	20	 Infusion set, 150 mL burette (needle-free valve), 3 needle-free valves 85", 60" and 6" from 2-piece male luer lock Approximate set length: 126" 320 cm. approx. Approximate priming volume: 28 mL	10
2441-0007	60	 Infusion set, 150 mL burette (needle-free valve), smallbore tubing segment, 3 needle-free valves 86", 49" and 7" from 2-piece male luer lock Approximate set length: 130" 330 cm. approx. Approximate priming volume: 21 mL	10 
2441-0600	60	 Infusion set, 150 mL burette (needle-free valve), smallbore tubing segment, 3 needle-free valves 86", 60" and 7" from 2-piece male luer lock Approximate set length: 132" 335 cm. approx. Approximate priming volume: 21 mL	10
2442-0007	60	 Infusion set, 150 mL burette (split septum port), smallbore tubing segment, 3 needle-free valves 86", 49", and 7" from 2-piece male luer lock Approximate set length: 130" 329 cm. approx. Approximate priming volume: 21 mL	10 
2443-0600	60	 Infusion set, 150 mL burette (needle-free valve), smallbore tubing segment, 0.2 micron (low-protein binding) filter, 3 needle-free valves 81", 55" and 6" from 2-piece male luer lock Approximate set length: 132" 335 cm. approx. Approximate priming volume: 24 mL	10

06.00 Alaris Pump module

06.02.01 Burettes | SmartSite valve sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports.



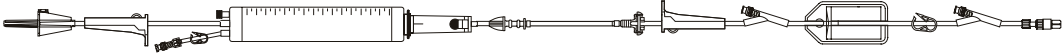
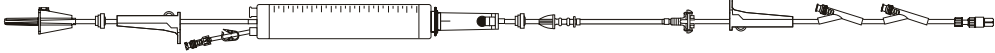
Model #	Drops	Description	Case quantity
2447-0007	60	 Infusion set, 150 mL burette (needle-free valve), ball valve in drip chamber, 2 needle-free valves 60" and 7" from 2-piece male luer lock Approximate set length: 129" 328 cm. approx. Approximate priming volume: 24 mL	10
2447-0600	60	 Infusion set, 150 mL burette (needle-free valve), ball valve in drip chamber, smallbore tubing segment, 2 needle-free valves 60" and 7" from 2-piece male luer lock Approximate set length: 131" 333 cm. approx. Approximate priming volume: 19 mL	10
2448-0600	20	 Infusion set, 150 mL burette (needle-free valve), 2 needle-free valves 60" and 6" from 2-piece male luer lock Approximate set length: 126" 320 cm. approx. Approximate priming volume: 30 mL	10
2449-0600	60	 Infusion set, 150 mL burette (needle-free valve), smallbore tubing segment, 2 needle-free valves 60" and 7" from 2-piece male luer lock Approximate set length: 131" 333 cm. approx. Approximate priming volume: 20 mL	10
10010541	20	 Infusion set, 150 mL burette (needle-free valve), check valve, 3 needle-free valves 82", 66" and 6" from 2-piece male luer lock Approximate set length: 125" 318 cm. approx. Approximate priming volume: 29 mL	10

06.00 Alaris Pump module

06.02.01 Burettes | SmartSite valve sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports.



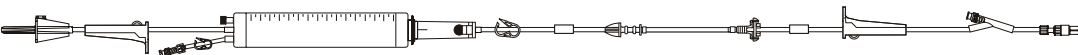
Model #	Drops	Description	Case quantity
10010871	20	 Infusion set, 150 mL burette (needle-free valve), low sorbing tubing segment, 1 needle-free valve 12" from 2-piece male luer lock (Gamma Radiation sterilization). Approximate set length: 69" 175 cm. approx. Approximate priming volume: 16 mL	10
10012283	60	 Infusion set, 150 mL burette (needle-free valve), ball valve in drip chamber, 0.2 micron (low-protein binding) filter, 2 needle-free valves 52" and 7" from 2-piece male luer lock Approximate set length: 129" 328 cm. approx. Approximate priming volume: 27 mL	10
10012293	60	 Infusion set, 150 mL burette (needle-free valve), ball valve in drip chamber, smallbore tubing segment, 2 check valves, 2 needle-free valves 64" and 8" from 2-piece male luer lock Approximate set length: 137" 348 cm. approx. Approximate priming volume: 20 mL	10
10014035	60	 Infusion set, 150 mL burette (needle-free valve), smallbore tubing segment, check valve, 3 needle-free valves 84", 60" and 7" from 2-piece male luer lock Approximate set length: 132" 335 cm. approx. Approximate priming volume: 21 mL	10

06.00 Alaris Pump module

06.02.01 Burettes | SmartSite valve sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports.



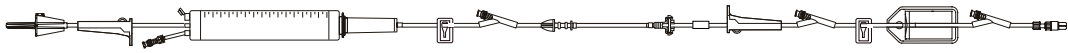
Model #	Drops	Description	Case quantity
10015012	20	 Infusion set, 150 mL burette (needle-free valve), low sorbing and smallbore tubing segments, 0.2 micron (low-protein binding) filter, 1 needle-free valve 6" from 2-piece male luer lock Approximate set length: 125" 318 cm. approx. Approximate priming volume: 26 mL	 10
10074281	20	 Low sorbing infusion set, 150 mL burette (needle-free valve), 1 needle-free valve 32" from 2-piece male luer lock Approximate set length: 130" 330 cm. approx. Approximate priming volume: 26 mL	 20
10817920	60	 Infusion set, 150 mL burette (needle-free valve), ball valve in drip chamber, smallbore tubing segment, 1 needle-free valve 7" from 2-piece male luer lock Approximate set length: 123" 312 cm. approx. Approximate priming volume: 19 mL	 10
10821753	20	 Infusion set, 150 mL burette (needle-free valve), low sorbing tubing segment, 3 needle-free valves 24", 18" and 6" from 2-piece male luer lock Approximate set length: 126" 320 cm. approx. Approximate priming volume: 25 mL	 10

06.00 Alaris Pump module

06.02.01 Burettes | SmartSite valve sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports.



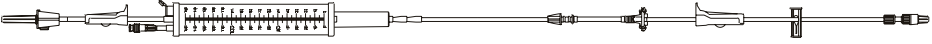
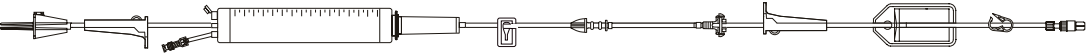
Model #	Drops	Description	Case quantity
11118122	60	 Infusion set, 150 mL burette (needle-free valve), 0.2 micron (low-protein binding) filter, 3 needle-free valves 89", 62" and 6" from 2-piece male luer lock Approximate set length: 129" 328 cm. approx. Approximate priming volume: 28 mL	 10
11607787	60	 Infusion set, 150 mL burette set (needle-free valve), smallbore tubing segment, 1 needle-free valve 86" from 2-piece male luer lock Approximate set length: 129" 328 cm. approx. Approximate priming volume: 21 mL	 10
11613191	60	 Infusion set, 150 mL burette (needle-free valve), ball valve in drip chamber, low sorbing tubing segment, 0.2 micron (low-protein binding) filter, 2 needle-free valves 51" and 7" from 2-piece male luer lock Approximate set length: 130" 330 cm. approx. Approximate priming volume: 27 mL	 10

06.00 Alaris Pump module

06.02.02 Burettes | No port sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports.



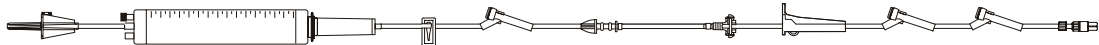
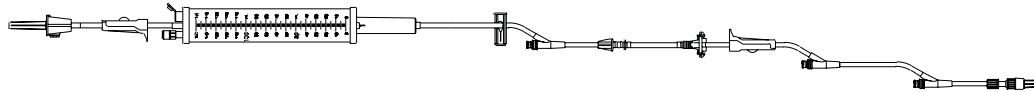
Model #	Drops	Description	Case quantity
2444-0007	20	 Low sorbing infusion set, burette (needle-free valve), 2-piece male luer lock Approximate set length: 123" 312 cm. approx. Approximate priming volume: 24 mL	 10
11045328	20	 Infusion set, 150 mL burette (needle-free valve), 0.2 micron (low-protein binding) filter, 2-piece male luer lock Approximate set length: 123" 312 cm. approx. Approximate priming volume: 30 mL	10
11063735	20	 Infusion set, 150 mL burette (needle-free valve), 2-piece male luer lock Approximate set length: 125" 318 cm. approx. Approximate priming volume: 28 mL	10

06.00 Alaris Pump module

06.02.03 Burettes | VersaSafe split septum sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports.



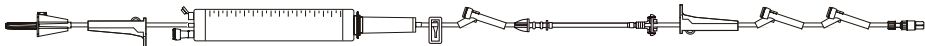
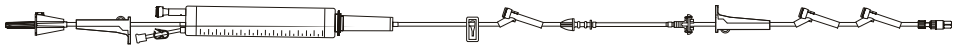
Model #	Drops	Description	Case quantity
2140-0600	20	 Infusion set, 150 mL burette (split septum port), 3 split septum ports 82", 56" and 6" from 2-piece male luer lock Approximate set length: 126" 320 cm. approx. Approximate priming volume: 28 mL	10
2141-0600	60	 Infusion set, 150 mL burette (split septum port), smallbore tubing segment, 3 split septum ports 86", 60" and 7" from 2-piece male luer lock Approximate set length: 132" 335 cm. approx. Approximate priming volume: 21 mL	10
2147-0600	60	 Infusion set, 150 mL burette (split septum port), ball valve in drip chamber, smallbore tubing segment, 3 split septum ports 86", 60" and 7" from 2-piece male luer lock Approximate set length: 132" 335 cm. approx. Approximate priming volume: 20 mL	10
2442-0007	60	 Infusion set, 150 mL burette (split septum port), smallbore tubing segment, 3 needle-free valves 86", 49", and 7" from 2-piece male luer lock Approximate set length: 130" 329 cm. approx. Approximate priming volume: 21 mL	10

06.00 Alaris Pump module

06.02.03 Burettes | VersaSafe split septum sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports.



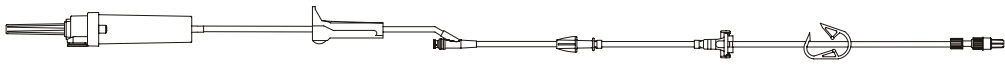
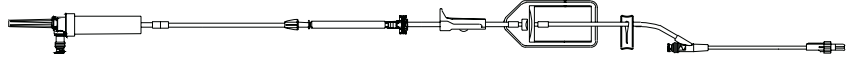
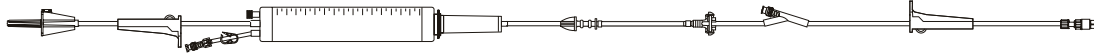
Model #	Drops	Description	Case quantity
10779313		 Infusion set, 150 mL burette (split septum port), 3 split septum ports 81" , 55" and 6" from 2-piece male luer lock Approximate set length: 125" 317 cm. approx. Approximate priming volume: 25 mL	 10
11401542		 Infusion set, 150 mL burette (split septum port), smallbore tubing segment, 3 split septum ports 81" , 62" and 8" from 2-piece male luer lock Approximate set length: 135" 343 cm. approx. Approximate priming volume: 19 mL	20

06.00 Alaris Pump module

06.03.01 Low sorbing tubing | SmartSite valve sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports.



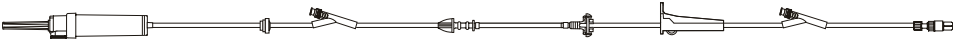
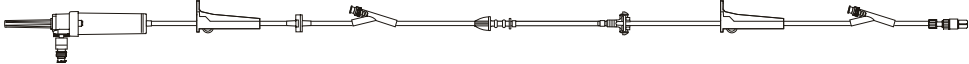
Model #	Drops	Description	Case quantity
2463-0007 30-day lead time	20	 Low sorbing infusion set, 1 needle-free valve 78" from 2-piece male luer lock Approximate set length: 119" 302 cm. approx. Approximate priming volume: 24 mL	 20
2465-0007	20	 Low sorbing infusion set, non-vented, needle-free bag access port, 20 drop, 0.2 micron (low protein binding) filter, 1 needle-free valve 6" from 2-piece male luer lock Approximate set length: 116" 293 cm. approx. Approximate priming volume: 26 mL	 20
10010453	20	 Low sorbing infusion set, 1.2 micron (low-protein binding) filter, 1 needle-free valve 6" from 2-piece male luer lock Approximate set length: 115" 291 cm. approx. Approximate priming volume: 26 mL	 20
10010454	20	 Low sorbing infusion set, 0.2 micron (low-protein binding) filter, 1 needle-free valve 6" from 2-piece male luer lock Approximate set length: 115" 291 cm. approx. Approximate priming volume: 26 mL	 20
10010871 30-day lead time	20	 Infusion set, 150 mL burette (needle-free valve), low sorbing tubing segment, 1 needle-free valve 12" from 2-piece male luer lock (Gamma Radiation sterilization) Approximate set length: 69" 175 cm. approx. Approximate priming volume: 16 mL	10

06.00 Alaris Pump module

06.03.01 Low sorbing tubing | SmartSite valve sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports.



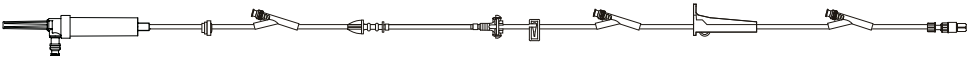
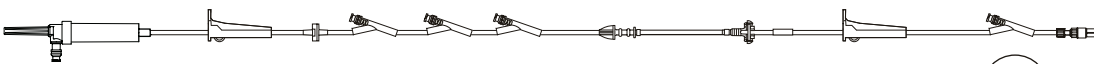
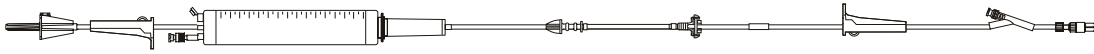
Model #	Drops	Description	Case quantity
10013072	20	 Low sorbing infusion set, check valve, 2 needle-free valves 78" and 6" from 2-piece male luer lock Approximate set length: 103" 262 cm. approx. Approximate priming volume: 22 mL	 20
10013889	20	 Infusion set, low sorbing tubing segment, check valve, 3 needle-free valves 103", 78" and 6" from 2-piece male luer lock Approximate set length: 138" 350 cm. approx. Approximate priming volume: 28 mL	20
10014855	20	 Low sorbing infusion set, non-vented, needle-free valve bag access port, check valve, 2 needle-free valves 95" and 6" from 2-piece male luer lock Approximate set length: 116" 295 cm. approx. Approximate priming volume: 24 mL	 20
10015012	20	 Infusion set, 150 mL burette (needle-free valve), low sorbing and smallbore tubing segments, 0.2 micron (low-protein binding) filter, 1 needle-free valve 6" from 2-piece male luer lock Approximate set length: 125" 318 cm. approx. Approximate priming volume: 26 mL	 10
10015048	20	 Low sorbing infusion set, check valve, 2 needle-free valves 78" and 6" from 2-piece male luer lock Approximate set length: 103" 262 cm. approx. Approximate priming volume: 22 mL	 20

06.00 Alaris Pump module

06.03.01 Low sorbing tubing | SmartSite valve sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports.



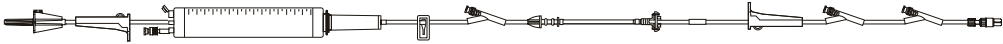
Model #	Drops	Description	Case quantity
10015861	20	 Low sorbing infusion set, non-vented, needle-free valve bag access port, check valve, 3 needle-free valves 95", 75" and 6" from 2-piece male luer lock Approximate set length: 128" 325 cm. approx. Approximate priming volume: 26 mL	20
10015862	20	 Low sorbing infusion set, non-vented, needle-free valve bag access port, 2-piece male luer lock Approximate set length: 44" 112 cm. approx. Approximate priming volume: 14 mL	20
10061661	20	 Low sorbing infusion set, non-vented, needle-free valve bag access port, check valve, 4 needle-free valves 94", 92", 90" and 6" from 2-piece male luer lock Approximate set length: 121" 307 cm. approx. Approximate priming volume: 25 mL	20
10074281	20	 Low sorbing infusion set, 150 mL burette (needle-free valve), 1 needle-free valve 32" from 2-piece male luer lock Approximate set length: 130" 330 cm. approx. Approximate priming volume: 26 mL	20
10137405	20	 Infusion set, low sorbing tubing segment, check valve, 3 needle-free valves 85", 67" and 6" from 2-piece male luer lock Approximate set length: 119" 302 cm. approx. Approximate priming volume: 25 mL	20

06.00 Alaris Pump module

06.03.01 Low sorbing tubing | SmartSite valve sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports.



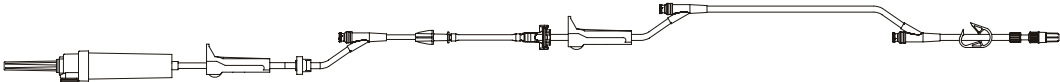
Model #	Drops	Description	Case quantity
10406194	20	 Infusion set, low sorbing and smallbore tubing segments, check valve, 0.2 micron (low-protein binding) filter, 3 needle-free valves 66", 46" and 6" from 2-piece male luer lock Approximate set length: 154" 391 cm. approx. Approximate priming volume: 28 mL	10
10821753	20	 Infusion set, 150 mL burette (needle-free valve), low sorbing tubing segment, 3 needle-free valves 24", 18" and 6" from 2-piece male luer lock Approximate set length: 126" 320 cm. approx. Approximate priming volume: 25 mL	10
11532269	20	 Low sorbing infusion set, check valve, 0.2 micron (low-protein binding) filter, 2 needle-free valves 87" and 6" from 2-piece male luer lock Approximate set length: 124" 315 cm. approx. Approximate priming volume: 28 mL	20
11609481	20	 Low sorbing infusion set, check valve, 0.2 micron (low-protein binding) filter, 2 needle-free valves 12" and 6" from 2-piece male luer lock Approximate set length: 124" 315 cm. approx. Approximate priming volume: 28 mL	20

06.00 Alaris Pump module

06.03.01 Low sorbing tubing | SmartSite valve sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports.



Model #	Drops	Description	Case quantity
11613191	60	 Infusion set, 150 mL burette (needle-free valve), ball valve in drip chamber, low sorbing tubing segment, 0.2 micron (low-protein binding) filter, 2 needle-free valves 51" and 7" from 2-piece male luer lock Approximate set length: 130" 330 cm. approx. Approximate priming volume: 27 mL	 10
2461-0007	20	 Infusion set, low sorbing tubing segments, check valve, 3 needle-free valves 96", 66" and 6" from 2-piece male luer lock Approximate set length: 137" 348 cm. approx. Approximate priming volume: 27 mL	 20
2462-0007	20	 Low sorbing half set, 1 needle-free valve, 6" from 2-piece male luer lock Approximate set length: 93" 236 cm. approx. Approximate priming volume: 13 mL	 20

06.00 Alaris Pump module

06.03.02 Low sorbing tubing | No port sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports.



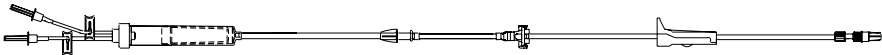
Model #	Drops	Description	Case quantity
2260-0500	20	 Low sorbing infusion set, 2-piece male luer lock Approximate set length: 113" 287 cm. approx. Approximate priming volume: 23 mL	 20
2444-0007	20	 Low sorbing infusion set, burette (needle-free valve), 2-piece male luer lock Approximate set length: 123" 312 cm. approx. Approximate priming volume: 24 mL	 10
2466-0007	20	 Low sorbing infusion set, non-vented, needle-free bag access port, spin male luer lock Approximate set length: 86" 219 cm. approx. Approximate priming volume: 20 mL	 20
10013854	20	 Low sorbing infusion set, non-vented, 15 micron filter, needle-free valve bag access port, luer lock connector Approximate set length: 113" 287 cm. approx. Approximate priming volume: 20 mL	 20
10013890	20	 Low sorbing infusion set, 2-piece male luer lock Approximate set length: 44" 112 cm. approx. Approximate priming volume: 13 mL	 20

06.00 Alaris Pump module

06.03.02 Low sorbing tubing | No port sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports.



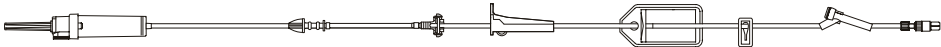
Model #	Drops	Description	Case quantity
10062818	15	 Non-vented blood set with blunt short spike, 180 micron filter, low sorbing tubing segment, 2-piece male luer lock Approximate set length: 122" 310 cm. approx. Approximate priming volume: 33 mL	 10
11426964	20	 Low sorbing infusion set, 2-piece male luer lock Approximate set length: 47" 119 cm. approx. Approximate priming volume: 14 mL	 20
C24117	20	 Low sorbing infusion set, 2-piece male luer lock Approximate set length: 113" 287 cm. approx. Approximate priming volume: 23 mL	 20

06.00 Alaris Pump module

06.03.03 Low sorbing tubing | VersaSafe split septum sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports.



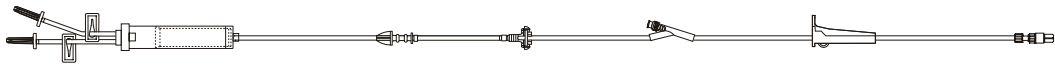
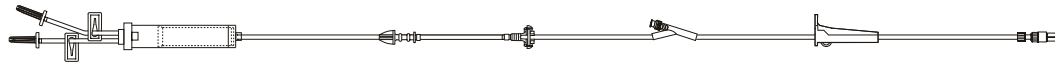
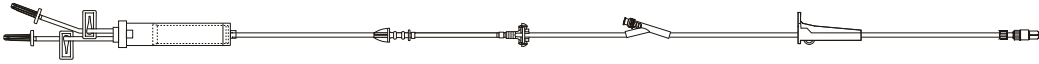
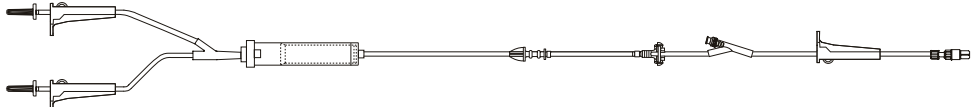
Model #	Drops	Description	Case quantity
2131-0500	20	 Low sorbing infusion set, check valve, 0.2 micron (low-protein binding) filter, 1 split septum port 6" from 2-piece male luer lock Approximate set length: 115" 292 cm. approx. Approximate priming volume: 26 mL	20
11130148	20	 Low sorbing infusion set, 0.2 micron (low-protein binding) filter, 1 split septum port 6" from 2-piece male luer lock Approximate set length: 115" 292 cm. approx. Approximate priming volume: 26 mL	20

06.00 Alaris Pump module

06.04.01.01 Filters | Blood filters | SmartSite valve sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports.



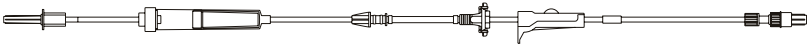
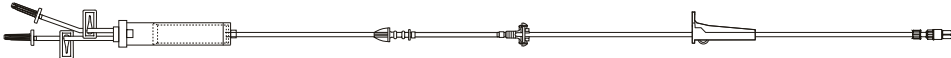
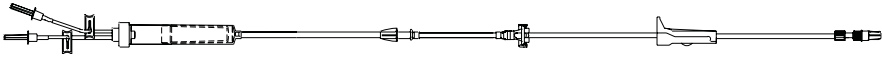
Model #	Drops	Description	Case quantity
2477-0000	15	 Non-vented blood set, 180 micron filter, 1 needle-free valve 65" from 2-piece male luer lock Approximate set length: 122" 310 cm. approx. Approximate priming volume: 34 mL	10
2477-0007	15	 Non-vented blood set, 180 micron filter, low sorbing tubing segment, 1 needle-free valve 65" from 2-piece male luer lock Approximate set length: 122" 310 cm. approx. Approximate priming volume: 33 mL	 10
2478-0000	15	 Non-vented blood set with blunt short spike, 180 micron filter, 1 needle-free valve 43" from 2-piece male luer lock Approximate set length: 100" 254 cm. approx. Approximate priming volume: 31 mL	10
10013037	15	 Non-vented blood set, 180 micron filter, 1 needle-free valve 65" from 2-piece male luer lock Approximate set length: 125" 318 cm. approx. Approximate priming volume: 34 mL	10

06.00 Alaris Pump module

06.04.01.02 Filters | Blood filters | No port sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports.



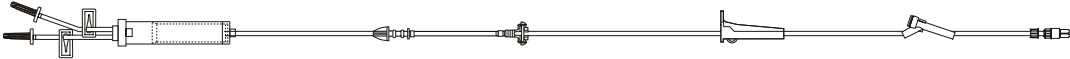
Model #	Drops	Description	Case quantity
2278-0500	20	 Non-vented blood set with single spike, 200 micron filter, smallbore tubing segment, 2-piece male luer lock Approximate set length: 121" 308 cm. approx. Approximate priming volume: 22 mL	20
10015414	15	 Non-vented blood set, 180 micron filter, 2-piece male luer lock Approximate set length: 126" 320 cm. approx. Approximate priming volume: 38 mL	10
10062818	15	 Non-vented blood set with blunt short spike, 180 micron filter, low sorbing tubing segment, 2-piece male luer lock Approximate set length: 122" 310 cm. approx. Approximate priming volume: 33 mL	 10

06.00 Alaris Pump module

06.04.01.03 Filters | Blood filters | VersaSafe split septum sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports.



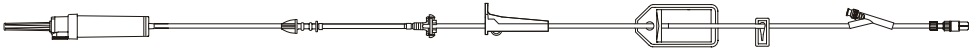
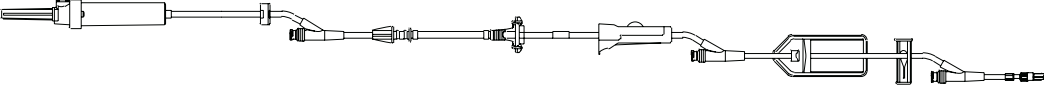
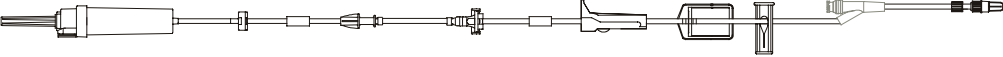
Model #	Drops	Description	Case quantity
2177-0000	15	 Non-vented blood set, 180 micron filter, 1 split septum port 65" from 2-piece male luer lock Approximate set length: 122" 310 cm. approx. Approximate priming volume: 34 mL	20

06.00 Alaris Pump module

06.04.02.01 Filters | 0.2 micron filters | SmartSite valve sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports.



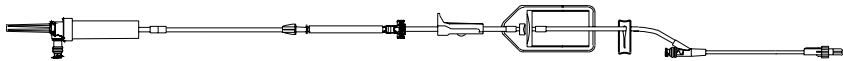
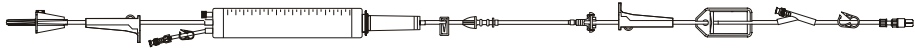
Model #	Drops	Description	Case quantity
2412-0500	20	 Infusion set, 0.2 micron (low-protein binding) filter, 1 needle-free valve 6" from 2-piece male luer lock Approximate set length: 109" 277 cm. approx. Approximate priming volume: 26 mL	20
2430-0500	20	 Infusion set, check valve, 0.2 micron (low-protein binding) filter, 2 needle-free valves 85" and 6" from 2-piece male luer lock Approximate set length: 116" 294 cm. approx. Approximate priming volume: 27 mL	20
2432-0007	20	 Infusion set, check valve, 0.2 micron (low protein binding) filter, 3 needle-free valves 86", 20" and 6" from 2-piece male luer lock Approximate set length: 118" 299 cm. approx. Approximate priming volume: 28 mL	 20
2434-0007	20	 Infusion set, vented/non-vented, 0.2 micron (low protein binding) filter, 1 needle-free valve 6" from 2-piece male luer lock Approximate set length: 115" 292 cm. approx. Approximate priming volume: 28 mL	 20
2443-0600	60	 Infusion set, 150 mL burette (needle-free valve), smallbore tubing segment, 0.2 micron (low-protein binding) filter, 3 needle-free valves 81", 55" and 6" from 2-piece male luer lock Approximate set length: 132" 335 cm. approx. Approximate priming volume: 24 mL	10

06.00 Alaris Pump module

06.04.02.01 Filters | 0.2 micron filters | SmartSite valve sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports.



Model #	Drops	Description	Case quantity
2465-0007	20	 Low sorbing infusion set, non-vented, needle-free bag access port, 20 drop, 0.2 micron (low protein binding) filter, 1 needle-free valve 6" from 2-piece male luer lock Approximate set length: 116" 293 cm. approx. Approximate priming volume: 26 mL	 20
10010454	20	 Low sorbing infusion set, 0.2 micron (low-protein binding) filter, 1 needle-free valve 6" from 2-piece male luer lock Approximate set length: 115" 291 cm. approx. Approximate priming volume: 26 mL	 20
10012283	60	 Infusion set, 150 mL burette (needle-free valve), ball valve in drip chamber, 0.2 micron (low-protein binding) filter, 2 needle-free valves 52" and 7" from 2-piece male luer lock Approximate set length: 129" 328 cm. approx. Approximate priming volume: 27 mL	10
10015012	20	 Infusion set, 150 mL burette (needle-free valve), low sorbing and smallbore tubing segments, 0.2 micron (low-protein binding) filter, 1 needle-free valve 6" from 2-piece male luer lock Approximate set length: 125" 318 cm. approx. Approximate priming volume: 26 mL	 10

06.00 Alaris Pump module

06.04.02.01 Filters | 0.2 micron filters | SmartSite valve sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports.



Model #	Drops	Description	Case quantity
10406194 30-day lead time	20	 Infusion set, low sorbing and smallbore tubing segments, check valve, 0.2 micron (low-protein binding) filter, 3 needle-free valves 66", 46" and 6" from 2-piece male luer lock Approximate set length: 154" 391 cm. approx. Approximate priming volume: 28 mL 	10
11118122 30-day lead time	60	 Infusion set, 150 mL burette (needle-free valve), 0.2 micron (low-protein binding) filter, 3 needle-free valves 89", 62" and 6" from 2-piece male luer lock Approximate set length: 129" 328 cm. approx. Approximate priming volume: 28 mL 	10
11532269	20	 Low sorbing infusion set, check valve, 0.2 micron (low-protein binding) filter, 2 needle-free valves 87" and 6" from 2-piece male luer lock Approximate set length: 124" 315 cm. approx. Approximate priming volume: 28 mL 	20
11609481 30-day lead time	20	 Low sorbing infusion set, check valve, 0.2 micron (low-protein binding) filter, 2 needle-free valves 12" and 6" from 2-piece male luer lock Approximate set length: 124" 315 cm. approx. Approximate priming volume: 28 mL 	20

06.00 Alaris Pump module

06.04.02.01 Filters | 0.2 micron filters | SmartSite valve sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports.



Model #	Drops	Description	Case quantity
11613191	60	 Infusion set, 150 mL burette (needle-free valve), ball valve in drip chamber, low sorbing tubing segment, 0.2 micron (low-protein binding) filter, 2 needle-free valves 51" and 7" from 2-piece male luer lock Approximate set length: 130" 330 cm. approx. Approximate priming volume: 27 mL	10
C24101E	20	 Infusion set, amber tubing, 15 micron filter (drip chamber), 0.2 micron (low-protein binding) filter, 1 needle-free valve 6" from 2-piece male luer lock Approximate set length: 106" 269 cm. approx. Approximate priming volume: 25 mL	20
C24103E	20	 Infusion set, check valve, 0.2 micron (low-protein binding) filter, 3 needle-free valves 90", 21" and 6" from 2-piece male luer lock Approximate set length: 117" 297 cm. approx. Approximate priming volume: 28 mL	20

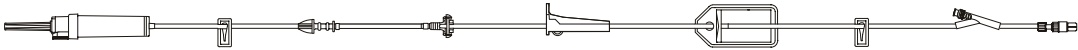
06.00 Alaris Pump module

06.04.02.01 Filters | 0.2 micron filters | SmartSite valve sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports.



Model #	Drops	Description	Case quantity
C24119E	20	Infusion set, 0.2 micron (low-protein binding) filter, 1 needle-free valve 6" from 2-piece male luer lock	20
30-day lead time		Approximate set length: 115" 292 cm. approx.	
		Approximate priming volume: 26 mL	

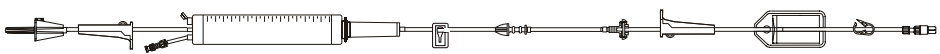


06.00 Alaris Pump module

06.04.02.02 Filters | 0.2 micron filters | No port sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports.



Model #	Drops	Description	Case quantity
11045328	20	 Infusion set, 150 mL burette (needle-free valve), 0.2 micron (low-protein binding) filter, 2-piece male luer lock Approximate set length: 123" 312 cm. approx. Approximate priming volume: 30 mL	10
11612593	20	 Infusion set, 0.2 micron (low-protein binding) filter, 2-piece male luer lock Approximate set length: 112" 284 cm. approx. Approximate priming volume: 27 mL	20

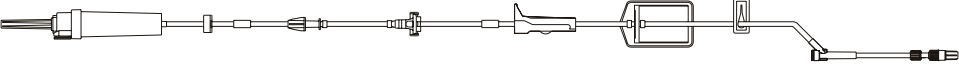
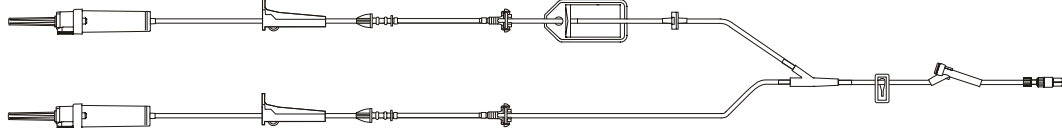
30-day lead time

06.00 Alaris Pump module

06.04.02.03 Filters | 0.2 micron filters | VersaSafe split septum sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports.



Model #	Drops	Description	Case quantity
2121-0500	20	 Infusion set, vented/non-vented, 0.2 micron (low-protein binding) filter, check valve, 1 split septum port 6" from 2-piece male luer lock Approximate set length: 115" 293 cm. approx. Approximate priming volume: 27 mL	20
2130-0500	20	 Infusion set, check valve, 0.2 micron (low-protein binding) filter, 2 split septum ports 85" and 6" from 2-piece male luer lock Approximate set length: 116" 295 cm. approx. Approximate priming volume: 28 mL	20
11130148	20	 Low sorbing infusion set, 0.2 micron (low-protein binding) filter, 1 split septum port 6" from 2-piece male luer lock Approximate set length: 115" 292 cm. approx. Approximate priming volume: 26 mL	20
11202653	20	 Infusion set, 0.2 micron (low-protein binding) filter, low sorbing tubing segment on non-filter side, 1 split septum port 12" from 2-piece male luer lock Approximate set length: 133" 338 cm. approx. Approximate priming volume: 51 mL	10

06.00 Alaris Pump module

06.04.03.01 Filters | 1.2 micron filters | SmartSite valve sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports.



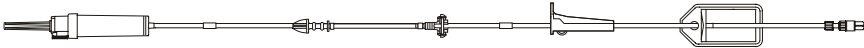
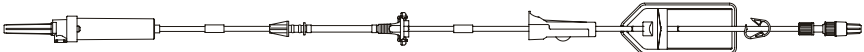
Model #	Drops	Description	Case quantity
10010453		 Low sorbing infusion set, 1.2 micron (low-protein binding) filter, 1 needle-free valve 6" from 2-piece male luer lock Approximate set length: 115" 291 cm. approx. Approximate priming volume: 26 mL	 20

06.00 Alaris Pump module

06.04.03.02 Filters | 1.2 micron filters | No port sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports.



Model #	Drops	Description	Case quantity
2202-0500	20	 Infusion set, 1.2 micron (low-protein binding) filter, 2-piece male luer lock Approximate set length: 122" 310 cm. approx. Approximate priming volume: 28 mL	20
2202-0007	20	 Infusion set, vented/non-vented, 1.2 micron (low-protein binding) filter, 2-piece male luer lock Approximate set length: 122" 309 cm. approx. Approximate priming volume: 27 mL	 20

06.00 Alaris Pump module

06.04.03.03 Filters | 1.2 micron filters | VersaSafe split septum sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports.



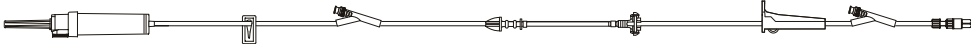
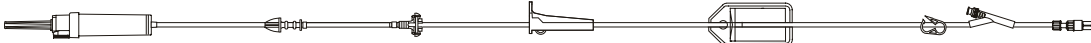
Model #	Drops	Description	Case quantity
2131-0500	20	 Low sorbing infusion set, check valve, 0.2 micron (low-protein binding) filter, 1 split septum port 6" from 2-piece male luer lock Approximate set length: 115" 292 cm. approx. Approximate priming volume: 26 mL	20

06.00 Alaris Pump module

06.04.04.01 Filters | 15 micron filters | SmartSite valve sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports.



Model #	Drops	Description	Case quantity
2414-0500	20	 Infusion set, 15 micron filter (drip chamber), 2 needle-free valves 78" and 6" from 2-piece male luer lock Approximate set length: 115" 292 cm. approx. Approximate priming volume: 24 mL	20
10561554	20	 Infusion set, 15 micron filter (drip chamber), check valve, 3 needle-free valves 86", 68" and 6" from 2-piece male luer lock Approximate set length: 127" 322 cm. approx. Approximate priming volume: 27 mL	20
C24101E	20	 Infusion set, amber tubing, 15 micron filter (drip chamber), 0.2 micron (low-protein binding) filter, 1 needle-free valve 6" from 2-piece male luer lock Approximate set length: 106" 269 cm. approx. Approximate priming volume: 25 mL	20

06.00 Alaris Pump module

06.04.04.02 Filters | 15 micron filters | No port sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports.



Model #	Drops	Description	Case quantity
2203-0500	20	 Infusion set, amber tubing, 15 micron filter (drip chamber), spin male luer lock Approximate set length: 110" 279 cm. approx. Approximate priming volume: 24 mL	20
10863358	20	 Infusion set, 15 micron filter (drip chamber), 2-piece male luer lock Approximate set length: 117" 297 cm. approx. Approximate priming volume: 24 mL	 20

30-day lead time

06.00 Alaris Pump module

06.04.04.03 Filters | 15 micron filters | VersaSafe split septum sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports.



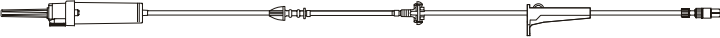
Model #	Drops	Description	Case quantity
11133698	20	 Infusion set, 15 micron filter (drip chamber), 2 split septum ports 83" and 6" from 2-piece male luer lock Approximate set length: 115" 292 cm. approx. Approximate priming volume: 25 mL	20

06.00 Alaris Pump module

06.05.01 Special applications | Yellow striped tubing sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports.



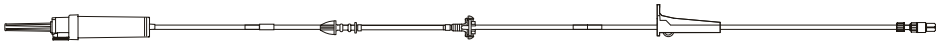
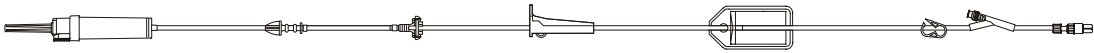
Model #	Drops	Description	Case quantity
2206-0007	20	 Infusion set, microbore yellow striped tubing, 2-piece male luer lock Approximate set length: 97" 247 cm. approx. Approximate priming volume: 13 mL	 20
10010761	20	 Infusion set, microbore yellow striped tubing, 2-piece male luer lock Approximate set length: 99" 251 cm. approx. Approximate priming volume: 14 mL	20
10011301	20	 Infusion set, check valve, microbore yellow striped tubing, 2-piece male luer lock Approximate set length: 118" 300 cm. approx. Approximate priming volume: 15 mL	20

06.00 Alaris Pump module

06.05.02 Special applications | Amber tubing sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports.



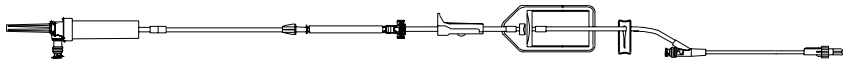
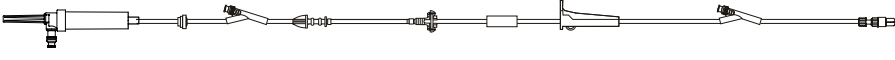
Model #	Drops	Description	Case quantity
2203-0500	20	 Infusion set, amber tubing, 15 micron filter (drip chamber), spin male luer lock Approximate set length: 110" 279 cm. approx. Approximate priming volume: 24 mL	20
C24101E	20	 Infusion set, amber tubing, 15 micron filter (drip chamber), 0.2 micron (low-protein binding) filter, 1 needle-free valve 6" from 2-piece male luer lock Approximate set length: 106" 269 cm. approx. Approximate priming volume: 25 mL	20

06.00 Alaris Pump module

06.05.03 Special applications | SmartSite bag access port sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports.



Model #	Drops	Description	Case quantity
2465-0007	20	 Low sorbing infusion set, non-vented, needle-free bag access port, 20 drop, 0.2 micron (low protein binding) filter, 1 needle-free valve 6" from 2-piece male luer lock Approximate set length: 116" 293 cm. approx. Approximate priming volume: 26 mL	 20
2466-0007	20	 Low sorbing infusion set, non-vented, needle-free bag access port, spin male luer lock Approximate set length: 86" 219 cm. approx. Approximate priming volume: 20 mL	 20
10013361	20	 Infusion set, non-vented, needle-free valve bag access port, check valve, 2 needle-free valves 78" and 6" from 2-piece male luer lock Approximate set length: 117" 297 cm. approx. Approximate priming volume: 25 mL	 20
10014855	20	 Low sorbing infusion set, non-vented, needle-free valve bag access port, check valve, 2 needle-free valves 95" and 6" from 2-piece male luer lock Approximate set length: 116" 295 cm. approx. Approximate priming volume: 24 mL	 20

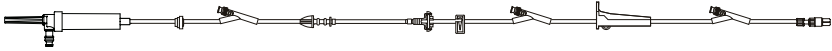
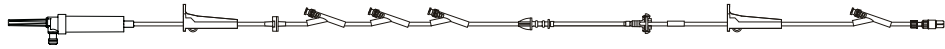
30-day lead time

06.00 Alaris Pump module

06.05.03 Special applications | SmartSite bag access port sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports.



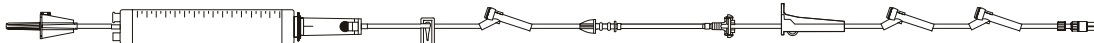
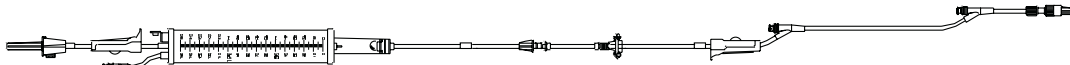
Model #	Drops	Description	Case quantity
10015861 30-day lead time	20	 Infusion set, non-vented, needle-free valve bag access port, low sorbing tubing, check valve, 3 needle-free valves 95", 75" and 6" from 2-piece male luer lock Approximate set length: 128" 325 cm. approx. Approximate priming volume: 26 mL	 20
10015862	20	 Infusion set, non-vented, needle-free valve bag access port, low sorbing tubing, 2-piece male luer lock Approximate set length: 44" 112 cm. approx. Approximate priming volume: 14 mL	 20
10061661 30-day lead time	20	 Infusion set, non-vented, needle-free valve bag access port, low sorbing tubing, check valve, 4 needle-free valves 94", 92", 90" and 6" from 2-piece male luer lock Approximate set length: 121" 307 cm. approx. Approximate priming volume: 25 mL	 20
10321213	20	 Infusion set, non-vented, needle-free valve bag access port, fixed male luer lock Approximate set length: 106" 269 cm. approx. Approximate priming volume: 23 mL	 20

06.00 Alaris Pump module

06.05.04 Special applications | Ball valve sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports.



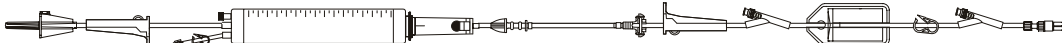
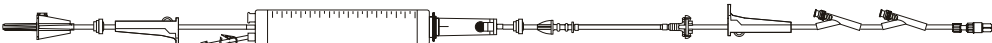
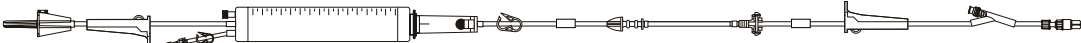
Model #	Drops	Description	Case quantity
2125-0500	20	 Infusion set, check valve, ball valve in drip chamber, 2 split septum ports 86" and 6" from 2-piece male luer lock Approximate set length: 118" 300 cm. approx. Approximate priming volume: 24 mL	20
2147-0600	60	 Infusion set, 150 mL burette (split septum port), ball valve in drip chamber, smallbore tubing segment, 3 split septum ports 86", 60" and 7" from 2-piece male luer lock Approximate set length: 132" 335 cm. approx. Approximate priming volume: 20 mL	10
2425-0500	20	 Infusion set, ball valve in drip chamber, check valve, 2 needle-free valves 85" and 6" from 2-piece male luer lock Approximate set length: 117" 297 cm. approx. Approximate priming volume: 24 mL	20
2447-0007	60	 Infusion set, 150 mL burette (needle-free valve), ball valve in drip chamber, 2 needle-free valves 60" and 7" from 2-piece male luer lock Approximate set length: 129" 328 cm. approx. Approximate priming volume: 24 mL	 10
2447-0600	60	 Infusion set, 150 mL burette set (needle-free valve), ball valve in drip chamber, smallbore tubing segment, 2 needle-free valves 60" and 7" from 2-piece male luer lock Approximate set length: 131" 333 cm. approx. Approximate priming volume: 19 mL	10

06.00 Alaris Pump module

06.05.04 Special applications | Ball valve sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports.



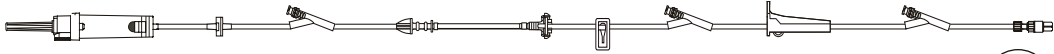
Model #	Drops	Description	Case quantity
10011652	20	 Infusion set, ball valve in drip chamber, 2 needle-free valves 83" and 6" from 2-piece male luer lock Approximate set length: 117" 297 cm. approx. Approximate priming volume: 23 mL	20
10012283	60	 Infusion set, 150 mL burette (needle-free valve), ball valve in drip chamber, 0.2 micron (low-protein binding) filter, 2 needle-free valves 52" and 7" from 2-piece male luer lock Approximate set length: 129" 328 cm. approx. Approximate priming volume: 27 mL	10
10012293	60	 Infusion set, 150 mL burette (needle-free valve), ball valve in drip chamber, smallbore tubing segment, 2 check valves, 2 needle-free valves 64" and 8" from 2-piece male luer lock Approximate set length: 137" 348 cm. approx. Approximate priming volume: 20 mL	10
10817920	60	 Infusion set, 150 mL burette (needle-free valve), ball valve in drip chamber, smallbore tubing segment, 1 needle-free valve 7" from 2-piece male luer lock Approximate set length: 123" 312 cm. approx. Approximate priming volume: 19 mL	10

06.00 Alaris Pump module

06.05.04 Special applications | Ball valve sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports.



Model #	Drops	Description	Case quantity
11522558	20	 Infusion set, ball valve in drip chamber, check valve, 3 needle-free valves 96", 75" and 6" from 2-piece male luer lock Approximate set length: 126" 320 cm. approx. Approximate priming volume: 25 mL	 20
11613191	60	 Infusion set, 150 mL burette (needle-free valve), ball valve in drip chamber, low sorbing tubing segment, 0.2 micron (low-protein binding) filter, 2 needle-free valves 51" and 7" from 2-piece male luer lock Approximate set length: 130" 330 cm. approx. Approximate priming volume: 27 mL	 10

30-day lead time

06.00 Alaris Pump module

06.05.05 Special applications | Syringe adapter sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports. Syringe clip holder 8101A available for use with Alaris Pump module syringe adapter sets.



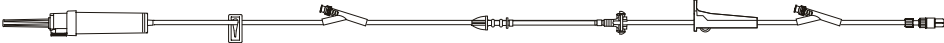
Model #	Drops	Description	Case quantity
2280-0000			20
	Syringe adapter set, microbore tubing segment, 2-piece male luer lock		
	Approximate set length:	83" 211 cm. approx.	
	Approximate priming volume:	4 mL	
10010483			 20
	Syringe adapter set, smallbore tubing, 2-piece male luer lock		
	Approximate set length:	83" 211 cm. approx.	
	Approximate priming volume:	4 mL	

06.00 Alaris Pump module

06.06.01 Primary administration sets | SmartSite valve sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports.



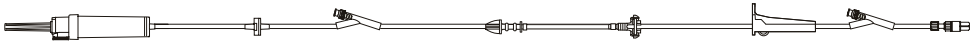
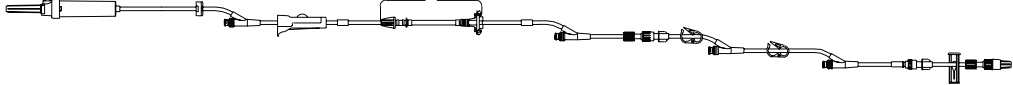
Model #	Drops	Description	Case quantity
2401-0500	20	 Infusion set, 1 needle-free valve 6" from 2-piece male luer lock Approximate set length: 109" 277 cm. approx. Approximate priming volume: 23 mL	20
2403-0000		 Infusion half set, 1 needle-free valve 6" from 2-piece male luer lock Approximate set length: 86" 218 cm. approx. Approximate priming volume: 13 mL	20
2403-0007		 Infusion half set, 1 needle-free valve 6" from 2-piece male luer lock Approximate set length: 86" 218 cm. approx. Approximate priming volume: 13 mL	20
30-day lead time			DEHP
2410-0500	20	 Infusion set, 2 needle-free valves 84" and 6" from 2-piece male luer lock Approximate set length: 115" 292 cm. approx. Approximate priming volume: 25 mL	20
2411-0500	60	 Infusion set, smallbore tubing segment, 2 needle-free valves 86" and 7" from 2-piece male luer lock Approximate set length: 116" 295 cm. approx. Approximate priming volume: 19 mL	20
2420-0007	20	 Infusion set, check valve, 2 needle-free valves 83" and 6" from 2-piece male luer lock Approximate set length: 117" 297 cm. approx. Approximate priming volume: 25 mL	20
			DEHP

06.00 Alaris Pump module

06.06.01 Primary administration sets | SmartSite valve sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports.



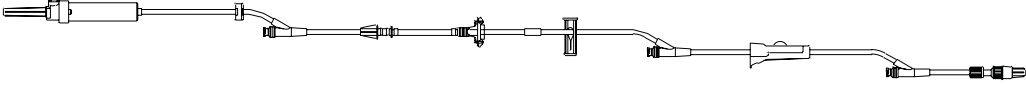
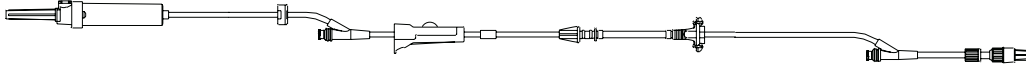
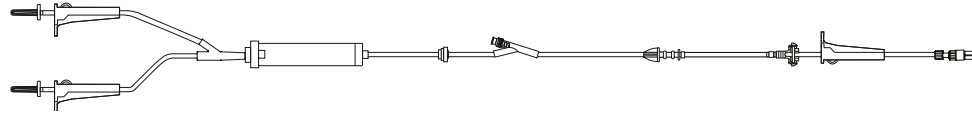
Model #	Drops	Description	Case quantity
2420-0500	20	 Infusion set, check valve, 2 needle-free valves 79" and 6" from 2-piece male luer lock Approximate set length: 117" 297 cm. approx. Approximate priming volume: 25 mL	20
2421-0500	60	 Infusion set, check valve, 2 needle-free valves 78" and 6" from 2-piece male luer lock Approximate set length: 117" 297 cm. approx. Approximate priming volume: 25 mL	20
2426-0007	20	 Infusion set, check valve, 3 needle-free valves 95", 75" and 6" from 2-piece male luer lock Approximate set length: 127" 322 cm. approx. Approximate priming volume: 26 mL	 20
2426-0500	20	 Infusion set, check valve, 3 needle-free valves 83", 63" and 6" from 2-piece male luer lock Approximate set length: 126" 321 cm. approx. Approximate priming volume: 26 mL	20
2429-0007	20	 Infusion set, check valve, 4 needle-free valves 134", 46", 33" and 13" from 2-piece male luer lock Approximate set length: 156" 396 cm. approx. Approximate priming volume: 30 mL	 20

06.00 Alaris Pump module

06.06.01 Primary administration sets | SmartSite valve sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports.



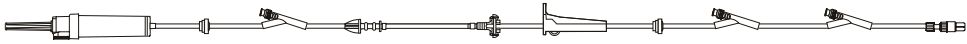
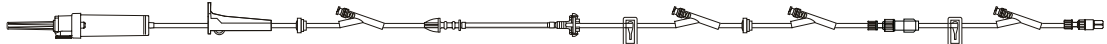
Model #	Drops	Description	Case quantity
2429-0500	20		20
		Infusion set, check valve, 3 needle-free valves 81", 42" and 6" from 2-piece male luer lock	
		Approximate set length: 110" 278 cm. approx. Approximate priming volume: 24 mL	
2433-0007	20		20
		Infusion set, check valve, 2 needle-free valves 88" and 6" from 2-piece male luer lock	
		Approximate set length: 117" 298 cm. approx. Approximate priming volume: 25 mL	
2455-0500	20		20
		Infusion set, 1 needle-free valve 12" from 2-piece male luer lock	
		Approximate set length: 56" 142 cm. approx. Approximate priming volume: 16 mL	
10012144	20		20
		Infusion set, check valve, 3 needle-free valves 76", 42" and 6" from 2-piece male luer lock	
		Approximate set length: 125" 318 cm. approx. Approximate priming volume: 26 mL	
10012645	20		10
		Non-vented infusion set with in-line drip chamber, check valve, 1 needle-free valve 65" from 2-piece male luer lock	
		Approximate set length: 122" 309 cm. approx. Approximate priming volume: 29 mL	
10013186	60		20
		Infusion set, check valve, 3 needle-free valves 84", 37" and 6" from 2-piece male luer lock	
		Approximate set length: 118" 300 cm. approx. Approximate priming volume: 26 mL	

06.00 Alaris Pump module

06.06.01 Primary administration sets | SmartSite valve sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports.



Model #	Drops	Description	Case quantity
10015294	20	 Infusion set, 2 check valves, 3 needle-free valves 118", 43" and 6" from 2-piece male luer lock Approximate set length: 140" 356 cm. approx. Approximate priming volume: 29 mL	20
10015645	20	 Infusion set, 2 needle-free valves 86" and 6" from 2-piece male luer lock Approximate set length: 117" 297 cm. approx. Approximate priming volume: 25 mL	 20
10031773	20	 Infusion set, 2 check valves, 4 needle-free valves 109", 61", 35" and 5" from 2-piece male luer lock Approximate set length: 139" 353 cm. approx. Approximate priming volume: 29 mL	20
10178573	20	 Infusion set, check valve, 4 needle-free valves 95", 42", 13" and 6" from 2-piece male luer lock Approximate set length: 128" 325 cm. approx. Approximate priming volume: 27 mL	20
10404198	20	 Infusion set, check valve, 3 needle-free valves 59", 49" and 6" from 2-piece male luer lock Approximate set length: 154" 391 cm. approx. Approximate priming volume: 31 mL	 20

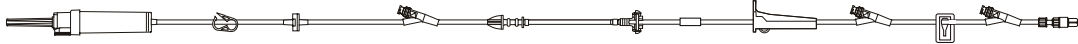
30-day lead time

06.00 Alaris Pump module

06.06.01 Primary administration sets | SmartSite valve sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports.



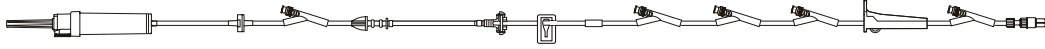
Model #	Drops	Description	Case quantity
10408730	20	 Infusion set, check valve, 3 needle-free valves 72" , 42" and 6" from 2-piece male luer lock Approximate set length: 125" 318 cm. approx. Approximate priming volume: 26 mL	20
10879047	10	 Infusion set, check valve, 2 needle-free valves 85" and 6" from 2-piece male luer lock Approximate set length: 117" 297 cm. approx. Approximate priming volume: 25 mL	 20
10879391	20	 Infusion set, 2 check valves, 3 needle-free valves 91" , 45" and 6" from 2-piece male luer lock Approximate set length: 116" 295 cm. approx. Approximate priming volume: 25 mL	20
10927766	20	 Infusion set, check valve, 3 needle-free valves 94" , 52" and 6" from 2-piece male luer lock Approximate set length: 125" 318 cm. approx. Approximate priming volume: 26 mL	20
10933805	20	 Infusion set, check valve, 2 roller clamps, 2 needle-free valves 85" and 6" from 2-piece male luer lock Approximate set length: 117" 297 cm. approx. Approximate priming volume: 25 mL	20

06.00 Alaris Pump module

06.06.01 Primary administration sets | SmartSite valve sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports.



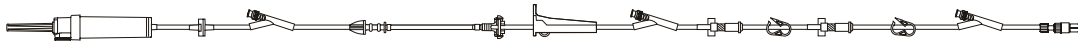
Model #	Drops	Description	Case quantity
11171447	20	 Infusion set, 2 check valves, 3 needle-free valves 96", 42" and 6" from 2-piece male luer lock Approximate set length: 126" 320 cm. approx. Approximate priming volume: 26 mL	DEHP 20
11287205	20	 Infusion set, 4 needle-free valves 97", 42", 13" and 6" from 2-piece male luer lock Approximate set length: 128" 325 cm. approx. Approximate priming volume: 26 mL	DEHP 20
11404930	10	 Infusion set, check valve, 3 needle-free valves 87", 37" and 6" from 2-piece male luer lock Approximate set length: 118" 300 cm. approx. Approximate priming volume: 25 mL	DEHP 20
11426965	20	 Infusion set, check valve, 5 needle-free valves 104", 75", 71", 67" and 6" from 2-piece male luer lock Approximate set length: 138" 351 cm. approx. Approximate priming volume: 28 mL	DEHP 20
11484001	20	 Infusion set, check valve, 1 needle-free valve 74" from 2-piece male luer lock Approximate set length: 106" 269 cm. approx. Approximate priming volume: 23 mL	DEHP 20

06.00 Alaris Pump module

06.06.01 Primary administration sets | SmartSite valve sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports.



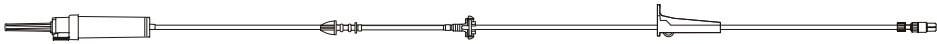
Model #	Drops	Description	Case quantity
11590100	60	 Infusion set, check valve, 2 needle-free valves 76" and 6" from 2-piece male luer lock Approximate set length: 107" 272 cm. approx. Approximate priming volume: 23 mL	 20
11607704	20	 Infusion set, check valve, 2 needle-free valves 88" and 6" from 2-piece male luer lock Approximate set length: 120" 305 cm. approx. Approximate priming volume: 25 mL	 20
C24105E	20	 Infusion set, check valve, 5 needle-free valves 92", 60", 49", 26" and 6" from 2-piece male luer lock Approximate set length: 111" 281 cm. approx. Approximate priming volume: 24 mL	20
C24111E	20	 Infusion set, check valve, 3 needle-free valves 96", 63" and 6" from 2-piece male luer lock Approximate set length: 120" 305 cm. approx. Approximate priming volume: 26 mL	20
C24116E	20	 Infusion set, check valve, 2 needle-free valves 83" and 6" from 2-piece male luer lock Approximate set length: 117" 297 cm. approx. Approximate priming volume: 25 mL	20

06.00 Alaris Pump module

06.06.02 Primary administration sets | No port sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports.



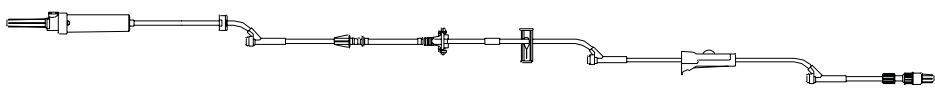
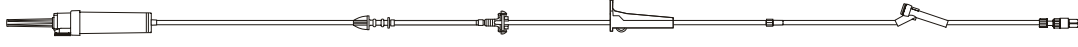
Model #	Drops	Description	Case quantity
2200-0500	20	 Infusion set, 2-piece male luer lock Approximate set length: 109" 277 cm. approx. Approximate priming volume: 24 mL	20
2204-0007	20	 Infusion set, vented/non-vented, 2-piece male luer lock Approximate set length: 47" 119 cm. approx. Approximate priming volume: 14 mL	 20
2941-0500	60	 Infusion set, 2-piece male luer lock Approximate set length: 40" 102 cm. approx. Approximate priming volume: 13 mL	20
2942-0500	20	 Infusion set, 2-piece male luer lock Approximate set length: 40" 102 cm. approx. Approximate priming volume: 13 mL	20
10942011		 Infusion set, 2-piece male luer lock Approximate set length: 109" 277 cm. approx. Approximate priming volume: 23 mL	 20
C24112		 Infusion set, 2-piece male luer lock Approximate set length: 26" 66 cm. approx. Approximate priming volume: 4 mL	50

06.00 Alaris Pump module

06.06.03 Primary administration sets | VersaSafe split septum sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports.



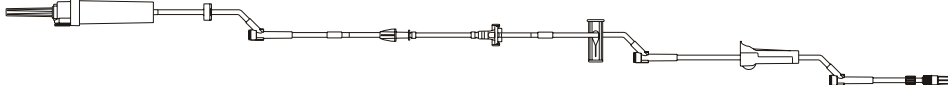
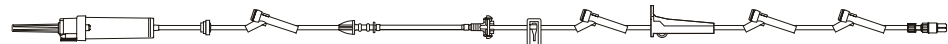
Model #	Drops	Description	Case quantity
2110-0500	20	 Infusion set, 2 split septum ports 85" and 6" from 2-piece male luer lock Approximate set length: 115" 292 cm. approx. Approximate priming volume: 25 mL	20
2111-0500	60	 Infusion set, smallbore tubing segment, 2 split septum ports 86" and 7" from 2-piece male luer lock Approximate set length: 117" 297 cm. approx. Approximate priming volume: 19 mL	20
2120-0500	20	 Infusion set, check valve, 2 split septum ports 85" and 6" from 2-piece male luer lock Approximate set length: 117" 297 cm. approx. Approximate priming volume: 26 mL	20
2126-0500	20	 Infusion set, check valve, 3 split septum ports 83", 63" and 6" from 2-piece male luer lock Approximate set length: 107" 272 cm. approx. Approximate priming volume: 24 mL	20
2129-0500	20	 Infusion set, check valve, 3 split septum ports 81", 42", and 6" from 2-piece male luer lock Approximate set length: 110" 280 cm. approx. Approximate priming volume: 25 mL	20
2929-0500	20	 Infusion set, smallbore tubing segment, 1 split septum port 6" from 2-piece male luer lock Approximate set length: 149" 378 cm. approx. Approximate priming volume: 16 mL	20

06.00 Alaris Pump module

06.06.03 Primary administration sets | VersaSafe split septum sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports.



Model #	Drops	Description	Case quantity
2120-0007	20	 Infusion set, vented/non-vented, check valve, 2 split septum ports 85" and 6" from 2-piece male luer lock Approximate set length: 118" 300 cm. approx. Approximate priming volume: 25 mL	DEHP 20
2121-0500	20	 Infusion set, vented/non-vented, 0.2 micron (low-protein binding) filter, check valve, 1 split septum port 6" from 2-piece male luer lock Approximate set length: 115" 293 cm. approx. Approximate priming volume: 27 mL	DEHP 20
2126-0007	20	 Infusion set, vented/non-vented, check valve, 3 split septum port 83", 63" and 6" from 2-piece male luer lock Approximate set length: 107" 272 cm. approx. Approximate priming volume: 23 mL	DEHP 20
10011462	60	 Infusion set, check valve, 4 split septum ports 97", 79", 36" and 6" from 2-piece male luer lock Approximate set length: 124" 315 cm. approx. Approximate priming volume: 27 mL	20
11147991	20	 Infusion set, 1 split septum port 12" from 2-piece male luer lock Approximate set length: 55" 140 cm. approx. Approximate priming volume: 16 mL	20

06.00 Alaris Pump module

06.06.03 Primary administration sets | VersaSafe split septum sets

Administration sets for use with the Alaris Pump module. All sets are latex free and contain an integrated safety clamp fitment to help prevent accidental free flow. Configurations are available with integrated SmartSite needle-free valves and VersaSafe split septum ports.



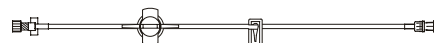
Model #	Drops	Description	Case quantity
11258588		 Infusion half set, 1 split septum port 6" from 2-piece male luer lock Approximate set length: 86" 218 cm. approx. Approximate priming volume: 13 mL	20
11325478		 Infusion set, 2 check valves, 3 split septum ports 80", 44" and 6" from 2-piece male luer lock Approximate set length: 109" 277 cm. approx. Approximate priming volume: 25 mL	20

07.00 Alaris Syringe module

07.01 Alaris Syringe module sets | With pressure sensing disc

Administration sets for use with the Alaris Syringe module. All sets are latex free. Configurations are available with the exclusive pressure sensing disc which enables the following Alaris Syringe module features: fast start, occlusion back off, numeric pressure display, auto-pressure and configurable pressure alarm settings. The Alaris Syringe module will accept non-dedicated administration sets, but we recommend using administration sets with the pressure sensing disc.



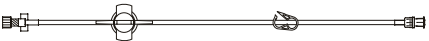
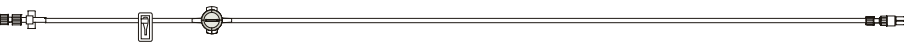
Model #	Drops	Description	Case quantity
			
30916-07		Syringe set, needle-free valve, microbore tubing, pressure-sensing disc, fixed male luer lock Approximate set length: 60" 152 cm. approx. Approximate priming volume: 0.8 mL	 100
			
30920-07		Syringe set, needle-free valve, smallbore tubing, pressure-sensing disc, fixed male luer lock Approximate set length: 60" 152 cm. approx. Approximate priming volume: 1.5 mL	 100
			
10014913		Microbore tubing, pressure sensing disc, fixed male luer lock Approximate set length: 36" 91 cm. approx. Approximate priming volume: 0.6 mL	 100
			
10014914		Microbore tubing, pressure sensing disc, fixed male luer lock Approximate set length: 60" 152 cm. approx. Approximate priming volume: 0.8 mL	 100
			
10014915		Microbore tubing, pressure sensing disc, 0.2 micron (low-protein binding) filter, fixed male luer lock Approximate set length: 36" 91 cm. approx. Approximate priming volume: 0.8 mL	 100

07.00 Alaris Syringe module

07.01 Alaris Syringe module sets | With pressure sensing disc

Administration sets for use with the Alaris Syringe module. All sets are latex free. Configurations are available with the exclusive pressure sensing disc which enables the following Alaris Syringe module features: fast start, occlusion back off, numeric pressure display, auto-pressure and configurable pressure alarm settings. The Alaris Syringe module will accept non-dedicated administration sets, but we recommend using administration sets with the pressure sensing disc.



Model #	Drops	Description	Case quantity	
10014916				100
	Microbore tubing, pressure sensing disc, 0.2 micron (low-protein binding) filter, fixed male luer lock			
	Approximate set length:	60" 152 cm. approx.		
	Approximate priming volume:	1 mL		
10014917				100
	Smallbore tubing, pressure sensing disc, fixed male luer lock			
	Approximate set length:	60" 152 cm. approx.		
	Approximate priming volume:	1.7 mL		
10014918				100
	Microbore low sorbing tubing, pressure sensing disc, fixed male luer lock			
	Approximate set length:	60" 152 cm. approx.		
	Approximate priming volume:	1.3 mL		
10015612				100
	Microbore tubing, pressure sensing disc, fixed male luer lock			
	Approximate set length:	78" 198 cm. approx.		
	Approximate priming volume:	1 mL		
10321214				100
	Microbore tubing, pressure sensing disc, spin male luer lock			
	Approximate set length:	79" 201 cm. approx.		
	Approximate priming volume:	1 mL		

07.00 Alaris Syringe module

07.01 Alaris Syringe module sets | With pressure sensing disc

Administration sets for use with the Alaris Syringe module. All sets are latex free. Configurations are available with the exclusive pressure sensing disc which enables the following Alaris Syringe module features: fast start, occlusion back off, numeric pressure display, auto-pressure and configurable pressure alarm settings. The Alaris Syringe module will accept non-dedicated administration sets, but we recommend using administration sets with the pressure sensing disc.



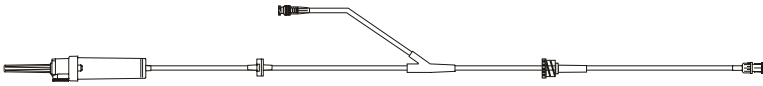
Model #	Drops	Description	Case quantity
10798696	20	 Syringe set, microbore tubing, check valve, needle-free valve Y-extension, 0.2 micron (low-protein binding) filter, anti-siphon valve, pressure sensing disc, fixed male luer lock Approximate set length: 96" 244 cm. approx. Approximate priming volume: 14 mL	 50
10798697		 Microbore amber tubing, pressure sensing disc, spin male luer lock Approximate set length: 63" 160 cm. approx. Approximate priming volume: 0.8 mL	 100
10798703	20	 Syringe set, microbore tubing, check valve, needle-free valve Y-extension, anti-siphon valve, pressure sensing disc, fixed male luer lock Approximate set length: 96" 244 cm. approx. Approximate priming volume: 14 mL	 50
10912433	30-day lead time	 Microbore tubing, pressure sensing disc, spin male luer lock Approximate set length: 47" 119 cm. approx. Approximate priming volume: 0.7 mL	 100

07.00 Alaris Syringe module

07.02 Alaris Syringe module sets | Without pressure sensing disc

Administration sets for use with the Alaris Syringe module. All sets are latex free. Configurations are available with the exclusive pressure sensing disc which enables the following Alaris Syringe module features: fast start, occlusion back off, numeric pressure display, auto-pressure and configurable pressure alarm settings. The Alaris Syringe module will accept non-dedicated administration sets, but we recommend using administration sets with the pressure sensing disc.



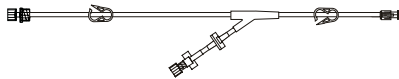
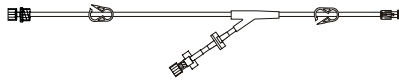
Model #	Drops	Description	Case quantity
10689602	20	 Syringe set, microbore tubing, check valve, needle-free valve Y-extension, anti-siphon valve, fixed male luer lock Approximate set length: 95" 241 cm. approx. Approximate priming volume: 14 mL	 50

08.00 Alaris PCA module

08.00 Alaris PCA module sets

The Alaris PCA module uses standard, single-use, disposable syringes (with luer-lock connectors) and administration sets with anti-siphon valves, designed for use on syringe-type PCA pumps. The following extension and administration sets are compatible with the Alaris PCA module. All sets are latex free.



Model #	Drops	Description	Case quantity	
30843E				100
		PCA extension set, smallbore tubing segment, check valve, 1 needle-free valve 7" from 2-piece male luer lock		
		Approximate set length: 13" 33 cm. approx. Approximate priming volume: 1 mL		
30853				50
		Administration set, Y-connector, 2 female luer lock ports, anti-siphon valve, check valve, microbore tubing segment, fixed male luer lock		
		Approximate set length: 70" 178 cm. approx. Approximate priming volume: 2.8 mL		
30863	30-day lead time			50
		Microbore administration set, anti-siphon valve, fixed male luer lock		
		Approximate set length: 62" 157 cm. approx. Approximate priming volume: 1.5 mL		
30873				50
		Administration set, Y-connector, 2 female luer locks, anti-siphon valve, check valve, microbore tubing segment, fixed male luer lock		
		Approximate set length: 93" 236 cm. approx. Approximate priming volume: 3 mL		

08.00 Alaris PCA module

08.00 Alaris PCA module sets

The Alaris PCA module uses standard, single-use, disposable syringes (with luer-lock connectors) and administration sets with anti-siphon valves, designed for use on syringe-type PCA pumps. The following extension and administration sets are compatible with the Alaris PCA module. All sets are latex free.



Model #	Drops	Description	Case quantity	
30883		Microbore administration set, anti-siphon valve, fixed male luer lock		50
		Approximate set length: 92" 234 cm. approx.		
		Approximate priming volume: 2.1 mL		
30893		Microbore administration set with yellow identification stripe, anti-siphon valve, fixed male luer lock		50
		Approximate set length: 112" 284 cm. approx.		
		Approximate priming volume: 2.7 mL		
10800173		Administration set, Y-connector, 2 female luer locks, anti-siphon valve, check valve, microbore tubing segment, fixed male luer lock with reusable plunger		50
		Approximate set length: 70" 178 cm. approx.		
		Approximate priming volume: 2.8 mL		
10800174		Administration set, anti-siphon valve, microbore tubing segment, fixed male luer lock with reusable plunger		50
		Approximate set length: 62" 157 cm. approx.		
		Approximate priming volume: 1.6 mL		
10800175		Administration set, Y-connector, 2 female luer locks, anti-siphon valve, check valve, microbore tubing segment, fixed male luer lock with reusable plunger		50
		Approximate set length: 90" 229 cm. approx.		
		Approximate priming volume: 4 mL		

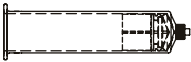
10800174
30-day lead time

08.00 Alaris PCA module

08.00 Alaris PCA module sets

The Alaris PCA module uses standard, single-use, disposable syringes (with luer-lock connectors) and administration sets with anti-siphon valves, designed for use on syringe-type PCA pumps. The following extension and administration sets are compatible with the Alaris PCA module. All sets are latex free.



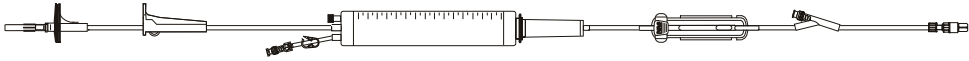
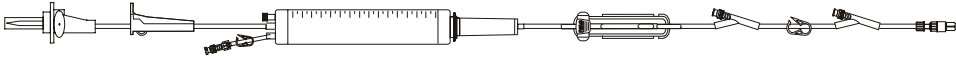
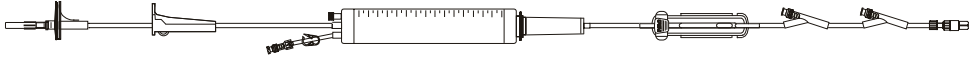
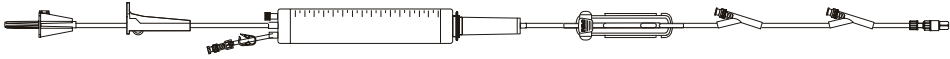
Model #	Drops	Description	Case quantity
10800176			 50
		Microbore administration set, anti-siphon valve, fixed male luer lock with reusable plunger	
		Approximate set length: 92" 234 cm. approx. Approximate priming volume: 2.1 mL	
10800177	30-day lead time		 50
		Microbore administration set with yellow identification stripe, anti-siphon valve, fixed male luer lock with reusable plunger	
		Approximate set length: 112" 284 cm. approx. Approximate priming volume: 2.8 mL	
8881135609		 Empty 35mL Monoject syringe barrel. 10/pack	 150
8881682101		 Monoject syringe barrel tip cap. 10/pack	 1000
8881682119		 Monoject syringe barrel tip cap. 25/pack	 1000

09.00 Alaris SE pump

09.01.01 Burettes | SmartSite valve sets

Administration sets for use with Alaris SE pumps. All sets are latex free and contain the AccuSlide® flow regulator, which replaces the roller clamp and minimizes the risk of accidental free flow. Set configurations are available with one or more of the SmartSite needle-free valves and VersaSafe split septum ports.



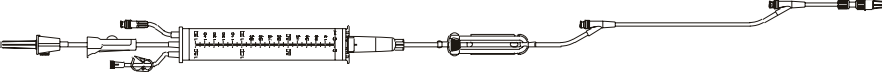
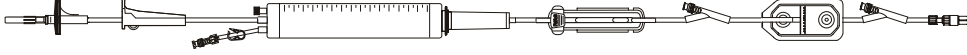
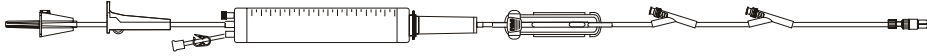
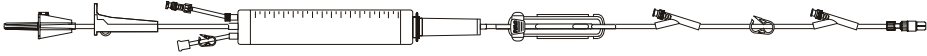
Model #	Drops	Description	Case quantity
72103E	20	 Infusion set, 150 mL burette (needle-free valve), 1 needle-free valve 9" from 2-piece male luer lock Approximate set length: 120" 305 cm. approx. Approximate priming volume: 22 mL	20
72104KE	20	 Infusion set, 150 mL burette (needle-free valve), 2 needle-free valves 62" and 9" from 2-piece male luer lock Approximate set length: 122" 310 cm. approx. Approximate priming volume: 22 mL	20
72163E	60	 Infusion set, 150 mL burette (needle-free valve), smallbore tubing segment, 2 needle-free valves 60" and 9" from 2-piece male luer lock Approximate set length: 122" 310 cm. approx. Approximate priming volume: 16 mL	20
72163E-0003	60	 Infusion set, 150 mL burette (needle-free valve), hat valve in bag spike, smallbore tubing segment, 2 needle-free valves 60" and 9" from spin male luer. Approximate set length: 122" 310 cm. approx. Approximate priming volume: 17 mL	20

09.00 Alaris SE pump

09.01.01 Burettes | SmartSite valve sets

Administration sets for use with Alaris SE pumps. All sets are latex free and contain the AccuSlide flow regulator, which replaces the roller clamp and minimizes the risk of accidental free flow. Set configurations are available with one or more of the SmartSite needle-free valves and VersaSafe split septum ports.



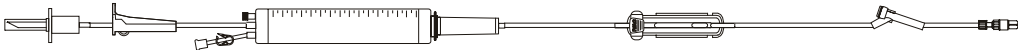
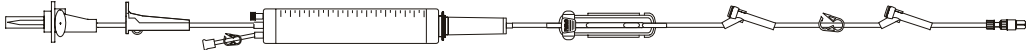
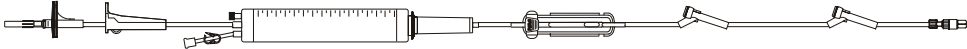
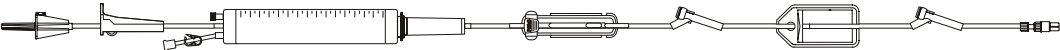
Model #	Drops	Description	Case quantity
72165E	60	 Infusion set, burette (needle-free valve), 2 needle-free valves 61" and 9" from 2-piece male luer lock Approximate set length: 122" 304 cm. approx. Approximate priming volume: 18 mL	20
72193E	60	 Infusion set, 150 mL burette (needle-free valve), smallbore tubing segment, 0.2 micron (low-protein binding) filter, 2 needle-free valves 62" and 9" from 2-piece male luer lock Approximate set length: 123" 312 cm. approx. Approximate priming volume: 20 mL	20
C72110E	60	 Infusion set, 150 mL burette (needle-free valve), smallbore tubing segment, 2 needle-free valves 83" and 9" from 2-piece male luer lock Approximate set length: 143" 363 cm. approx. Approximate priming volume: 19 mL	20
C72113E	60	 Infusion set, 150 mL burette (needle-free valve), smallbore tubing segment, 2 needle-free valves 62" and 9" from 2-piece male luer lock Approximate set length: 121" 307 cm. approx. Approximate priming volume: 17 mL	20

09.00 Alaris SE pump

09.01.02 Burettes | VersaSafe split septum sets

Administration sets for use with Alaris SE pumps. All sets are latex free and contain the AccuSlide flow regulator, which replaces the roller clamp and minimizes the risk of accidental free flow. Set configurations are available with one or more of the SmartSite needle-free valves and VersaSafe split septum ports.



Model #	Drops	Description	Case quantity
72103M	20	 Infusion set, 150 mL burette (split septum port), 1 split septum port 9" from 2-piece male luer lock Approximate set length: 121" 307 cm. approx. Approximate priming volume: 22 mL	20
72104KM	20	 Infusion set, 150 mL burette (split septum port), 2 split septum ports 63" and 9" from 2-piece male luer lock Approximate set length: 122" 310 cm. approx. Approximate priming volume: 23 mL	20
72163M	60	 Infusion set, 150 mL burette (needle-free valve), smallbore tubing segment, 2 split septum ports 63" and 9" from 2-piece male luer lock Approximate set length: 122" 310 cm. approx. Approximate priming volume: 18 mL	20
11141556	60	 Infusion set, 150 mL burette (split septum port), smallbore tubing segment, 0.2 micron (low-protein binding) filter, 2 split septum ports 60" and 9" from 2-piece male luer lock Approximate set length: 124" 315 cm. approx. Approximate priming volume: 21 mL	 20

09.00 Alaris SE pump

09.02.01 Low sorbing tubing | SmartSite valve sets

Administration sets for use with Alaris SE pumps. All sets are latex free and contain the AccuSlide flow regulator, which replaces the roller clamp and minimizes the risk of accidental free flow. Set configurations are available with one or more of the SmartSite needle-free valves and VersaSafe split septum ports.



Model #	Drops	Description	Case quantity
72313E 30-day lead time	20	Infusion set, low sorbing tubing segment, 1.2 micron (low-protein binding) filter, 1 needle-free valve 9" from 2-piece male luer lock Approximate set length: 118" 300 cm. approx. Approximate priming volume: 26 mL	50
72923E	20	Infusion set, low sorbing tubing segment, 0.2 micron (low-protein binding) filter, 1 needle-free valve 9" from 2-piece male luer lock Approximate set length: 119" 302 cm. approx. Approximate priming volume: 26 mL	50
10015863 30-day lead time	20	Infusion set, low sorbing tubing segment, non-vented, needle-free valve bag access port, check valve, 3 needle-free valves 83", 37" and 9" from 2-piece male luer lock Approximate set length: 101" 257 cm. approx. Approximate priming volume: 22 mL	50

09.00 Alaris SE pump

09.02.02 Low sorbing tubing | No port sets

Administration sets for use with Alaris SE pumps. All sets are latex free and contain the AccuSlide flow regulator, which replaces the roller clamp and minimizes the risk of accidental free flow. Set configurations are available with one or more of the SmartSite needle-free valves and VersaSafe split septum ports.



Model #	Drops	Description	Case quantity
72923	20	 Infusion set, low sorbing tubing segment, 0.2 micron (low-protein binding) filter, 2-piece male luer lock Approximate set length: 104" 264 cm. approx. Approximate priming volume: 24 mL	50
72953	20	 Infusion set, low sorbing tubing segment, 2-piece male luer lock Approximate set length: 110" 279 cm. approx. Approximate priming volume: 22 mL	20
10026645		 Infusion set, low sorbing tubing segment, non-vented, needle-free valve bag access port, 2-piece male luer lock Approximate set length: 88" 224 cm. approx. Approximate priming volume: 20 mL	20
10784614	20	 Infusion set, low sorbing tubing segment, non-vented, needle-free valve bag access port, spin male luer lock Approximate set length: 88" 224 cm. approx. Approximate priming volume: 20 mL	20
11258587	20	 Infusion set, low sorbing tubing segment, 2-piece male luer lock Approximate set length: 111" 282 cm. approx. Approximate priming volume: 23 mL	20

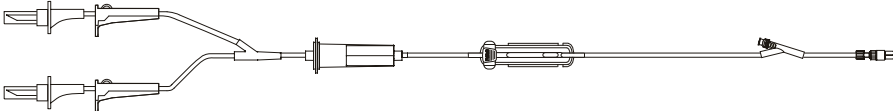
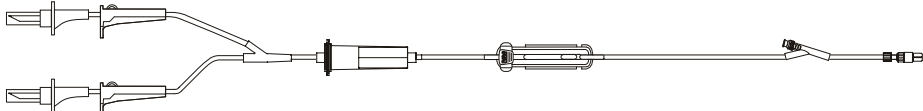
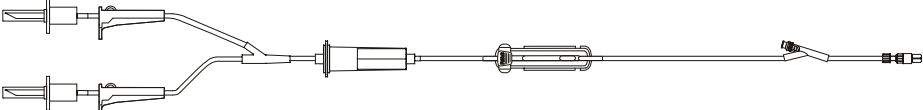


09.00 Alaris SE pump

09.03.01.01 Filters | Blood filters | SmartSite valve sets

Administration sets for use with Alaris SE pumps. All sets are latex free and contain the AccuSlide flow regulator, which replaces the roller clamp and minimizes the risk of accidental free flow. Set configurations are available with one or more of the SmartSite needle-free valves and VersaSafe split septum ports.



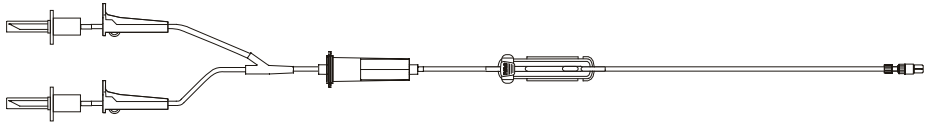
Model #	Drops	Description	Case quantity
72980E	15		20
		Non-vented blood set, 180 micron filter, 1 needle-free valve 9" from 2-piece male luer lock	
		Approximate set length: 105" 267 cm. approx. Approximate priming volume: 31 mL	
72981E	20		20
		Non-vented blood set, 200 micron filter, 1 needle-free valve 62" from 2-piece male luer lock	
		Approximate set length: 106" 269 cm. approx. Approximate priming volume: 31 mL	
79980E	20		20
		Non-vented blood set, blunt short spike, 200 micron filter, 1 needle-free valve 59" from 2-piece male luer lock	
		Approximate set length: 106" 269 cm. approx. Approximate priming volume: 30 mL	

09.00 Alaris SE pump

09.03.01.02 Filters | Blood filters | No port sets

Administration sets for use with Alaris SE pumps. All sets are latex free and contain the AccuSlide flow regulator, which replaces the roller clamp and minimizes the risk of accidental free flow. Set configurations are available with one or more of the SmartSite needle-free valves and VersaSafe split septum ports.



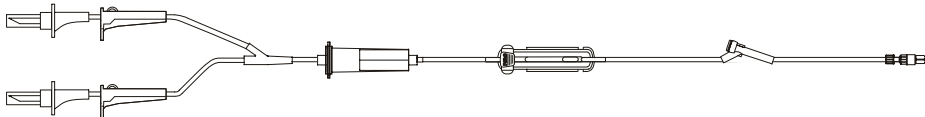
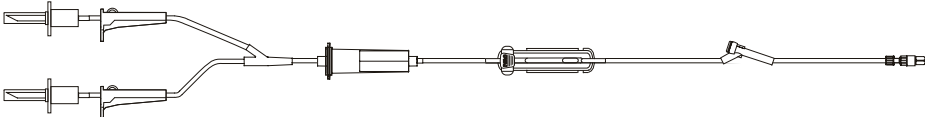
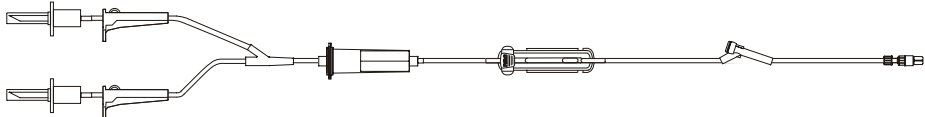
Model #	Drops	Description	Case quantity
71980-0500	20		50
		Non-vented blood set with single spike, 200 micron filter, smallbore tubing segment, 2-piece male luer lock	
		Approximate set length: 108" 274 cm. approx.	
		Approximate priming volume: 21 mL	
10015415	15		20
		Non-vented blood set, 180 micron filter, 2-piece male luer lock	
		Approximate set length: 104" 264 cm. approx.	
		Approximate priming volume: 31 mL	

09.00 Alaris SE pump

09.03.01.03 Filters | Blood filters | VersaSafe split septum sets

Administration sets for use with Alaris SE pumps. All sets are latex free and contain the AccuSlide flow regulator, which replaces the roller clamp and minimizes the risk of accidental free flow. Set configurations are available with one or more of the SmartSite needle-free valves and VersaSafe split septum ports.



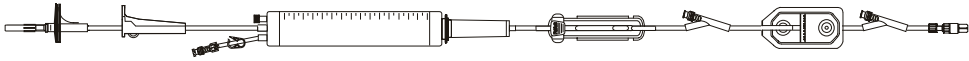
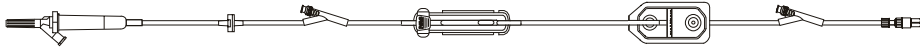
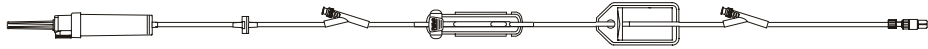
Model #	Drops	Description	Case quantity
72980M	15		20
		Non-vented blood set, 180 micron filter, 1 split septum port 9" from 2-piece male luer lock	
		Approximate set length: 106" 269 cm. approx. Approximate priming volume: 31 mL	
10012212	20		20
		Non-vented blood set, 200 micron filter, 1 split septum port 9" from 2-piece male luer lock	
		Approximate set length: 106" 269 cm. approx. Approximate priming volume: 31 mL	
11499817	15		 20
		Non-vented blood set, 180 micron filter, 1 split septum port 9" from 2-piece male luer lock	
		Approximate set length: 106" 269 cm. approx. Approximate priming volume: 31 mL	

09.00 Alaris SE pump

09.03.02.01 Filters | 0.2 micron filters | SmartSite valve sets

Administration sets for use with Alaris SE pumps. All sets are latex free and contain the AccuSlide flow regulator, which replaces the roller clamp and minimizes the risk of accidental free flow. Set configurations are available with one or more of the SmartSite needle-free valves and VersaSafe split septum ports.



Model #	Drops	Description	Case quantity
72193E	60	 Infusion set, 150 mL burette (needle-free valve), smallbore tubing segment, 0.2 micron (low-protein binding) filter, 2 needle-free valves 62" and 9" from 2-piece male luer lock Approximate set length: 123" 312 cm. approx. Approximate priming volume: 20 mL	20
72323E	20	 Infusion set, check valve, 0.2 micron (low-protein binding) filter, 2 needle-free valves 83" and 9" from 2-piece male luer lock Approximate set length: 100" 254 cm. approx. Approximate priming volume: 25 mL	50
10014854	20	 Infusion set, 0.2 micron (low-protein binding) filter, 1 needle-free valve 9" from 2-piece male luer lock Approximate set length: 101" 257 cm. approx. Approximate priming volume: 25 mL	50
11225954	20	 Infusion set, check valve, 0.2 micron (low-protein binding filter), 2 needle-free valves 83" and 9" from 2-piece male luer lock Approximate set length: 100" 254 cm. approx. Approximate priming volume: 25 mL	 50

09.00 Alaris SE pump

09.03.02.02 Filters | 0.2 micron filters | No port sets

Administration sets for use with Alaris SE pumps. All sets are latex free and contain the AccuSlide flow regulator, which replaces the roller clamp and minimizes the risk of accidental free flow. Set configurations are available with one or more of the SmartSite needle-free valves and VersaSafe split septum ports.



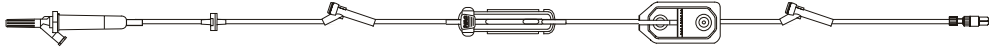
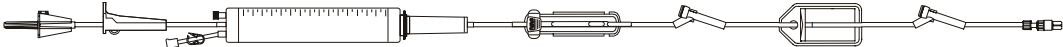
Model #	Drops	Description	Case quantity
72923	20	 Infusion set, low sorbing tubing segment, 0.2 micron (low-protein binding) filter, 2-piece male luer lock Approximate set length: 104" 264 cm. approx. Approximate priming volume: 24 mL	50

09.00 Alaris SE pump

09.03.02.03 Filters | 0.2 micron filters | VersaSafe split septum sets

Administration sets for use with Alaris SE pumps. All sets are latex free and contain the AccuSlide flow regulator, which replaces the roller clamp and minimizes the risk of accidental free flow. Set configurations are available with one or more of the SmartSite needle-free valves and VersaSafe split septum ports.



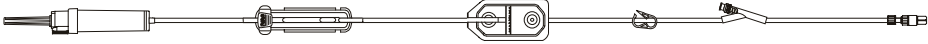
Model #	Drops	Description	Case quantity
72323M	20	 Infusion set, check valve, 0.2 micron (low-protein binding) filter, 2 split septum ports 73" and 9" from 2-piece male luer lock Approximate set length: 100" 254 cm. approx. Approximate priming volume: 25 mL	50
11141556	60	 Infusion set, 150 mL burette (split septum port), smallbore tubing segment, 0.2 micron (low-protein binding) filter, 2 split septum ports 60" and 9" from 2-piece male luer lock Approximate set length: 124" 315 cm. approx. Approximate priming volume: 21 mL	20

09.00 Alaris SE pump

09.03.03.01 Filters | 1.2 micron filters | SmartSite valve sets

Administration sets for use with Alaris SE pumps. All sets are latex free and contain the AccuSlide flow regulator, which replaces the roller clamp and minimizes the risk of accidental free flow. Set configurations are available with one or more of the SmartSite needle-free valves and VersaSafe split septum ports.



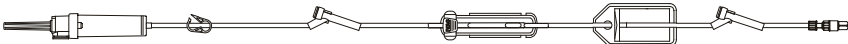
Model #	Drops	Description	Case quantity
72313E	20	 Infusion set, low sorbing tubing segment, 1.2 micron (low-protein binding) filter, 1 needle-free valve 9" from 2-piece male luer lock Approximate set length: 118" 300 cm. approx. Approximate priming volume: 26 mL	50

09.00 Alaris SE pump

09.03.03.02 Filters | 1.2 micron filters | VersaSafe split septum sets

Administration sets for use with Alaris SE pumps. All sets are latex free and contain the AccuSlide flow regulator, which replaces the roller clamp and minimizes the risk of accidental free flow. Set configurations are available with one or more of the SmartSite needle-free valves and VersaSafe split septum ports.



Model #	Drops	Description	Case quantity
11132982	 20	 Infusion set, 1.2 micron (low-protein binding) filter, 2 split septum ports 87" and 9" from 2-piece male luer lock Approximate set length: 105" 267 cm. approx. Approximate priming volume: 26 mL	 50

09.00 Alaris SE pump

09.03.04.01 Filters | 15 micron filters | No port sets

Administration sets for use with Alaris SE pumps. All sets are latex free and contain the AccuSlide flow regulator, which replaces the roller clamp and minimizes the risk of accidental free flow. Set configurations are available with one or more of the SmartSite needle-free valves and VersaSafe split septum ports.



Model #	Drops	Description	Case quantity
72303-0500	20	 Infusion set, 15 micron filter (drip chamber), 2-piece male luer lock Approximate set length: 96" 244 cm. approx. Approximate priming volume: 20 mL	50
72544	20	 Infusion set, amber tubing, 15 micron filter (drip chamber), fixed male luer lock Approximate set length: 108" 274 cm. approx. Approximate priming volume: 24 mL	20
10012529	20	 Infusion set, amber tubing, 15 micron filter (drip chamber), 2-piece male luer lock Approximate set length: 86" 218 cm. approx. Approximate priming volume: 20 mL	50

09.00 Alaris SE pump

09.04.01 Special applications | Yellow striped tubing sets

Administration sets for use with Alaris SE pumps. All sets are latex free and contain the AccuSlide flow regulator, which replaces the roller clamp and minimizes the risk of accidental free flow. Set configurations are available with one or more of the SmartSite needle-free valves and VersaSafe split septum ports.



Model #	Drops	Description	Case quantity
10012812	20	Infusion set, microbore yellow striped tubing, 2-piece male luer lock	50
30-day lead time		Approximate set length: 100" 254 cm. approx.	
		Approximate priming volume: 12 mL	



09.00 Alaris SE pump

09.04.02 Special applications | Amber tubing sets

Administration sets for use with Alaris SE pumps. All sets are latex free and contain the AccuSlide flow regulator, which replaces the roller clamp and minimizes the risk of accidental free flow. Set configurations are available with one or more of the SmartSite needle-free valves and VersaSafe split septum ports.



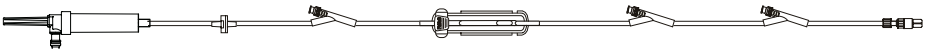
Model #	Drops	Description	Case quantity
72544	20	 Infusion set, amber tubing, 15 micron filter (drip chamber), fixed male luer lock Approximate set length: 108" 274 cm. approx. Approximate priming volume: 24 mL	20
10012529	20	 Infusion set, amber tubing, 15 micron filter (drip chamber), 2-piece male luer lock Approximate set length: 86" 218 cm. approx. Approximate priming volume: 20 mL	50

09.00 Alaris SE pump

09.04.03 Special applications | SmartSite bag access port sets

Administration sets for use with Alaris SE pumps. All sets are latex free and contain the AccuSlide flow regulator, which replaces the roller clamp and minimizes the risk of accidental free flow. Set configurations are available with one or more of the SmartSite needle-free valves and VersaSafe split septum ports.



Model #	Drops	Description	Case quantity
10013363	20	 Infusion set, non-vented, needle-free valve bag access port, smallbore tubing segment, check valve, 2 needle-free valves 90" and 9" from 2-piece male luer lock Approximate set length: 100" 254 cm. approx. Approximate priming volume: 22 mL	50
10015863	20	 Infusion set, low sorbing tubing segment, non-vented, needle-free valve bag access port, check valve, 3 needle-free valves 83", 37" and 9" from 2-piece male luer lock Approximate set length: 101" 257 cm. approx. Approximate priming volume: 22 mL	50
10026645	20	 Infusion set, low sorbing tubing segment, non-vented, needle-free valve bag access port, 2-piece male luer lock Approximate set length: 88" 224 cm. approx. Approximate priming volume: 20 mL	20
10784614	20	 Infusion set, low sorbing tubing segment, non-vented, needle-free valve bag access port, spin male luer lock Approximate set length: 88" 224 cm. approx. Approximate priming volume: 20 mL	20

09.00 Alaris SE pump

09.04.04 Special applications | Syringe adapter sets

Administration sets for use with Alaris SE pumps. All sets are latex free and contain the AccuSlide flow regulator which replaces the roller clamp and minimizes the risk of accidental free flow. Set configurations are available with one or more of the SmartSite needle-free valves and VersaSafe split septum ports. Syringe clip holder 3011 available for use with Alaris SE pump syringe adapter sets.



Model #	Drops	Description	Case quantity
72983			
	Syringe adapter set, microbore tubing, 2-piece male luer lock		50
	Approximate set length:	94" 239 cm. approx.	
	Approximate priming volume:	3 mL	

09.00 Alaris SE pump

09.05.01 Primary administration sets | SmartSite valve sets

Administration sets for use with Alaris SE pumps. All sets are latex free and contain the AccuSlide flow regulator, which replaces the roller clamp and minimizes the risk of accidental free flow. Set configurations are available with one or more of the SmartSite needle-free valves and VersaSafe split septum ports.



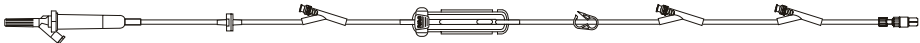
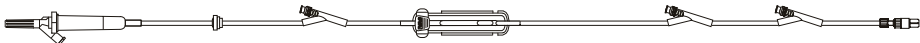
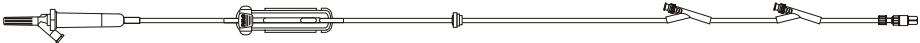
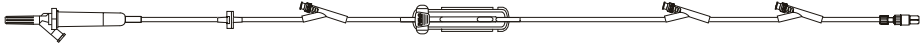
Model #	Drops	Description	Case quantity
71633E	10	 Infusion set, check valve, 3 needle-free valves 99", 39" and 9" from 2-piece male luer lock Approximate set length: 138" 351 cm. approx. Approximate priming volume: 25 mL	50
72001E	20	 Infusion set, 1 needle-free valve 9" from 2-piece male luer lock Approximate set length: 102" 259 cm. approx. Approximate priming volume: 22 mL	50
72013E	20	 Infusion set, 2 needle-free valves 83" and 9" from 2-piece male luer lock Approximate set length: 102" 259 cm. approx. Approximate priming volume: 22 mL	50
72013KE	20	 Infusion set, 2 needle-free valves 68" and 9" from 2-piece male luer lock Approximate set length: 120" 305 cm. approx. Approximate priming volume: 25 mL	50
72023E	20	 Infusion set, check valve, 2 needle-free valves 83" and 9" from 2-piece male luer lock Approximate set length: 100" 254 cm. approx. Approximate priming volume: 22 mL	50

09.00 Alaris SE pump

09.05.01 Primary administration sets | SmartSite valve sets

Administration sets for use with Alaris SE pumps. All sets are latex free and contain the AccuSlide flow regulator, which replaces the roller clamp and minimizes the risk of accidental free flow. Set configurations are available with one or more of the SmartSite needle-free valves and VersaSafe split septum ports.



Model #	Drops	Description	Case quantity
72023KE	20	 Infusion set, check valve, 2 needle-free valves 56" and 9" from 2-piece male luer lock Approximate set length: 118" 300 cm. approx. Approximate priming volume: 25 mL	50
72033E	20	 Infusion set, check valve, 3 needle-free valves 83", 39" and 9" from 2-piece male luer lock Approximate set length: 101" 257 cm. approx. Approximate priming volume: 22 mL	50
72433E	60	 Infusion set, check valve, 3 needle-free valves 83", 39" and 9" from 2-piece male luer lock Approximate set length: 102" 259 cm. approx. Approximate priming volume: 23 mL	50
72623KE	60	 Infusion set, check valve, 2 needle-free valves 67" and 9" from 2-piece male luer lock Approximate set length: 118" 300 cm. approx. Approximate priming volume: 25 mL	50
30-day lead time			
72633E	20	 Infusion set, check valve, 3 needle-free valves 119", 39" and 9" from 2-piece male luer lock Approximate set length: 138" 351 cm. approx. Approximate priming volume: 28 mL	50

09.00 Alaris SE pump

09.05.01 Primary administration sets | SmartSite valve sets

Administration sets for use with Alaris SE pumps. All sets are latex free and contain the AccuSlide flow regulator, which replaces the roller clamp and minimizes the risk of accidental free flow. Set configurations are available with one or more of the SmartSite needle-free valves and VersaSafe split septum ports.



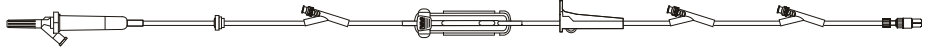
Model #	Drops	Description	Case quantity
72900E		 Infusion half set, 1 needle-free valve 9" from 2-piece male luer lock Approximate set length: 85" 216 cm. approx. Approximate priming volume: 12 mL	50
72963E	60	 Infusion set, 2 needle-free valves 83" and 9" from 2-piece male luer lock Approximate set length: 103" 262 cm. approx. Approximate priming volume: 22 mL	50
72964E	60	 Infusion set, check valve, 2 needle-free valves 83" and 9" from spin male luer lock Approximate set length: 101" 257 cm. approx. Approximate priming volume: 23 mL	50
10013182	20	 Infusion set, check valve, 2 needle-free valves 83" and 9" from 2-piece male luer lock Approximate set length: 100" 254 cm. approx. Approximate priming volume: 22 mL	 50
10942968	20	 Infusion set, check valve, 1 needle-free valve 81" from 2-piece male luer lock Approximate set length: 99" 252 cm. approx. Approximate priming volume: 22 mL	20

09.00 Alaris SE pump

09.05.01 Primary administration sets | SmartSite valve sets

Administration sets for use with Alaris SE pumps. All sets are latex free and contain the AccuSlide flow regulator, which replaces the roller clamp and minimizes the risk of accidental free flow. Set configurations are available with one or more of the SmartSite needle-free valves and VersaSafe split septum ports.



Model #	Drops	Description	Case quantity
C72102E 30-day lead time	10	 Infusion set, check valve, 3 needle-free valves 83", 42" and 9" from 2-piece male luer lock Approximate set length: 112" 284 cm. approx. Approximate priming volume: 22 mL	50
C72108E 30-day lead time	20	 Infusion set, smallbore tubing segment, 2 needle-free valves 69" and 26" from 2-piece male luer lock Approximate set length: 107" 272 cm. approx. Approximate priming volume: 14 mL	50
C72109E	20	 Infusion set, check valve, 3 needle-free valves 121", 98" and 9" from 2-piece male luer lock Approximate set length: 138" 351 cm. approx. Approximate priming volume: 28 mL	50

09.00 Alaris SE pump

09.05.02 Primary administration sets | No port sets

Administration sets for use with Alaris SE pumps. All sets are latex free and contain the AccuSlide flow regulator, which replaces the roller clamp and minimizes the risk of accidental free flow. Set configurations are available with one or more of the SmartSite needle-free valves and VersaSafe split septum ports.



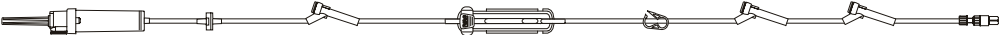
Model #	Drops	Description	Case quantity
72003	20	 Infusion set, 2-piece male luer lock Approximate set length: 86" 218 cm. approx. Approximate priming volume: 20 mL	50
72943	20	 Infusion set, smallbore tubing segment, 2-piece male luer lock Approximate set length: 100" 254 cm. approx. Approximate priming volume: 14 mL	50
10657768	20	 Infusion set, 1.2 micron (low-protein binding) filter, 2-piece male luer lock Approximate set length: 102" 259 cm. approx. Approximate priming volume: 25 mL	20
C72101	20	 Infusion set, 2-piece male luer lock Approximate set length: 136" 345 cm. approx. Approximate priming volume: 22 mL	50

09.00 Alaris SE pump

09.05.03 Primary administration sets | VersaSafe split septum sets

Administration sets for use with Alaris SE pumps. All sets are latex free and contain the AccuSlide flow regulator, which replaces the roller clamp and minimizes the risk of accidental free flow. Set configurations are available with one or more of the SmartSite needle-free valves and VersaSafe split septum ports.



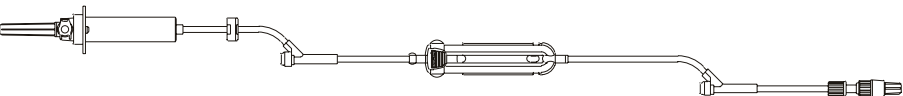
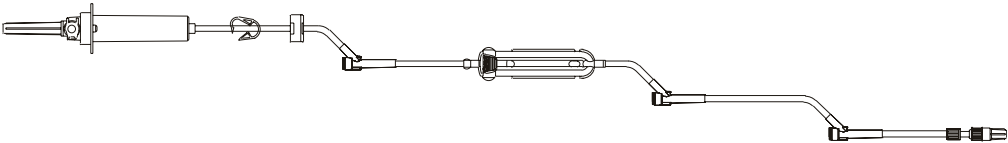
Model #	Drops	Description	Case quantity
11325479	60	 Infusion set, 2 split septum ports 83" and 9" from 2-piece male luer lock Approximate set length: 103" 262 cm. approx. Approximate priming volume: 23 mL	 50
72012M	20	 Infusion set, 2 split septum ports 56" and 9" from 2-piece male luer lock Approximate set length: 100" 254 cm. approx. Approximate priming volume: 22 mL	50
72013KM	20	 Infusion set, 2 split septum ports 55" and 6" from 2-piece male luer lock Approximate set length: 121" 307 cm. approx. Approximate priming volume: 25 mL	50
30-day lead time			
72023M	20	 Infusion set, check valve, 2 split septum ports 83" and 9" from 2-piece male luer lock Approximate set length: 102" 259 cm. approx. Approximate priming volume: 23 mL	50
72033M	20	 Infusion set, check valve, 3 split septum ports 83", 40" and 9" from 2-piece male luer lock Approximate set length: 103" 262 cm. approx. Approximate priming volume: 23 mL	50

09.00 Alaris SE pump

09.05.03 Primary administration sets | VersaSafe split septum sets

Administration sets for use with Alaris SE pumps. All sets are latex free and contain the AccuSlide flow regulator, which replaces the roller clamp and minimizes the risk of accidental free flow. Set configurations are available with one or more of the SmartSite needle-free valves and VersaSafe split septum ports.



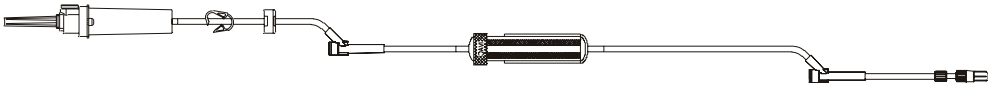
Model #	Drops	Description	Case quantity
72900M 30-day lead time		 Infusion half set, 1 split septum port 9" from 2-piece male luer lock Approximate set length: 85" 216 cm. approx. Approximate priming volume: 13 mL	50
10010721	20	 Infusion set, check valve, 3 split septum ports 71", 58" and 6" from 2-piece male luer lock Approximate set length: 102" 259 cm. approx. Approximate priming volume: 23 mL	50
11499816	20	 Infusion set, check valve, 2 split septum ports 83" and 9" from 2-piece male luer lock Approximate set length: 102" 258 cm. approx. Approximate priming volume: 22 mL	 50
11501182	20	 Infusion set, check valve, Roberts clamp, 3 split septum ports 83", 62" and 9" from 2-piece male luer lock Approximate set length: 103" 261 cm. approx. Approximate priming volume: 23 mL	 50

09.00 Alaris SE pump

09.05.03 Primary administration sets | VersaSafe split septum sets

Administration sets for use with Alaris SE pumps. All sets are latex free and contain the AccuSlide flow regulator, which replaces the roller clamp and minimizes the risk of accidental free flow. Set configurations are available with one or more of the SmartSite needle-free valves and VersaSafe split septum ports.



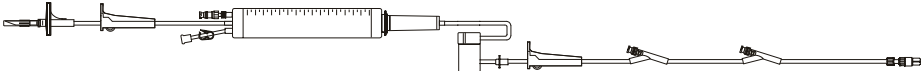
Model #	Drops	Description	Case quantity
11501183			 50
		Infusion set, check valve, 2 split septum ports 83" and 9" from 2-piece male luer lock	
		Approximate set length: 102" 258 cm. approx. Approximate priming volume: 23 mL	

10.00 MedSystem III infusion pump

10.01.01 Burettes | SmartSite valve sets

Administration sets for use with the MedSystem III infusion pump. All sets are latex free and contain an integrated mechanism to help prevent accidental free flow. Set configurations are available with SmartSite needle-free valves and standard injection ports.



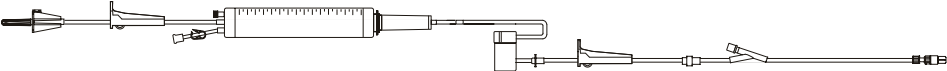
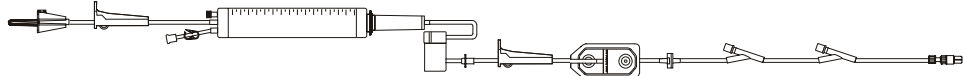
Model #	Drops	Description	Case quantity
28063E		 Infusion set, 150 mL burette (needle-free valve), smallbore tubing segment, 2 needle-free valves 71" and 9" from 2-piece male luer lock Approximate set length: 122" 310 cm. approx. Approximate priming volume: 18 mL	20

10.00 MedSystem III infusion pump

10.01.02 Burettes | Standard injection port sets

Administration sets for use with the MedSystem III infusion pump. All sets are latex free and contain an integrated mechanism to help prevent accidental free flow. Set configurations are available with SmartSite needle-free valves and standard injection ports.



Model #	Drops	Description	Case quantity
28208	60	 Infusion set, 150 mL burette (injection port), smallbore tubing segment, 1 injection port 9" from 2-piece male luer lock Approximate set length: 119" 302 cm. approx. Approximate priming volume: 18 mL	20
28216	60	 Infusion set, 150 mL burette (injection port), check valve, 0.2 micron (low-protein binding) filter, smallbore tubing segment, 2 injection ports 57" and 9" from 2-piece male luer lock Approximate set length: 122" 310 cm. approx. Approximate priming volume: 21 mL	20

10.00 MedSystem III infusion pump

10.02.01 Low sorbing tubing | SmartSite valve sets

Administration sets for use with the MedSystem III infusion pump. All sets are latex free and contain an integrated mechanism to help prevent accidental free flow. Set configurations are available with SmartSite needle-free valves and standard injection ports.



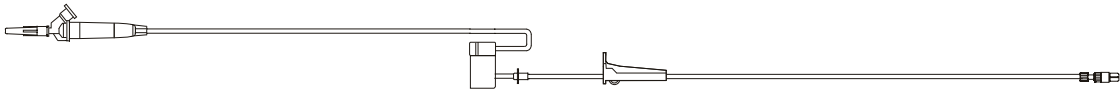
Model #	Drops	Description	Case quantity
10011433	 20	 Low sorbing infusion set, check valve, 2 needle-free valves 96" and 9" from 2-piece male luer lock Approximate set length: 115" 292 cm. approx. Approximate priming volume: 25 mL	50

10.00 MedSystem III infusion pump

10.02.02 Low sorbing tubing | No port sets

Administration sets for use with the MedSystem III infusion pump. All sets are latex free and contain an integrated mechanism to help prevent accidental free flow. Set configurations are available with SmartSite needle-free valves and standard injection ports.



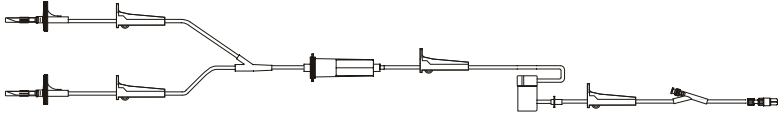
Model #	Drops	Description	Case quantity
28053		 Low sorbing infusion set, 2-piece male luer lock Approximate set length: 118" 300 cm. approx. Approximate priming volume: 24 mL	20

10.00 MedSystem III infusion pump

10.03.01.01 Filters | Blood filters | SmartSite valve sets

Administration sets for use with the MedSystem III infusion pump. All sets are latex free and contain an integrated mechanism to help prevent accidental free flow. Set configurations are available with SmartSite needle-free valves and standard injection ports.



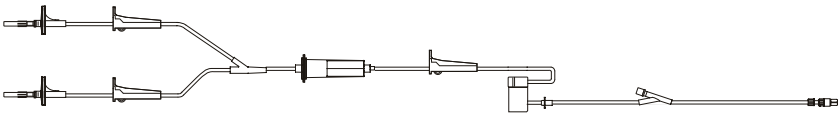
Model #	Drops	Description	Case quantity
28080E	20	 Non-vented blood set, 200 micron filter, 1 needle-free valve 9" from 2-piece male luer lock Approximate set length: 106" 269 cm. approx. Approximate priming volume: 32 mL	20

10.00 MedSystem III infusion pump

10.03.01.02 Filters | Blood filters | Standard injection port sets

Administration sets for use with the MedSystem III infusion pump. All sets are latex free and contain an integrated mechanism to help prevent accidental free flow. Set configurations are available with SmartSite needle-free valves and standard injection ports.



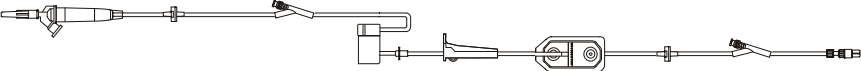
Model #	Drops	Description	Case quantity
28080	20	 Non-vented blood set, 200 micron filter, 1 injection port 9" from 2-piece male luer lock Approximate set length: 106" 269 cm. approx. Approximate priming volume: 32 mL	20

10.00 MedSystem III infusion pump

10.03.02.01 Filters | 0.2 micron filters | SmartSite valve sets

Administration sets for use with the MedSystem III infusion pump. All sets are latex free and contain an integrated mechanism to help prevent accidental free flow. Set configurations are available with SmartSite needle-free valves and standard injection ports.



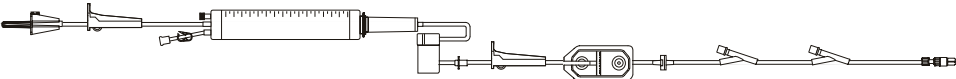
Model #	Drops	Description	Case quantity
28273E		 Infusion set, smallbore tubing segment, 2 check valves, 0.2 micron (low-protein binding) filter, 2 needle-free valves 97" and 9" from 2-piece male luer lock Approximate set length: 114" 290 cm. approx. Approximate priming volume: 23 mL	50

10.00 MedSystem III infusion pump

10.03.02.02 Filters | 0.2 micron filters | Standard injection port sets

Administration sets for use with the MedSystem III infusion pump. All sets are latex free and contain an integrated mechanism to help prevent accidental free flow. Set configurations are available with SmartSite needle-free valves and standard injection ports.



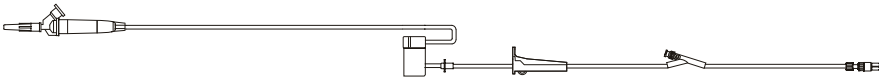
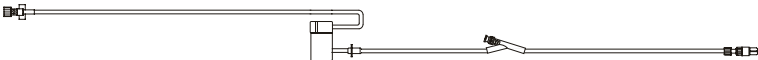
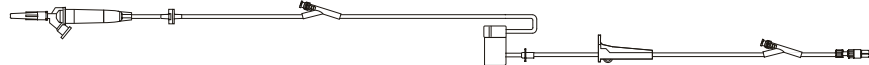
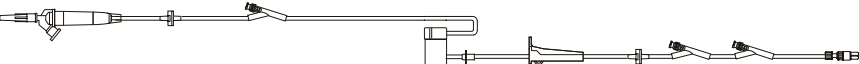
Model #	Drops	Description	Case quantity
28216	60	 Infusion set, 150 mL burette (injection port), check valve, 0.2 micron (low-protein binding) filter, smallbore tubing segment, 2 injection ports 57" and 9" from 2-piece male luer lock Approximate set length: 122" 310 cm. approx. Approximate priming volume: 21 mL	20

10.00 MedSystem III infusion pump

10.04.01 Primary administration sets | SmartSite valve sets

Administration sets for use with the MedSystem III infusion pump. All sets are latex free and contain an integrated mechanism to help prevent accidental free flow. Set configurations are available with SmartSite needle-free valves and standard injection ports.



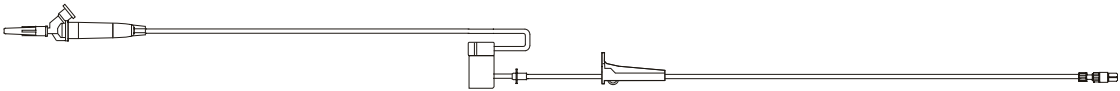
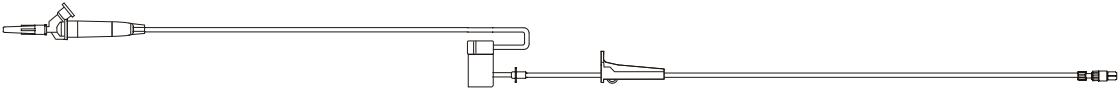
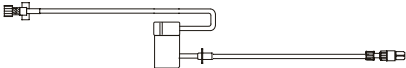
Model #	Drops	Description	Case quantity
28004E	20	 Infusion set, 1 needle-free valve 9" from 2-piece male luer lock Approximate set length: 115" 292 cm. approx. Approximate priming volume: 25 mL	50
28117E		 Infusion half set, smallbore tubing segment, 1 needle-free valve 9" from 2-piece male luer lock Approximate set length: 86" 218 cm. approx. Approximate priming volume: 6 mL	50
28293E	20	 Infusion set, check valve, 2 needle-free valves 96" and 9" from 2-piece male luer lock Approximate set length: 118" 300 cm. approx. Approximate priming volume: 25 mL	50
28493E	20	 Infusion set, 2 check valves, 3 needle-free valves 101", 44" and 9" from 2-piece male luer lock Approximate set length: 118" 300 cm. approx. Approximate priming volume: 26 mL	50

10.00 MedSystem III infusion pump

10.04.02 Primary administration sets | No port sets

Administration sets for use with the MedSystem III infusion pump. All sets are latex free and contain an integrated mechanism to help prevent accidental free flow. Set configurations are available with SmartSite needle-free valves and standard injection ports.



Model #	Drops	Description	Case quantity
28018	20	 Infusion set, smallbore tubing segment, 2-piece male luer lock Approximate set length: 117" 297 cm. approx. Approximate priming volume: 25 mL	50
28026	60	 Infusion half set, smallbore tubing segment, 2-piece male luer lock Approximate set length: 116" 295 cm. approx. Approximate priming volume: 20 mL	50
28125		 Infusion half set, smallbore tubing segment, 2-piece male luer lock Approximate set length: 16" 41 cm. approx. Approximate priming volume: 2 mL	50

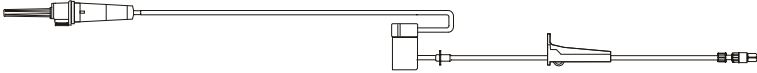
30-day lead time

10.00 MedSystem III infusion pump

10.04.02 Primary administration sets | No port sets

Administration sets for use with the MedSystem III infusion pump. All sets are latex free and contain an integrated mechanism to help prevent accidental free flow. Set configurations are available with SmartSite needle-free valves and standard injection ports.



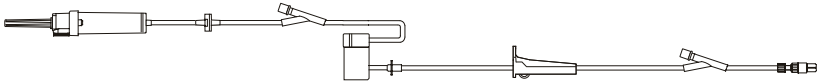
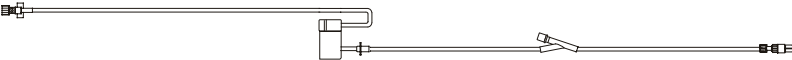
Model #	Drops	Description	Case quantity
11089893	20	Infusion set, 2-piece male luer lock Approximate set length: 116" 295 cm. approx. Approximate priming volume: 25 mL	50

10.00 MedSystem III infusion pump

10.04.03 Primary administration sets | Standard injection port sets

Administration sets for use with the MedSystem III infusion pump. All sets are latex free and contain an integrated mechanism to help prevent accidental free flow. Set configurations are available with SmartSite needle-free valves and standard injection ports.



Model #	Drops	Description	Case quantity
28035-0500	20	 Infusion set, check valve, 2 injection ports 100" and 9" from 2-piece male luer lock Approximate set length: 118" 299 cm. approx. Approximate priming volume: 26 mL	50
28117		 Infusion half set, smallbore tubing segment, 1 injection port 9" from 2-piece male luer lock Approximate set length: 86" 218 cm. approx. Approximate priming volume: 6 mL	50
28265	60	 Infusion set, smallbore tubing segment, check valve, 2 injection ports 96" and 9" from 2-piece male luer lock Approximate set length: 115" 292 cm. approx. Approximate priming volume: 20 mL	50

11.00 Secondary administration sets

11.01.01 Filters | Blood filters | No port sets

Secondary sets provide access to primary infusion sets via MaxPlus needleless connectors and y-sites, SmartSite needle-free valves and VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
10010985	20	 Non-vented secondary blood set, 200 micron filter, 2-piece male luer lock Approximate set length: 33" 83 cm. approx. Approximate priming volume: 16 mL	50
10014081	20	 Non-vented secondary blood set, blunt short spike, 200 micron filter, 2-piece male luer lock Approximate set length: 32" 81 cm. approx. Approximate priming volume: 16 mL	50
10016242	20	 Non-vented secondary blood set, 200 micron filter, 2-piece male luer lock and hanger Approximate set length: 33" 84 cm. approx. Approximate priming volume: 16 mL	50

11.00 Secondary administration sets

11.01.02 Filters | 15 micron filters | No port sets

Secondary sets provide access to primary infusion sets via MaxPlus needleless connectors and y-sites, SmartSite needle-free valves and VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
C70101		 Vented/non-vented secondary set, 15 micron filter (drip chamber), 2-piece male luer lock Approximate set length: 37" 94 cm. approx. Approximate priming volume: 12 mL	30

11.00 Secondary administration sets

11.02.01 Special applications | SmartSite bag access port sets

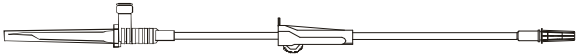
Secondary sets provide access to primary infusion sets via MaxPlus needleless connectors and y-sites, SmartSite needle-free valves and VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
10013364		 Non-vented secondary set, needle-free valve bag access port, fixed male luer lock and hanger Approximate set length: 32" 81 cm. approx. Approximate priming volume: 12 mL	 100
10014881		 Low sorbing non-vented secondary set, needle-free valve bag access port, fixed male luer lock and hanger Approximate set length: 31" 79 cm. approx. Approximate priming volume: 12 mL	 100
11610341	 30-day lead time	 Non-vented secondary set, needle-free valve bag access port, spin male luer lock and hanger Approximate set length: 31" 79 cm. approx. Approximate priming volume: 12 mL	 100

11.00 Secondary administration sets

11.02.02 Special applications | Bag access transfer sets

Secondary sets provide access to primary infusion sets via MaxPlus needleless connectors and y-sites, SmartSite needle-free valves and VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
MT6000 30-day lead time		 Secondary transfer set, vented bag access spike with cap, roller clamp, male luer slip Approximate set length: 22" 55.9 cm. approx.	 50

11.00 Secondary administration sets

11.02.03 Special applications | Syringe adapter sets

Secondary sets provide access to primary infusion sets via MaxPlus needleless connectors and y-sites, SmartSite needle-free valves and VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
10015501		Syringe adapter secondary set, microbore tubing, 2-piece male luer lock, syringe holder and hanger Approximate set length:28"71 cm. approx. Approximate priming volume:1.3 mL	50

11.00 Secondary administration sets

11.03 Secondary administration sets

Secondary sets provide access to primary infusion sets via MaxPlus needleless connectors and y-sites, SmartSite needle-free valves and VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
4900-0000	20	 Vented/non-vented secondary set, spin male luer lock and hanger Approximate set length: 41" 104 cm. approx. Approximate priming volume: 10 mL	 50
A 492	20	 Vented/non-vented secondary set, spin male luer lock and 2 hangers. Approximate set length: 41" 104 cm. approx. Approximate priming volume: 10 mL	 50
9385-0500	20	 Vented/non-vented secondary set, 18 gauge shielded needle and hanger Approximate set length: 45" 114 cm. approx. Approximate priming volume: 14 mL	30
30-day lead time			
9387-0500	20	 Vented/non-vented secondary set, 11 gauge blunt plastic cannula and hanger Approximate set length: 44" 111 cm. approx. Approximate priming volume: 14 mL	30
9987-0000	20	 Vented/non-vented secondary set, spin male luer lock and hanger Approximate set length: 43" 108 cm. approx. Approximate priming volume: 14 mL	30

11.00 Secondary administration sets

11.03 Secondary administration sets

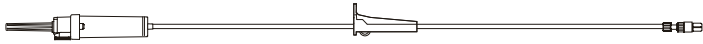
Secondary sets provide access to primary infusion sets via MaxPlus needleless connectors and y-sites, SmartSite needle-free valves and VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
70027	20	 Vented/non-vented secondary set, 18 gauge protected needle and hanger Approximate set length: 37" 94 cm. approx. Approximate priming volume: 12 mL	100
72213	20	 Vented/non-vented secondary set, 18 gauge needle and hanger Approximate set length: 33" 84 cm. approx. Approximate priming volume: 11 mL	100
72213N	20	 Vented/non-vented secondary set, spin male luer lock and hanger Approximate set length: 31" 79 cm. approx. Approximate priming volume: 11 mL	100
72215N	15	 Vented/non-vented secondary set, spin male luer lock and hanger Approximate set length: 40" 102 cm. approx. Approximate priming volume: 13 mL	 100
10016073	20	 Vented/non-vented secondary set, spin male luer lock and hanger Approximate set length: 31" 79 cm. approx. Approximate priming volume: 11 mL	 100

11.00 Secondary administration sets

11.03 Secondary administration sets

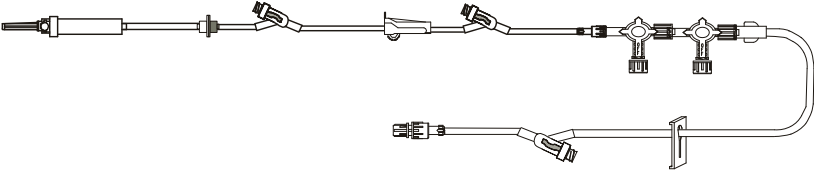
Secondary sets provide access to primary infusion sets via MaxPlus needleless connectors and y-sites, SmartSite needle-free valves and VersaSafe split septum ports.

Model #	Drops	Description		Case quantity
11448964	20	 Vented/non-vented secondary set, spin male luer lock and hanger Approximate set length: 36" 91 cm. approx. Approximate priming volume: 12 mL		100
11582774	20	 Vented/non-vented secondary set, spin male luer lock and hanger Approximate set length: 36" 91 cm. approx. Approximate priming volume: 12 mL		100
MS3500	20	 Secondary set, roller clamp, male spin lock, bag hanger Approximate set length: 36" 92 cm. approx.		100
MS3500-15	15	 Secondary set, roller clamp, male spin lock, bag hanger Approximate set length: 36" 92 cm. approx.		100

12.00 Gravity sets

12.01.01 Stopcocks and manifolds | MaxPlus needleless connectors and y-site sets

Administration sets for use as gravity sets. All sets are latex free. Some sets include MaxPlus needleless connectors and y-sites, SmartSite needle-free valves or VersaSafe split septum ports.

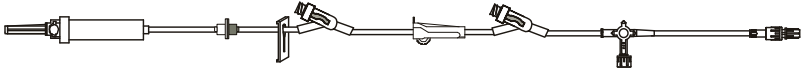
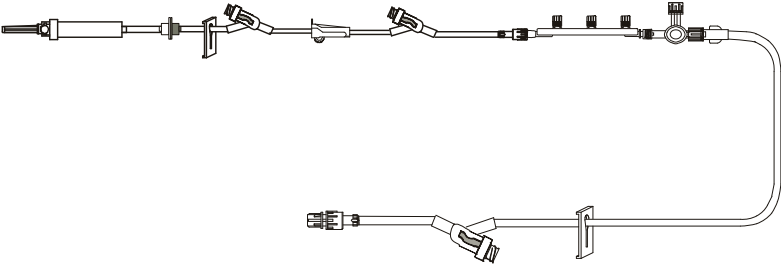
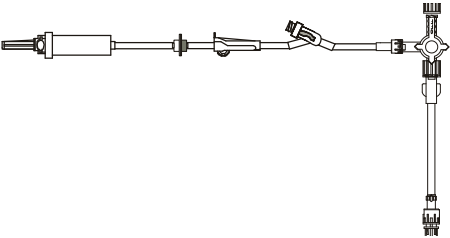
Model #	Drops	Description	Case quantity
MX4449	15	 Administration set, needleless y-site, three 4-way stopcocks ("OFF" Indicator), back check valve, roller clamp, male spin lock Approximate set length: 119" 302 cm. approx.	 25
MX4452	15	 Administration set, 3 needleless y-sites, two 4-way stopcocks ("OFF" Indicator), back check valve, roller clamp, slide clamp, male spin lock Approximate set length: 137" 348 cm. approx.	 25

30-day lead time

12.00 Gravity sets

12.01.01 Stopcocks and manifolds | MaxPlus needleless connectors and y-site sets

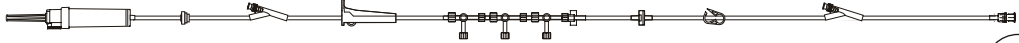
Administration sets for use as gravity sets. All sets are latex free. Some sets include MaxPlus needleless connectors and y-sites, SmartSite needle-free valves or VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
MX4410	15	 Administration set, 2 needleless y-sites, 4-way stopcock ("OFF" Indicator), back check valve, slide clamp, roller clamp, male spin lock Approximate set length: 145" 368.3 cm. approx.	 25
MX4412 30-day lead time	20	 Administration set, back check valve, 2 needleless y-sites, roller clamp, 3-port manifold, male spin lock Approximate set length: 90" 229 cm. approx.	 25
MX4436 30-day lead time	15	 Administration set, 3 needleless y-sites, removable 3-port manifold, removable 4-way stopcock ("OFF" Indicator), back check valve, roller clamp, slide clamp, male spin lock Approximate set length: 155" 393 cm. approx.	 25
MX4441	15	 Administration set, back check valve, roller clamp, needleless y-site, 4-way stopcock ("OFF" Indicator), male spin lock Approximate set length: 119" 302 cm. approx.	 50

12.00 Gravity sets

12.01.02 Stopcocks and manifolds | SmartSite valve sets

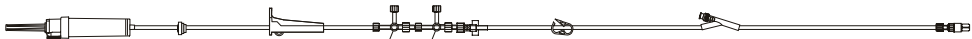
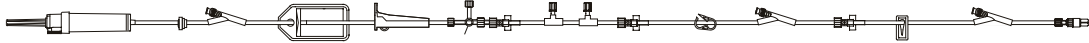
Administration sets for use as gravity sets. All sets are latex free. Some sets include MaxPlus needleless connectors and y-sites, SmartSite needle-free valves or VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
42502E 30-day lead time	10	 Gravity set, check valve, 4-way stopcock, two 1-way valves, 2 needle-free valves 107" and 9" from spin male luer lock Approximate set length: 124" 315 cm. approx. Approximate priming volume: 26 mL	 50
42503E	10	 Gravity set, check valve, 4-way stopcock, 3 needle-free valves 79", 46" and 6" from spin male luer lock Approximate set length: 104" 264 cm. approx. Approximate priming volume: 22 mL	 100
42504E	10	 Gravity set, check valve, 3-ganged 4-way stopcock, 2 needle-free valves 98" and 6" from spin male luer lock Approximate set length: 117" 297 cm. approx. Approximate priming volume: 24 mL	 50
42511E	20	 Gravity set, check valve, 4-way stopcock, two 1-way valves, 2 needle-free valves 105" and 6" from spin male luer lock Approximate set length: 121" 307 cm. approx. Approximate priming volume: 23 mL	50
42519E	20	 Gravity set, check valve, manifold with three 1-way valves, 2 needle-free valves 96" and 6" from 2-piece male luer lock Approximate set length: 119" 302 cm. approx. Approximate priming volume: 23 mL	20

12.00 Gravity sets

12.01.02 Stopcocks and manifolds | SmartSite valve sets

Administration sets for use as gravity sets. All sets are latex free. Some sets include MaxPlus needleless connectors and y-sites, SmartSite needle-free valves or VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
42521E	15	 Gravity set, check valve, 2-ganged 4-way stopcock ("OFF" Indicator), 3 needle-free valves 111", 36" and 6" from 2-piece male luer lock Approximate set length: 136" 346 cm. approx. Approximate priming volume: 28 mL	 20
10011265	10	 Gravity set, check valve, 2-ganged 3-way stopcock, 1 needle-free valve 6" from 2-piece male luer lock Approximate set length: 126" 320 cm. approx. Approximate priming volume: 24 mL	50
10011611	20	 Gravity set, check valve, 0.2 micron (low-protein binding) filter, 4-way stopcock, two 1-way valves, 3 needle-free valves 106", 14" and 5" from spin male luer lock Approximate set length: 123" 312 cm. approx. Approximate priming volume: 27 mL	20
10050186	60	 Gravity set, 2-ganged 4-way stopcock, 1 needle-free valve 8" from 2-piece male luer lock Approximate set length: 65" 165 cm. approx. Approximate priming volume: 15 mL	50

12.00 Gravity sets

12.01.02 Stopcocks and manifolds | SmartSite valve sets

Administration sets for use as gravity sets. All sets are latex free. Some sets include MaxPlus needleless connectors and y-sites, SmartSite needle-free valves or VersaSafe split septum ports.

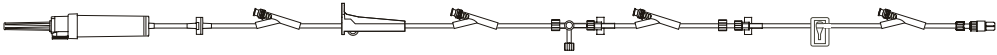
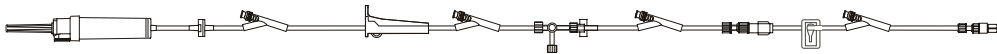
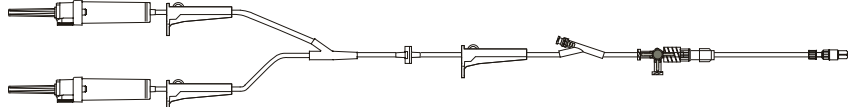
Model #	Drops	Description	Case quantity
10050187 30-day lead time	60	 Gravity set, 4-ganged 4-way stopcock, 2 needle-free valves 57" and 32" from 2-piece male luer lock Approximate set length: 74" 188 cm. approx. Approximate priming volume: 17 mL	20
10425228 30-day lead time	60	 Gravity set, check valve, 4-way stopcock ("OFF" Indicator), 4 needle-free valves 79", 45", 28" and 4" from 2-piece male luer lock Approximate set length: 99" 251 cm. approx. Approximate priming volume: 20 mL	20
10425229 30-day lead time	20	 Gravity set, check valve, 4-way stopcock ("OFF" Indicator), 4 needle-free valves 79", 45", 28" and 4" from 2-piece male luer lock Approximate set length: 98" 249 cm. approx. Approximate priming volume: 20 mL	20
10879046 30-day lead time	15	 Gravity set, check valve, 2-ganged 3-way stopcock, 1 needle-free valve 107" from 2-piece male luer lock Approximate set length: 135" 343 cm. approx. Approximate priming volume: 27 mL	50



12.00 Gravity sets

12.01.02 Stopcocks and manifolds | SmartSite valve sets

Administration sets for use as gravity sets. All sets are latex free. Some sets include MaxPlus needleless connectors and y-sites, SmartSite needle-free valves or VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
11059727 30-day lead time	60	 Gravity set, check valve, 4-way stopcock ("OFF" Indicator), 4 needle-free valves 84", 50", 31" and 6" from 2-piece male luer lock Approximate set length: 101" 257 cm. approx. Approximate priming volume: 21 mL	20
11059728 30-day lead time	20	 Gravity set, check valve, 4-way stopcock ("OFF" Indicator), 4 needle-free valves 84", 50", 31" and 6" from 2-piece male luer lock Approximate set length: 102" 259 cm. approx. Approximate priming volume: 21 mL	20
11627697 30-day lead time	10	 Gravity set, 2 drip chambers, check valve, 4-way stopcock ("OFF" Indicator), 1 needle-free valve 49" from 2-piece male luer lock Approximate set length: 131" 333 cm. approx. Approximate priming volume: 36 mL	20
C42006E	10	 Gravity set, check valve, 2-ganged 4-way stopcock ("OFF" Indicator), 3 needle-free valves 113", 35" and 6" from 2-piece male luer lock Approximate set length: 139" 352 cm. approx. Approximate priming volume: 26 mL	20

12.00 Gravity sets

12.01.02 Stopcocks and manifolds | SmartSite valve sets

Administration sets for use as gravity sets. All sets are latex free. Some sets include MaxPlus needleless connectors and y-sites, SmartSite needle-free valves or VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
C42010E	10	 Gravity set, check valve, 4-way stopcock ("OFF" Indicator), 3 needle-free valves 118", 40" and 6" from 2-piece fixed male luer lock Approximate set length: 141" 358 cm. approx. Approximate priming volume: 25 mL	20
C42012E	10	 Gravity set, 2 check valves, three 1-way valves, 2 needle-free valves 88" and 7" from spin male luer lock Approximate set length: 104" 264 cm. approx. Approximate priming volume: 23 mL	50
C42013E	20	 Gravity set, check valve, two 1-way valves, 2 split septum ports 114" and 59" from spin male luer lock, 1 needle-free valve 6" from spin male luer lock Approximate set length: 131" 333 cm. approx. Approximate priming volume: 26 mL	50

12.00 Gravity sets

12.01.03 Stopcocks and manifolds | VersaSafe split septum sets

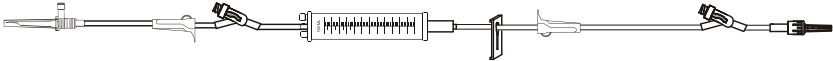
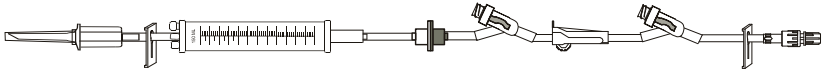
Administration sets for use as gravity sets. All sets are latex free. Some sets include MaxPlus needleless connectors and y-sites, SmartSite needle-free valves or VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
11147994	10	 Gravity set, check valve, 3-way stopcock, 2 split septum ports 113" and 44" from 2-piece male luer lock Approximate set length: 134" 340 cm. approx. Approximate priming volume: 27 mL 	20
C42013E	20	 Gravity set, check valve, two 1-way valves, 2 split septum ports 114" and 59" from spin male luer lock, 1 needle-free valve 6" from spin male luer lock Approximate set length: 131" 333 cm. approx. Approximate priming volume: 26 mL	50

12.00 Gravity sets

12.02.01 Burettes | MaxPlus needleless connectors and y-site sets

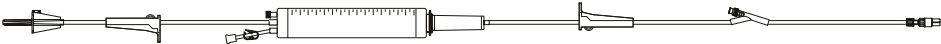
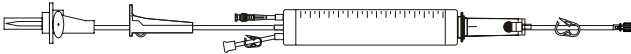
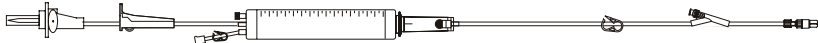
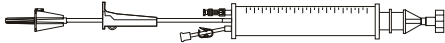
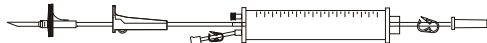
Administration sets for use as gravity sets. All sets are latex free. Some sets include MaxPlus needleless connectors and y-sites, SmartSite needle-free valves or VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
MX4416 30-day lead time		Administration set, 150 mL burette, 2 needleless y-sites, 2 roller clamps, slide clamp, male luer lock Approximate set length: 84" 213.3 cm. approx.	25
MX9407 30-day lead time		Administration set, 150 mL burette, 2 needleless y-sites, back check valve, roller clamp, slide clamp, male spin lock Approximate set length: 90" 234 cm. approx.	25

12.00 Gravity sets

12.02.02 Burettes | SmartSite valve sets

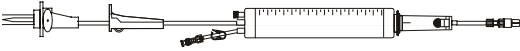
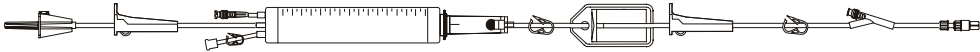
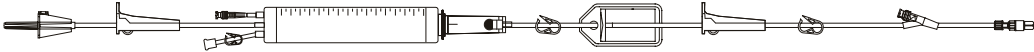
Administration sets for use as gravity sets. All sets are latex free. Some sets include MaxPlus needleless connectors and y-sites, SmartSite needle-free valves or VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
42163E 30-day lead time	60	 Gravity set, 150 mL burette (needle-free valve), 1 needle-free valve 10" from 2-piece male luer lock Approximate set length: 116" 295 cm. approx. Approximate priming volume: 22 mL	10
82005E 30-day lead time	20	 150 mL add-on burette set (needle-free valve), ball valve in drip chamber, pigtail spike adapter connector. Approximate set length: 26" 66 cm. approx.	20
82104E 30-day lead time	60	 Gravity set, 150 mL burette (needle-free valve), ball valve in drip chamber, 1 needle-free valve 10" from 2-piece male luer lock Approximate set length: 100" 254 cm. approx. Approximate priming volume: 18 mL	20
82113E		 150 mL add-on burette (needle-free valve), spike adapter. Approximate set length: 21" 53 cm. approx.	DEHP 20
82114E		 150 mL add-on burette (needle-free valve), pigtail spike adapter. Approximate set length: 24" 61 cm. approx.	DEHP 20

12.00 Gravity sets

12.02.02 Burettes | SmartSite valve sets

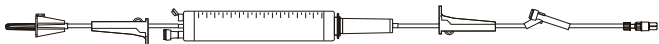
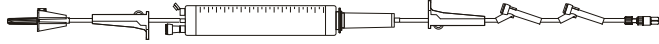
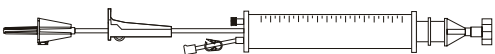
Administration sets for use as gravity sets. All sets are latex free. Some sets include MaxPlus needleless connectors and y-sites, SmartSite needle-free valves or VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
82115E	20	 150 mL add-on burette (needle-free valve), ball valve in drip chamber, pigtail spike adapter. Approximate set length: 25" 64 cm. approx.	20
10011087	60	 Gravity set, 150 mL burette (needle-free valve), ball valve in drip chamber, smallbore tubing segment, 2 needle-free valves 47" and 4" from male luer lock, 4 split septum ports 67", 49", 32" and 6" from spin male luer lock Approximate set length: 127" 323 cm. approx. Approximate priming volume: 18 mL	10
10012564	60	 Gravity set, 150 mL burette (needle-free valve), ball valve in drip chamber, 2 needle-free valves 43" and 6" from 2-piece male luer lock Approximate set length: 105" 265 cm. approx. Approximate priming volume: 18 mL	10
10037031	60	 Gravity set, 150 mL burette (needle-free valve), ball valve in drip chamber, 1.2 micron (low-protein binding) filter, 1 needle free valve 10" from 2 piece male luer lock Approximate set length: 100" 254 cm. approx. Approximate priming volume: 21 mL	20
10037032	20	 Gravity set, 150 mL burette (needle-free valve), ball valve in drip chamber, 1.2 micron (low-protein binding) filter, 1 needle free valve 10" from 2 piece male luer lock Approximate set length: 100" 254 cm. approx. Approximate priming volume: 21 mL	20

12.00 Gravity sets

12.02.03 Burettes | VersaSafe split septum sets

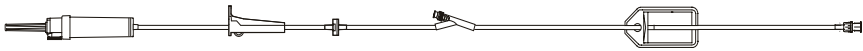
Administration sets for use as gravity sets. All sets are latex free. Some sets include MaxPlus needleless connectors and y-sites, SmartSite needle-free valves or VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
10807857	60	 Gravity set, 150 mL burette (split septum port), 1 split septum port 6" from 2-piece male luer lock Approximate set length: 116" 295 cm. approx. Approximate priming volume: 25 mL	 10
10849550	60	 Gravity set, 150 mL burette (split septum port), 2 split septum ports 25" and 10" from 2-piece male luer lock Approximate set length: 100" 254 cm. approx. Approximate priming volume: 21 mL	 10
11607532		 150 mL add-on burette (split septum port), spike adapter. Approximate set length: 21" 53 cm. approx. Approximate priming volume: 35 mL	 20

12.00 Gravity sets

12.03 Low sorbing tubing | SmartSite valve sets

Administration sets for use as gravity sets. All sets are latex free. Some sets include MaxPlus needleless connectors and y-sites, SmartSite needle-free valves or VersaSafe split septum ports.

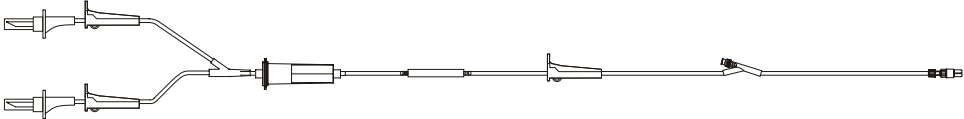
Model #	Drops	Description	Case quantity
10015104	10	 Low sorbing gravity set, check valve, 0.2 micron (low-protein binding) filter, 1 needle-free valve 82" from fixed male luer lock Approximate set length: 106" 270 cm. approx. Approximate priming volume: 25 mL	 50

30-day lead time

12.00 Gravity sets

12.04.01.01 Filters | Blood filters | SmartSite valve sets

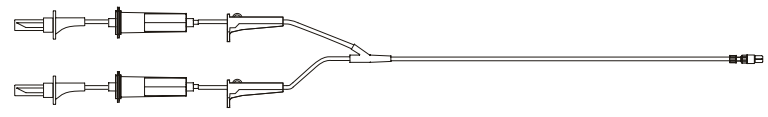
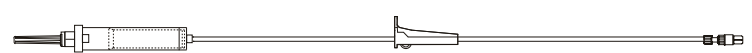
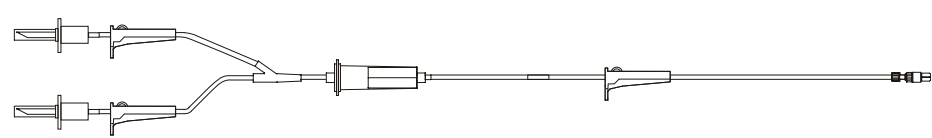
Administration sets for use as gravity sets. All sets are latex free. Some sets include MaxPlus needleless connectors and y-sites, SmartSite needle-free valves or VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
42081E	15	 Non-vented gravity blood set, 180 micron blood filter, 1 needle-free valve 9" from 2-piece male luer lock Approximate set length: 104" 264 cm. approx. Approximate priming volume: 30 mL	 20
42082E	20	 Non-vented gravity blood set, blunt short spike, 200 micron filter, 2 needle-free valves 50" and 10" from 2-piece male luer lock Approximate set length: 108" 274 cm. approx. Approximate priming volume: 29 mL	20
10010903	15	 Non-vented gravity blood set with pressure pump, 180 micron filter, 1 needle-free valve 9" from 2-piece male luer lock Approximate set length: 105" 267 cm. approx. Approximate priming volume: 67 mL	20

12.00 Gravity sets

12.04.01.02 Filters | Blood filters | No port sets

Administration sets for use as gravity sets. All sets are latex free. Some sets include MaxPlus needleless connectors and y-sites, SmartSite needle-free valves or VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
42085	20	 Non-vented add-on blood set, single spike, 200 micron filter, spike adapter. Approximate set length: 11" 28 cm. approx. Approximate priming volume: 14 mL	100
30-day lead time			
10015492	15	 Non-vented gravity blood set, 180 micron filter, 2-piece male luer lock Approximate set length: 103" 263 cm. approx. Approximate priming volume: 29 mL	10
10137407	20	 Non-vented gravity blood set, two 200 micron filters, 2-piece male luer lock Approximate set length: 34" 86 cm. approx. Approximate priming volume: 32 mL	20
30-day lead time			
10387509	20	 Non-vented gravity blood set, single spike, 200 micron filter, 2-piece male luer lock Approximate set length: 105" 267 cm. approx. Approximate priming volume: 27 mL	50
10844551	20	 Non-vented gravity blood set, single spike, 200 micron filter, 2-piece male luer lock Approximate set length: 55" 140 cm. approx. Approximate priming volume: 19 mL	20
10869591	15	 Non-vented gravity blood set, 180 micron filter, 2-piece male luer lock Approximate set length: 103" 262 cm. approx. Approximate priming volume: 29 mL	10
30-day lead time			



12.00 Gravity sets

12.04.02.01 Filters | 0.2 micron filters | SmartSite valve sets

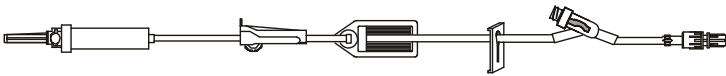
Administration sets for use as gravity sets. All sets are latex free. Some sets include MaxPlus needleless connectors and y-sites, SmartSite needle-free valves or VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
42933E	20	 Gravity set, check valve, 0.2 micron (low-protein binding) filter, 2 needle-free valves 42" and 10" from 2-piece male luer lock Approximate set length: 105" 267 cm. approx. Approximate priming volume: 24 mL	50
10011611	20	 Gravity set, check valve, 0.2 micron (low-protein binding) filter, 4-way stopcock, two 1-way valves, 3 needle-free valves 106", 14" and 5" from spin male luer lock Approximate set length: 123" 312 cm. approx. Approximate priming volume: 27 mL	20
10015104	10	 Low sorbing gravity set, check valve, 0.2 micron (low-protein binding) filter, 1 needle-free valve 82" from fixed male luer lock Approximate set length: 106" 270 cm. approx. Approximate priming volume: 25 mL	 50

12.00 Gravity sets

12.04.02.02 Filters | 0.2 micron filters | MaxPlus needleless connector and y-site sets

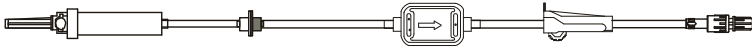
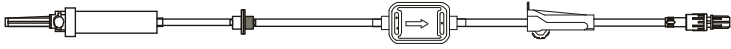
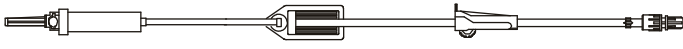
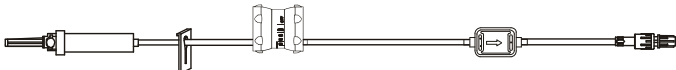
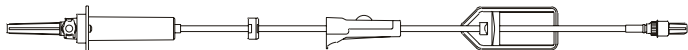
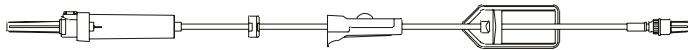
Administration sets for use as gravity sets. All sets are latex free. Some sets include MaxPlus needleless connectors and y-sites, SmartSite needle-free valves or VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
MS3542 30-day lead time	15	 Administration set, needleless y-site, 0.2 micron filter, roller clamp, slide clamp, male spin lock Approximate set length: 100" 254 cm. approx.	 50

12.00 Gravity sets

12.04.02.03 Filters | 0.2 micron filters | No port sets

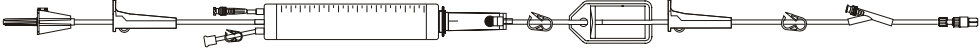
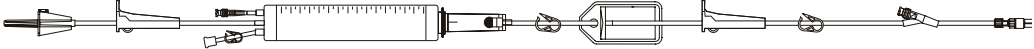
Administration sets for use as gravity sets. All sets are latex free. Some sets include MaxPlus needleless connectors and y-sites, SmartSite needle-free valves or VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
MA3137 30-day lead time	20	 Administration set, 0.2 micron filter, check valve, roller clamp, male spin lock Approximate set length: 47" 119 cm. approx.	 50
MA3138 30-day lead time	60	 Administration set, 0.2 micron filter, check valve, roller clamp, male spin lock Approximate set length: 47" 119 cm. approx.	 50
MB3110 30-day lead time	10	 Administration set, 0.2 micron filter, roller clamp, male spin lock Approximate set length: 95" 241.3 cm. approx.	 50
MFS120 30-day lead time	15	 Administration set, flow regulator, 0.2 micron filter, slide clamp, male spin lock Approximate set length: 84" 213 cm. approx.	 50
42002-07 30-day lead time	20	 Vented/non vented gravity set, 0.2 micron filter, check valve, spin male luer lock Approximate set length: 47" 119 cm. approx. Approximate priming volume: 16 mL	 100
42006-07 30-day lead time	60	 Vented/non vented gravity set, 0.2 micron filter, check valve, spin male luer lock Approximate set length: 47" 119 cm. approx. Approximate priming volume: 16 mL	 100

12.00 Gravity sets

12.04.03 Filters | 1.2 micron filters

Administration sets for use as gravity sets. All sets are latex free. Some sets include MaxPlus needleless connectors and y-sites, SmartSite needle-free valves or VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
10037031	60	 Gravity set, 150 mL burette (needle-free valve), ball valve in drip chamber, 1.2 micron (low-protein binding) filter, 1 needle-free valve 10" from 2-piece male luer lock Approximate set length: 100" 254 cm. approx. Approximate priming volume: 21 mL	20
10037032	20	 Gravity set, 150 mL burette (needle-free valve), ball valve in drip chamber, 1.2 micron (low-protein binding) filter, 1 needle-free valve 10" from 2-piece male luer lock Approximate set length: 100" 254 cm. approx. Approximate priming volume: 21 mL	20

30-day lead time

12.00 Gravity sets

12.04.04 Filters | 15 micron filters

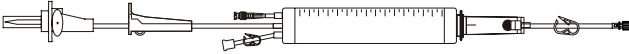
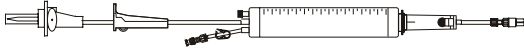
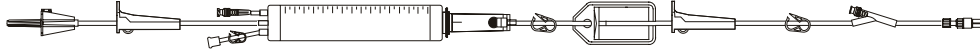
Administration sets for use as gravity sets. All sets are latex free. Some sets include MaxPlus needleless connectors and y-sites, SmartSite needle-free valves or VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
C42001	20	 Gravity set, 15 micron filter (drip chamber), 2-piece male luer lock	100
30-day lead time		Approximate set length: 92" 233 cm. approx. Approximate priming volume: 19 mL	

12.00 Gravity sets

12.05.01 Special applications | Ball valve sets

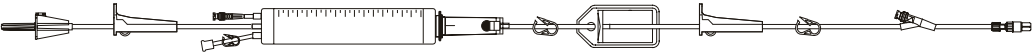
Administration sets for use as gravity sets. All sets are latex free. Some sets include MaxPlus needleless connectors and y-sites, SmartSite needle-free valves or VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
82005E 30-day lead time	20	 150 mL add-on burette set (needle-free valve), ball valve in drip chamber, pigtail spike adapter connector. Approximate set length: 26" 66 cm. approx.	20
82115E	20	 150 mL add-on burette (needle-free valve), ball valve in drip chamber, pigtail spike adapter. Approximate set length: 25" 64 cm. approx.	20
10011087	60	 Gravity set, 150 mL burette (needle-free valve), ball valve in drip chamber, smallbore tubing segment, 2 needle-free valves 47" and 4" from spin male luer lock, 4 split septum ports 67", 49" 32" and 6" from spin male luer lock Approximate set length: 127" 323 cm. approx. Approximate priming volume: 18 mL	10
10012564 30-day lead time	60	 Gravity set, 150 mL burette (needle-free valve), ball valve in drip chamber, 2 needle-free valves 43" and 6" from 2-piece male luer lock Approximate set length: 105" 265 cm. approx. Approximate priming volume: 18 mL	10
10037031	60	 Gravity set, 150 mL burette (needle-free valve), ball valve in drip chamber, 1.2 micron (low-protein binding) filter, 1 needle-free valve 10" from 2-piece male luer lock Approximate set length: 100" 254 cm. approx. Approximate priming volume: 21 mL	20

12.00 Gravity sets

12.05.01 Special applications | Ball valve sets

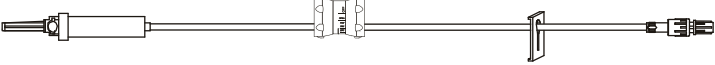
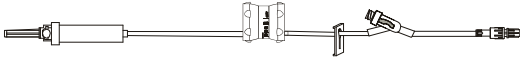
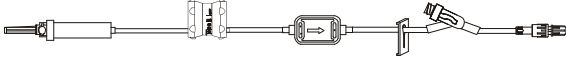
Administration sets for use as gravity sets. All sets are latex free. Some sets include MaxPlus needleless connectors and y-sites, SmartSite needle-free valves or VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
10037032	20	 Gravity set, 150 mL burette (needle-free valve), ball valve in drip chamber, 1.2 micron (low-protein binding) filter, 1 needle-free valve 10" from 2-piece male luer lock Approximate set length: 100" 254 cm. approx. Approximate priming volume: 21 mL	20

12.00 Gravity sets

12.05.02 Special applications | Flow regulator sets

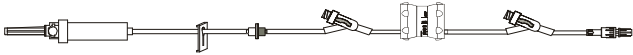
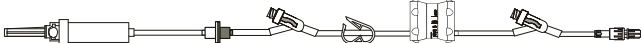
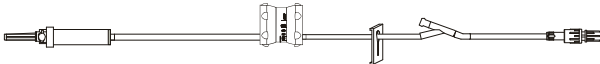
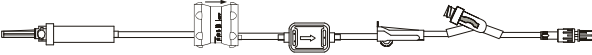
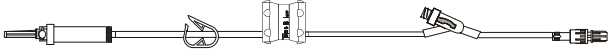
Administration sets for use as gravity sets. All sets are latex free. Some sets include MaxPlus needleless connectors and y-sites, SmartSite needle-free valves, injection sites or VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
MFS100	10		 50
		Administration set, flow regulator, slide clamp, male spin clamp Approximate set length: 100" 254 cm. approx.	
MFS102	10		 50
		Administration set, flow regulator, slide clamp, needleless y-site, male spin lock Approximate set length: 100" 254 cm. approx.	
MFS120	15		 50
		Administration set, flow regulator, 0.2 micron filter, slide clamp, male spin lock Approximate set length: 84" 213 cm. approx.	
MFS121	15		 50
		Administration set, flow regulator, 0.2 micron filter, slide clamp, needleless y-site, male spin lock Approximate set length: 90" 228.6 cm. approx.	
MFS124	15		 50
		Administration set, flow regulator, 2 needleless y-sites, slide clamp, back check valve, male spin lock Approximate set length: 75" 191 cm. approx.	

12.00 Gravity sets

12.05.02 Special applications | Flow regulator sets

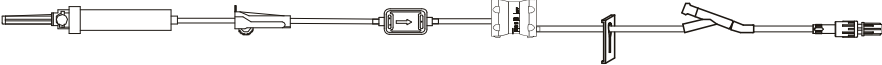
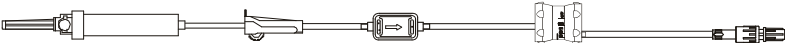
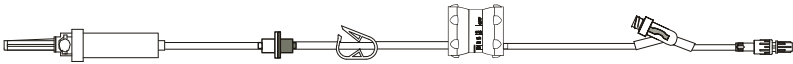
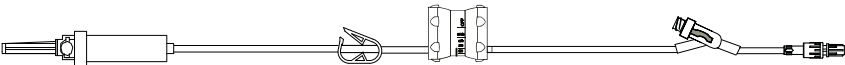
Administration sets for use as gravity sets. All sets are latex free. Some sets include MaxPlus needleless connectors and y-sites, SmartSite needle-free valves, injection sites or VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
MFS125	20	 Administration set, flow regulator, slide clamp, injection site, male spin lock Approximate set length: 100" 254 cm. approx.	 50
MFS132	20	 Administration set, flow controller, 2 needleless y-sites, back check valve, pinch clamp, male spin lock Approximate set length: 83" 211 cm. approx.	 50
MFS133	20	 Administration set, flow regulator, slide clamp, injection site, male spin lock Approximate set length: 100" 254 cm. approx.	 50
MFS134	15	 Administration set, flow controller, 0.2 micron filter, needleless y-site, roller clamp, male spin lock Approximate set length: 90" 229 cm. approx.	 25
MFS135	20	 Administration set, pinch clamp, flow regulator, 15 micron filter, needleless y-site Approximate set length: 86" 218 cm. approx.	 50

12.00 Gravity sets

12.05.02 Special applications | Flow regulator sets

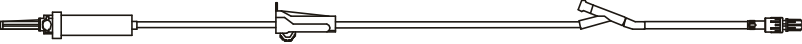
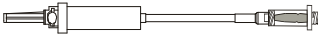
Administration sets for use as gravity sets. All sets are latex free. Some sets include MaxPlus needleless connectors and y-sites, SmartSite needle-free valves, injection sites or VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
MFS136	20	 Administration set, flow regulator, 0.2 micron filter, injection site Approximate set length: 64" 162.4 cm. approx.	 50
MFS137	10	 Administration set, flow regulator, 0.2 micron filter Approximate set length: 74" 188 cm. approx.	 50
MFS139	20	 Gravity set, check valve, flow regulator, 1 needleless y-site 5" from 2-piece male luer lock Approximate set length: 84" 216 cm. approx. Approximate priming volume: 13 mL	 50
MFS140	20	 Gravity set, flow regulator, 1 needleless y-site 5" from 2-piece male luer lock Approximate set length: 84" 216 cm. approx. Approximate priming volume: 13 mL	 50

12.00 Gravity sets

12.06.01 Primary administration sets | MaxPlus needleless connectors and y-site sets

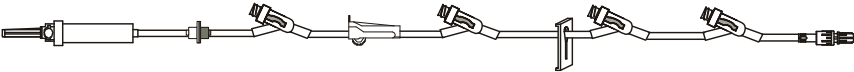
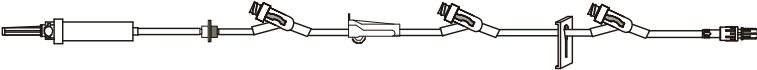
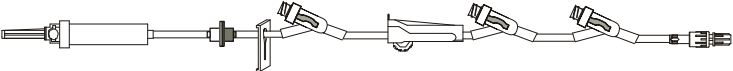
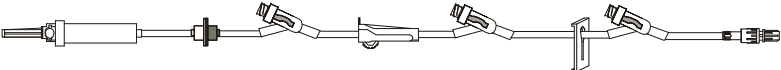
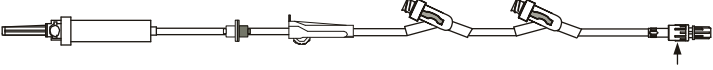
Administration sets for use as gravity sets. All sets are latex free. Some sets include MaxPlus needleless connectors and y-sites, SmartSite needle-free valves or VersaSafe split septum ports.

Model #	Drops	Description	Case quantity	
MA3090	10			50
		Administration set, injection site, male spin lock Approximate set length: 83" 83 cm. approx.		
MB3105	10			50
		Administration set, 2 needleless y-sites, roller clamp, male spin lock Approximate set length: 96" 243.8 cm. approx.		
MB3107	10			50
		Administration set, needleless y-site, roller clamp, male spin lock Approximate set length: 95" 241.3 cm. approx.		
MC3102	15			50
		Administration set, needleless y-site, roller clamp, male spin lock Approximate set length: 84" 213 cm. approx.		
MS3101	15			50
		Administration set, 2 needleless y-sites, back check valve, slide clamp, roller clamp, male spin lock Approximate set length: 110" 279.4 cm. approx.		
MT6004	15			50
		Administration set, standard needleless connector Approximate set length: 8.5" 21.59 cm. approx.		

12.00 Gravity sets

12.06.01 Primary administration sets | MaxPlus needleless connectors and y-site sets

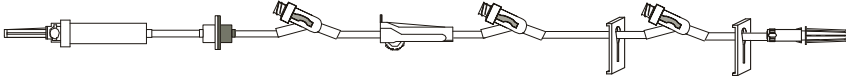
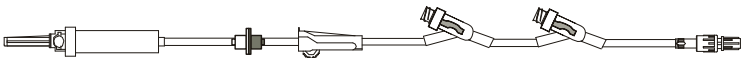
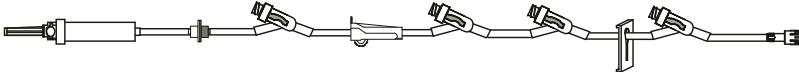
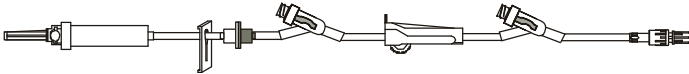
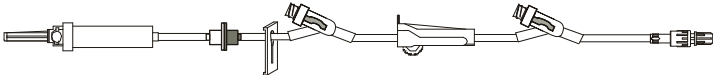
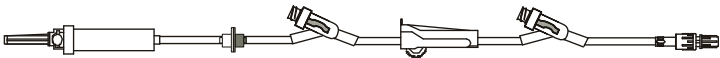
Administration sets for use as gravity sets. All sets are latex free. Some sets include MaxPlus needleless connectors and y-sites, SmartSite needle-free valves or VersaSafe split septum ports.

Model #	Drops	Description		Case quantity
MX4443	15	 Administration set, 4 needleless y-sites, back check valve, roller clamp, slide clamp, male spin lock Approximate set length: 170" 432 cm. approx.		25
MX9107	10	 Administration set, needleless y-site, roller clamp, male spin lock Approximate set length: 90" 229 cm. approx.		50
MX9128	15	 Administration set, 3 needleless y-sites, back check valve, roller clamp, slide clamp, male spin lock Approximate set length: 112" 284.5 cm. approx.		50
MX9412	10	 Administration set, 3 needleless y-sites, back check valve, slide clamp, roller clamp, male spin lock Approximate set length: 110" 279.4 cm. approx.		50
MX9414	60	 Administration set, 3 needleless y-sites, back check valve, roller clamp, slide clamp, male spin lock Approximate set length: 112" 284.4 cm. approx.		50
MX9416	15	 Administration set, 2 needleless y-sites, back check valve, roller clamp, male spin lock Approximate set length: 110" 279 cm. approx.		50

12.00 Gravity sets

12.06.01 Primary administration sets | MaxPlus needleless connectors and y-site sets

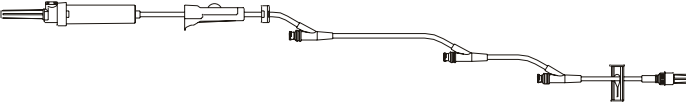
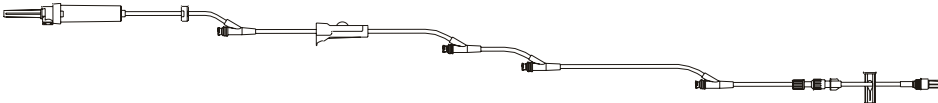
Administration sets for use as gravity sets. All sets are latex free. Some sets include MaxPlus needleless connectors and y-sites, SmartSite needle-free valves or VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
MX9417 30-day lead time	15	 Administration set, 3 needleless y-sites, back check valve, roller clamp, 2 slide clamps, male luer slip Approximate set length: 112" 284.5 cm. approx.	 50
MX9424 30-day lead time	15	 Administration set, 2 needleless y-sites, back check valve, roller clamp, male spin lock Approximate set length: 98" 248.9 cm. approx.	 50
MX9433	15	 Administration set, 4 needleless y-sites, back check valve, roller clamp, slide clamp, male spin lock Approximate set length: 127" 322 cm. approx.	 50
MX9963	60	 Administration set, 2 needleless y-sites, back check valve, slide clamp, roller clamp, male spin lock Approximate set length: 106" 269.2 cm. approx.	 50
MX9968	15	 Administration set, 2 needleless connectors, back check valve, slide clamp, roller clamp, male spin lock Approximate set length: 110" 279 cm. approx.	 50
MX9998	20	 Administration set, 2 needleless y-sites, back check valve, slide clamp, roller clamp, male spin lock Approximate set length: 82" 208.3 cm. approx.	 50

12.00 Gravity sets

12.06.02 Primary administration sets | SmartSite valve sets

Administration sets for use as gravity sets. All sets are latex free. Some sets include MaxPlus needleless connectors and y-sites, SmartSite needle-free valves or VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
41134E	10	 Gravity set, check valve, 3 needle-free valves 83", 23" and 4" from 2-piece male luer lock Approximate set length: 104" 264 cm. approx. Approximate priming volume: 21 mL	100
41135E	10	 Gravity set, 3 needle-free valves 55", 25" and 12" from spin male luer lock Approximate set length: 88" 224 cm. approx. Approximate priming volume: 18 mL	100
41137E	10	 Gravity set, 4 needle-free valves 118", 60", 52" and 15" from 2-piece male luer lock Approximate set length: 136" 345 cm. approx. Approximate priming volume: 27 mL	DEHP 50
41173E	20	 Gravity set, 1 needle-free valve 9" from 2-piece male luer lock Approximate set length: 104" 264 cm. approx. Approximate priming volume: 23 mL	DEHP 100
41177E	10	 Gravity set, check valve, 2 needle-free valves 83" and 4" from fixed male luer lock Approximate set length: 105" 267 cm. approx. Approximate priming volume: 21 mL	100
42133E	60	 Gravity set, 2 needle-free valves 87" and 10" from 2-piece male luer lock Approximate set length: 104" 264 cm. approx. Approximate priming volume: 21 mL	100

12.00 Gravity sets

12.06.02 Primary administration sets | SmartSite valve sets

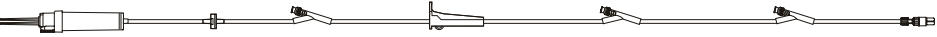
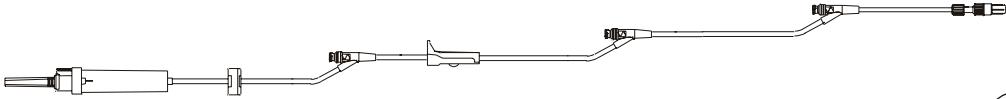
Administration sets for use as gravity sets. All sets are latex free. Some sets include MaxPlus needleless connectors and y-sites, SmartSite needle-free valves or VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
42193E-0500	20	 Gravity set, check valve, 1 needle-free valve 10" from 2-piece male luer lock Approximate set length: 104" 264 cm. approx. Approximate priming volume: 20 mL	100
42233E	60	 Gravity set, check valve, 2 needle-free valves 87" and 10" from 2-piece male luer lock Approximate set length: 104" 264 cm. approx. Approximate priming volume: 21 mL	100
42273E	20	 Gravity set, 2 needle-free valves 87" and 10" from 2-piece male luer lock Approximate set length: 104" 264 cm. approx. Approximate priming volume: 21 mL	100
42274E	10	 Gravity set, check valve, 1 needle-free valve 84" from male luer lock, 1 split septum port 6" from 2-piece male luer lock Approximate set length: 104" 264 cm. approx. Approximate priming volume: 21 mL	50
42293E	20	 Gravity set, check valve, 2 needle-free valves 84" and 10" from 2-piece male luer lock Approximate set length: 104" 264 cm. approx. Approximate priming volume: 21 mL	100

12.00 Gravity sets

12.06.02 Primary administration sets | SmartSite valve sets

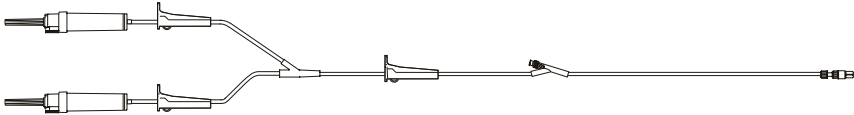
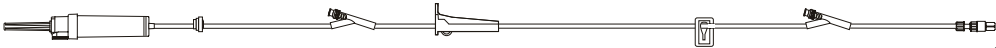
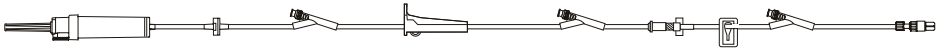
Administration sets for use as gravity sets. All sets are latex free. Some sets include MaxPlus needleless connectors and y-sites, SmartSite needle-free valves or VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
42433E	60	 Gravity set, check valve, 3 needle-free valves 88", 42" and 10" from 2-piece male luer lock Approximate set length: 105" 267 cm. approx. Approximate priming volume: 21 mL	100
42434E	60	 Gravity set, check valve, 3 needle-free valves 87", 42" and 9" from 2-piece male luer lock Approximate set length: 104" 264 cm. approx. Approximate priming volume: 23 mL	 100
42493E	20	 Gravity set, check valve, 3 needle-free valves 88", 42" and 10" from 2-piece male luer lock Approximate set length: 105" 267 cm. approx. Approximate priming volume: 21 mL	100
47177E	10	 Gravity set, check valve, 2 needle-free valves 78" and 4" from fixed male luer lock Approximate set length: 104" 264 cm. approx. Approximate priming volume: 23 mL	 100
47233E	60	 Gravity set, check valve, 2 needle-free valves 85" and 9" from 2-piece male luer lock Approximate set length: 104" 264 cm. approx. Approximate priming volume: 22 mL	 100
30-day lead time			
10010741	10	 Gravity set, 2 needle-free valves 26" and 6" from 2-piece male luer lock Approximate set length: 76" 193 cm. approx. Approximate priming volume: 17 mL	100
30-day lead time			

12.00 Gravity sets

12.06.02 Primary administration sets | SmartSite valve sets

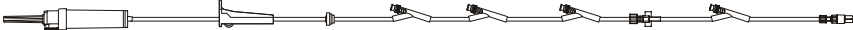
Administration sets for use as gravity sets. All sets are latex free. Some sets include MaxPlus needleless connectors and y-sites, SmartSite needle-free valves or VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
10015171 30-day lead time	20	 Gravity set, 2 drip chambers, 1 needle-free valve 6" from 2-piece male luer lock Approximate set length: 105" 265 cm. approx. Approximate priming volume: 31 mL	20
10053857	15	 Gravity set, check valve, 2 needle-free valves 74" and 7" from 2-piece male luer lock Approximate set length: 101" 257 cm. approx. Approximate priming volume: 22 mL	DEHP 100
10073124	10	 Gravity set, check valve, 3 needle-free valves 88", 42" and 6" from 2-piece male luer lock Approximate set length: 110" 279 cm. approx. Approximate priming volume: 22 mL	50
10802688	10	 Gravity set, check valve, 3 needle-free valves 41", 36" and 6" from 2-piece male luer lock Approximate set length: 107" 272 cm. approx. Approximate priming volume: 23 mL	DEHP 100
11609826 30-day lead time	20	 Gravity set, check valve, 3 needle-free valves 94", 16" and 4" from spin male luer lock Approximate set length: 111" 282 cm. approx. Approximate priming volume: 22 mL	50

12.00 Gravity sets

12.06.02 Primary administration sets | SmartSite valve sets

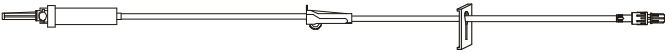
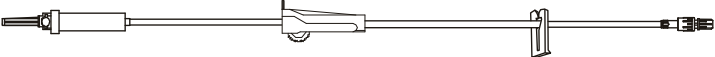
Administration sets for use as gravity sets. All sets are latex free. Some sets include MaxPlus needleless connectors and y-sites, SmartSite needle-free valves or VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
11619427 30-day lead time	10	 Gravity set, check valve, 4 needle-free valves 118", 45", 32" and 7" from spin male luer lock Approximate set length: 140" 356 cm. approx. Approximate priming volume: 25 mL	20
C42002E 30-day lead time	20	 Gravity set, check valve, 2 needle-free valves 104" and 26" from fixed male luer lock Approximate set length: 129" 328 cm. approx. Approximate priming volume: 24 mL	100
C42007E	10	 Gravity set, check valve, 4 needle-free valves 77", 52", 47" and 18" from 2-piece male luer lock Approximate set length: 119" 302 cm. approx. Approximate priming volume: 22 mL	50
C42011E 30-day lead time	20	 Gravity set, 1 needle-free valve 9" from 2-piece male luer lock Approximate set length: 104" 264 cm. approx. Approximate priming volume: 23 mL	100

12.00 Gravity sets

12.06.03 Primary administration sets | No port sets

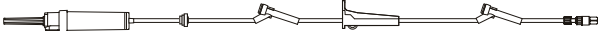
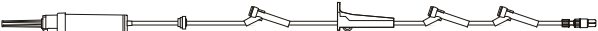
Administration sets for use as gravity sets. All sets are latex free. Some sets include MaxPlus needleless connectors and y-sites, SmartSite needle-free valves or VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
MA3127 30-day lead time	15	 Administration set, male spin lock Approximate set length: 85" 216 cm. approx.	 50
MA3131	10	 Administration set, male spin lock Approximate set length: 84" 213.5 cm. approx.	 50
42000	20	 Gravity set, 2-piece male luer lock Approximate set length: 107" 271 cm. approx. Approximate priming volume: 20 mL	100
11619231 30-day lead time	60	 Gravity set, check valve, spin male luer lock Approximate set length: 47" 119 cm. approx. Approximate priming volume: 14 mL	 100
11620791 30-day lead time	20	 Gravity set, check valve, spin male luer lock Approximate set length: 47" 119 cm. approx. Approximate priming volume: 14 mL	 100

12.00 Gravity sets

12.06.04 Primary administration sets | VersaSafe split septum sets

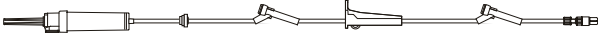
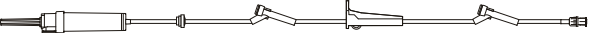
Administration sets for use as gravity sets. All sets are latex free. Some sets include MaxPlus needleless connectors and y-sites, SmartSite needle-free valves or VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
42274E 30-day lead time	10	 Gravity set, check valve, 1 needle-free valve 84" from male luer lock, 1 split septum port 6" from 2-piece male luer lock Approximate set length: 104" 264 cm. approx. Approximate priming volume: 21 mL	50
10793510 30-day lead time	20	 Gravity set, 1 split septum port 9" from 2-piece male luer lock Approximate set length: 104" 264 cm. approx. Approximate priming volume: 22 mL	 100
10796811	20	 Gravity set, check valve, 2 split septum ports 87" and 9" from 2-piece male luer lock Approximate set length: 104" 264 cm. approx. Approximate priming volume: 22 mL	 100
10796812 30-day lead time	20	 Gravity set, check valve, 3 split septum ports 45", 32" and 9" from 2-piece male luer lock Approximate set length: 105" 267 cm. approx. Approximate priming volume: 23 mL	 100

12.00 Gravity sets

12.06.04 Primary administration sets | VersaSafe split septum sets

Administration sets for use as gravity sets. All sets are latex free. Some sets include MaxPlus needleless connectors and y-sites, SmartSite needle-free valves or VersaSafe split septum ports.

Model #	Drops	Description	Case quantity
10796813	 60	 Gravity set, check valve, 2 split septum ports 87" and 9" from 2-piece male luer lock Approximate set length: 104" 264 cm. approx. Approximate priming volume: 23 mL	 100
10807856	 10	 Gravity set, check valve, 2 split septum ports 78" and 4" from fixed male luer lock Approximate set length: 104" 264 cm. approx. Approximate priming volume: 23 mL	 100

Index

Set #	Page #	Set #	Page #	Set #	Page #	Set #	Page #
19MMTP.....	5	2430-0500.....	124	20337E.....	86	30920-07.....	156
2000E.....	1, 5, 55	2432-0007.....	124	20338E.....	86	31202E.....	63
2106-0500.....	104	2433-0007.....	147	20350E.....	21, 44	31262E.....	63
2110-0500.....	153	2434-0007.....	124	20511E.....	30	37262E.....	63
2111-0500.....	153	2440-0600.....	105	20519E.....	30	4900-0000.....	206
2120-0007.....	154	2441-0007.....	105	20558E.....	1	41134E.....	237
2120-0500.....	153	2441-0600.....	105	21028E.....	21	41135E.....	237
2121-0500.....	130, 154	2442-0007.....	105, 111	21059E.....	61	41137E.....	237
2125-0500.....	141	2443-0600.....	105, 124	22059E.....	61	41173E.....	237
2126-0007.....	154	2444-0007.....	110, 118	25202E.....	30	41177E.....	237
2126-0500.....	153	2447-0007.....	106, 141	25203.....	34	42000.....	242
2129-0500.....	153	2447-0600.....	106, 141	25205.....	34	42002-07.....	226
2130-0500.....	130	2448-0600.....	106	25206.....	34	42006-07.....	226
2131-0500.....	120, 133	2449-0600.....	106	25207E.....	30	42081E.....	222
2140-0600.....	111	2450-0500.....	99	25208E.....	30	42082E.....	222
2141-0600.....	111	2455-0500.....	147	28004E.....	197	42085.....	223
2147-0600.....	111, 141	2461-0007.....	117	28018.....	198	42133E.....	237
2177-0000.....	123	2462-0007.....	117	28026.....	198	42163E.....	218
2200-0500.....	152	2463-0007.....	113	28035-0500.....	200	42193E-0500.....	238
2201.....	5	2465-0007.....	11, 113, 125, 139	28053.....	192	42233E.....	238
2202-0007.....	132	2466-0007.....	11, 118, 139	28063E.....	189	42273E.....	238
2202-0500.....	132	2477-0000.....	121	28080.....	194	42274E.....	238, 243
2202E.....	5	2477-0007.....	121	28080E.....	193	42293E.....	238
2203-0500.....	135, 138	2478-0000.....	121	28117.....	200	42433E.....	239
2203E.....	5	2929-0500.....	153	28117E.....	197	42434E.....	239
2204-0007.....	152	2941-0500.....	152	28125.....	198	42493E.....	239
2205E.....	5	2942-0500.....	152	28208.....	190	42502E.....	211
2206-0007.....	137	20019E.....	55, 85	28216.....	190, 196	42503E.....	211
2260-0500.....	118	20021E.....	85	28265.....	200	42504E.....	211
2278-0500.....	122	20022.....	97	28273E.....	195	42511E.....	211
2280-0000.....	144	20027E.....	20	28293E.....	197	42519E.....	211
2300E-0500.....	7	20028E.....	20	28493E.....	197	42521E.....	212
2309E.....	7	20029E.....	20	30202E.....	61	42933E.....	224
2401-0500.....	145	20035E.....	85	30203E.....	86	47177E.....	239
2403-0000.....	145	20037E.....	85	30204E.....	62	47233E.....	239
2403-0007.....	145	20038E.....	85	30205E.....	62	70027.....	207
2406-0500.....	99	20039E.....	55, 85	30212E.....	62	70200.....	3, 7
2408-0500.....	99	20041E.....	40, 55	30262E.....	62	70804.....	3, 7
2410-0500.....	145	20043E.....	40	30263E.....	62	71633E.....	181
2411-0500.....	145	20049E.....	86	30822.....	97	71980-0500.....	169
2412-0500.....	124	20051E.....	40	30842E.....	62	72001E.....	181
2414-0500.....	134	20053E.....	40	30843E.....	160	72003.....	185
2420-0007.....	145	20057E.....	60	30853.....	160	72012M.....	186
2420-0500.....	146	20059E.....	60	30862-0500.....	98	72013E.....	181
2421-0500.....	146	20060E.....	60	30863.....	160	72013KE.....	181
2425-0500.....	141	20061E.....	60	30873.....	160	72013KM.....	186
2426-0007.....	146	20062E.....	61	30883.....	161	72023E.....	181
2426-0500.....	146	20127E.....	20	30893.....	45, 161	72023KE.....	182
2428-0007.....	99	20128E.....	20	30910.....	98	72023M.....	186
2429-0007.....	146	20129E.....	25	30914.....	98	72033E.....	182
2429-0500.....	147	20159E.....	61	30916-07.....	156	72033M.....	186

Index

Set #	Page #	Set #	Page #	Set #	Page #	Set #	Page #
72103E.....	163	10010761.....	137	10013890.....	118	10061661.....	13, 115, 140
72103M.....	165	10010871.....	107, 113	10013902.....	21, 44	10062818.....	119, 122
72104KE.....	163	10010903.....	222	10014035.....	107	10073124.....	240
72104KM.....	165	10010912.....	63	10014081.....	201	10074281.....	108, 115
72163E.....	163	10010971.....	64	10014472.....	35	10108961.....	101
72163E-0003.....	163	10010977.....	64	10014692.....	64	10108964.....	31
72163M.....	165	10010983.....	87	10014766.....	87	10137405.....	115
72165E.....	164	10010985.....	201	10014813.....	42	10137407.....	223
72193E.....	164, 171	10011087.....	219, 229	10014854.....	171	10142568.....	101
72213.....	207	10011253.....	55, 87	10014855.....	12, 114, 139	10178573.....	148
72213N.....	207	10011265.....	212	10014881.....	10, 203	10237425.....	27
72215N.....	207	10011301.....	137	10014881T.....	10	10241342.....	45
72303-0500.....	176	10011433.....	191	10014913.....	156	10321213.....	13, 140
72313E.....	166, 174	10011462.....	154	10014914.....	156	10321214.....	157
72323E.....	171	10011611.....	212, 224	10014915.....	156	10373772.....	93
72323M.....	173	10011652.....	142	10014916.....	157	10382963.....	101
72433E.....	182	10011865.....	25	10014917.....	157	10387509.....	223
72544.....	176, 178	10011881.....	92	10014918.....	157	10404198.....	148
72623KE.....	182	10012143.....	64	10015012.....	108, 114, 125	10406194.....	116, 126
72633E.....	182	10012144.....	147	10015048.....	114	10406890.....	87
72900E.....	183	10012182.....	100	10015104.....	221, 224	10408730.....	149
72900M.....	187	10012212.....	170	10015171.....	240	10425228.....	213
72923.....	167, 172	10012213.....	68	10015294.....	148	10425229.....	213
72923E.....	166	10012240-0500.....	4	10015312.....	100	10561554.....	134
72943.....	185	10012241-0500.....	4	10015414.....	122	10657768.....	185
72953.....	167	10012283.....	107, 125, 142	10015415.....	169	10689602.....	159
72963E.....	183	10012293.....	107, 142	10015485.....	69	10779313.....	112
72964E.....	183	10012392-0500.....	4	10015487.....	35	10784614.....	14, 167, 179
72980E.....	168	10012463.....	27	10015488.....	87	10793510.....	243
72980M.....	170	10012476.....	40	10015492.....	223	10794983.....	93
72981E.....	168	10012496.....	27	10015501.....	205	10796811.....	243
72983.....	180	10012529.....	176, 178	10015612.....	157	10796812.....	243
79980E.....	168	10012562.....	34	10015645.....	148	10796813.....	244
82005E.....	218, 229	10012564.....	219, 229	10015649.....	100	10796814.....	66
82104E.....	218	10012645.....	147	10015817.....	92	10796815.....	22
82113E.....	218	10012812.....	177	10015861.....	12, 115, 140	10798696.....	158
82114E.....	218	10012866.....	25	10015862.....	12, 115, 140	10798697.....	158
82115E.....	219, 229	10013034.....	100	10015863.....	14, 166, 179	10798703.....	158
9385-0500.....	206	10013037.....	121	10015896.....	101	10800173.....	161
9387-0500.....	206	10013072.....	114	10016072.....	31	10800174.....	161
9391-0200.....	3	10013182.....	183	10016073.....	207	10800175.....	161
9987-0000.....	206	10013186.....	147	10016242.....	201	10800176.....	162
10010453.....	113, 131	10013203.....	34	10026645.....	14, 167, 179	10800177.....	162
10010454.....	113, 125	10013361.....	11, 139	10026685.....	69	10802688.....	240
10010483.....	144	10013363.....	14, 179	10031773.....	148	10807851.....	22
10010511.....	86	10013364.....	10, 203	10031776.....	93	10807853.....	22
10010525.....	92	10013364T.....	10	10037031.....	219, 227, 229	10807854.....	66
10010541.....	106	10013365.....	4	10037032.....	219, 227, 230	10807855.....	89
10010570.....	25	10013784.....	41	10050186.....	212	10807856.....	244
10010721.....	187	10013854.....	11, 118	10050187.....	213	10807857.....	220
10010741.....	239	10013889.....	114	10053857.....	240	10807859.....	89

Index

Set #	Page #	Set #	Page #	Set #	Page #	Set #	Page #
10813621	102	11499816	187	C42001	228	ME1232	43
10817920	108, 142	11499817	170	C42002E	241	ME1233	43
10821753	108, 116	11501182	187	C42006E	214	ME1234	91
10839681	93	11501183	188	C42007E	241	ME1242	68
10844551	223	11522558	143	C42010E	215	ME1246	68
10849550	220	11532269	116, 126	C42011E	241	ME1247	68
10863358	135	11582773	102	C42012E	215	ME1248	68
10869591	223	11582774	208	C42013E	215, 216	ME1250	23
10879046	213	11590099	64	C70101	202	ME1251	91
10879047	149	11590100	151	C72101	185	ME1253	91
10879391	149	11607532	220	C72102E	184	ME1254	23
10896354	93	11607704	151	C72108E	184	ME1255	23
10912433	158	11607787	109	C72109E	184	ME1256	23
10927766	149	11608375	69	C72110E	164	ME2010	92
10933805	149	11609481	116, 126	C72113E	164	ME2017	92
10934345	102	11609826	240	M4018	26	ME2020	97
10942011	152	11610341	10, 203	M4020	26	ME2050	92
10942968	183	11612593	129	M4025	33	ME2059	97
11045328	110, 129	11613191	109, 117, 127, 143	M4026	33	ME2064	24
11059727	214	11619231	242	M4058	33	ME2068	68
11059728	214	11619427	241	M4060	33	ME2069	24
11059729	102	11620791	242	MA3090	234	ME2070	24
11063735	110	11627697	214	MA3127	242	ME2071	24
11088483	94	8881135609	162	MA3131	242	ME5301	56
11089893	199	8881682101	162	MA3137	226	ME5302	56
11118122	109, 126	8881682119	162	MA3138	226	ME5303	56
11130148	120, 130	A492	206	MB3105	234	ME5304	56
11132982	175	C20014	45, 69	MB3107	234	MFS100	231
11133698	136	C20016	94	MB3110	226	MFS102	231
11141556	165, 173	C20022	69	MBC6000	2	MFS104	44
11147991	154	C20025	69	MBC6008	2	MFS106	44
11147992	104	C20032	98	MC3102	234	MFS107	44
11147994	216	C24101E	127, 134, 138	ME1001	90	MFS120	226, 231
11171447	150	C24103E	127	ME1021	90	MFS121	231
11202651	89	C24104E	103	ME1028	90	MFS124	231
11202652	89	C24105E	151	ME1032	90	MFS125	232
11202653	130	C24106E	103	ME1046	43	MFS132	232
11225954	171	C24107E	103	ME1047	43	MFS133	232
11258587	167	C24111E	151	ME1048	90	MFS134	232
11258588	155	C24112	152	ME1050	91	MFS135	232
11275183	22	C24116E	151	ME1055	67	MFS136	233
11287205	150	C24117	119	ME1069	67	MFS137	233
11325478	155	C24119E	128	ME1110	91	MFS139	233
11325479	186	C25002E	88	ME1214	43	MFS140	233
11401542	112	C25004E	65	ME1222	97	MFS141	44
11404930	150	C25005E	88	ME1223	67	MM4000	7, 26
11419365	102	C25006	98	ME1224	67	MM4021	33
11426964	119	C25012	94	ME1225	23	MM4024	32
11426965	150	C25013E	41	ME1227	91	MM4300-1	28
11448964	208	C25015E	65	ME1229	67	MP1000	1
11484001	150	C30006	94	ME1230	67	MP1000A	1

Index

Set #	Page #	Set #	Page #	Set #	Page #	Set #	Page #
MP1000-C.....	1, 46	MP9017-C.....	77	MP9268-C.....	96	MX4439.....	29
MP2000.....	74	MP9027.....	77	MP9269-C.....	17	MX4440.....	29
MP2000-C.....	74	MP9027-C.....	78	MP9270-C.....	18	MX4441.....	210
MP2002.....	74	MP9081-A.....	71	MP9271-C.....	18	MX4443.....	235
MP2002-C.....	74	MP9081-C.....	71	MP9272-C.....	18	MX4449.....	209
MP2005.....	70	MP9081-SF.....	71	MP9273-C.....	18	MX4450.....	29
MP2005-C.....	70	MP9101.....	78	MP9274-C.....	19	MX4452.....	209
MP2202.....	74	MP9202.....	57	MP9275-C.....	96	MX5301.....	53
MP2202-C.....	75	MP9202-C.....	57	MPC-1A.....	3	MX5302.....	54
MP4000-C.....	28	MP9204.....	16	MPX5300-C.....	51	MX9037.....	58
MP4001-C.....	15, 28	MP9208-C.....	38	MPX5301-C.....	51	MX9059.....	58
MP5301.....	46	MP9209.....	16	MPX5302-C.....	51	MX9060.....	73
MP5301-A.....	46	MP9212-C.....	16	MPX5303-C.....	52	MX9076.....	58
MP5301-C.....	46	MP9213-C.....	71	MPX5304-C.....	52	MX9107.....	235
MP5302.....	47	MP9216-C.....	39	MPX5305-C.....	52	MX9128.....	235
MP5302-A.....	47	MP9218-C.....	78	MPX9100.....	72	MX9164.....	73
MP5302-C.....	47	MP9220-C.....	78	MPX9101.....	82	MX9165.....	83
MP5303.....	47	MP9221-C.....	39	MPX9101-C.....	82	MX9166.....	19
MP5303-A.....	48	MP9224.....	78	MPX9102-C.....	72	MX9167.....	84
MP5303-C.....	48	MP9229.....	79	MPX9102-SF-C.....	72	MX9171.....	19
MP5304-C.....	48	MP9230.....	57	MPX9103.....	72	MX9172.....	58
MP5305-C.....	48	MP9232-A.....	79	MPX9104.....	82	MX9175.....	59
MP5310-A.....	49	MP9232-C.....	79	MPX9105.....	83	MX9178.....	59
MP5310-C.....	49	MP9233-C.....	79	MPX9105-C.....	83	MX9181.....	84
MP5311-C.....	49	MP9234-C.....	17	MPX9107-C.....	83	MX9182.....	59
MP5312-A.....	49	MP9235-C.....	80	MPXT1001-C.....	36, 52	MX9407.....	217
MP5312-C.....	50	MP9236-A.....	80	MPXT1003-C.....	36, 53	MX9412.....	235
MP5313-A.....	50	MP9236-C.....	80	MPXT1004-C.....	36	MX9414.....	235
MP5313-C.....	50	MP9238-C.....	57	MPXT1005-C.....	36	MX9416.....	235
MP5314-C.....	50	MP9240-C.....	80	MPXT1006.....	37	MX9417.....	236
MP5315-C.....	51	MP9246-C.....	57	MPXT5301-C.....	37, 53	MX9424.....	236
MP9001.....	75	MP9247-C.....	39	MPXT5302-C.....	37, 53	MX9433.....	236
MP9001-A.....	75	MP9249-C.....	95	MS3101.....	234	MX9963.....	236
MP9001-C.....	75	MP9250-C.....	80	MS3500.....	208	MX9968.....	236
MP9002.....	70	MP9251-C.....	81	MS3500-15.....	208	MX9998.....	236
MP9002-A.....	70	MP9252-C.....	81	MS3542.....	225	MXT1003.....	37, 54
MP9002-C.....	70	MP9253-C.....	81	MT6000.....	204	MXT1004.....	38
MP9004.....	75	MP9254-C.....	17	MT6004.....	234	MXT1005.....	38
MP9004-A.....	76	MP9255-C.....	81	MV0400.....	4	MXT1006.....	38, 54
MP9004-C.....	76	MP9256-C.....	71	MV0500.....	8	MY0240.....	3
MP9004-NRC.....	76	MP9257-C.....	81	MV0510.....	8	MY3001.....	9
MP9008.....	15	MP9258-C.....	95	MV0600.....	8	MY3003.....	9
MP9009.....	15	MP9259-C.....	72	MV0700.....	8	MY3004.....	9
MP9009-C.....	16	MP9260-A.....	82	MX1002.....	1	MY8003.....	6
MP9014.....	76	MP9262-C.....	82	MX4410.....	210	MY8005.....	6
MP9014-A.....	76	MP9264-C.....	17	MX4412.....	210	MY8010.....	6
MP9014-C.....	77	MP9265-C.....	95	MX4414.....	29	MY8020.....	6
MP9017.....	77	MP9266-C.....	95	MX4416.....	217	MY8060.....	6
MP9017-B.....	77	MP9267-C.....	96	MX4436.....	210		

© 2013 CareFusion Corporation or one of its subsidiaries.
All rights reserved. AccuSlide, Alaris, MedSystem III, MaxPlus,
MaxPlus clear, MaxGuard, SmartSite, Texium and VersaSafe are
trademarks or registered trademarks of CareFusion Corporation
or one of its subsidiaries. All other trademarks are the property
of their respective owners. IV0033 (0113)

CareFusion
San Diego, CA

carefusion.com

