

Using CareFusion Surgical Clippers

Remember:

- Wet clip whenever possible to help prevent airborne skin cells and loose hair dispersal, which can contaminate the wound site.
- Hold the skin taut with one hand while clipping with the other, ensuring a smooth surface for clipping. This technique is especially important when clipping loose skin areas. Avoid skin tags and other inconsistencies with the skin’s surface.
- Always rest the base of the clipping blade flat on the patient’s skin. Do not change the angle or “toe in” the blade in an attempt to get a closer clip.
- Maintain a consistent grip on the clipper handle. Go over the area thoroughly on the first attempt to ensure clipping accuracy and precision.
- Always move the clippers away from your body with a gentle push. Do not pull the clippers toward you across the patient’s skin, and never apply greater force than needed.



Ordering information

Cat. no.	Description	Qty. cs.
5513E	Rechargeable surgical clipper	1
5514A	Charging adapter	1
4406	General purpose blade	50
4412A	Neuro blade	20
4403A	SensiClip® blade	20

To order, contact your local representative or call 800.523.0502.

* CareFusion Surgical Clippers can be cleaned with a mild disinfectant or a disinfectant wipe. Disinfectants containing benzyl alcohol can cause chemical damage to the external surface of the clipper handle or charger. This could void the warranty on the device. Submersion cannot exceed one meter in depth for up to 30 minutes.

** It is recommended that clinicians unplug the charger from a power source when the solid orange light is no longer illuminated, indicating the device is fully charged.

carefusion.com/surgicalclippers



To use CareFusion Surgical Clippers:

1. Attach the blade

With gloves on position a new blade on the top of the clipper head. Align the mark on the clipper blade to the arrow. Slide the blade toward the dot to secure it in place until you hear a click.



2. Clip the hair

Hold the clipper using a pencil grip or natural overhand grip with the CareFusion logo facing up. Stretch the skin, rest the clipping blade flat on the skin and clip against the grain of the hair.



3. Remove the blade

Turn the clipper off, position the clipper blade facing downward over a sharps trash receptacle and use your thumb to push the blade forward along the blade frame.



4. Clean the clipper handle

Wash the clipper handle thoroughly with soap and water, and then wipe the device with surface disinfectant or soak the handle in an antimicrobial agent for no more than 30 minutes.*



5. Charge and store

Place the clipper handle in an upright position in the charging adapter. Lithium-ion battery technology facilitates longer runtime and improves energy efficiency, and a battery life indicator light alerts clinicians when to charge the unit.**

Instructions apply to all CareFusion Surgical Clipper blades.

