



BD Vacutainer® Eclipse™ Blood Collection Needle

with Pre-Attached Holder

Usage of Product

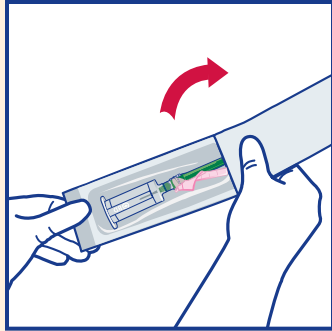


1. Ready to use right out of the package, with no assembly required!

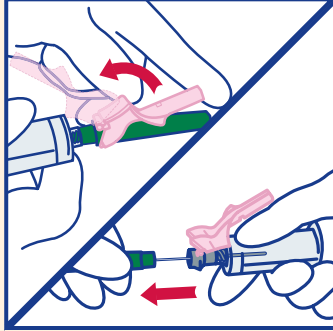


2a. Gently position pink safety shield straight back toward the holder.

2b. Twist and pull colored needle cap straight off.

Note: The needle bevel is always in position for venipuncture when the pink safety shield is facing up. **DO NOT** twist or rotate the pink safety shield.

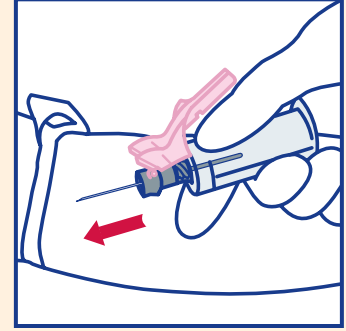


3. Perform venipuncture according to your facility's established procedures.

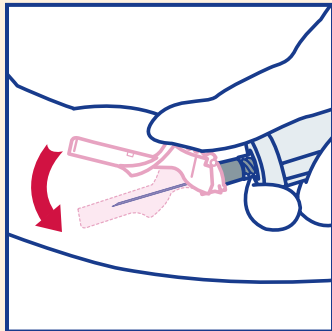


4. Immediately after removing needle from vein, position thumb squarely on pink safety shield thumb pad and push pink safety shield forward to cover needle. An audible click may be heard. Lock shield into place and inspect.

DO NOT attempt to engage safety shield by pressing against a hard surface.

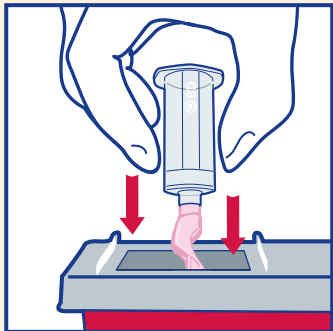

5. Discard immediately into an approved sharps disposal container. **DO NOT** remove needle from holder. Dispose of the needle and holder as one unit into nearest sharps container. **DO NOT REUSE.**

BD Vacutainer® Eclipse Blood Collection Needle with Pre-Attached Holder

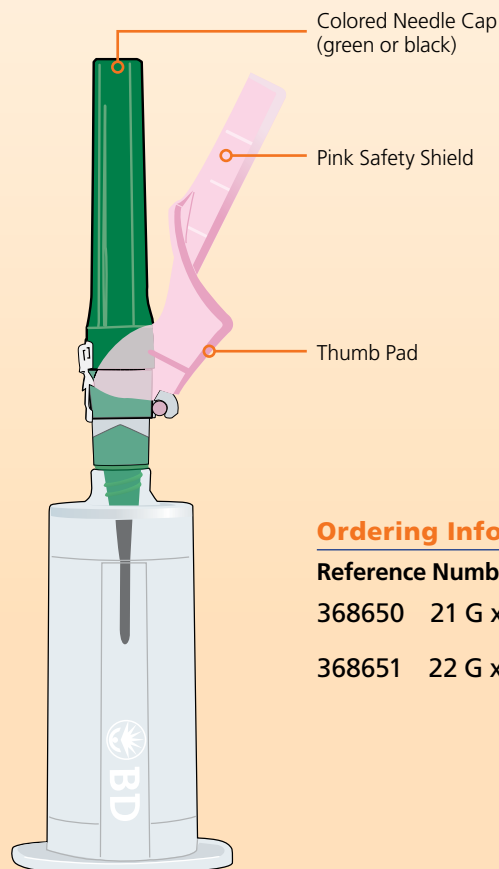

Ordering Information

Reference Numbers:

368650 21 G x 1 ¼"

368651 22 G x 1 ¼"

FOR SINGLE USE ONLY

BD Global Technical Services: 1.800.631.0174


BD Customer Service: 1.888.237.2762

www.bd.com/vacutainer


BD Vacutainer® Eclipse™ Blood Collection Needle

with BD Vacutainer® One Use Holder


Usage of Product

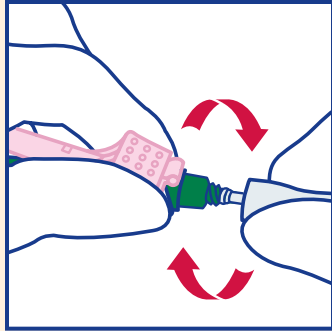

1. Holding both pink safety shield and colored needle cap, twist and remove white needle cap.

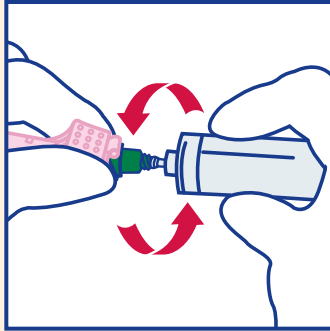


2. While holding the pink safety shield and the colored needle cap firmly, screw holder onto needle until it fits securely and the needle is fully seated onto the holder.

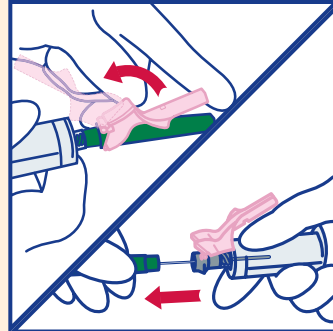


3a. Gently position pink safety shield straight back toward the holder.

3b. Twist and pull colored needle cap straight off.


Note: The needle bevel is always in position for venipuncture when the pink safety shield is facing up. **DO NOT** twist or rotate the pink safety shield.

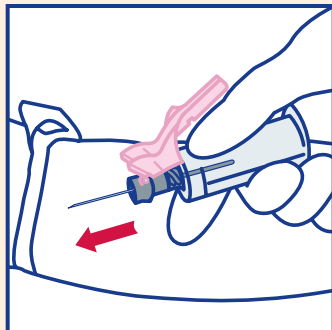

4. Perform venipuncture according to your facility's established procedures.

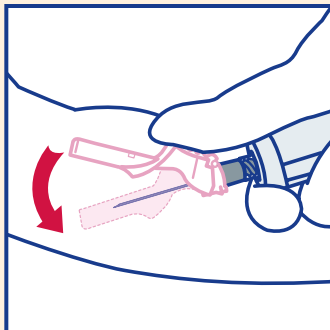


5. Immediately after removing needle from vein, position thumb squarely on pink safety shield thumb pad and push pink safety shield forward to cover needle. An audible click may be heard. Lock shield into place and inspect. **DO NOT** attempt to engage safety shield by pressing against a hard surface.

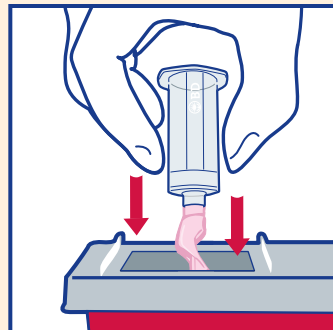

6. Discard immediately into an approved sharps disposal container. **DO NOT** remove needle from holder. Dispose of the needle and holder as one unit into nearest sharps container. **DO NOT REUSE.**

BD Vacutainer® Eclipse Needle and One Use Holder

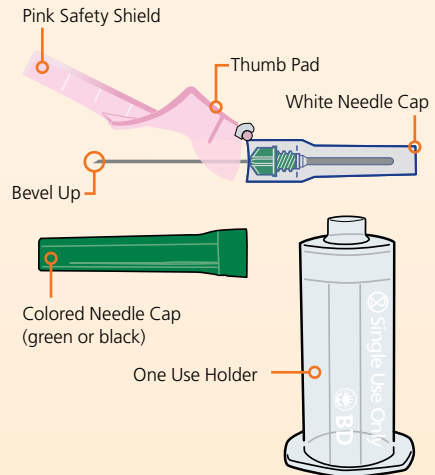

Ordering Information

Reference Numbers:

BD Vacutainer® One Use Holder


364815

BD Vacutainer® Eclipse™
Blood Collection Needles

368607 21 G x 1 ¼"

368608 22 G x 1 ¼"

Product Storage

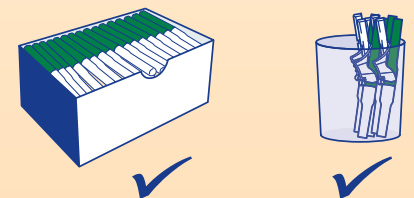

It is recommended that product be dispensed directly from BD packaging. If you choose to dispense outside of BD packaging, please ensure that all green or black colored caps are facing up in container. Fit product loosely in container.

FOR SINGLE USE ONLY

BD Global Technical Services: 1.800.631.0174

BD Customer Service: 1.888.237.2762

www.bd.com/vacutainer

IMPORTANT SAFETY TIPS

- **DO NOT REUSE HOLDER**
- Holder threads are designed to be used **ONLY ONCE**.
- Reuse of holder is potentially unsafe and could result in serious injury or exposure to bloodborne pathogens.
- Dispose of needle and holder as a single unit.