


BD Eclipse™ Needle

Important usage guidelines

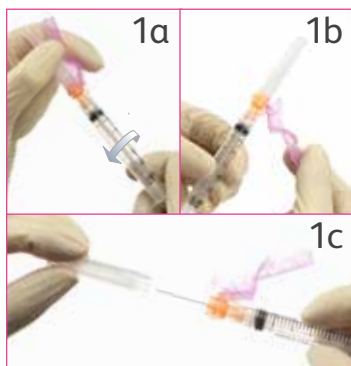

1. Attachment

- 1a. Attach the BD Eclipse needle to any standard luer lock hypodermic syringe by twisting until firmly seated.
- 1b. Pull back the safety cover toward the syringe and away from the needle.
- 1c. Grasp the syringe with one hand and with the other pull the clear needle shield straight off.



2. Aspiration


Draw up medication in accordance with your institution's protocol.

NOTE: If changing needles, activate safety cover prior to removing the needle from the syringe.



3. Injection

Administer medication in accordance with your institution's protocol. For user convenience, the needle's "bevel up" position is oriented to the safety cover.

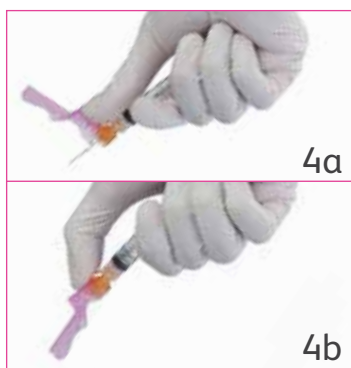

4. Activation

After injection, immediately activate safety cover as follows:

- 4a. Using the hand holding the syringe, center thumb or forefinger on the finger pad area of the safety cover.
- 4b. Push the cover forward over the needle until you hear or feel it lock.

Use one-handed technique and activate away from self and others. Visually confirm that the needle tip is covered. After activation or if unable to activate, immediately discard into an approved sharps collector after single use.

Remember

- The BD Eclipse needle is not to be used with luer slip syringes as the needle may become disengaged from the syringe when activating the safety cover.
- Follow your institution's protocol for aseptic technique and skin preparation.
- Follow your institution's policy for safe disposal of all medical waste.
- Activation of the protective mechanism may cause minimal splatter of any fluid that is remaining in the needle after injection.