

BD Insyte™ Autoguard™ BC

SHIELDED IV CATHETER WITH BLOOD CONTROL TECHNOLOGY

>>> Tips for Success




Blood Return Visualization

Initial visualization
along catheter
(20 G - 24 G)

Secondary flash in
flash chamber

Blood will fill up to the septum
after the needle is no longer
in the lumen of the catheter

Secure Connection

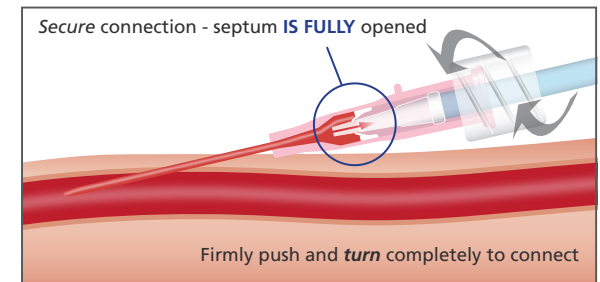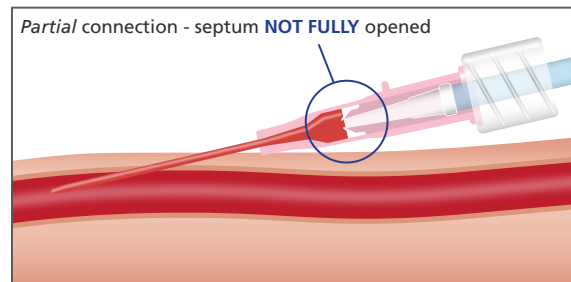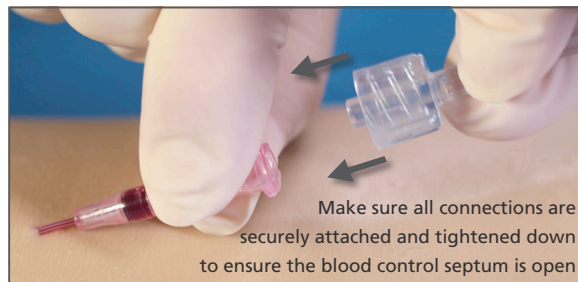

Blood Collection

- > Luer locking devices are preferred
- > Drawing via an extension allows you control of blood flow
- > If dripping blood into the collection tube, a male Luer must be connected and then disconnected in order to open the fluid path and allow blood flow
(once a male Luer has been inserted, the one-time valve is open)



BD Medical
9450 South State Street
Sandy, Utah 84070
1.888.237.2762
www.bd.com/iagbc