

# BD Twinpak™

## Dual Cannula Device



Helping all people  
live healthy lives

## Simplifying Needleless IV Access with a Truly Universal Device

The BD Twinpak is a truly universal IV access device, offering two methods of performance over traditional stand-alone devices. It features two cannulas in one shield: one red hub cannula for syringe filling and one clear blunt plastic cannula for safe split-septum access. The BD Twinpak Device is designed to help streamline IV work flow and reduce the number of individual pieces, thereby saving time and reducing costs.



# BD Twinpak™

## Dual Cannula Device

Working Together  
for Simplified,  
Convenient IV Access

### The Self-Contained Solution



**BD™ Red Hub  
Cannula Syringe  
Filling Device:**

*Provides Access to Vials*



**BD Twinpak Shield:**  
*Holds Both Cannulas Securely*



**BD™ Blunt  
Plastic Cannula:**  
*Allows for Safe, Needleless  
Access to Split-Septum  
Injection Sites*



**Green Cap:**  
*Protects / Covers  
BD Blunt Plastic  
Cannula Prior to Use*

### Red Hub Cannula

(Syringe Filling Device):

- **20 G Metal Cannula**  
provides versatility in syringe filling from vials, and for other multiple vial access needs (e.g., reconstitution)
- **Blunt Tip Design**  
provides a safer alternative to a traditional, sharp needle for syringe filling
- **Red Hub**  
easily identifies the syringe filling device

### Blunt Plastic Cannula

(with Green Cap):

- **Dual Side Ports**  
provide greater fluid dispersion in flushing and drug delivery
- **Center-point Design**  
features tapered tip for less resistance during insertion
- **Universally Compatible**  
with split-septum cannula access devices, such as Baxter Interlink®, Hospira LifeShield®, B. Braun SafeLine®, and Alaris VersaSafe® injection sites as well as vials designed for needleless access
- **Green Cap**  
designed to provide easy identification and a protective cover for clear BD Blunt Plastic Cannula Hub, and can later be used to cover the BD Red Hub Cannula



*Secure Red Hub Cannula to the syringe. Remove shield.*



*After filling, recap the the Red Hub Cannula into standing BD Twinpak Shield or use "scoop" method.*



*Flip the BD Twinpak and attach the clear BD Blunt Plastic Cannula for split-septum IV access.*

**BD Medical**  
9450 South State Street  
Sandy, Utah 84070  
1.888.237.2762  
[www.bd.com](http://www.bd.com)

Reference No.	Description	Case Quantity	
303390	BD Twinpak	100/Box	1000/Case
303391	3 mL Syringe with BD Twinpak	100/Box	800/Case
303392	5 mL Syringe with BD Twinpak	100/Box	400/Case
303393	10 mL Syringe with BD Twinpak	100/Box	400/Case

Interlink is a registered trademark of Baxter International, Inc.  
LifeShield is a registered trademark of Hospira.  
SafeLine is a registered trademark of B. Braun Medical, Inc.  
VersaSafe is a registered trademark of Alaris Medical Systems, Inc.

BD, BD Logo and BD Twinpak are trademarks of Becton, Dickinson and Company. ©2006 BD.

BD 2/06-0320 D15729-1