

BD Microtainer™ Tubes with Microgard™ Closure Tube Guide and Order of Draw

Catalog #/Closure Color	Additive	Mix by Inverting	Laboratory Use
-------------------------	----------	------------------	----------------



365974 Lavender

K₂ EDTA

10x

For whole blood hematology determinations. Tube inversions prevent clotting.



365965 Green

Lithium Heparin

10x

For plasma determinations in chemistry. Tube inversions prevent clotting.



365985 Mint Green

Lithium Heparin and Gel for plasma separation

10x

For plasma determinations in chemistry. Tube inversions prevent clotting.



365987 Mint Green



365992 Grey

NaF/Na₂ EDTA

10x

For glucose determinations. Tube inversions ensure proper mixing of additive and blood.



365967 Gold

Clot Activator and Gel for serum separation

5x

For serum determinations in chemistry.



365978 Gold



365963 Red

No additive

0x

For serum determinations in chemistry, serology and blood banking.


365976
Tube Extender
