

Ergonomics & 360° Dynamic Motion:

A Review of The Brewer Company's 135AS/135JS Series Saddle Stools



As dental hygienists, we all know the importance of ergonomics. Hygiene program instructors stressed its importance every moment they could. Ergonomics was stressed for a good reason; to prevent musculoskeletal disorders (MSD) and injury.

Dental hygienists are preventive specialists, yet often some don't do as much as they can to prevent injury to themselves until an injury has already occurred and the pain sets in. Instead of being proactive, many dental professionals are reactive, when it comes to the importance of ergonomics. There are stunning statistics that back up just how large a problem MSDs are for dental professionals:

- Depending on which population of dentists were studied (Australian, North American (Canada and USA), Greek, Swedish, Danish, and Saudi Arabian), the reported prevalence of MSD pain ranges from 29% to 72%. Numerous studies report that on average 2 out of 3 dentists experienced musculoskeletal pain.¹
- Nearly 1/3 of dentists who had to retire early were forced to do so due to disability (MSD).¹
- The reported prevalence of MSD pain and neuropathy in dental hygienists ranges from 60% to 96% depending on the research measures employed and specific population studied.²
- On average, 30% of dental assistants report neck pain.¹

Taking preventive measures before you have musculoskeletal disorder or injury is not only essential to your comfort, it's essential to your overall health, livelihood, and quality of life. As shown by the statistics above, MSDs can lead to premature retirement and shorten a dental professional's career.

MSDs also have a direct effect on a dental practice's production and revenue.

Regardless of if you are a dentist or dental hygienist, think what your daily production is and how much revenue can be lost due to just one day out of the office due to MSDs. Further, consider the numbers if you were out for an extended period of time due to surgery related to MSDs; this can be weeks or even months of lost production and revenue, not to mention loss of take home pay. Doctors, consider how smoothly your day of patient treatment will go if your assistant is out of the office due to MSD pain and/or injury; especially if you have a small practice and employ only one dental assistant. Even if a clinician does not take time off for pain or injury caused by MSDs, constant pain can limit daily production, which lowers total production, on an ongoing basis. ***Improper ergonomics and MSDs can have an effect on all members of the dental team.***

Beyond loss of production, if dental professionals are not actively addressing MSDs, it can lead to sub-standard productivity, disruptions, worker compensation claims, and lost referrals. Severe flare-ups of MSDs can lead to unexpected sick time to be taken, which means the rescheduling of patients and with that, especially when a hygienist isn't in-office, the potential loss of production from hygiene exams. MSDs can also lead to worker compensation claims that may have been avoided if an actively addressed. Further, extended absences by a member of the dental team might cost an office potential patient referrals due to the lack of consistency in staff members, and the loss of patient rapport and trust that ensues.



Kara Vavrosky, RDHEP, is a Co-founder & the Chief Content Officer of [*Today's RDH*](#), a digital dental hygiene publication for hygienist, by hygienists. She runs the popular Facebook page, [*Dental Hygiene with Kara RDH*](#).

A graduate of the Oregon Institute of Technology, Kara has a deep passion for spreading knowledge about the importance of oral health and how it relates to the entire body.


There are many ways to help improve ergonomics and lower the risk of MSDs

beyond proper patient positioning and proper posture. One of which is an ergonomic operator chair, and more specifically an operator chair that allows for a natural spine position and proper hip placement. The operator chair that can accomplish this is a saddle stool. The Brewer Company's 135AS/135JS Series saddle stools, in particular, are not only designed to ease the load on back muscles by tilting the pelvis forward to enhance posture and naturally align vertebrae and disks, which also relieves shoulder and neck tension, but it allows for thighs to be slanted downward at a 45-degree angle. This is important because dental professionals tend to lean forward. Leaning forward when the thighs are parallel to the floor, causes the lower back to flatten, causing pressure in spinal discs which can lead to MSDs.

Specs: I spent some time with the newest saddle stool models, the 135AS and 135JS by The Brewer Company. The stools are available in two seat sizes: the JS model with a wider seat, and the AS model with a narrower seat. The seats are a padded, one-piece design new to Brewer.

The wider seated, 135JS, is available in both a standard cylinder height and a high cylinder height. The narrower seated, 135AS, is available in both a standard cylinder height and a lower cylinder height. For this review, I tested both the JS model in the dove gray color (item 135JSG) and the AS model in the color of papyrus (item 135ASP), each with a standard cylinder height.

Fit: As a five foot, four-inch woman, with a small to medium build, I found the narrower 135AS stool fit my body size best. The 135JS stool with a wider seat was by no means uncomfortable; I just felt the larger seat size placed my feet and legs just a touch too far apart for my liking. At my shorter height, using the standard cylinder height, I needed to lower the stool as far as it could go to fit the length of my legs. Essentially, a shorter cylinder height would probably fit my height and leg length better.

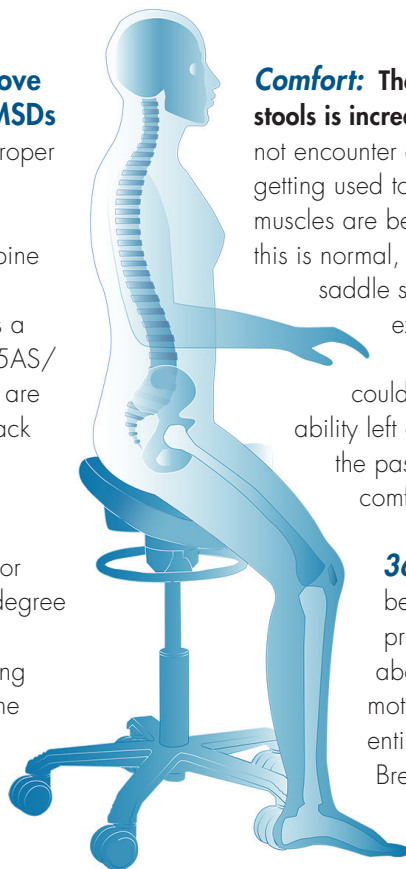

Comfort: The comfort of the 135AS and 135JS saddle stools is incredible. After four days of prolonged sitting, I did not encounter any sore spots or pressure points. Often, when getting used to a new saddle, some will feel a bit sore as muscles are being engaged that were not engaged before; this is normal, assuming you have a well fitted and adjusted saddle stool in the first place. Personally, I did not experience any soreness with the use of the saddle stools. I should note, not being sore could possibly be due to core-strength and balance ability left over from my ballet and horse riding days of the past. The padding on the seat also was perfectly comfortable for me.

360° Dynamic Motion: A huge reason I believe I did not feel sore or encounter any pressure points is due to the most amazing thing about these saddle stools: the 360° dynamic motion of the seat. 360° dynamic motion is entirely unique to the 135 saddle stools by Brewer. What this means is that the seat moves; not just tilting front to back and/or side to side, but in a full 360° motion.

The seat is not statically attached to the cylinder, so it moves with you. Even better, is that the 360° dynamic motion of the seat is adjustable. I started using the stool adjusted to slight dynamic motion. Each day I increased the motion as I became used to it.

In no way, did the 360° dynamic motion make the stool feel unstable. Instead, the dynamic motion made me feel like the stool was working and moving with me, not against me. When I moved and changed positions, the chair moved with me, without causing any pressure points.



Dynamic motion is important, especially to dental professionals, because "when you sit in a static operating posture without leaning on your chair's backrest, more than 50% of your body's muscles must contract to hold the body motionless while resisting gravity. The static forces resulting from these prolonged, static postures (PSPs) are much more taxing on the body than dynamic (moving) forces.

The resultant microtrauma from these PSPs includes muscle imbalances, muscle ischemia, trigger points, and spinal disc degeneration.³" The 360° dynamic motion of the Brewer saddle seats helps with circulation and improves the core muscles, reducing back pain and sitting fatigue. **Simply put, the 360° dynamic motion of the Brewer saddle stools is a game changer in ergonomic seating.**



Adjustability: Instead of reaching down and trying to find a lever to adjust the seat height and 360° dynamic movement, the 135AS and 135JS models have a ring under the upholstered seat. Pulling up on the ring, no matter which way I was facing, lowers or raises the stool height.



The ring easily turns clockwise & counter-clockwise to increase or decrease the amount of dynamic motion of the seat. I found adjusting the seat to be very simple.

Ergonomics: I found these stools to be very helpful in maintaining proper ergonomics and keeping my hips and spine in a natural, ergonomic position. With the natural curve my lower back remained in while sitting on the stool, it took more effort to slump than to sit up properly. With less ability to slump while sitting on the stool, I found that my personal struggle with the improper, head forward position I tend to fall into, was greatly alleviated. I cannot begin to describe the pain caused by the improper head forward position I experience; from headaches to neck & upper back pain, it's a real problem that I'm sure others can relate to. **For me, it was eye-opening to realize that allowing myself to keep the natural curve in my lower back, helped with keeping my head positioned as it should be.**

Aesthetics: In my opinion, the Brewer 135AS and 135JS are quality stools; upon sitting they feel sturdy and well-built. They not only feel like a quality piece of equipment but look like it too. The seats' upholstery on both models I tested, were well-stitched and made of quality material.

The 135AS and 135JS saddle stools are available in Brewer's three most popular UltraLeather upholstery colors: Raven Wing, which is a classic black; Dove Gray, which is a lighter gray; and Papyrus, which is a nice tan. The Dove Gray would look nice in an operatory decorated with cooler colors, whereas, the Papyrus would go nicely in an operatory with natural or warmer colors. Of course, the Raven Wing black will match any operatory.



Both series are available in your choice of three upholstery colors. The 135JS Jumper is shown in Raven Wing & Dove Gray. The 135AS Amazone is shown in Raven Wing & Papyrus.

In closing: As oral health professionals, we are all aware of the importance of ergonomics, as dentistry can be taxing on the body. The question is, are you doing all you can to prevent musculoskeletal disorders or just waiting and hoping “it won’t happen to you?” Beyond the importance of ergonomics as it relates to a clinician’s overall health, the “economics of ergonomics,” and the cost that MSDs can incur (lost production, medical expenses, replacement staff, workers’ comp, etc.), can significantly add up over time.

If you are ready to invest in your overall health and are in the market for ergonomic seating, such as a saddle stool, or if you want to upgrade your current saddle stool, in my opinion, The Brewer Company’s 135AS and 135JS saddle stools are a must try. ***I feel The Brewer Company hit the nail on the head with these saddle stools, especially with the 360° dynamic motion. As stated earlier, the 360° dynamic motion of the Brewer Score saddle stools are a game changer in ergonomic seating.***

For more information on the 135AS, [click here](#)



For more information on the 135JS, [click here](#)



References

1. Gupta, A., Ankola, A., Hebbal, M. (2013). Dental Ergonomics to Combat Musculoskeletal Disorders: A Review. International Journal of Occupational Safety and Ergonomics (JOSE) 2013, Vol. 19, No. 4, 561–571. Retrieved from <https://pdfs.semanticscholar.org/a4f7/98b3a6d1238a8d7d18ef40cb1e1f4cbbdcfc.pdf>.
2. Morse, T., et al. (2007). Musculoskeletal disorders of the neck and shoulder in dental hygienists and dental hygiene students. Journal of Dental Hygiene, Vol. 81, No. 1, January 2007. Retrieved from <https://pdfs.semanticscholar.org/b7d3/c7412378513ac2aec0f5a8062f5600f31fc9.pdf>.
3. Valachi, B. Move to Improve Your Health: The Research Behind Static Postures. (May, 2011). Dentistry Today. Retrieved from <http://www.dentistrytoday.com/ergonomics/5361-move-to-improve-your-health-the-research-behind-static-postures>

Brewer

The Power to Advance. Safely. Financially. Competitively.

www.brewercompany.com | P: 1.888.273.9371

The Brewer Company is an FDA registered & ISO 13485:2003 certified company.