

Summary



Tips	Maximum recommended modes	Fine setting*	Irrigation ml/min
BS1S	   	 1 to 3	
BS1L	   	 1 to 3	
BS2L	   	 1 to 3	
BS2R	   	 1 to 3	
BS4	   	 1 to 3	
BS5	   	 1 to 3	
BS6	   	 1 to 3	
SL1	   	 1 to 3	
SL2	   	 1 to 3	
SL3	   	 1 to 3	
SL4	   	 1 to 3	
SL5	   	 1 to 3	
TKW1	   	 3	
TKW2	   	 3	
TKW3	   	 3	
TKW4	   	 3	

*Not applicable to PIEZOTOME SOLO (LED).

Summary



Tips	Maximum recommended modes				Fine setting*	Irrigation ml/min
TKW5	D1	D2	D3	D4	 1	30-40
CS1	D1	D2	D3	D4	 1 to 3	80-100
CS2	D1	D2	D3	D4	 1 to 3	80-100
CS3	D1	D2	D3	D4	 1 to 3	80-100
CS4	D1	D2	D3	D4	 1 to 3	80-100
CS5	D1	D2	D3	D4	 1 to 3	80-100
CS6	D1	D2	D3	D4	 1 to 3	80-100
LC1	D1	D2	D3	D4	 1 to 3	60-80
LC190°	D1	D2	D3	D4	 1 to 3	60-80
LC2	D1	D2	D3	D4	 1 to 3	60-80
LC2L	D1	D2	D3	D4	 1 to 3	60-80
LC2R	D1	D2	D3	D4	 1 to 3	60-80
NINJA™	D1	D2	D3	D4	 1 to 3	60-80
CE1	D1	D2	D3	D4	 1 to 3	60-80
CE2	D1	D2	D3	D4	 1 to 3	60-80
CE3	D1	D2	D3	D4	 1 to 3	60-80

*Not applicable to PIEZOTOME SOLO (LED).