

# I AM VERSATILE

**airN GO**  
easy



**All clinical applications  
are easy with the  
all-in-one air polisher**

**ACTEON**

# SIMPLE AIR POLISHING

**airN GO**  
easy

DUAL FUNCTIONS

- "**SUPRA**" function" for supragingival polishing and stain removal
- "**PERIO**" function" for subgingival polishing, periodontal maintenance and peri-implant treatments



**A single handpiece to cover a range of care and treatment**

**AIR-N-GO® easy** combines essential functions and reliability:

- Fast start-up: direct connection to the high speed handpiece connector
- 360° swivel handpiece for fast and accurate treatment
- Parts can be dismantled for easy cleaning

## 4 Nozzles for Treatment

- Sodium bicarbonate "**CLASSIC**" powders
- Calcium carbonate "**PEARL**" powder
- Glycine "**PERIO**" powder



# A COMPLETE SOLUTION FOR IMPLANT MAINTENANCE



**NEWTRON®**



**NEWTRON**



**implantProtect**  
Pure Titanium

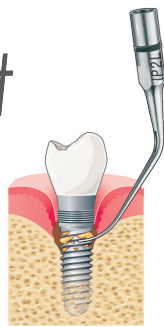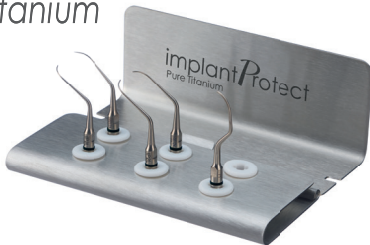

**IMPLANT PROTECT TIP KIT**



**AIR-N-GO®  
EASY + PERIO  
EASY KIT**

**DETECT  
DIAGNOSE  
TREAT**

**Contact ACTEON for a Detailed Implant Maintenance Protocol**

# PREVENT & FIGHT DISEASE

**AIR-N-GO®** nozzles provide continuous, accurate and controlled spray. These nozzles offer power and efficiency for the removal of supra and subgingival biofilm.

## AIR-N-GO® SUPRA NOZZLE

(standard)



*Clinical case Dr. P. ROUAS (France)*

- Suitable for any supra-gingival prophylactic treatment in combination with **AIR-N-GO®** "CLASSIC" or "PEARL" powders

## AIR-N-GO® PERIO easy NOZZLE

(optional Perio easy kit)



*Clinical case Dr. C. LIM (U.K.)*

- Innovative design in a "shoehorn" shape providing easy sub-gingival insertion
- For non-surgical, sub-gingival treatment of teeth and implants

## AIR-N-GO® PERIO MAINTENANCE NOZZLE

(optional)



*Clinical case Dr. C. LIM (U.K.)*

- Reduced consumption of **AIR-N-GO®** "PERIO" fine powder particles supragingivally
- For use on implants and appliances

## AIR-N-GO® PERIO NOZZLE

(optional)



*Clinical case Dr. A. PATEL (U.K.)*

- Sub-gingival use for surgical and non-surgical treatments (tunnelling and open flaps)
- Powerful double lateral spray
- For use on implants





# CUSTOMIZED SOLUTIONS FOR YOUR CLINICAL NEEDS

**airN GO**  
powder

## AIR-N-GO® "CLASSIC" POWDERS

(Sodium bicarbonate)  
(≈ 76µm average grain size)


### Complete range of active supragingival powders

- Cleaning provides instant results
- Remove stains and bacterial plaque, even in areas that are difficult to access such as pits and fissures
- Better patient comfort: gentle and efficient with limited bleeding during treatment



**5 fresh flavors based on 100% natural aromas or essential oils:**  
**Neutral, Cola, Raspberry, Peppermint and Lemon**

## AIR-N-GO® "PEARL" POWDER

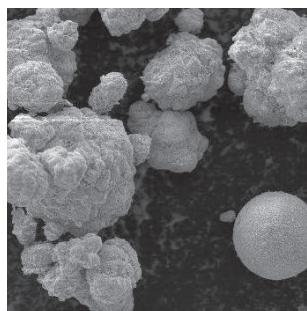


AIR-N-GO® "PEARL" (x1000)  
Source: ICMCB CNRS (Institute of Condensed Matter Chemistry - French National Centre for Scientific Research) S.E.M. photos

*Natural calcium carbonate-based*  
(≈ 55µm average grain size)

### Supragingival polishing powder for delicate tissue protection

- Reduction in pH acidity caused by tissue inflammation; return to clinically healthy buccal flora
- Restoration of the natural shine and pearliness of enamel
- Fast cleaning of difficult stains and composites

## AIR-N-GO® "PERIO" POWDER

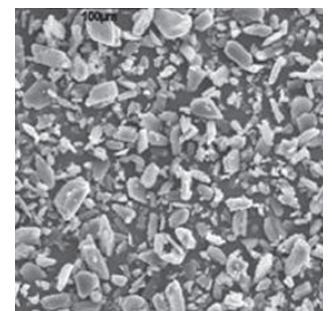

AIR-N-GO® "PERIO" (x1000)  
Source: ICMCB CNRS (Institute of Condensed Matter Chemistry - French National Centre for Scientific Research) S.E.M. photos

*Natural glycine-based*  
(≈ 25µm average grain size)

- Low abrasiveness preserving the treated surfaces and the most fragile anatomical areas
- Clinically proven to decrease biofilm subgingivally
- Safe to use on any implant surface

# airN GO








easy



GO SUPRA

Teeth and Implants

GO PERIO

Nozzles	Powders	Indications
 AIR-N-GO® easy SUPRA 120° Supplied as standard	 <b>"CLASSIC"</b> Sodium bicarbonate-based Average particle size: 76µm FLAVORS: Neutral, Cola, Raspberry, Peppermint and Lemon	<ul style="list-style-type: none"> <li>• Finishing after scaling</li> <li>• Preparation for whitening</li> <li>• Cleaning of interproximal areas, fissures and crevices</li> <li>• Preparation of the tooth surface before etching</li> <li>• Cleaning of orthodontic brackets</li> <li>• Removal of supragingival biofilm, plaque, stains and temporary cement residues</li> </ul>
	 <b>"PEARL"</b> Calcium carbonate-based Average particle size: 55µm FLAVOR : Neutral	Ultra-fast cleaning of difficult stains and composites with no risk of damaging them  Note: The absence of sodium makes it possible to treat sensitive patients or those following a salt-free diet
 AIR-N-GO® PERIO easy Supplied with the PERIO Kit (optional)	 <b>"PERIO"</b> Glycine-based Average particle size: 25µm FLAVOR : Neutral	<ul style="list-style-type: none"> <li>• Removal of the sub-gingival biofilm</li> <li>• Treatment of periodontal disease and peri-implantitis</li> </ul>
 AIR-N-GO® PERIO (optional)		Treatment of periodontal disease and peri-implantitis
 AIR-N-GO® PERIO MAINTENANCE (optional)		Periodontal maintenance