


■ Touches de contrôle

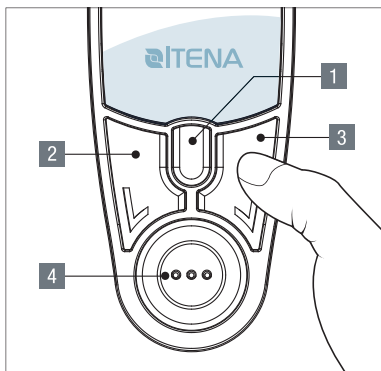

1 Touche TURBO MODE
Touche d'activation du Mode Turbo.

2 Touche TIME
Touche de sélection de la durée de l'exposition.

3 Touche MODE
Touche de sélection des modes.

4 Touche START
Touche de lancement de la polymérisation et de mise sous tension/hors tension de l'appareil.

Pressez n'importe quelle touche si vous souhaitez interrompre la polymérisation.


Six programmes •

Turbo
Normal
High
Soft start
Pulse
Pulse Soft start

Pressez la touche Mode pendant 1,5 seconde pour passer du mode graphique au mode textuel. Un signal sonore long retentit après la modification.

ITENA

■ Explanation Control Button

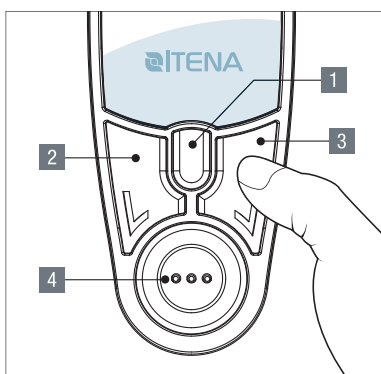

1 Turbo button
It is button for turbo output select.

2 Time button
It is button for lighting time select.

3 Mode button
It is button for mode select.

4 Start button
It is button for starting the exposure and power on / off.

Press any button if you want to cut off exposure.

Six program modes •

Turbo
Normal
High
Soft start
Pulse
Pulse Soft start

Press the Mode button for 1.5 seconds it would be changed to graph display and text display. A long beeps after converting is completed.

ITENA