# PROVITEMP

TEMPORARY CEMENT

## ProviTemp Temporary Cement

F

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## ProviTemp Temporary Cement

GB

Non-eugenol, resin based temporary cement containing Fluoride, Chlorhexidine and Potassium Nitrate.

### INDICATIONS:

- Temporary cementation of crowns, bridges, inlays, onlays and splints.

### PROPERTIES:

- ProviTemp has excellent flow to permit the restoration to be easily and completely seated. • It is strong enough to withstand the stresses of mastication, yet permits easy removal of the restoration. • ProviTemp does not contain essential oils so it will not harm temporary crowns or interfere with permanent cementation.

### INSTRUCTIONS FOR USE:

**1)** Complete tooth preparation and fabrication of the temporary restoration before extruding the mixed cement. **2)** Dry and isolate the prepared tooth. Be careful not to desiccate or over-dry the preparation. **3)** Remove the cap from double barrel automix syringe and set aside. Extrude a very small portion of the unmixed material to insure unobstructed flow from both the base and catalyst components of the respective barrels. **4)** Place the mixing tip onto syringe and lock in place with a quarter turn. **5)** Extrude a thin layer of the automixed cement onto the cleaned inside surface of the temporary restoration. You have approximately 45 seconds of working time after extruding the mixed cement. **6)** Firmly seat the temporary restoration in place. Initial gel-set occurs within 1.5 to 2 minutes after seating. Cleaning of excess cement should be initiated immediately after this gel-set phase. **7)** Final cement set occurs between 3 and 4 minutes after seating of the temporary restoration.

### TECHNICAL DATA:

Working time: 45 s\*  
Setting time: 3-4 min\*

\*These indications are obtained at the ideal mixing conditions of 24 °C (75 °F) and 40 % – 60 % relative humidity.

### RECOMMENDATIONS/ WARNINGS:

Some individuals may be sensitive to this product. If an allergic reaction occurs, discontinue use and remove from patient. The patient should consult a doctor.

### ESSENTIAL INGREDIENTS:

Fluoride • Chlorhexidine • Potassium Nitrate • Methacrylates • Urethan dimethacrylate • Polymerisation activator.

### STORAGE:

- Keep refrigerated. • For dental use only.

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