
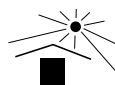
TempCanal™

enhanced
calcium hydroxide canal treatment paste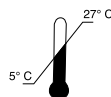

Non-setting, Radiopaque

PRODUCT DESCRIPTION AND INTENDED USE

TempCanal Enhanced is a calcium hydroxide-based paste used by dental professionals in root canal procedures and as pulpal dressing for all pulp capping, pulpotomy and pulpal curettage procedures. It is recommended for routine use between office visits to cleanse canals and for use in complicated cases when extended calcium hydroxide treatment is indicated.

The paste flows through 27-gauge x 1" (0.4 x 25mm), 2-side-vent, endodontic irrigation needles for direct, one-handed delivery into the root canal. Closed-end needles prevent dispensing beyond the apex. Non-drying formula will not clog 27-gauge needles.

TempCanal Enhanced has a pH>12 and provides the proven benefits of calcium hydroxide for successful root canal therapy. Use routinely as an intracanal dressing and for treatment of periapical lesions and abscesses, internal and external root resorption, traumatic injuries, root fractures and perforations, apexification, and weeping canals.

INSTRUCTIONS FOR USE

1. Prepare and cleanse canal in the usual manner.
2. Attach luer lock dispenser needle securely onto the syringe. (Alternatively, paste can be placed in the canal using a lentulo paste filler.)
3. Insert needle into the canal to the desired depth. The needle is closed end with two side vents.
4. Depress syringe plunger to dispense paste.
5. Fill canal while slowly withdrawing the needle.
6. Seal access opening with your preferred cement.
7. Discard needle and recap syringe immediately after use
8. TempCanal Enhanced is easily removed with file and irrigation.


Precautions: May be irritating to eyes, skin, respiratory system, gastrointestinal tract, or mucous membranes, especially in large quantities or with prolonged exposure. For use by dental professionals only. Wear gloves and safety glasses. Wash hands after use. Avoid unnecessary exposure.

Note: Apply disposable barrier sleeves/wraps over multiple-use dental dispensers before use with each patient.

For additional information, refer to the following websites:

<http://www.fda.gov/MedicalDevices/ProductsandMedicalProcedures/DentalProducts/ucm404472.htm>

CLINICAL SITUATIONS WHERE CALCIUM HYDROXIDE CAN BE USED

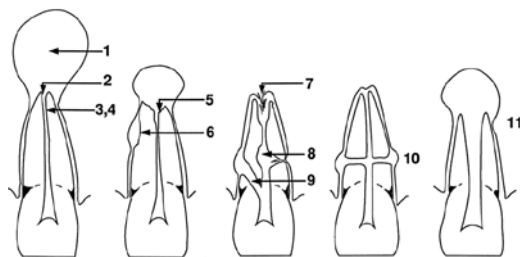- | | |
|---|---|
| 1. exudation control | 7. apical internal resorption |
| 2. large periapical lesions | 8. internal-external root resorption |
| 3. antibacterial dressing | 9. perforations |
| 4. temporary root filling | 10. transverse root fractures |
| 5. apical inflammatory resorption | 11. incompletely developed pulpless teeth |
| 6. inflammatory resorption following trauma | |

(Heithersay, G.S. Calcium Hydroxide in the Treatment of Pulpless Teeth with Associated Pathology, J. Brit Endo Society, 8[2]: 74-93 July 1975)

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