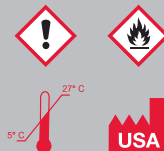
 4 x 1.2ml Dry Rite

 PULPDENT Corporation • 80 Oakland Street • Watertown, MA 02472 • U.S.A.
www.pulpdent.com • pulpdent@pulpdent.com

PULPDENT® Dry-Rite Drying Agent

Contains Ethyl Alcohol

For use in appropriate clinical situations where residual moisture may effect bonding or cementation.

PULPDENT DRY-RITE is a preparation of anhydrous ethyl alcohol with an organic solvent. It is packaged in 1.2 ml syringes for easy handling, and pre-bent needles are provided for controlled dispensing.

Note: Apply disposable barrier sleeves/wraps over multiple-use dental dispensers before use with each patient. For additional information, refer to: <http://www.fda.gov/MedicalDevices/ProductsandMedicalProcedures/DentalProducts/ucm404472.htm>

INDICATIONS

For use in clinical applications requiring totally dry surfaces. Use Dry-Rite prior to applying hydrophobic pit and fissure sealant or other materials adversely effected by the presence of moisture. It is also used to remove moisture from pits and fissures prior to acid etching.

PRECAUTIONS

1. Dry-Rite with ethyl alcohol is flammable. Handle with care and keep away from open flame.

2. Do not place Dry-Rite directly on dentin as this can desiccate the dentinal tubules.

DIRECTIONS FOR USE

1. Remove syringe cap and place a 22 gauge red dropper tip on the syringe.
2. Isolate the operative area in the mouth to maintain a clean dry field.
3. Rinse the operative area and air dry with clean, uncontaminated, oil-free and moisture-free compressed air.
4. Dispense a small amount of Dry-Rite on the desired surface. For maximum control of flow, do not push on plunger with thumb. It is recommended instead to hold syringe barrel between forefinger and thumb while resting the plunger in palm of hand. Gently pull back with thumb and forefinger to start flow. Use a new dropper tip for each patient.
5. To promote rapid drying, use a gentle flow of clean, uncontaminated air if desired.

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