

# Pulpdent<sup>®</sup> Cavity Liner

The original premixed calcium hydroxide-methylcellulose preparation in liquid form.

---

## THREE-WAY PROTECTION

### *Chemical – Physical – Biological*

---

#### CHEMICAL PROTECTION

Stops chemical irritation to pulp by neutralizing acids from all filling materials.

#### PHYSICAL PROTECTION

Creates protective barrier between filling and dentin.

#### BIOLOGICAL PROTECTION

Stimulates growth of sclerotic and reparative dentin.

#### USES

Pulpdent Cavity Liner is placed on all exposed dentin, under fillings, in indirect pulp capping procedures and on dentin after crown preparations.

#### DIRECTIONS

1. Shake bottle well making sure stainless steel mixing ball is moving.
2. Place a drop of Pulpdent Cavity Liner liquid onto slab and use explorer, spoon excavator or other instrument to place liner in cavity.
3. Dry with a slow stream of air. Pulpdent Cavity Liner appears chalk white when completely dry.
4. Wipe off excess from enamel margins with a damp pledget of cotton or with an explorer.



**PULPDENT Corporation**

80 Oakland Street • Watertown, MA 02472 • U.S.A.