


Administration of IV push medications or flushes

Alaris® System pump module

1. Pause the pump module first to administer an IV push medication or an IV flush through an injection port on an Alaris® System administration set.



2. Occlude the tubing above the injection port closest to the patient to allow the medication or flush to flow toward the patient without backing up the tubing toward the pump.

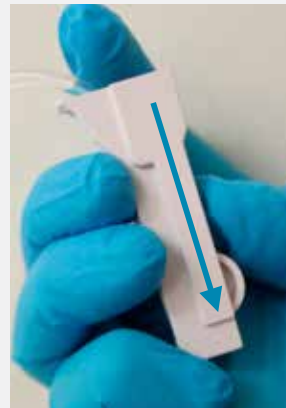

OR



3. Administer medication or flush.



4. Release the occluded tubing and select **RESTART**.



For product support, contact Customer Advocacy at **888.812.3266** or email customerfeedback@carefusion.com.

For technical support, contact Instrument Technical Support at **888.812.3229**.

For product orders, contact Customer Order Management at **888.482.4822**.

© 2013 CareFusion Corporation or one of its subsidiaries. All rights reserved. Alaris, CareFusion and the CareFusion logo are trademarks or registered trademarks of CareFusion Corporation or one of its subsidiaries. IF2204 (0713/QTY)

CareFusion
San Diego, CA

carefusion.com

