

CareFusion Surgical Clippers Neuro blade

Designed to easily remove thick hair on the scalp

Now, you can glide through removal of even the thickest hair on the scalp with the Neuro blade. As part of the CareFusion Surgical Clippers system, the Neuro blade provides fast, convenient removal of scalp hair, usually in a single pass, for lower chance for skin irritation. You and your patients will feel the difference.



The Neuro blade offers the following features:

- Wide blade for efficient cutting
- Utilizes existing clipper handle
- Disposable blade
- Wet/Dry capability
- Drop-in charging so clippers are always ready





General use blade shown.

1. Attach the blade

With gloves on, position a new blade on the top of the clipper head. Align the mark on the clipper blade to the arrow. Slide the blade toward the dot to secure it in place until you hear a click.



General use blade shown.

2. Remove the blade

Turn the clipper off, position the clipper blade facing downward over a sharps trash receptacle and use your thumb to push the blade forward along the blade frame.

Tips for clipping with the Neuro blade:

- Hold the clippers like a pen or pencil with the CareFusion logo facing up
- Rest the base of the blade flat on the patient's scalp
- Clip against the grain of the hair
- Clip only where necessary



General use blade shown.



Neuro blade

Model no.	4412A
Blade width	39.4 mm
Cutting ability (no. strands/second)	18,000
Blade cut height	.45 mm
Facility uses	Head and scalp procedures, including those in the ER

Ordering information

Cat. no.	Description	Qty. cs.
4412A	Neuro blade (for head hair and other very thick hair)	20
4406	General purpose blade	50
4403A	SensiClip blade (for sensitive areas)	20
5513E	Rechargeable surgical clipper	1
5514A	Charging adapter	1

For more information, including a video highlighting the features of the CareFusion Surgical Clippers, visit carefusion.com/surgicalclippers.

CareFusion
Vernon Hills, IL

carefusion.com

