

V.Mueller® Vascular Loops

For one time use in occluding, retracting, and identifying arteries, veins, tendons, and nerves in surgical procedures.

1. After removing loops from package, the surgeon may do as follows:
 - A. Retract - wrap around vessel. Clamp loop to drape or hang outside wound.
 - B. Occluding - wrap loop around vessel, nerve, ureter, or intestine.
 - C. Identify - wrap loop around vessel, nerve, ureter, or intestine.
2. At end of procedure, dispose of loop in an appropriate manner.

Note: Not Intended for Implantation



Do not use if package
is damaged.



Caution: For single use only.
Re-use may degrade product
performance or contribute to
cross contamination.

 Distributed by
CareFusion
McGaw Park, IL 60085 USA

 **CareFusion**

Made in USA

36-5005A